


R7254 - R7256 - R7257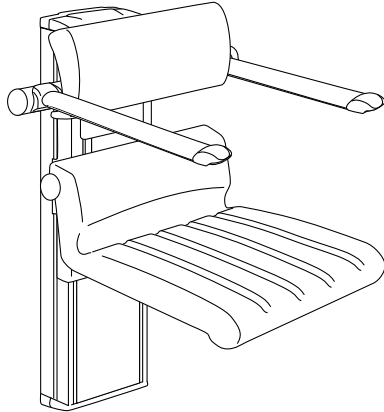


R7255 - R7258 - R7259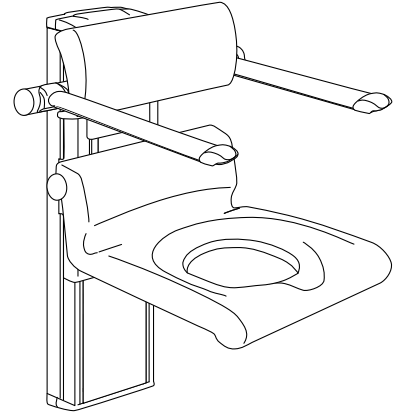

R7654 - R7656 - R7657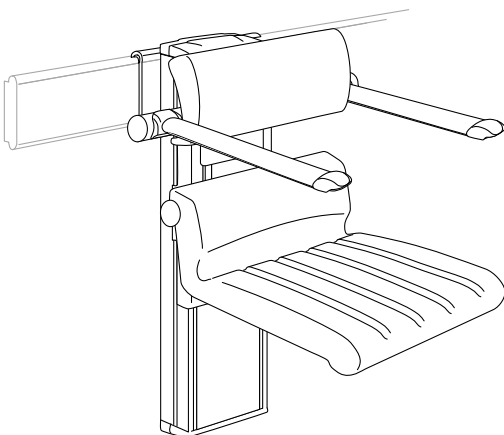





R7655 - R7658 - R7659

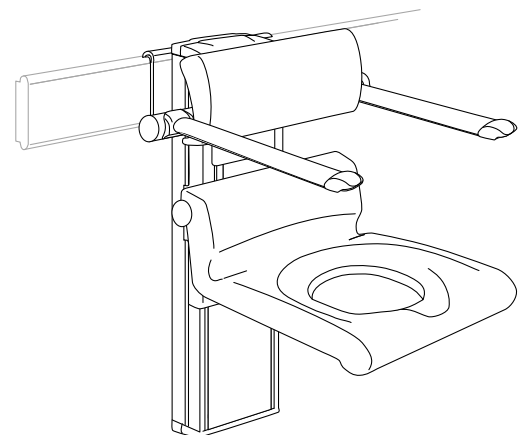Ret til ændringer forbeholdes. Recht auf Änderungen sind vorbehalten. Subject to alteration without further notice.
Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis.
Wijzigingen voorbehouden. Salvo variaciones. Salvo variazioni. Rätten till ändringar förbehålles. Vi förbeholder oss rett til endringer.
Kaikki oikeudet muutoksiin pidätetään. Rettur til breytinga áskilinn. Reservado o direito de fazer modificações sem prévio
Επιφύλασσόμεθα για εκτυπωτικά γάθια, μικρές τεχνικές αλλαγές και παρεκκλίσεις μοντέλων.

● keep living

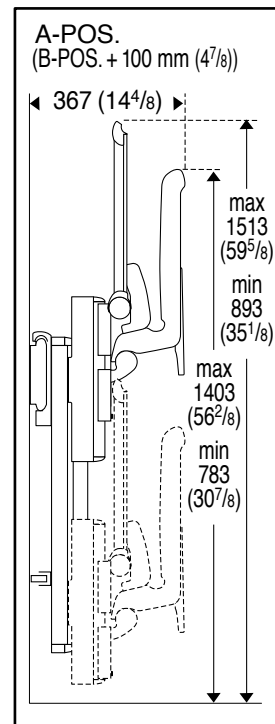
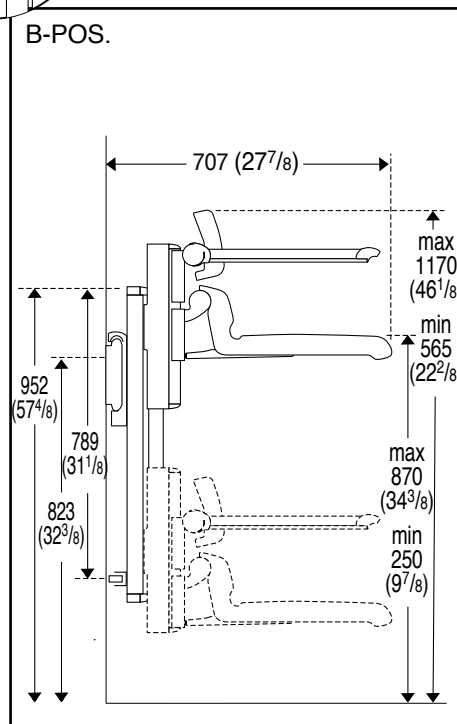
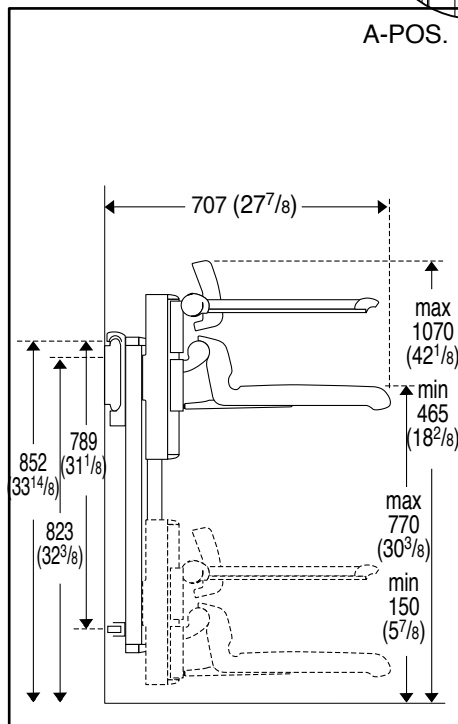
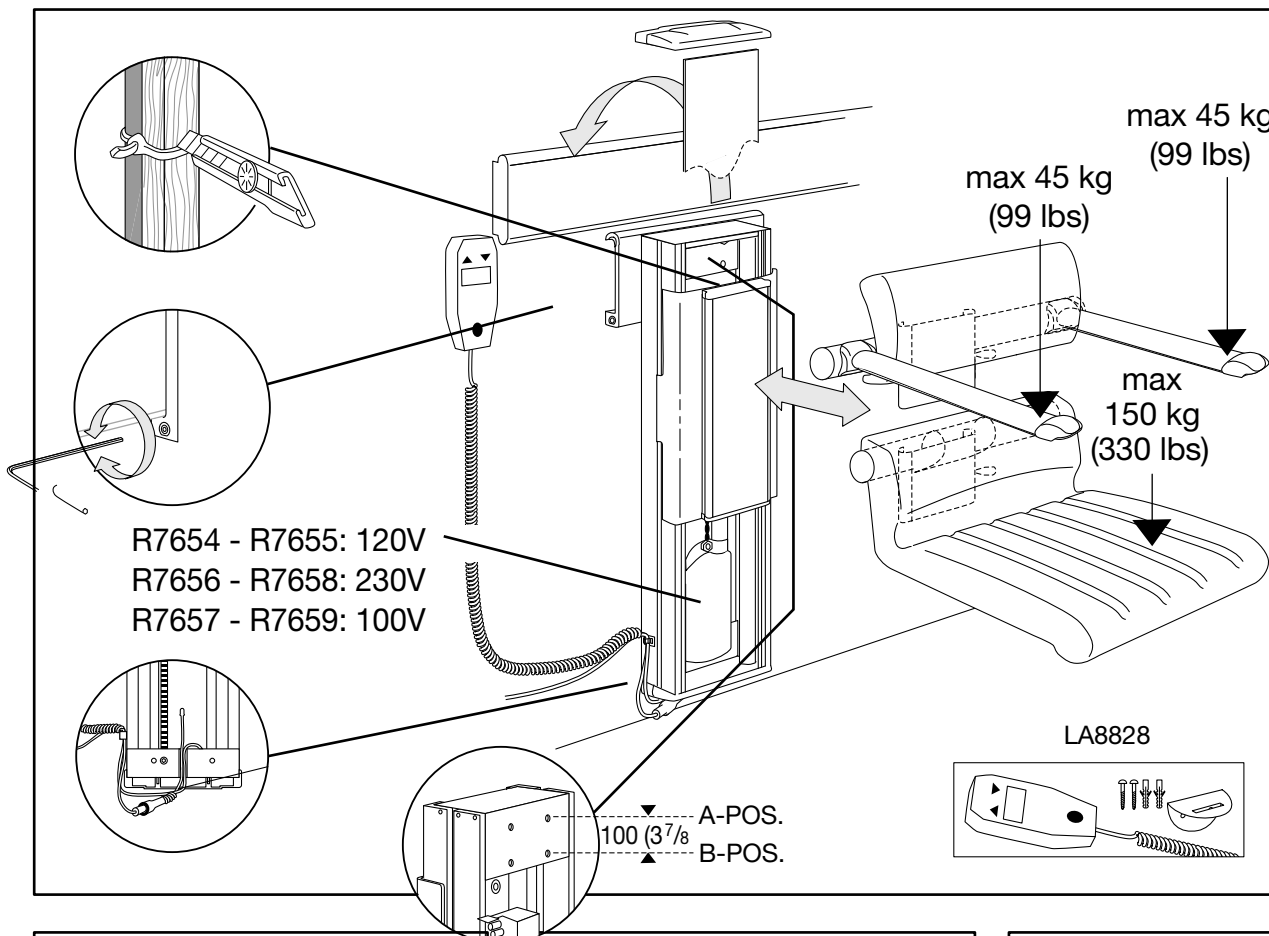
Ret til ændringer forbeholdes. Ret til ændringer sind vorbehalten. Subject to alteration without further notice. Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis. Wijzigingen voorbehouden. Salvo variaciones. Salvo variazioni. Rätten till ändringar förbehålls. Vi förbehåller oss rätt till ändringar. Kaikki oikeudet muutoksiin pidätetään. Rettur til breytinga áskilinn. Reservado o direito de fazer modificações sem prévio. Επιφυλασσομένη για εκπτώτικα γάθη, μικές τεχνικές αλλαγές και παρεκκλίσεις μοντέλων.

Ret til ændringer forbeholdes. Recht auf Änderungen sind vorbehalten. Subject to alteration without further notice. Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis. Wijzigingen voorbehouden. Salvo variaciones. Kätken til ändringar förbehålles. Vi förbehåller oss rätt till ändringar. Kaikki oikeudet muutoksiin pidätetään. Rettur til breytinga áskilinn. Reservado o direito de fazer modificações sem prévio. Επιφυλασσομέρα για εκτυπωτικά γάθη, μικρές τεχνικές αλλαγές και παρεκκλίσεις μοντέλων.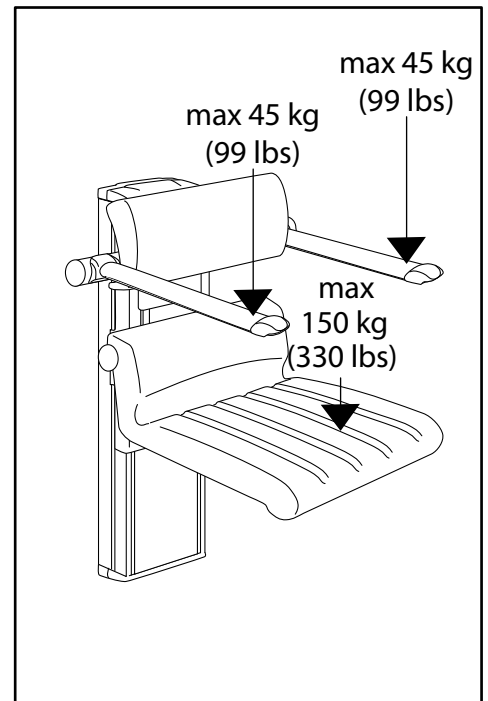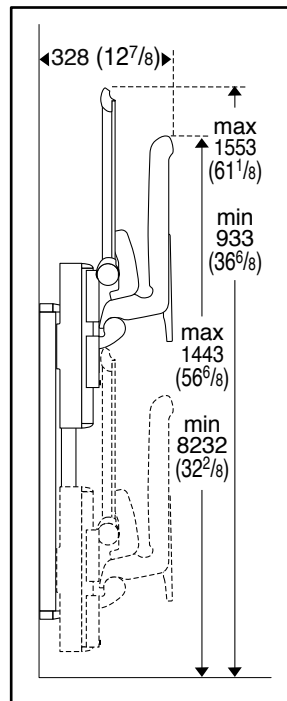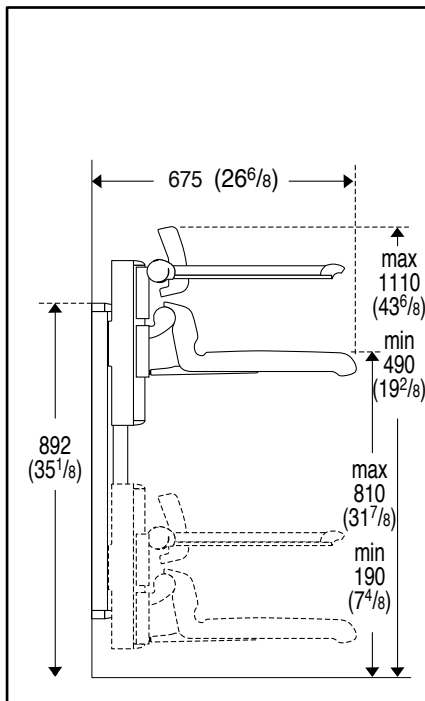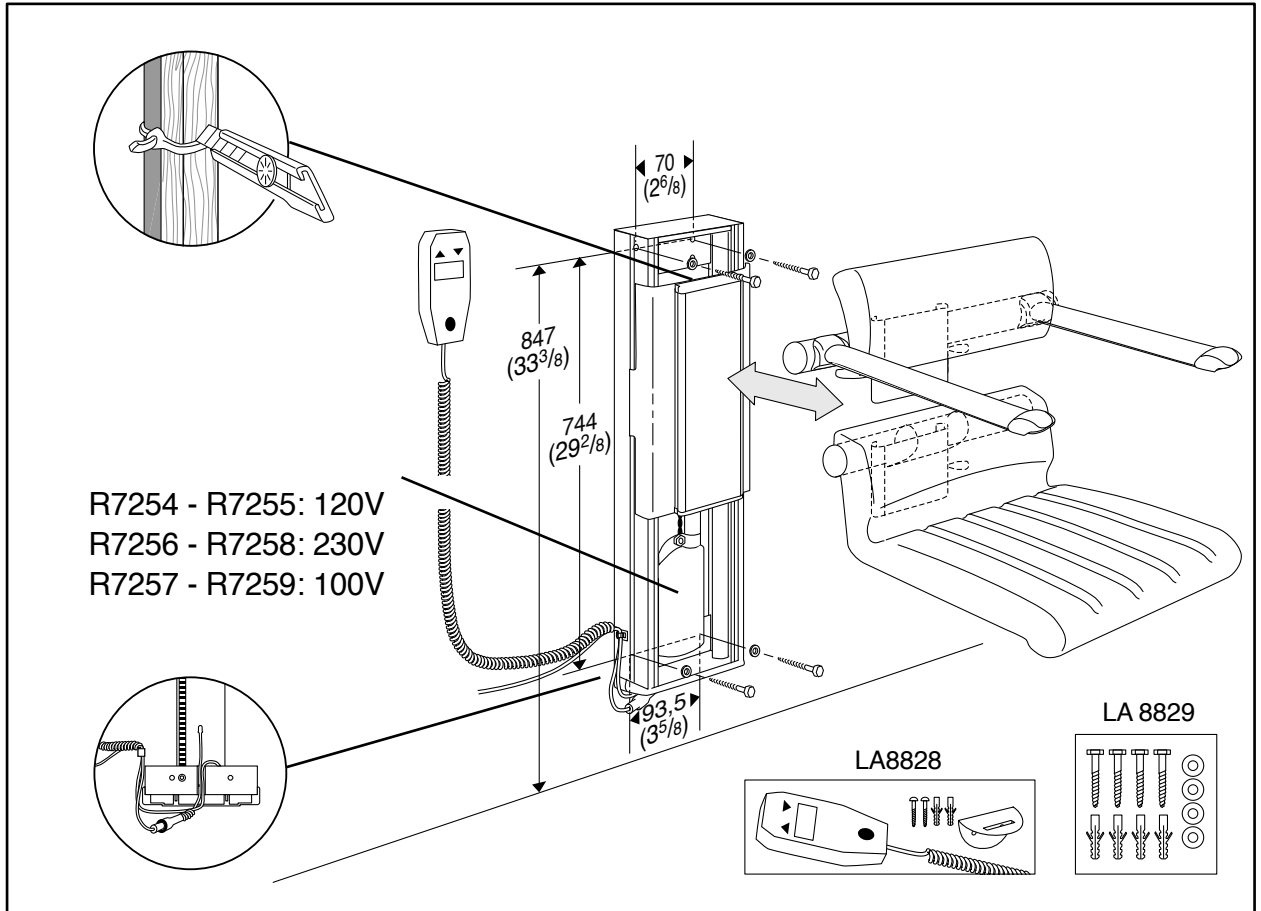

● keep living

Telephone: +45 87 888 777
Telefax: +45 87 888 669
E-mail: mail@pressalitcare.com
Web: www.pressalitcare.com

Pressalit Care
Pressalitvej 1
DK 8680 Ry
Denmark


Pressalit Care

R7254 - R7256 - R7257
 R7255 - R7258 - R7259
 R7654 - R7656 - R7657
 R7655 - R7658 - R7659



01/08 Mounting Info Page 4/9

R7254 - R7256 - R7258
 R7255 - R7258 - R7259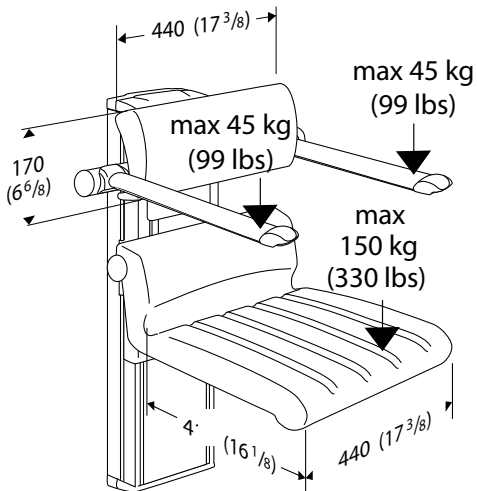

R7654 - R7656 - R7657
 R7655 - R7658 - R7659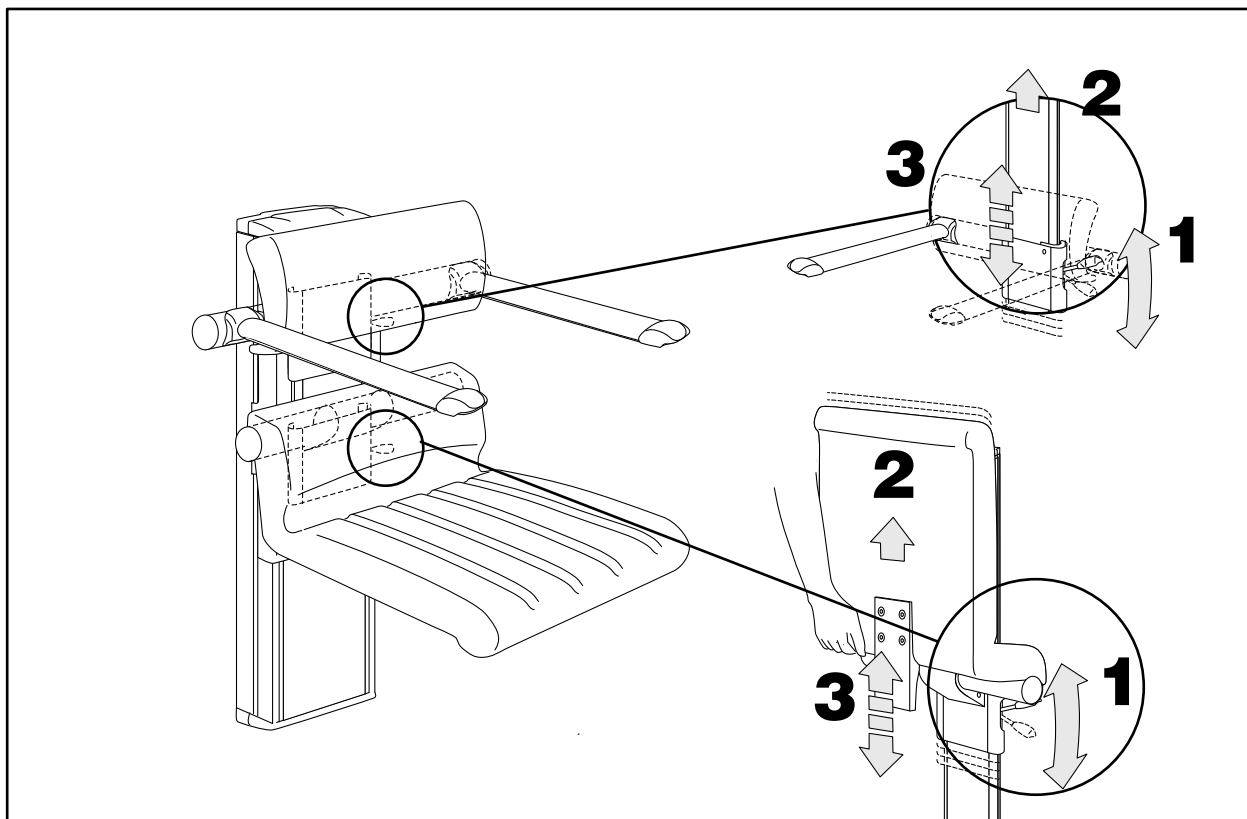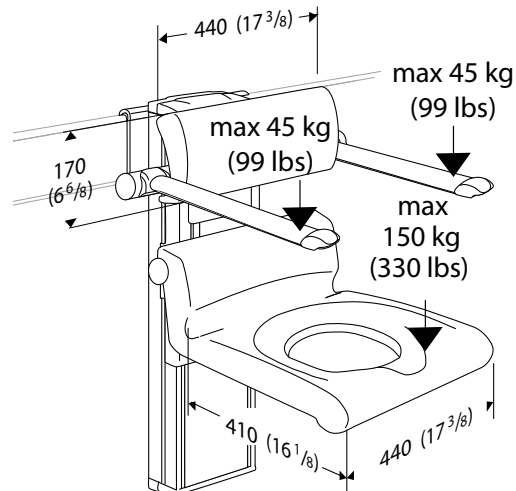

Ret til ændringer forbeholdes. Recht auf Änderungen sind vorbehalten. Subject to alteration without further notice.
 Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis.
 Wijzigingen voorbehouden. Salvo variaciones. Salvo variazioni. Rätten till ändringar förbehålles. Vi förbeholder oss rett til endringer.
 Kaikki oikeudet muutoksiin pidätetään. Rettur til breytinga áskilinn. Reservado o direito de fazer modificações sem prévio
 Επιφυλασσομένα για εκτυπωτικά γόθρη, μικές τεχνικές αλλαγές και παρεκκλίσεις μοντέλων.


keep living

Telephone: +45 87 888 777
 Telefax: +45 87 888 669
 E-mail: mail@pressalitcare.com
 Web: www.pressalitcare.com

Pressalit Care
 Pressalitvej 1
 DK 8680 Ry
 Denmark

Ret til ændringer forbeholdes. Recht auf Änderungen sind vorbehalten. Subject to alteration without further notice. Nous nous réservons le droit d'apporter toutes modifications pour l'amélioration de nos produits sans préavis. Wijzigingen voorbehouden. Salvo variaciones. Salvo variazioni. Rätten till ändringar förbehålles. Vi förbeholder oss rett til endringer. Kaikkii oikeudet muutoksiin pidätetään. Rettur til breytinga áskilinn. Reservado o direito de fazer modificações sem prévio. Επιφυλασσομέθο για εκτυπωτικά γάφθη, μικρές τεχνικές αλλαγές και παρεκκλίσεις μοντέλων.

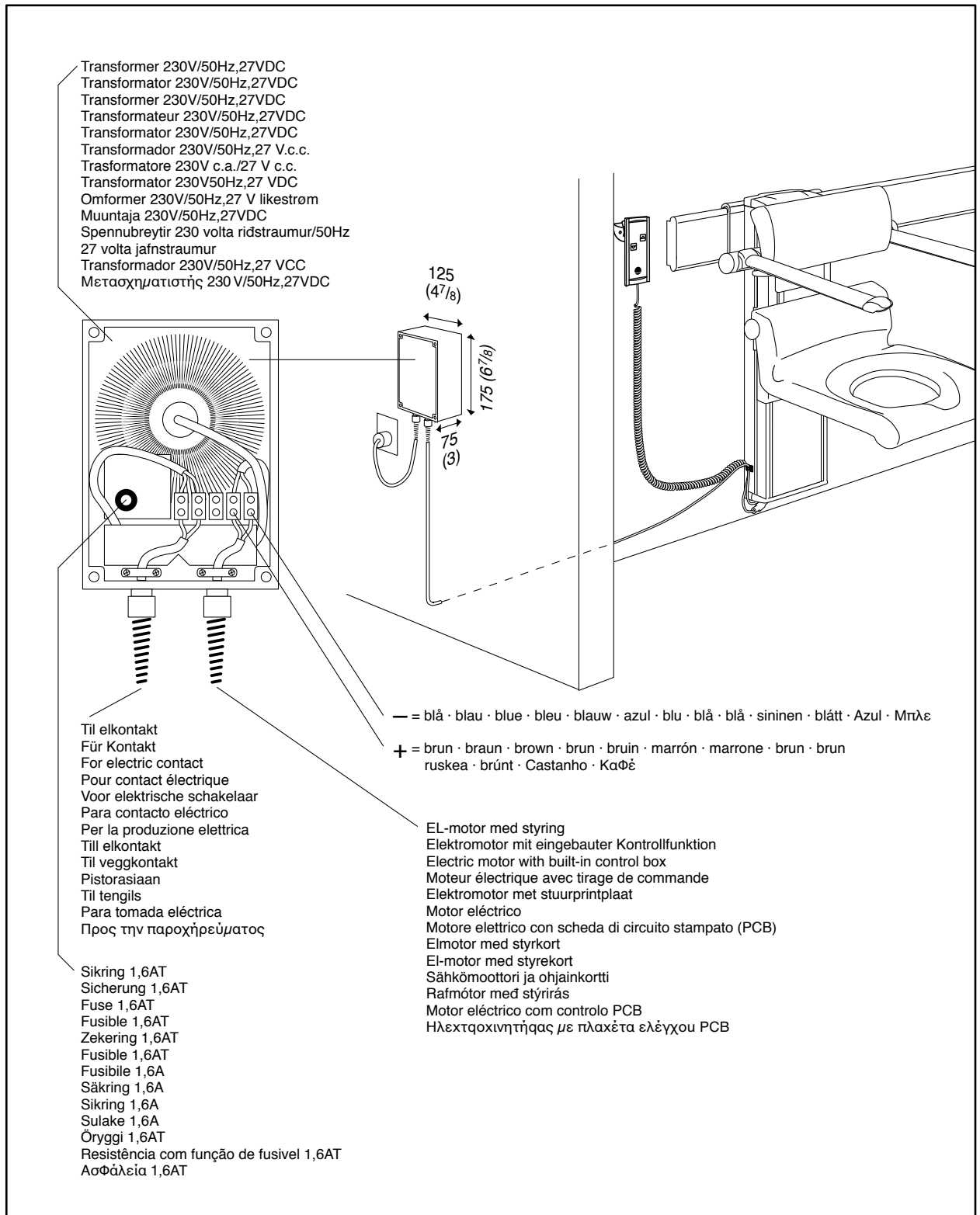Kun vejledende • Nur richtungsweisend • Guidelines only • Le guide n'a qu'une valeur indicative • Niet te beoordelen
Solamente a titulo indicativo • Solamente a titulo indicativo • Endast anvisning • Kun veiledende • Vain ohjeellinen • Adeins til leidbeinandi
Apenas linhas de orientação • Μόνο βοηθητικές γραμμές.

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El-installation

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GB Assembly Instruction

Electric installation

All electrical connections to be carried out by a qualified electrician. Mount the transformer 27 VDC 100 VA in a dry room separated from the bathroom. Test the wall mounting with a mounted shower chair when the connection has been completed. When the remote control is activated up and down respectively the shower chair must move up and down respectively. If the chair does not move:

1. check the fuse in the transformer, 1,6 AT or 2. reverse the wires + and - on the transformer.

⚠ WARNING! - The power must be disconnected when opening the box to the electric motor. Risk of personal injury.

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GB Maintenance

The chair (seat and back rest) is mounted on 2 sliding rods. In order to maintain a smooth-running function in connection with the height adjustment it is recommended to lubricate the rods and the chain once a year or as requested.

- 1) Remove end cap and front plate
- 2) Wipe the dirt off the rods, if any
- 3) Apply the lubricant with a brush
- 4) Move the carriage up and down until the lubricant is spread - apply more lubricant if necessary.

The shower chair R7656/R7658 is mounted on the track by way of a glider on which it is recommended to apply eg silicone according to requirement to ensure the chair to move smoothly on the track at all times.

Cleaning Instructions:
Use ordinary soapy water (avoid aggressive cleaners). Rinse off with fresh water and dry the surface with a soft cloth.

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GB

EC Declaration of conformity

The undersigned
Pressalit A/S
Pressalitvej 1
DK-8680 Ry
hereby declare that the following
product
Trade name: Shower chair with
electric motor
Model No.:
R7256/R7258-R7656/R7658
Category: Technical aids for the
disabled, class of risk I
which is comprehended by this
declaration, has been manufactured
in accordance with the Directives
89/392/EEC
93/42/EEC
The above-mentioned product
1. has been manufactured in
accordance with the following
standards:
IEC 601-1
EN 60601-1:1990
2. has been tested by a third party:

Technical documentation shall be
available to the national authorities.

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