
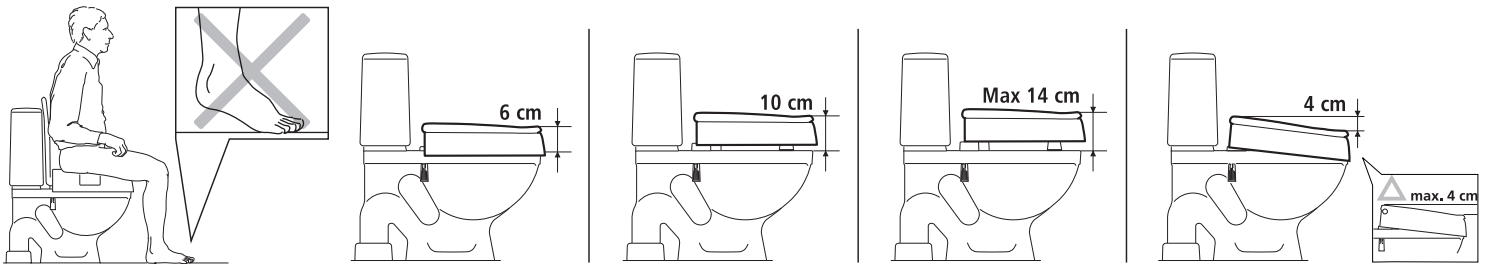
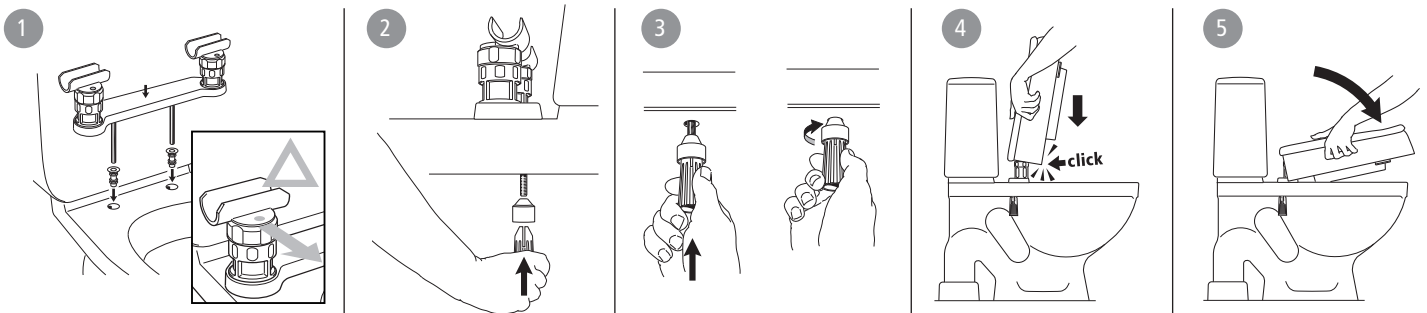
A


B

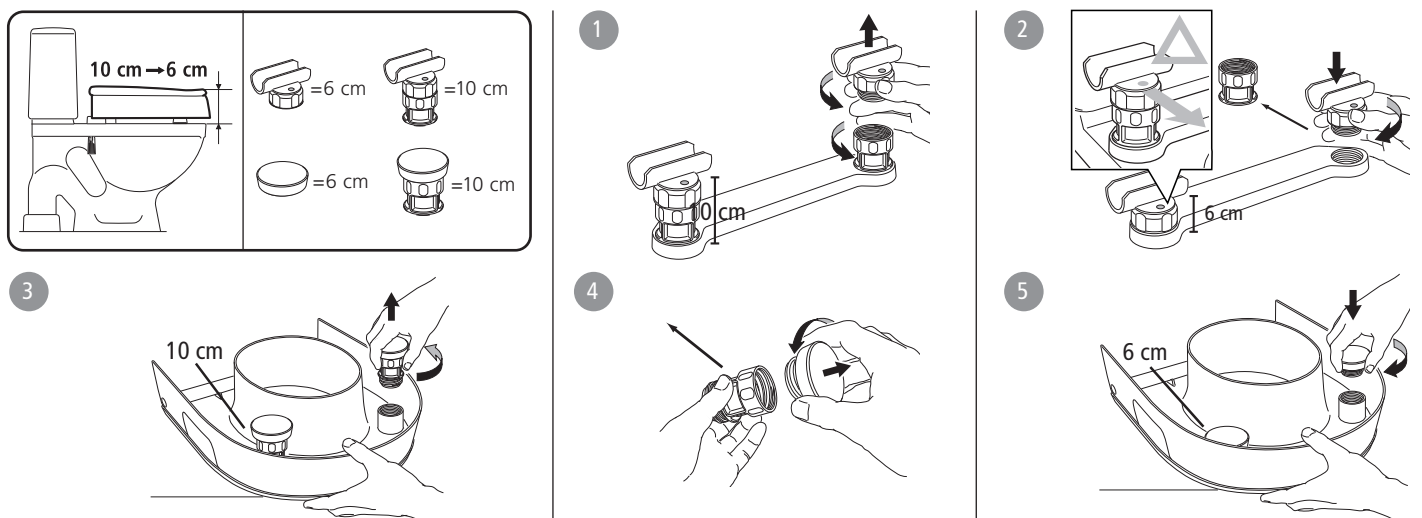
C


D

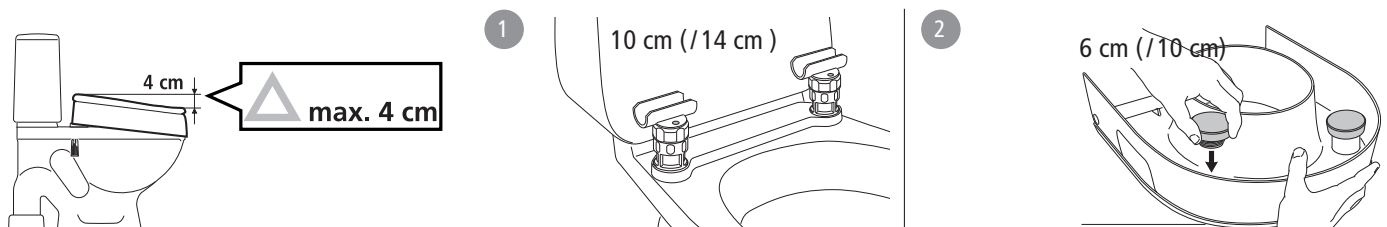
E

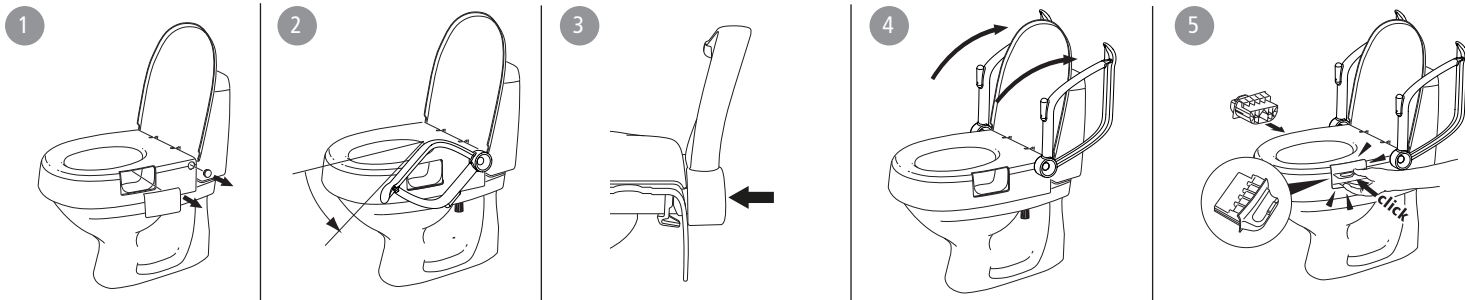
F

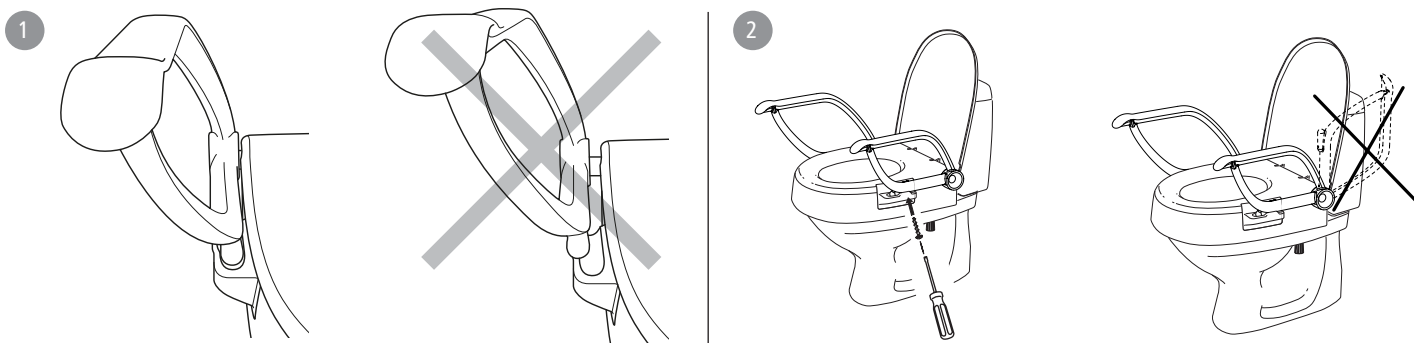G

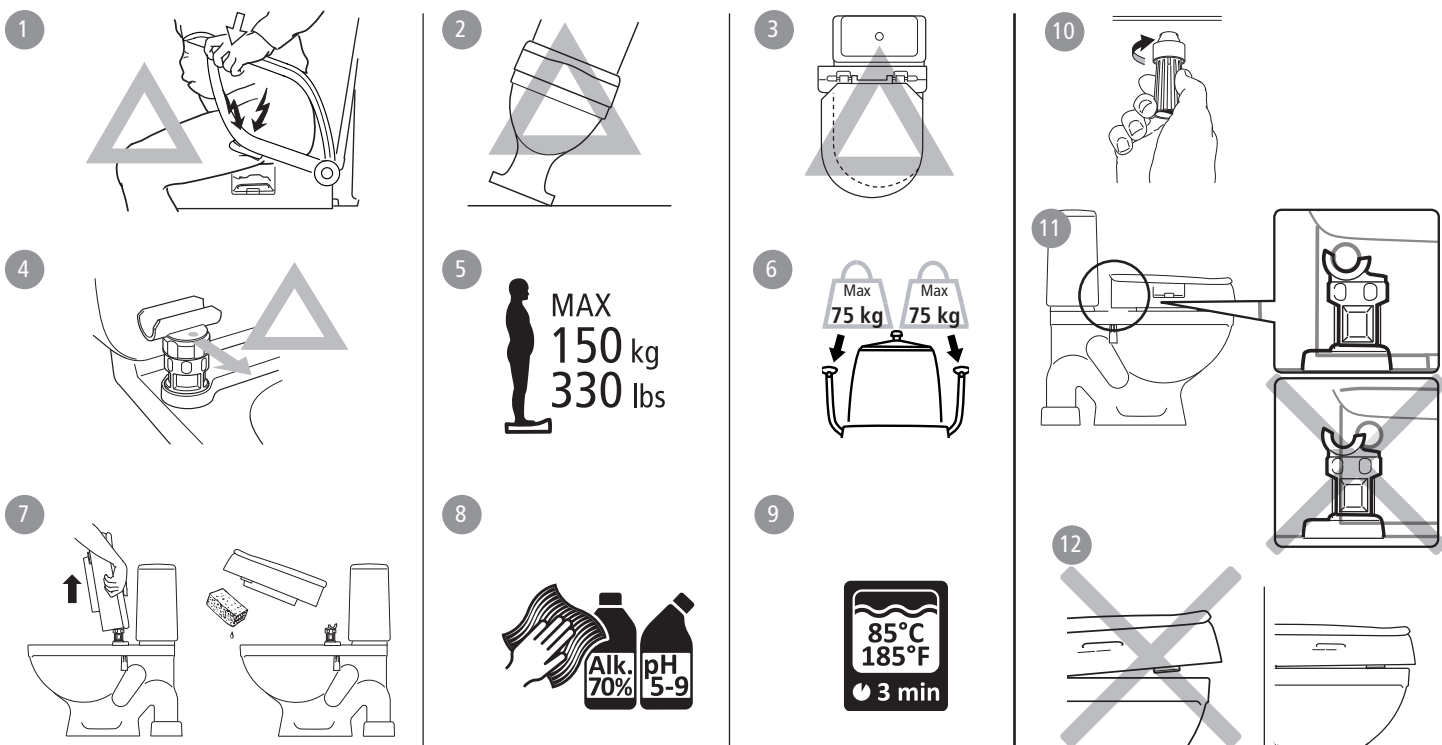H

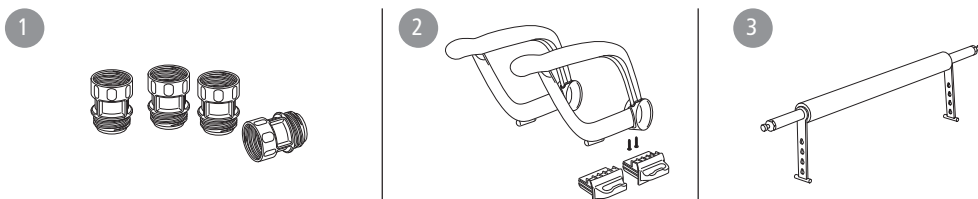SV

Svenska

Allmänt

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English

General information

Cloo is a height-adjustable toilet seat raiser that is fixed securely to the toilet. It is supplied with a 10 cm elevation. When armrests are included in the kit, these are not supplied already fitted. Cloo can also be fitted at an angle.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Warranty: 5 year warranty against material and manufacturing defects. For terms and conditions, see www.etac.com. A defective product must not be used.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

Choosing your height *Figure A*

Mounting *Figure B*

Height adjustment *Figure C*

Angled fitting *Figure D*

Armrest *Figure E*

Fixed mounting *Figure F*

Safety and Maintenance *Figure G*

G1. Risk of crushing **G2.** The commode must be securely anchored **G3.** The product should be mounted centrally on the commode **G4.** Mount so that the red dot is pointing forwards. **G5.** Maximum user weight 150 kg **G6.** The armrests have each been tested and approved for half of the maximum user weight **G7.** The seat ring can be removed for cleaning. Make sure it clicks in place completely when refitting **G8.** Clean the product with a solvent-free cleaning agent with a pH level between 5 and 9, or with a 70% disinfectant solution. **G9.** Can be decontaminated at max. 85°C if

so required. **G10.** Check the screw attachment.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Accessories *Figure H*
H1. Blocks for 14 cm elevation. **H2.** Armrest kit. Supplied in pairs. **H3.** Crossbar

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Etac A/S
R82, Inc.
Etac Supply Center AB