






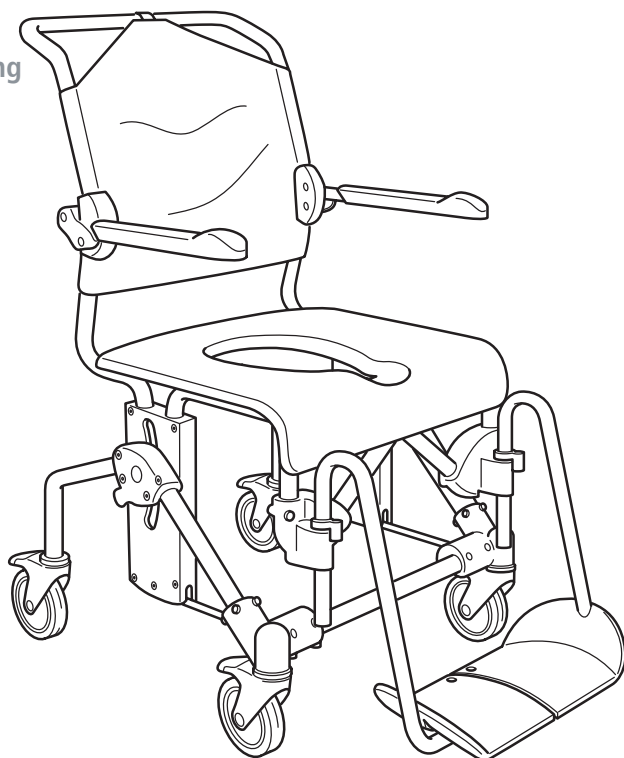
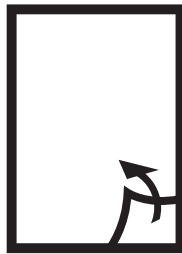
74873C 10-03-29

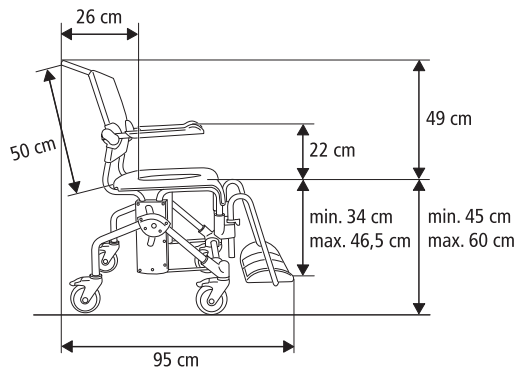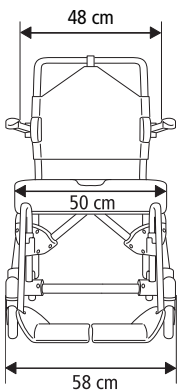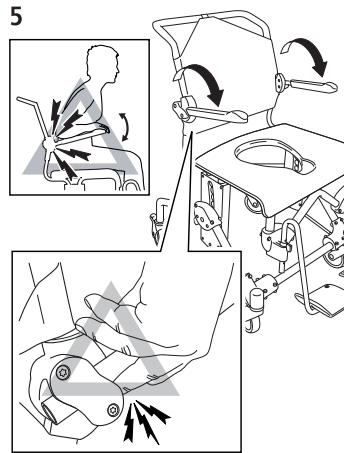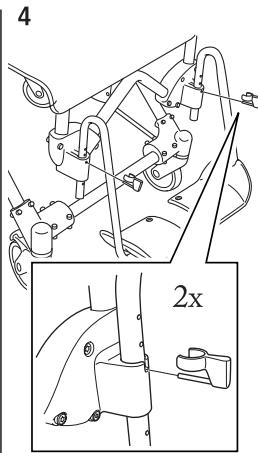
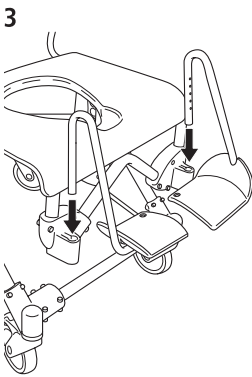
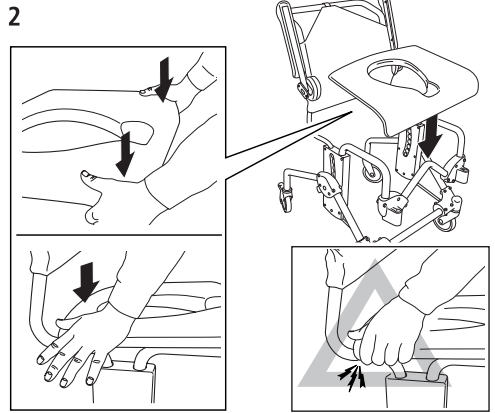
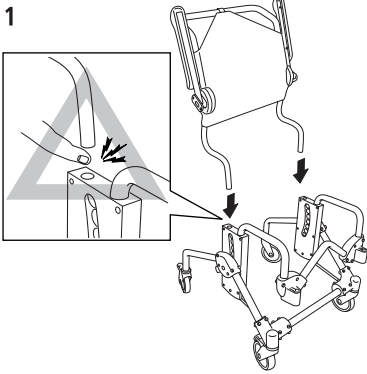
- SV** Manual
- NO** Bruksanvisning
- DA** Betjenings-  
vejledning
- FI** Käyttöopas
- EN** Manual
- DE** Gebrauchsanweisung
- FR** Manuel
- NL** Handleiding
- IT** Manuale
- ES** Manual


**A**

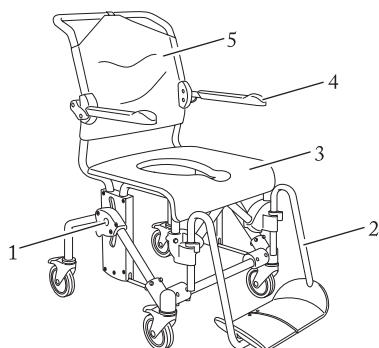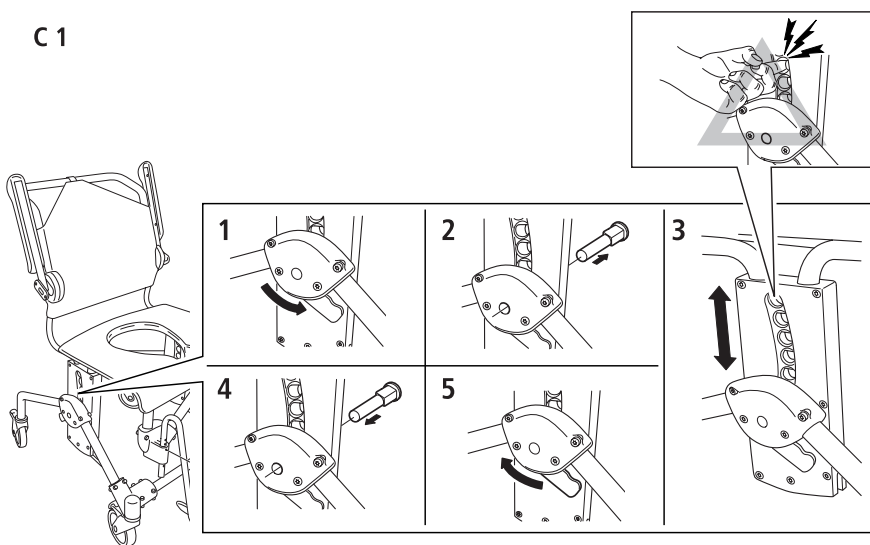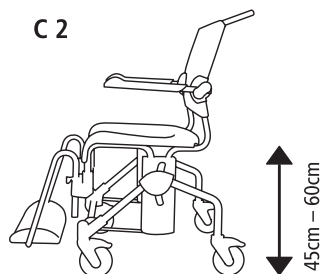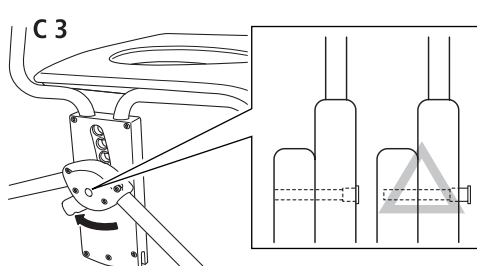
**B****C****C1****C2****C3**

D


















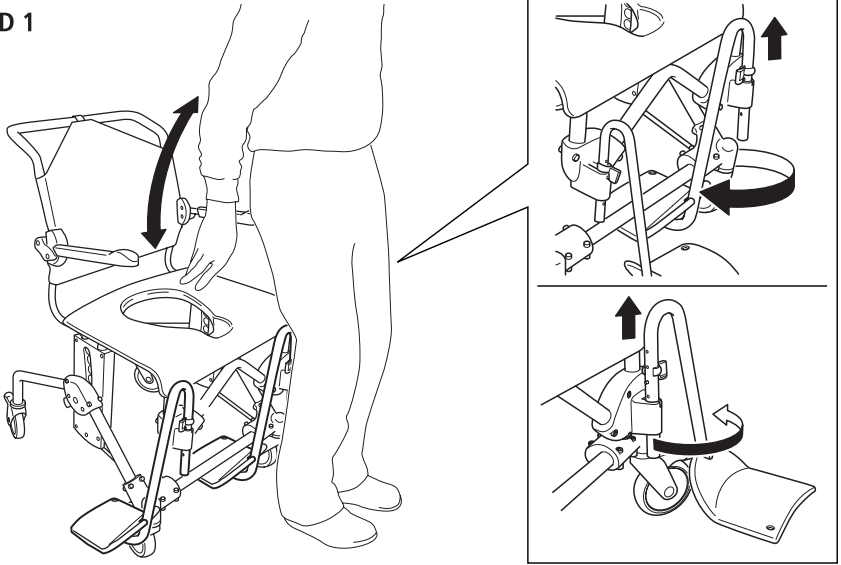
D 1

D 2

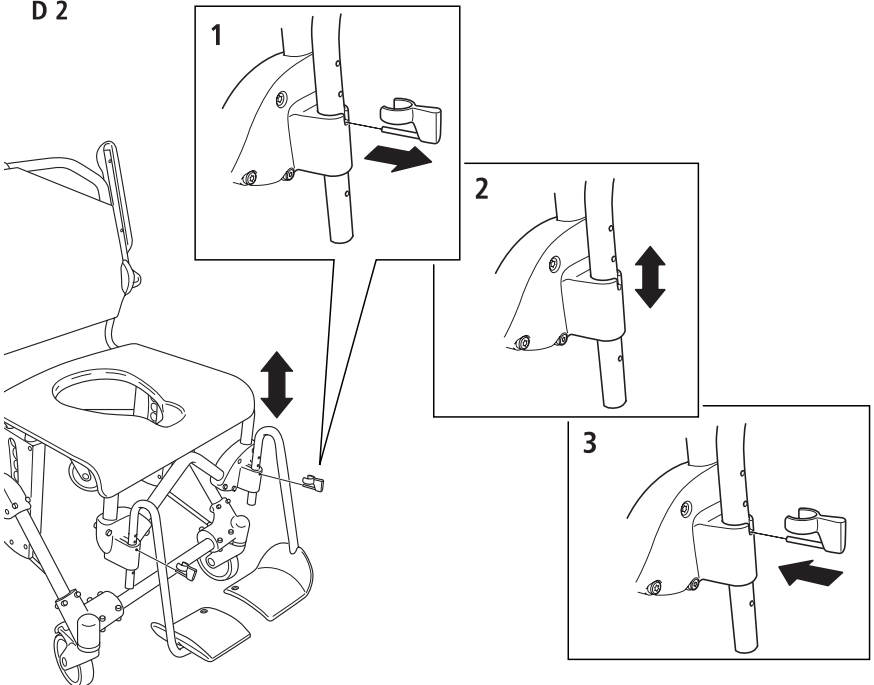E
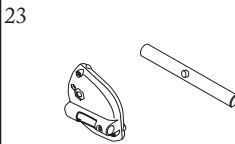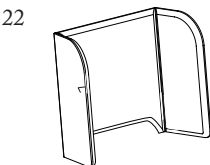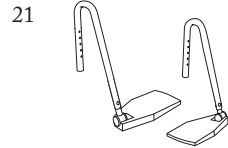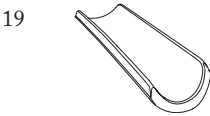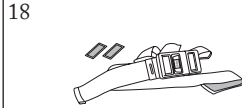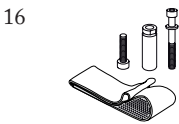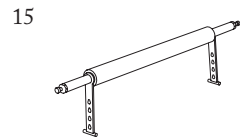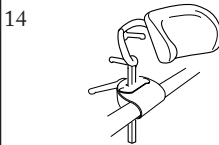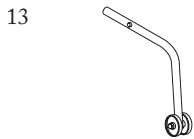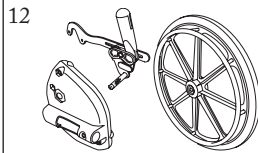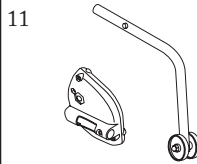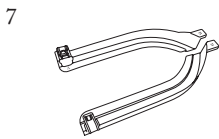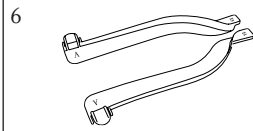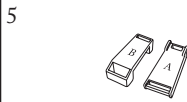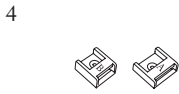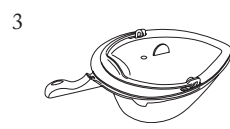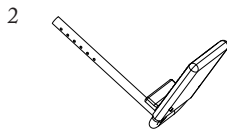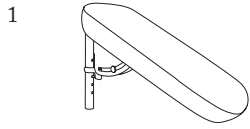
## Innehåll

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## General information

Thank you for choosing a shower chair from Etac.

In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

Swift Mobil is a shower and toilet chair intended to make hygiene care easier for people with reduced mobility and their carers.

Swift Mobil has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

It is designed for use in showers, with hand basins and over toilets, as well as for transport to and from the sanitary room. Every component of the chair is designed to make things easier for users and assistants.

**This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.**

## Assembly/Dimension table

fig. A

## Product description

fig. B

|           |                       |
|-----------|-----------------------|
| 1 Bolt    | 4 Armrest             |
| 2 Legrest | 5 Backrest upholstery |
| 3 Seat    |                       |

## Height adjustment

fig. C

The height can be adjusted with ease by positioning the bolt in the required position (fig. C 1).

Swift Mobil is height-adjustable between 45 and 60 cm (fig. C 2).

**Check that both sides are similarly adjusted, otherwise stability will be reduced. Do not use the chair if the bolt (orange) is not locked (fig. C 3).**

## Legrest


fig. D

The assistant can easily raise and swivel the legrests to the side for the user to sit down in or get up out of the chair (fig. D 1). The footrests are detachable.

Various height adjustment options are available by moving the locking pin (fig. D 2). The legrests have 6 different height settings.



## Transport

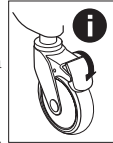1. Make sure that at least two of the wheels are locked whenever the user is sitting down in or getting up out of the chair.
2. The legrests should be swivelled to the side whenever the user is sitting down or getting up.

### Negotiating thresholds:

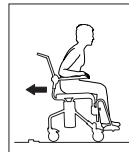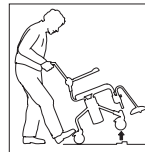1. Chair only: Place one foot on the rear chassis tube.
2. With a user: move the chair backwards. Ensure that the castors are facing in the direction of travel.

## Warranty/maintenance

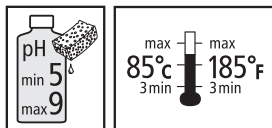
### Warranty

The warranty period for material and manufacturing defects is 5 years,

### Maintenance and reconditioning

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- The product can be cleaned at 85 °C for 3 minutes.
- Keep the castors free of hair and soap deposits, etc. If your shower and toilet chair is not in perfect working order, contact your dealer.

## Safety

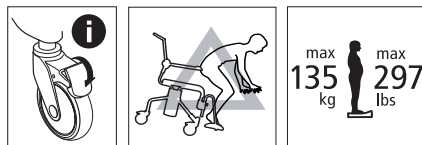
**A defective chair should not be used.**

Make sure that the wheels are locked whenever the user is sitting down in or getting up out of the chair.

Anti-tip stabilisers should be used if the user is a leg amputee.

Do not stand on the footrests, risk of tipping!

Swift Mobil can be used by people weighing a maximum of 135 kg.

## Recycling

This product should be recycled as per national regulations.

## Accessories

fig. E

1. Leg and calf rest, angle-variable.....Item no. 8020 9437
2. Footrest ..... Item no. 8020 9443
3. Commode pan .....Item no. 8020 9254
4. Pan-holder attachments .....Item no. 8020 9435
5. Attachments for low pan-holder mounting .....Item no. 8020 9259
6. Commode pan holders .....Item no. 8020 9257
7. Bucket holders .....Item no. 8020 9258
8. Splash guard .....Item no. 8020 9430
9. Comfort seat.....Item no. 8020 9433
10. Seat cover.....Item no. 8020 9434
11. Anti-tip stabilisers including bracket...Item no. 8020 9439
12. Rear wheel kit.....Item no. 8020 9436
13. Anti-tip stabilisers excluding bracket...Item no. 8020 9438
14. Headrest and bracket .....Item no. 8020 9447
15. Crossbar 605 mm ..Item no. 8030 3034
16. Heel straps.....Item no. 8020 9432
17. Chest belt .....Item no. 8020 9442
18. Positioning belt.....Item no. 8020 9440
19. Hemiplegic armrest.....Item no. 8020 9035
20. Armrest lock..... Item no. 8020 9445
21. Angle adj. footrest . Item no. 8020 9446
22. Soft back pad ..... Item no. 8020 9448
23. Tilter ..... Item no. 8020 9449

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Etac Sverige AB
Etac AB (export)
Etac A/S