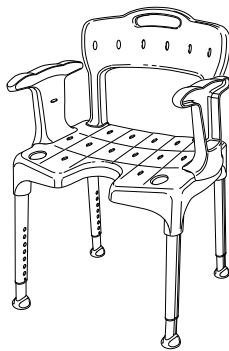
Swift

Duschstol/pall

Shower chair/stool

Art.nr. 8170 1010 / 8170 1020 / 8170 1030

Brukarmanual / User manual

Beskrivning:

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Description:

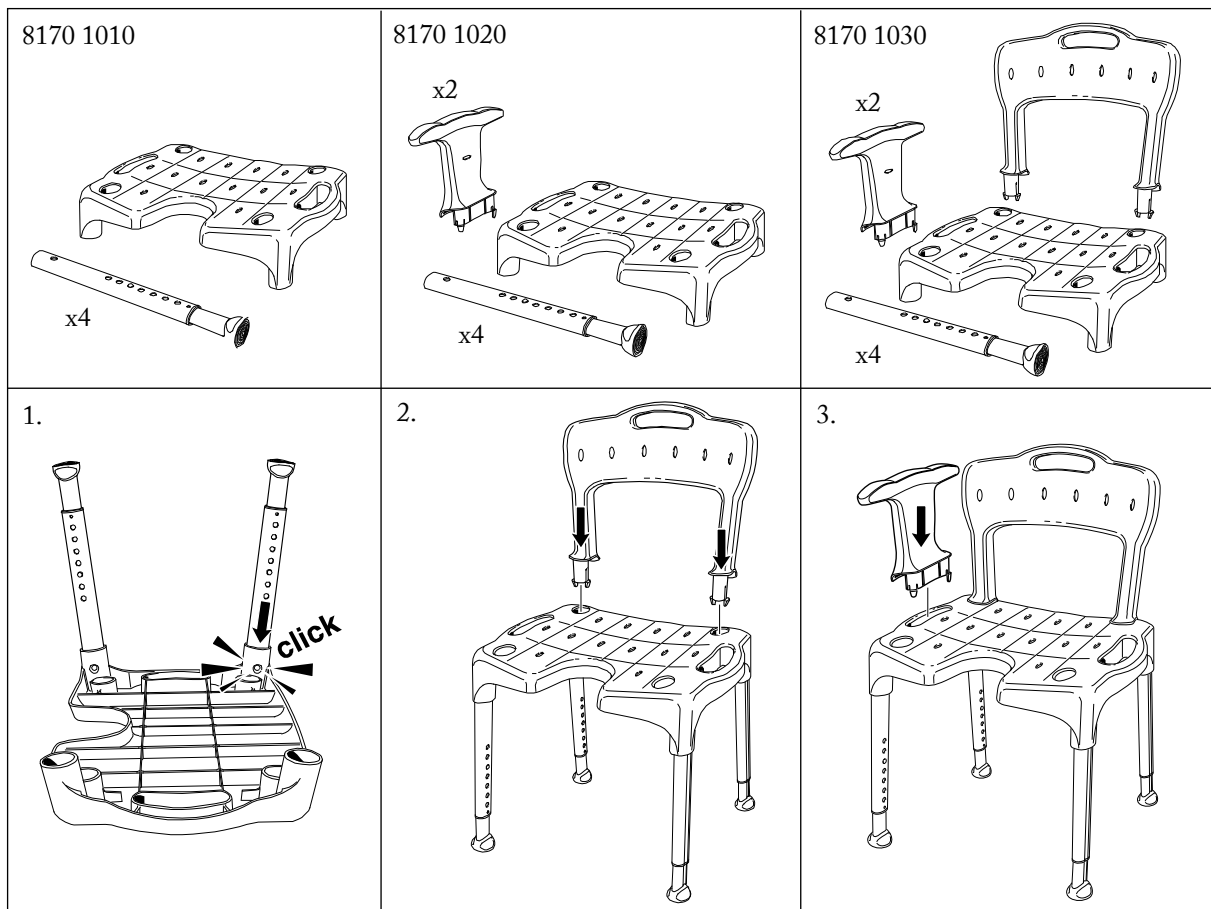
Swift is a stool/chair developed for use in the shower or in the bathroom. The chair/stool is of a sturdy design that adapts to uneven surfaces. In showers where the floor slopes towards the drain Swift stands securely on all four legs thanks to its built in flexibility. The ferrules also provide an excellent grip, and the anti-slip pattern on the seat provides security and stability, important elements that help you to feel safe and secure. Swift can be anything from a simple shower stool to a complete shower chair, depending on how you choose to equip it.

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Mounting

Swift is easy to assemble. No tools are needed.

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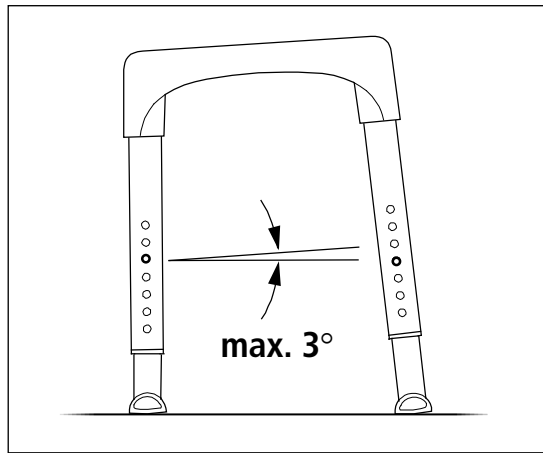
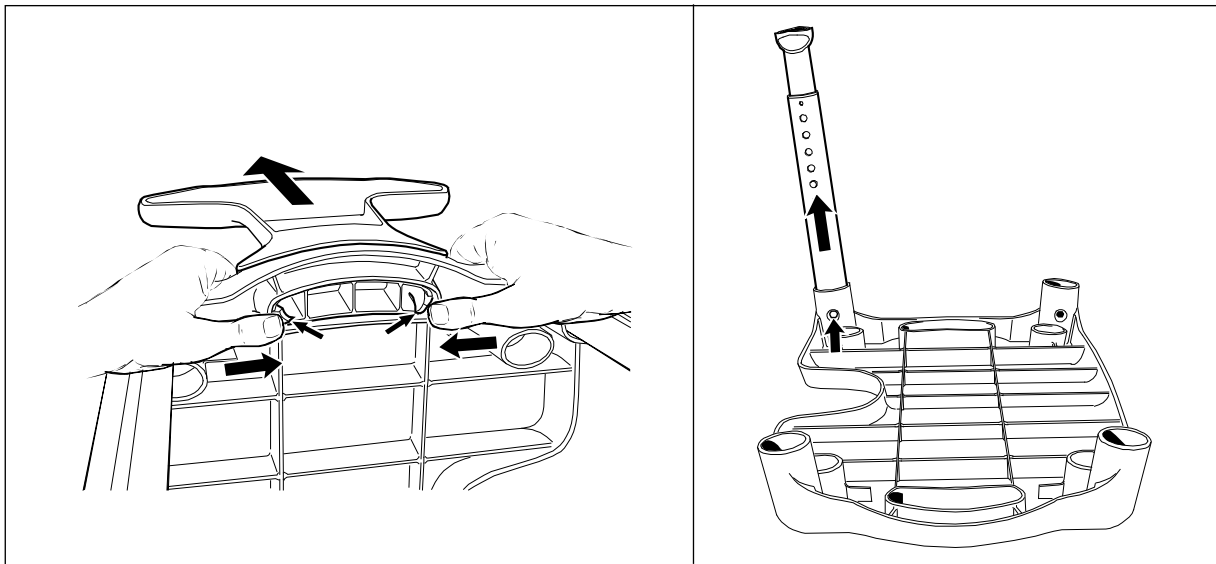
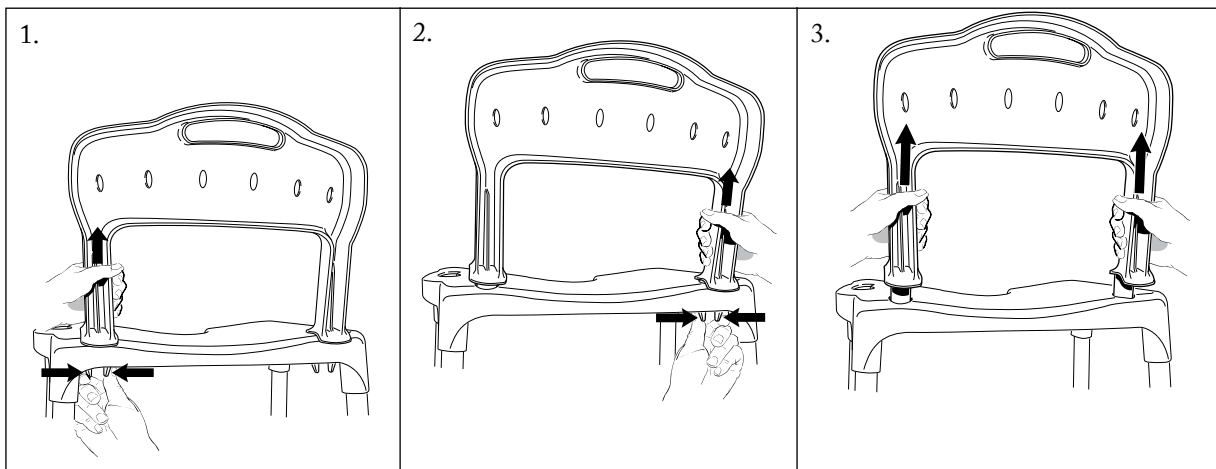
Accessories:

Armrest/pair: Swift can be easily fitted with armrests for better support when standing up.

There is a backrest for people who require more support. This is fitted simply by pressing it into place. It then stays securely in place.

Seat pad: A seat pad that provides both softness, warmth and friction.

Backrest pad: A back pad that provides softness, warmth and friction.

Soap dish: A small shelf to keep your shampoo, soap or shower gel, within convenient reach. There is also a holder for the shower handle.

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Safety:

Warning: Do not angle the chair more than 3° (difference of 1 hole).

Warning: When moving sideways, do not lean on just one armrest.

Max. weight: The chair must not be subjected to a load greater than 130 kg/286.7 lbs.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185°F for 3 minutes if required.

78390A

06-02-01

Etac, Box 203, SE-334 24 Anderstorp

Tel. +46/371-58 73 00, Fax. +46/371-58 73 90

www.etac.com

etac®