Nova 100

Lofthejs

Hubmotor

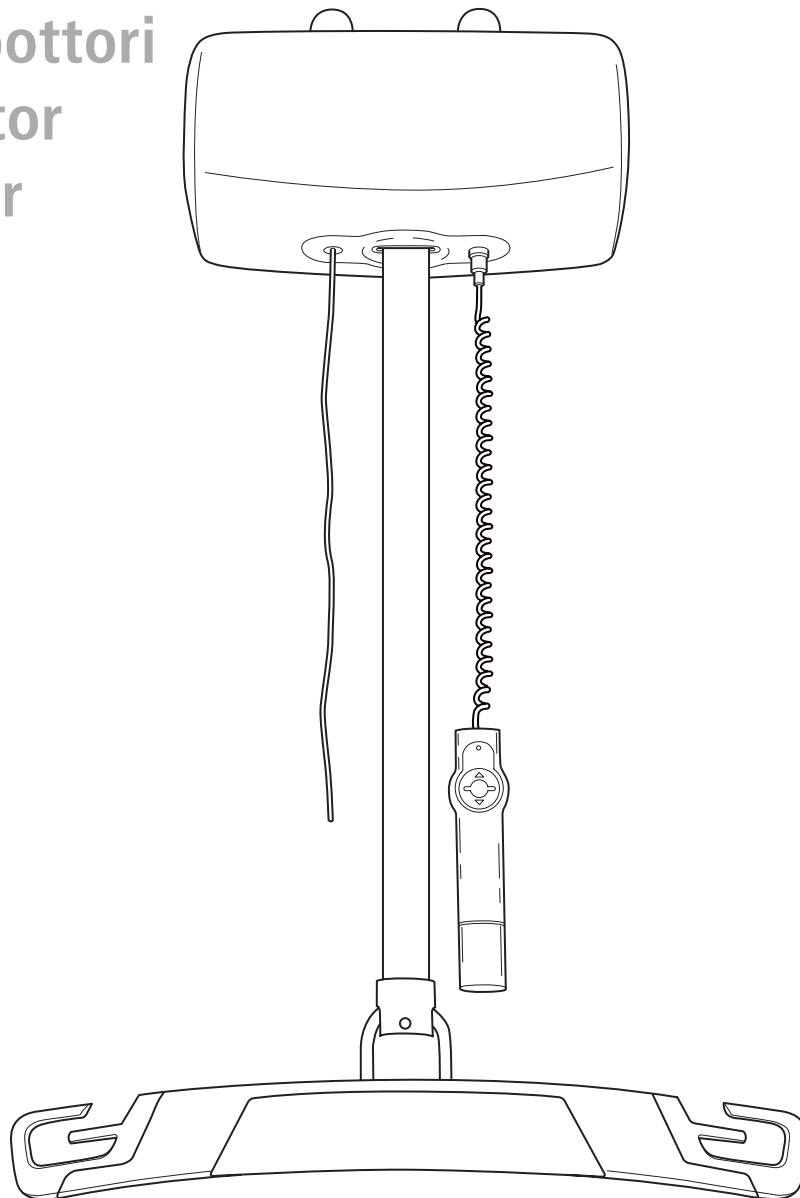
Lift Unit

Nostomoottori

Løftemotor


Lyftmotor

Manual



A


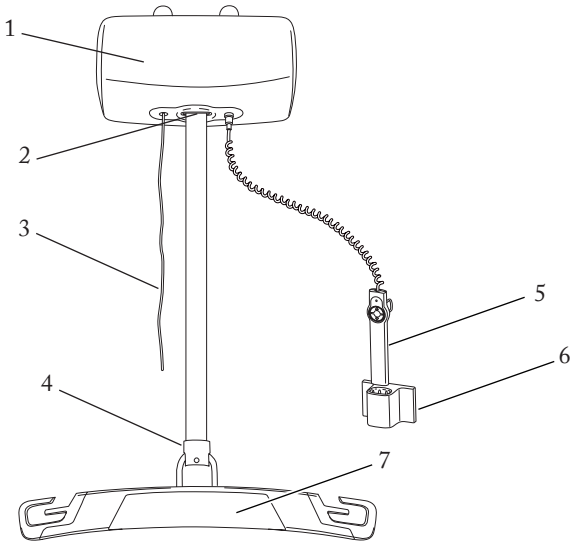
B

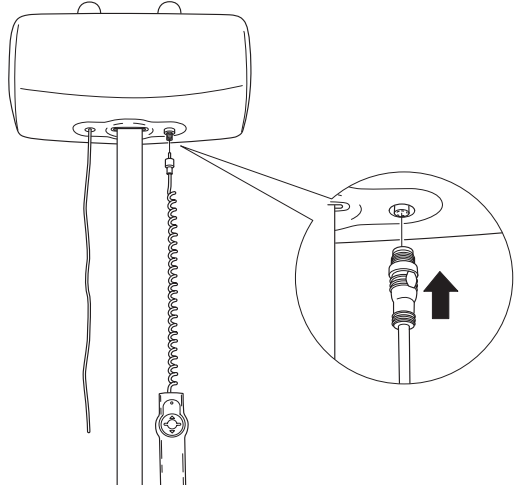
C

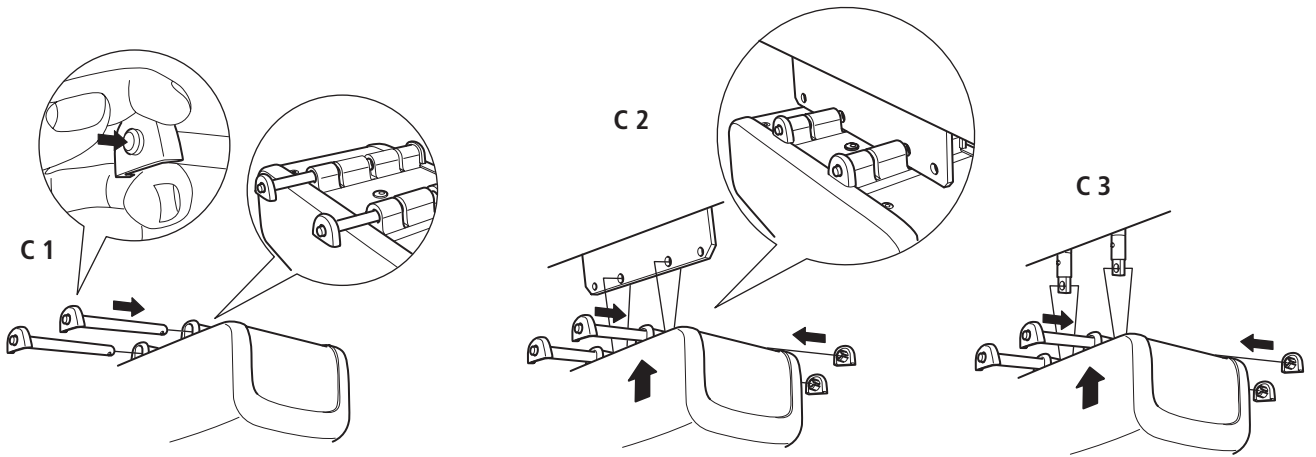
D

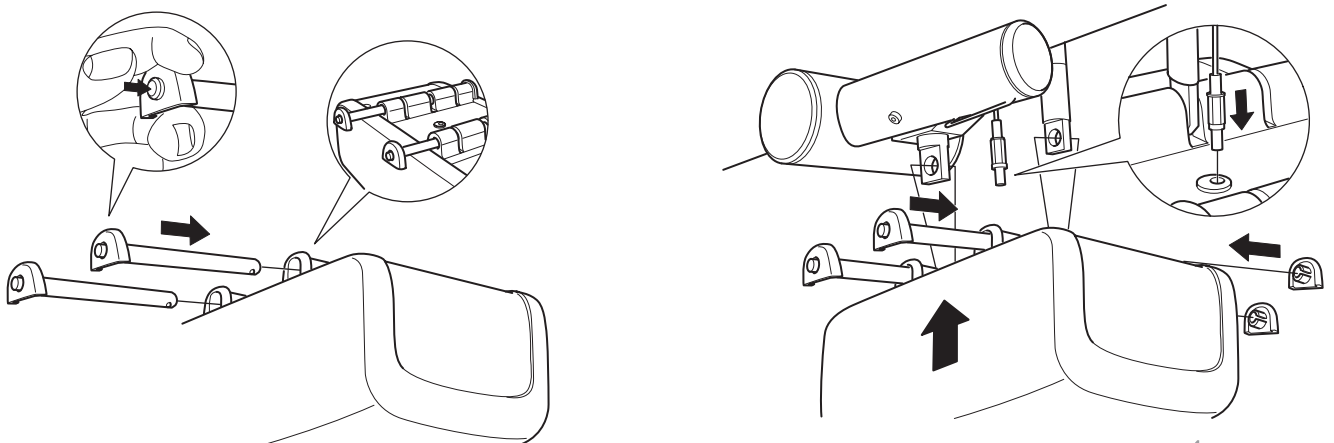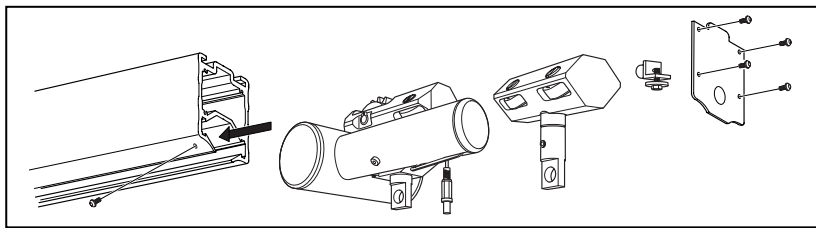E

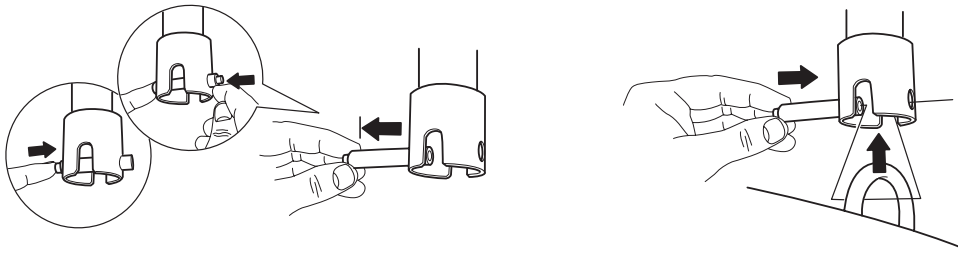
F

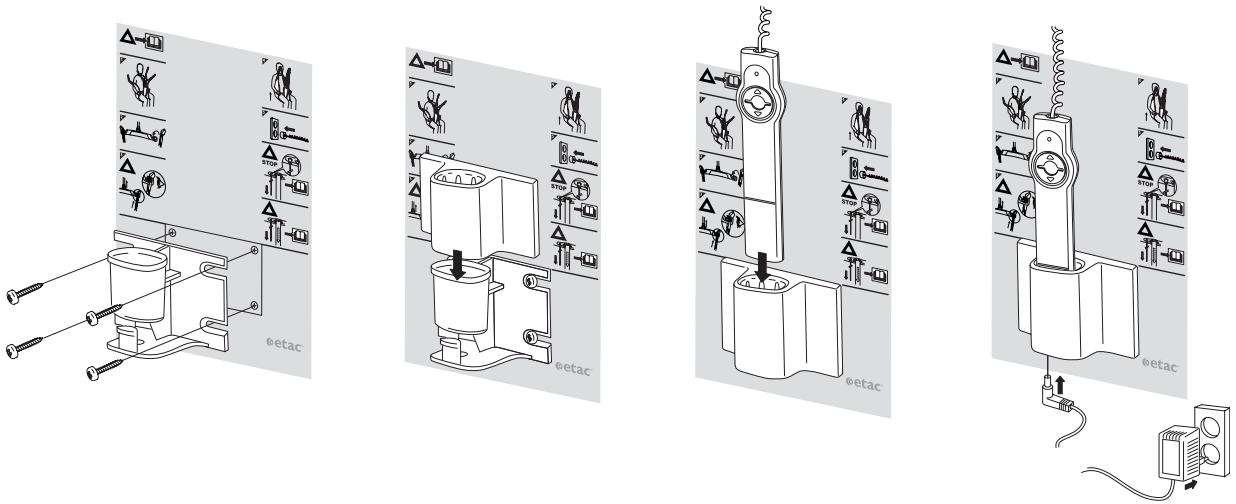
G

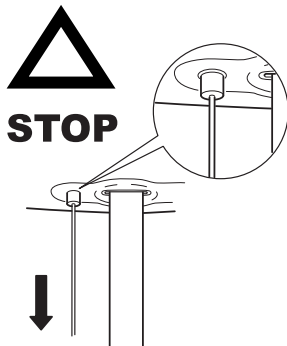G 1


G 2

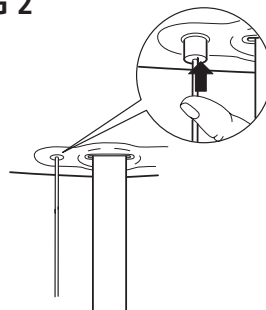
H

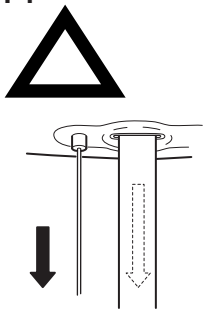
H 1


H 2

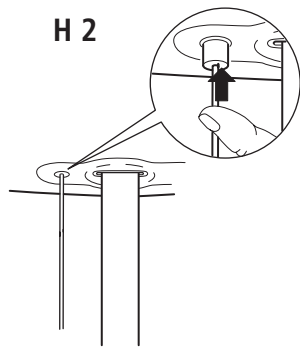
I

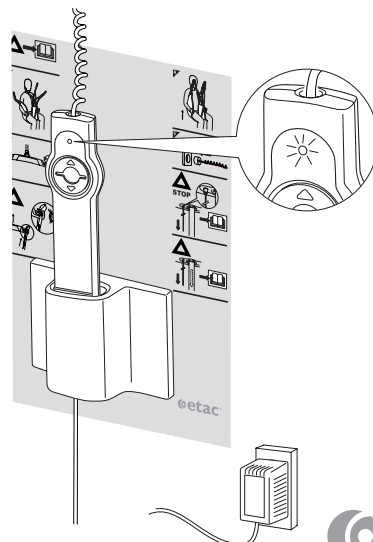
J

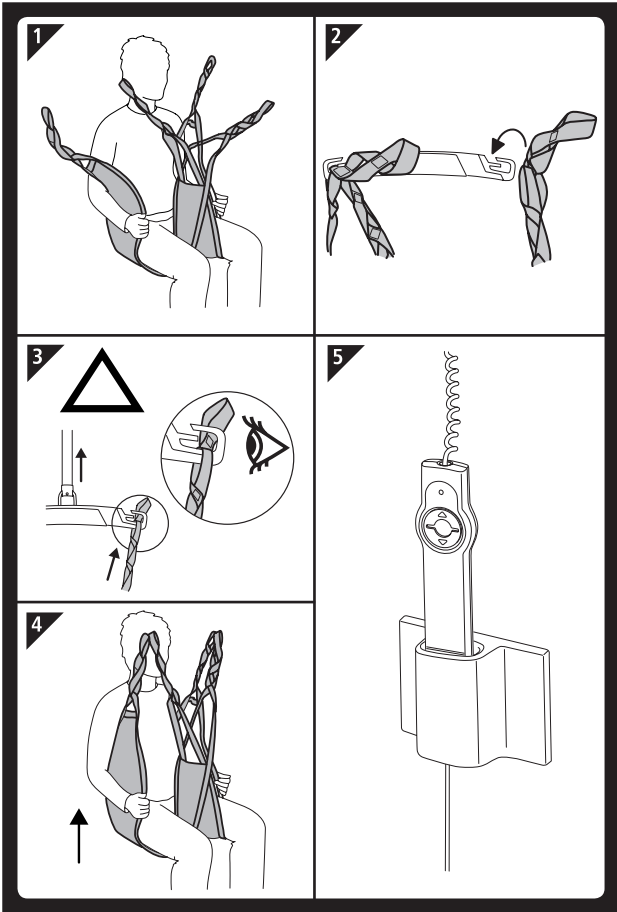K

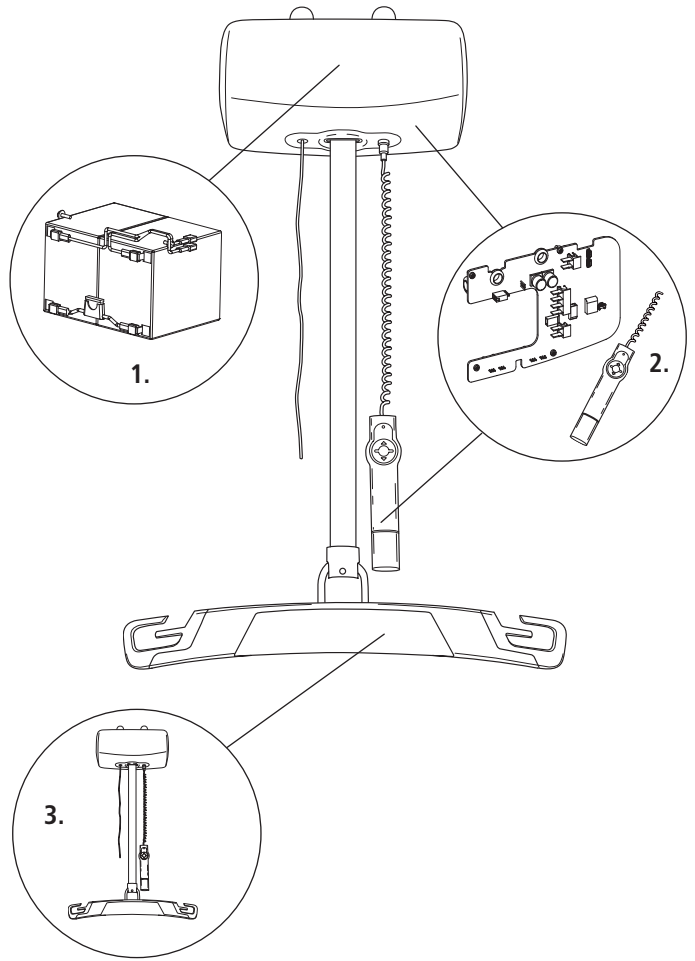L

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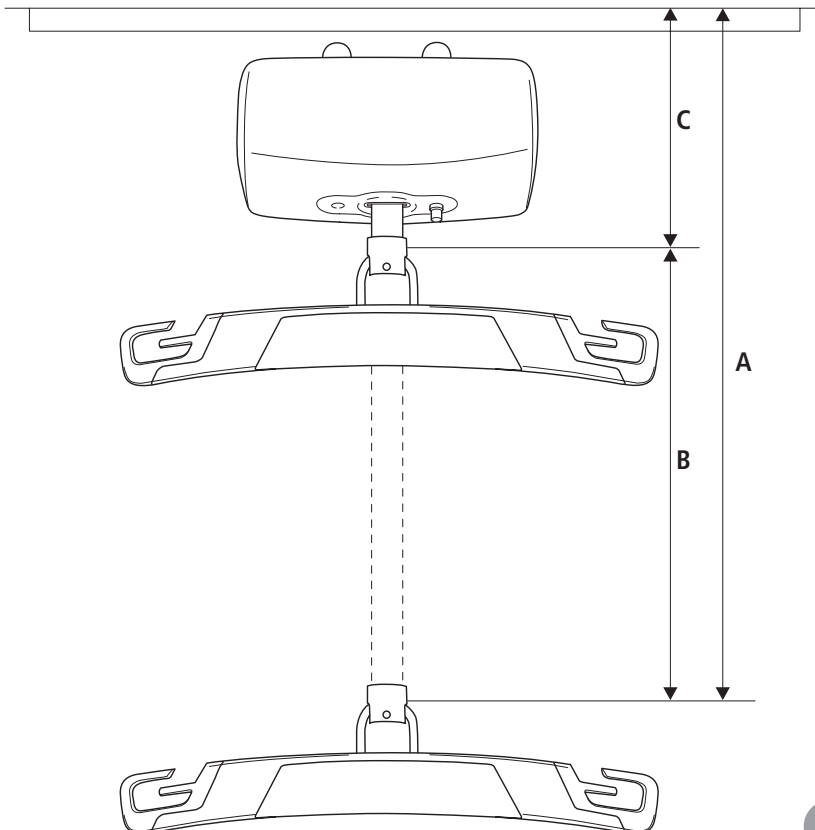General information

EN

Congratulations on your purchase of a lift from Etac. In order to avoid injury during handling and use, please read through the lift and sling manual.

In this manual, the user is the person being lifted. The assistant is the person operating the lift.

This symbol appears in the manual along with text. It indicates instances where the safety of the lift motor, the user or the assistant may be at risk.

The lift has been tested by a testing institute and complies with the requirements stated in ISO 10535:2006, EN 60601-1, EN 60601-1-2.

NB! With the installation of medical devices equipment, special care must be taken with regard to electromagnetic radiation (EMC).

Even radio frequency equipment, such as walkie-talkies and mobile phones, can affect medicotechnical equipment.

If in doubt, contact Etac or staff responsible for the equipment.

Etac is continually making improvements to its products. We therefore reserve the right to make changes to our products without prior notice. The dimensions shown in the drawings or other material are for guidance only.

We cannot be held responsible for any printing errors or omissions.

Permitted lifting slings/slingbars

EN

– Etac has a separate range of slingbars and lifting slings specially designed for Etac's lift motor.

– Etac does not accept responsibility for faults or accidents that occur as a result of using a slingbar/lifting sling made by a manufacturer other than Etac.

Loading capacity of your ceiling lift system

EN

See specified maximum load for each unit. The part, whether it be the lifting sling or some other component, with the lowest loading capacity will determine the maximum loading for the whole system.

Nova 100

Prod.nr 16019001

IP20

24V FUSE 20A

Intermittant operation 15s ON/85s OFF

Charger: **Nova 100**

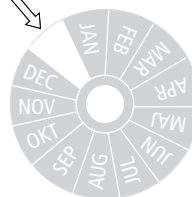
Max: 250 kg / 550 lbs

Etac, Box 203, SE-334 24 Anderstorp
Sweden, +46 371 58 73 00
www.etac.com

etac
Made in Sweden

Tested

CE mark

Type B according to EN 60601-1

Read instructions before use

Recycling according to national regulations

Description

EN

Description

fig. A

The lift unit is designed for indoor use, installed in the carriage which runs in rails attached to the ceiling. The motor has a maximum lifting capacity of 250 kg (550 lbs).

With loads of more than 250 kg (550 lbs), a buzzing sound will be heard as long as the button on the hand control is depressed.

The lift unit is powered by rechargeable batteries. Charging is effected via the hand control. A warning light will start flashing when the battery needs recharging.

- | | |
|--------------------------------------|-------------------|
| 1. Motor | 5. Hand control |
| 2. Anti-pinch guard | 6. Charger holder |
| 3. Emergency lowering/Emergency stop | 7. Slingbar |
| 4. Slingbar attachment | |

There are two versions of the hand control:

- With two functions for lifting/lowering.
- With four functions for lifting/lowering and moving sideways. This is only used when the lift is equipped with a transfer motor for operation in the rail system.

EN

Designed for use

The lift is designed to be used in the most frequently occurring lifting situations, for example, when moving to and from the toilet/shower chair, when moving between the bed and wheelchair or for lifting yourself on to and off the floor.

Before anyone is lifted using the lift, the helper must be familiar with all the necessary manoeuvring instructions and safety regulations. The Etac lift unit may only be used in accordance with this manual, by personnel who have been instructed in how to use the lift correctly.

- The lift must only be used to lift people.
- The assistant should check the lift and sling before using the lift.
- When a user is to be lifted in the lift for the first time, the assistant should first describe the process to the user.
- The lift must not be loaded with more than 250 kg (550 lbs).
- Only use accessories specially designed for Etac lifts or stipulated accessories.
- Eliminate all risks of crushing before and during lifting, for both user and assistant.
- Never lift a user if you suspect there may be something wrong with the lift.
- Etac recommends that you avoid lifting the user longer or higher than necessary.

Assembly

EN Installation of lift unit/general information

- The lift unit is delivered complete. It consists of a motor unit, standard slingbar, quick guide, hand control with holder and charging unit. Quick release pins for installing the motor unit in the carriage.
- The battery is fully charged on delivery but the hand control must nevertheless be placed in the charger holder to be sure of full battery power.

- On installation, the emergency stop string must be adjusted to the required length.

Before the rail system is used for the first time, the installation of the following must be inspected.

EN Installation of hand control

fig. B

The hand control is fitted using the screw attachment.

EN Installation of lift unit in carriage and curve carriage

fig. C 1, C 2 and C 3

Push in the quick release pins to the first position (fig. C1). Raise the lift unit with its tracks towards the attachment hole of the carriage (fig. C2) or curve carriage (fig. C3). Then push the quick release pins into the bottom position by holding the quick release

button down. Release the quick release button and attach the cover to the back.

Ensure that the quick release pins are in the locked position.

EN Installation of lift unit on transfer motor

fig. D

It is important that the active transfer motor and the passive carriage are installed in the rail system in the correct order. The driving wheel on the transfer motor should be positioned so that it points away from the passive carriage. See illustration. Push in the quick release pins to the first position. Raise the lift unit with its tracks towards the attachment hole of the transfer motor and the passive carriage. Then push the quick release pins into the bottom position by holding the quick release button down.

Release the quick release button and attach the cover to the back.

Connect the contact from the transfer motor to the lift unit. Attach the hand control designed for the transfer motor with additional sideways function (included in the purchase of the transfer motor) by means of the screw attachment.

Ensure that the quick release pins are in the locked position.

EN Assembly/replacement of slingbar

fig. E

Release the quick release pin on the slingbar attachment by pressing the quick release button and pushing with one finger on the other side. Fit the slingbar and push the quick release pin back into the

bottom position by holding in the quick release button. Then release the quick release button.

Ensure that the quick release pin is in the locked position.

EN

Installation of charging unit

fig. F

Fit the wall bracket to the quick guide and connect the charging socket to the charger holder. Connect the transformer to an earthed power outlet.

Ensure that the hand control's labelling is at the edge of the hand control holder.

Charger: Nova 100, FW7318M/24

Operation

EN

Emergency stop

fig. G 1 and G 2

Activate the emergency stop by pulling the red string (fig. G1).

To reset, push the emergency stop housing back to its original position (fig. G2).

When activated, the red part of the emergency stop housing will be visible. When the emergency stop is activated, this will be indicated by the light diode on the hand control which will flash rapidly.

Tip! If you cannot manage to reset the emergency stop, use the slingbar as an extension.

EN

Emergency lowering

fig. H 1 and H 2

If, for any reason, the lift stops working while in use, the user can always be lowered without any risk using the emergency lowering function.

To activate the emergency lowering function (fig. H1), pull the red string and keep pulling until the user is in the required position. The user can be lowered at any time because the battery will never lose its charge sufficiently to make this impossible.

To reset, push the emergency stop housing back to its original position (fig. H2).

EN

Charging the battery

fig. I

Make sure the transformer is plugged into a grounded wall socket. The lift unit's battery is charged by placing the hand control in the charger holder. If it needs recharging, the light diode on the hand control will start flashing. During charging, the diode will stop flashing and remain fixed. Once the battery is fully charged, the diode will go out and it will go into maintenance charging mode.

We recommend putting the hand control in the charger holder whenever the lift is not being used. Make sure the hand control is placed correctly in the charger holder (see marking). The diode will then indicate charging.

The hand control has a yellow light diode which lights up as follows:

Charging in progress:	Fixed light
Charging recommended:	Slowly flashing light
Battery fully charged:	Light off
Emergency stop activated:	Rapidly flashing light

Operation

EN Prior to use

The assistant is responsible for performing a visual check of the lift prior to use. If any problems are discovered, the lift unit must not be used until it has been repaired by a qualified technician. Damaged lifting slings must be discarded.

The following checks must be performed before every lift

Ceiling lift

- Check that all parts are in place.
- That the assistant is familiar with how the lift works.
- Check that the lift unit is not making any abnormal noises.
- Check that there are no signs of damage on the lift.
- Check the condition of the lifting strap.
- Check the attachment between lifting strap and bar.

Sling

- Check that the sling is undamaged, especially the straps.
- Check that a prescribed sling is chosen.

Slingbar

- Check that a prescribed slingbar is chosen.
- Check that there are no signs of damage on the slingbar.
- Check that the slingbar is secured correctly.

WARNING!

Check that a prescribed sling and slingbar are chosen.

EN Replacement of sling bar

See slingbar assembly (fig. E).

EN Operation

fig. J

- When raising or lowering, the lifting strap should be loaded with, for example, the slingbar.
- We recommend using the sling guide for choosing the correct sling.
- Place the sling on the user.
- Connect the strap loops on the sling to the slingbar. Check that the straps are in the slingbar hooks.
- Stretch the sling straps by pressing the button on the hand control marked with an arrow upwards. Check that the lift unit is positioned directly above the user, that the sling is sitting correctly and the user is sitting in a comfortable position.
- Press the hand control's arrow upwards to lift the user and effect the transfer. Make sure you always maintain eye contact with the user.
- If the rail system is fitted with a transfer motor, sideways transfer can be effected with the help of the hand control's arrow left or right.

Be particularly careful to find out which direction the transfer motor will run when the arrow left or arrow right is depressed on the hand control. As you can stand both in front of and behind the lift unit, you may occasionally make a mistake regarding the direction.

Also, be careful to ensure that there is nothing to impede the transfer motor when running so that the user or helper are not injured in any way.

- Lower the user to the intended location by pressing downwards on the hand control's arrow.
- Disconnect the lifting straps from the slingbar.
- Reset the slingbar to the parking position and put the hand control in the charging unit.
- NB! If overloaded, a buzzing sound will be heard as long as the button on the hand control is depressed.
- If it needs recharging, the light diode on the hand control will start flashing.

Maintenance/Service/Warranty

EN Cleaning

- Use a damp cloth and a mild cleaning agent or alcohol to clean the casing and slingbar externally.

NB! Do not use cleaning agents containing phenol or chlorine as they can damage aluminium and plastic materials.

EN General maintenance

A general inspection of the sling, the lifting strap's condition, attachment between lifting strap and bar as well as the rail system's attachment to ceiling/walls must be performed before each instance of use.

If preventative maintenance reveals a defect that might compromise the safety of the user, the lift must be taken out of service immediately. The lift must not be used again until the defect has been dealt with and function has been checked.

If any defect is detected in the ceiling lift, the lift must not be used until repairs have been carried out by a suitably qualified technician. Damaged slings must be discarded.

EN Annual service/inspection

The preventative maintenance service includes inspection of the lift's essential components such as the rail system, attachment devices, lifting strap and slingbar, as well as checking functionality and safety appliances. The maintenance service must be carried out by an individual authorized service partner. The test results must be documented with an inspection report.

Repairs and maintenance must only be carried out by an individual authorized service partner using original spare parts.

EN Warranty

- The motor has a six-year warranty; the battery a one-year warranty. For full warranty information, see Etac's "Warranty Ceiling Lift" at www.etac.com

Troubleshooting table

EN

Symptom	Possible cause	Troubleshooting	Corrective action
The lift does not function.	No power.	Check that the emergency stop is not activated.	See chapter emergency stop.
		Check that the battery is charged.	Check that the charger connected to a socket with power. See chapter charging the battery.
	Contact fault.	Check contact between motor and hand control.	See chapter installation of hand control.
		If the lift does not function after the above actions, contact your service partner.	
The lift produces a buzzing sound when lifting.	Maximum user weight exceeded.	Check that the user/sling is not trapped in something, e.g. stuck in their wheelchair.	Free the user/sling.
		Check the user's weight.	Choose a lift with a higher user weight.
The lift produces a jarring sound.			Contact your service partner.

Recycling instructions

EN

Recycling instructions

fig. K

All aluminium parts are recycled as aluminium or mixed metals.

Parts of the lift unit/transfer motor are recycled as electrical waste. Batteries as battery waste.

This product may contain substances that can harm the environment if not recycled according to national directives.

1. Lead acid battery
2. Waste from electrical/electronic equipment
3. Mixed metals

Technical information

EN	Maximum load:	250 kg (550 lbs)	
	Battery charger:	3-stage charger 24V DC 1.5A max.	
	Charging time:	7 hours	
	Lift speed:	6 cm/sec (2½")	
	Sound volume:	50 dB	
	Lift motor dimensions:	W 325 mm x H 165 mm x D 184 mm (13" x 6½ x 7¼")	
	Lift motor weight:	10.5 kg (23,2 lbs)	
	Lifting belt:	Width 35 mm, thickness 1 mm (Width 1½", thickness 1/16")	
	Intermittent operation:	15% operation = 15 sec., 85% rest = 85 sec. continuous operation	
	Safety class motor:	IP 20 (IP 43*)	
	Safety class hand control:	IP 54	
	Compressive force hand control buttons:	3.3 N	
	Emergency stop:	Yes	
	Emergency lowering:	Yes	
	Low battery voltage warning:	Yes	
	Exceeded max. load warning:	Yes	
	Max. dimension from ceiling (rail narrow):	2880 mm (113") (A)	
	Lifting range:	2500 mm (98") (B)	fig. L
	Min. dimension from ceiling (rail narrow):	380 mm (15") (C)	
	Electric connection of charger:	100-240V 50-60Hz	

* IP 43 classification attained on a standard product with accessories, protective cover and protective box for charger.

Storage/Transport

EN If the lift is to be stored for a prolonged period we recommend storing it in Etac's original box, indoors with normal humidity (not exceeding 60%).

We recommend transporting it in Etac's original box.

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Etac Sverige AB
Etac A/S