# Bruksanvisning

---

**EUROFLEX®**

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## INTRODUCTION

We congratulate you to the choice of work chair. We hope you will be satisfied with this product from Eurovema AB, constructed and built in Gislaved, Sweden. The work chair is developed to meet very high demands on comfort and function. The chair is suitable for disabled people and for those who spend a lot of time in their chair. The chair is fully adjustable in order to meet all the needs for adaptation. Read the manual carefully before use.

## CONTACT

If you have questions or need help with your product, please contact the local agent.

To get in touch with the manufacturer see info below:

Eurovema AB  
Tel: +46 371 390 100  
Fax: +46 371 189 82  
E-post: info@eurovema.se  
Baldersvägen 38  
SE-332 35 Gislaved  
Sweden  
www.eurovema.se

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## ASSEMBLY

- Open the packing and check that no transport damage has occurred.

- Also check that the delivery harmonize with the order.

The chair is delivered with backrest and armrest not mounted! Backrest and armrest are equipped with set screws (2, 3 and 7) to prevent that they can loosen the adjustment to the end positions. Requirements acc. SS-EN1335.

-Mount the backrest and the armrests as shown in the picture. Tighten knobs (1 and 4).

## TRANSPORTATION

During the transportation of office chair so you can easily remove the back and lower the armrests and lower seat lift to thereby reduce the transport size.

Unscrew the set screw on the backrest (2) and lift off the seat back.

Lower the armrest to the bottom position. Lower the seat lift to the bottom position.

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## SAFETY

• Carefully read the manual before using the wheel chair.

• The work chair is designed to be used on even surface in normal indoor climate.

• Be careful when adjusting the high backrest and neckrest seated. Could cause a risk of falling backwards.

• Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.

• Immediately contact service if damages or change in function of the chair is discovered.

• If the chair is used frequently it should have service regularly.

• Make sure to tighten all screws, knobs and levers after adjustments.

• Immediately contact service if damages or change in function of the chair is discovered.

• Service and maintenance should be performed by educated staff.

• Use only original parts from Eurovema at repair.

• Do not exceed maximum user weight.

• Avoid contact with a damage or leaking battery. Used batteries should be recycled. Make sure the screws are mounted in the armrests and back fastening. They prevent the armrest or back to loosen the adjustment of the end position. Requirements of SS-EN1335 standard.

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## BRAKES

### Brake

- Push the brake lever (12) forward. The rear wheels are locked.

### Release

- Pull the brake lever backward (12).

The brake lever can be fitted on the right side (standard) or the left.

- Loosen screw (17) and pull out the hexagon bar (16). Enter the bar from the other side. Mount in reverse order.

### RISK OF FALLING

OBS! Always engage the brakes before entering or leaving the chair. Make sure that the chair is not standing on loose carpets or similar. This can affect the brake ability in a negative way.

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- Loosen lever (1). Set the desired angle. Tighten the lever.

### Setting the height of the back rest

- Loosen the knobs (2). Set the desired height. Tighten the knobs.

### Setting the depth of the back rest

- Adjust the depth roughly by removing the screw (3). Tighten the screw.

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## SEAT HEIGHT - gas

### Raising

- Pull the gas-lift mechanism lever (1) upward. Do not load the seat. Release the lever at the required seat height.

### Lowering

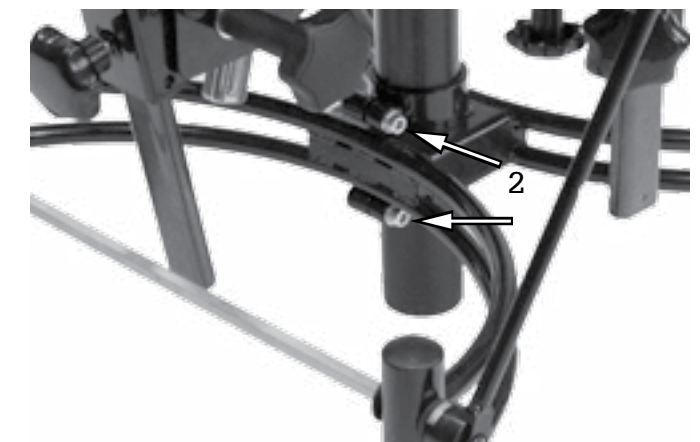
- Sit on the chair. Pull lever (1) upward. Release the lever at the required seat height.

Lever (1) can also be mounted on the left side.

Gas lift is set to standard size. To change the default setting, loosen the screws (2). Seat height is adjustable by 8 cm.

### NOTE!

Important to check the stability of the changing dimensions.

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english

### SEAT ANGLE - manual

Seat angle is possible to adjust from 0-20° forward. To adjust the angle of twist of the knob (5). Adjust the desired seat angle. Locks by tightening the knob.

#### FALL RISK!

NOTE! When the wheel is unloaded seat angle adjustment and movable seat can quickly fall forward if you sit in the chair. Use caution when adjusting!

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### ARMREST - adjustments

#### Height

- Loosen the knob (1) and set the desired height. Tighten the knob.

#### Width

- Loosen the knob (2) and set the desired width. Tighten the knob.

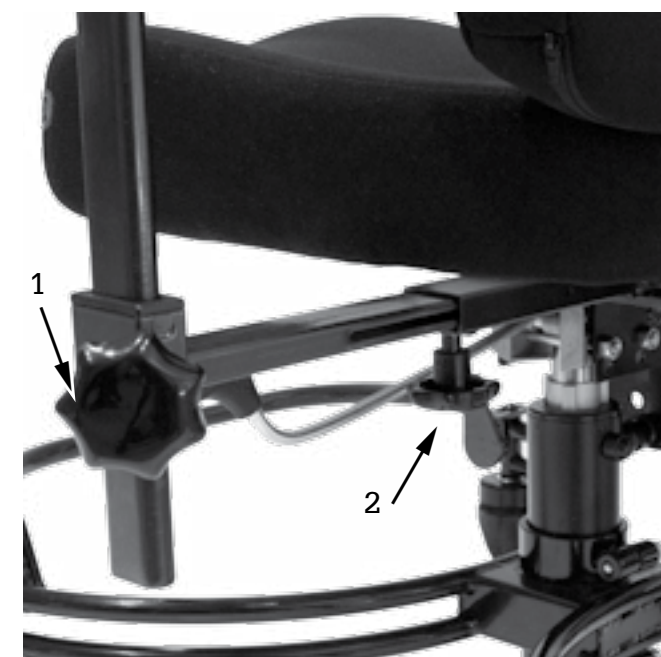
### FOOT RING

Foot ring is mounted with four screws in the front of the chassis. Foot ring is also hinged to facilitate movement (3).

Foot ring is not standard on Coxit. Is available as an option.

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## ROTATION

At the rotary (3) requires the chair to be braked. For the locking lever down (4) to apply the brakes. For the locking lever (4) up to release the brake. Slide the rotating lever (3) downward, the seat can now be rotated 180°. The seat can be locked in three fixed positions. For rotating the lever up to lock into position. The lever (3) can also be mounted on the right side.

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## COXIT SITS

Work chair with coxit-seat: to adjust the valves, loosen the locking knob (1) up to under the seat to the desired angle for each flap. Tighten the locking wheel (1) when the right angle achieved.

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\*Trinnløst justerbar.

\*Adjustable between min and max (actuators).

## TECHNICAL DATE

ISO	
chassis, width	
chassis, length	
weight (gas)	
user weight (max)	
wheel, diameter	
seat width, between armrests	
seat depth, front seat edge to back	
seat height - std gas spring	
seat, sizes, cm	
seat, angle - manual	
back, size - std WxH	
neckrest, size WxH	
armrest, height	
armrest, plate, size	
upholstery, color	

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## WARRANTY

We give a 2 year warranty on our products for fabrication fault / damage. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

CE - label 

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. Tested and approved according to Directive NSEN1335-1, -2, -3.

## MAINTENANCE

**Lift actuator:** The gas and electrical lift column (interior tube) can stick and move jerkily. Raise the seat to its highest position and dry off the column properly with a cloth. Then lubricate it with Teflon-based grease.  
**Fuse:** Located next to the battery behind the protective housing see lower picture. The fuse is 15A. If the chair stops suddenly, this can be due to the fuse having blown. Replace the fuse with a new one.  
**Upholstery:** The upholstery is made of 100% polyester and withstands spilling. It withstands high temperatures (60° machine washing – cover). Remove any spots with warm soapy water and a damp cloth.

NOTE! Ensure that all locking wheels and screws are well tightened when adjustments to the chair have been made.

*Eurovema*  
Creates mobility and ergonomics.