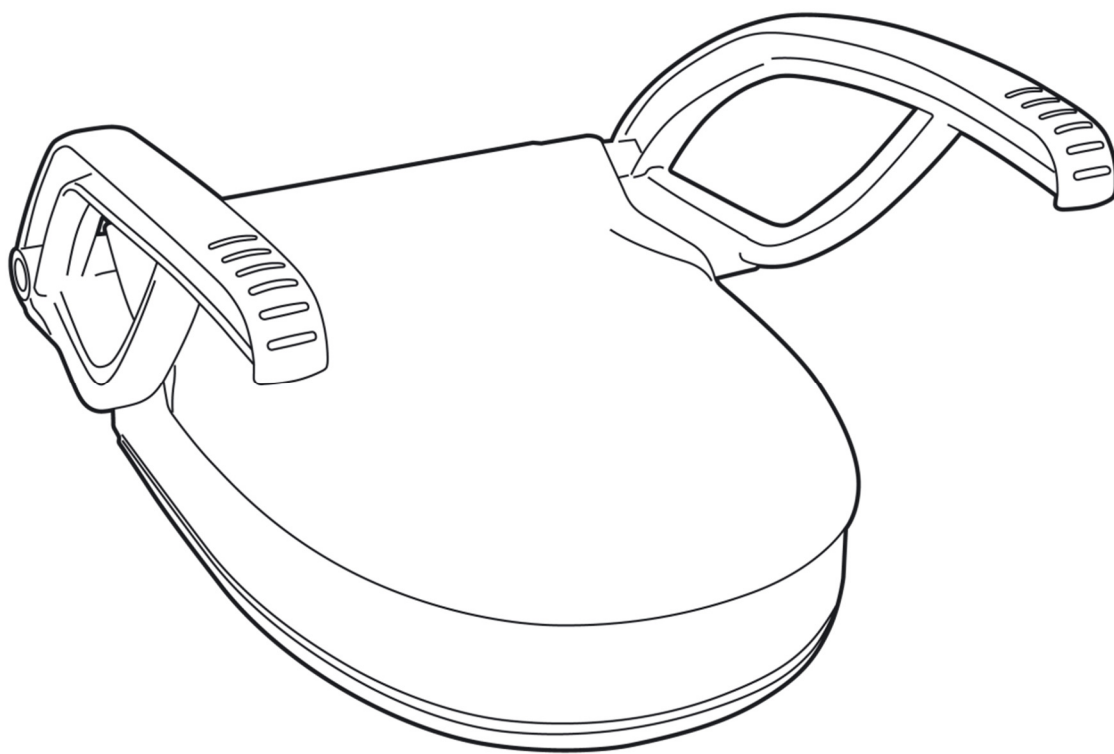
FLUSH

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Intended use: Flush is designed for use by individuals who need support handles and/or an elevated seat on the toilet.

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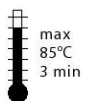
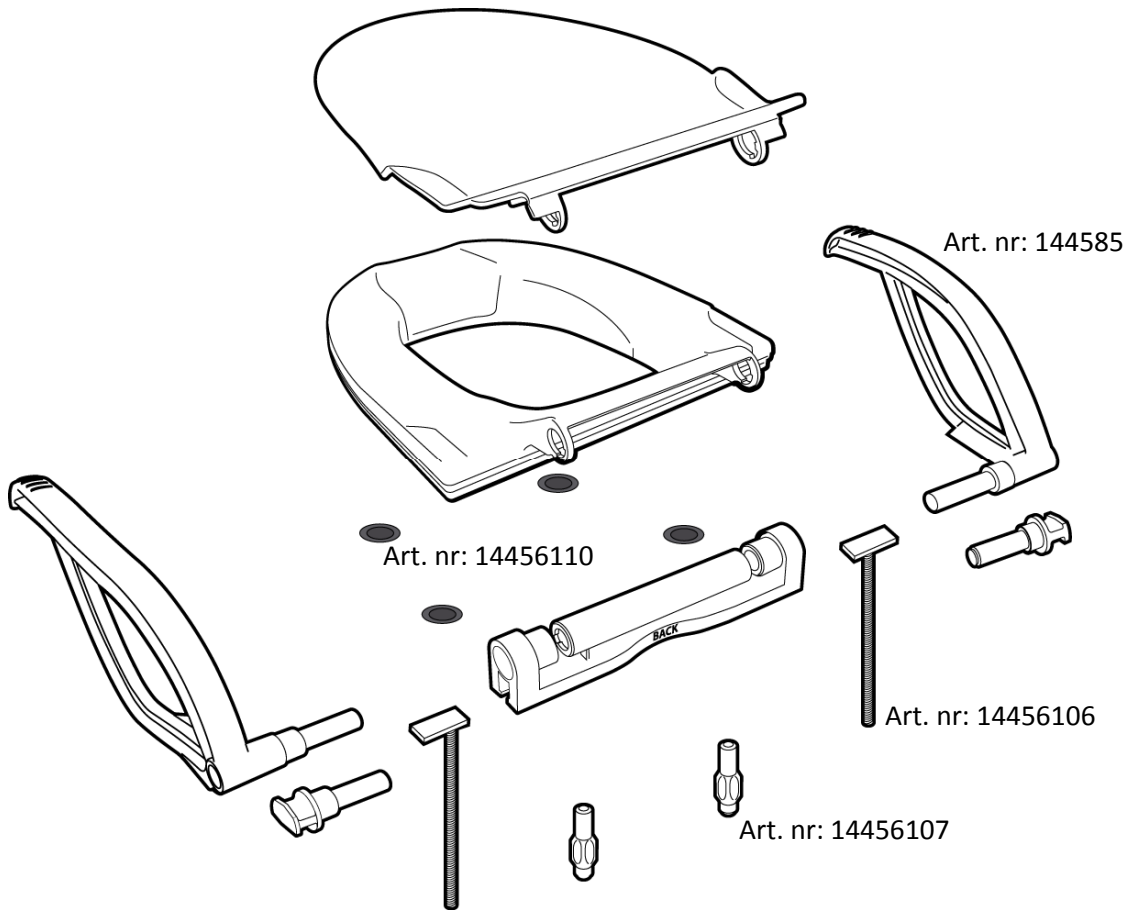
Safety precautions:

- The product should not be used otherwise than as intended.
- The toilet seat must be installed according to the supplied installation instructions.
- Before use, check that the toilet seat is securely mounted on the toilet.
- Check the screws regularly as they may be corroded by urine.
- Beware of pinching when the armrests are folded down.
- Beware of pinching by the armrests if the patient is spastic or anxious and may pull on the armrests.
- The armrests should not be used for movement. This is because the armrests are designed to be loaded from above and not from side to side. If the armrest is loaded from the side there is a risk that the armrest will come loose from the chair. If needed, combine the toilet seat with support handles on the wall.
- Max user weight 150 kg. Maximum weight is calculated using a distributed load on both armrests.

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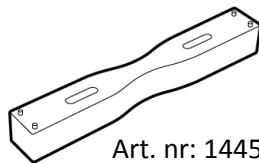
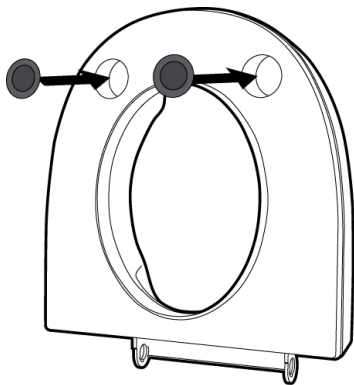
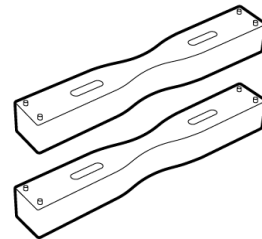
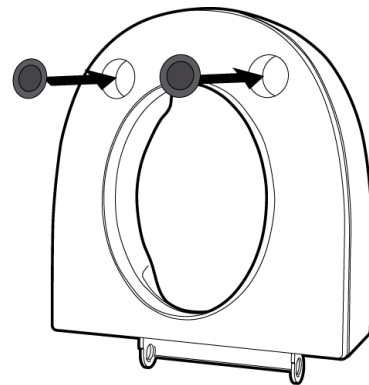
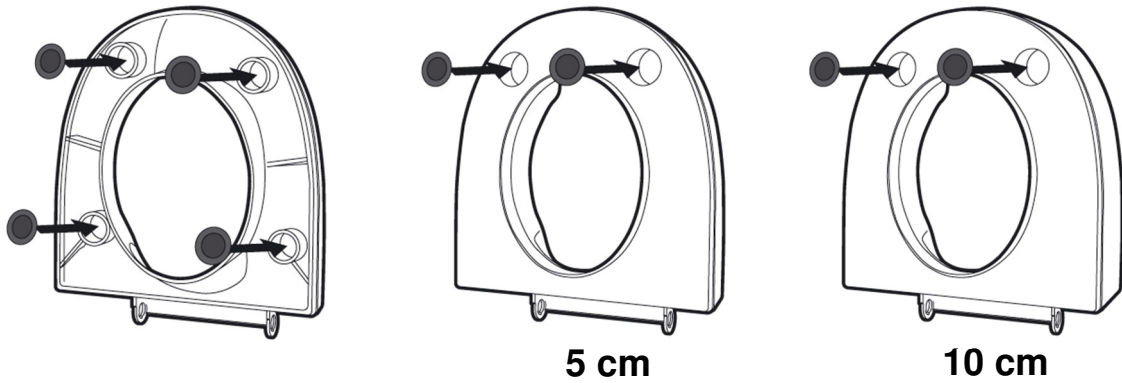
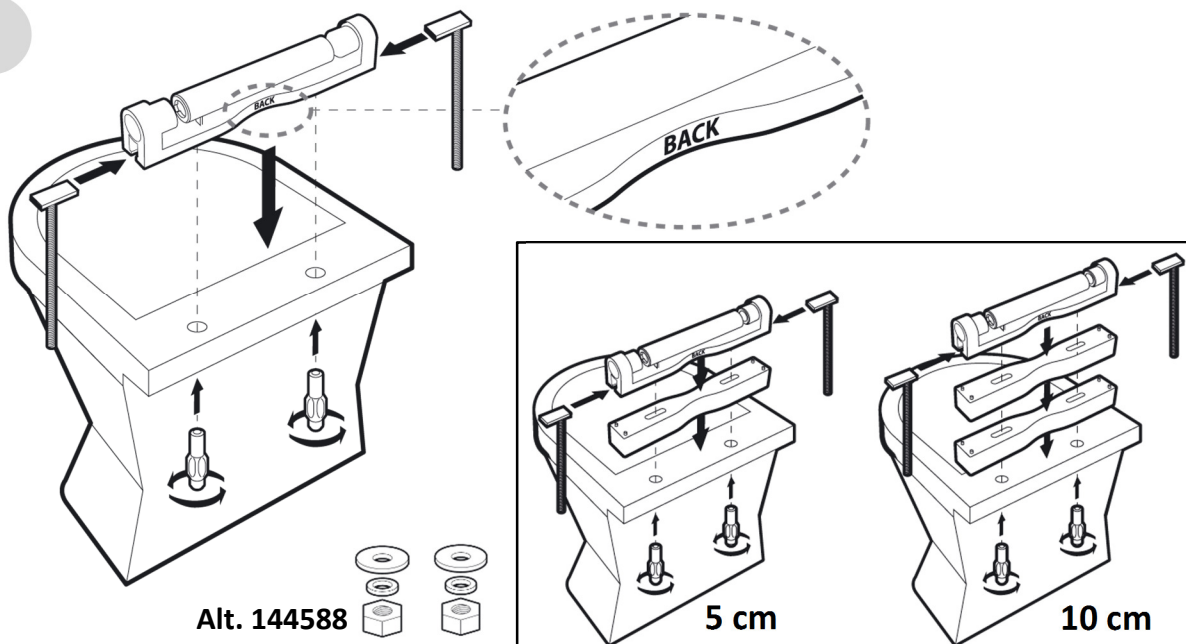
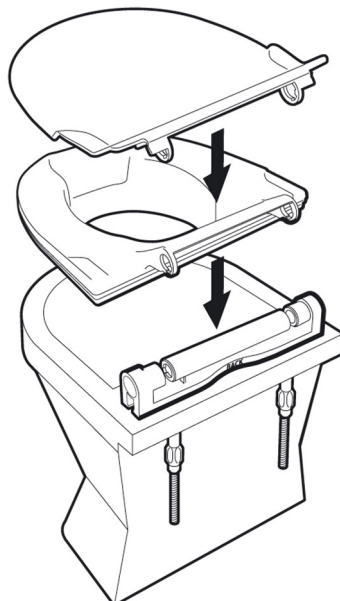
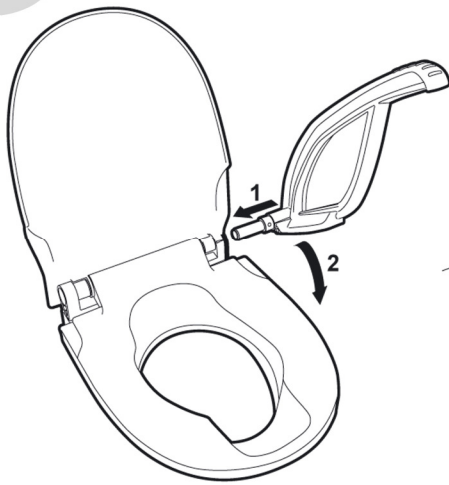
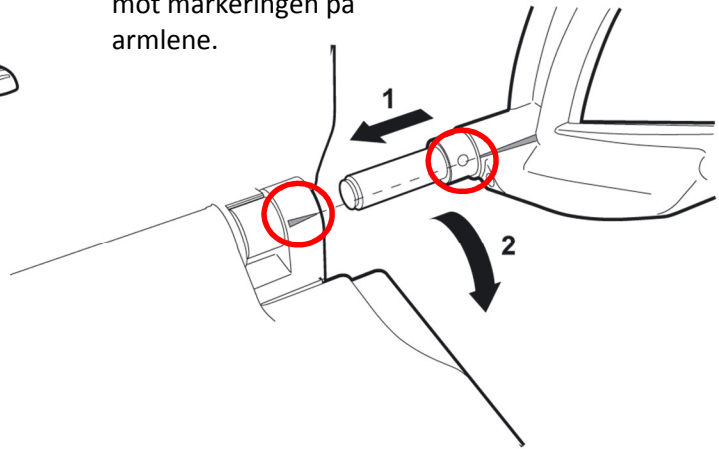
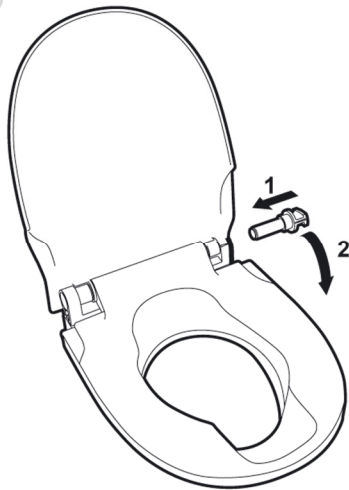
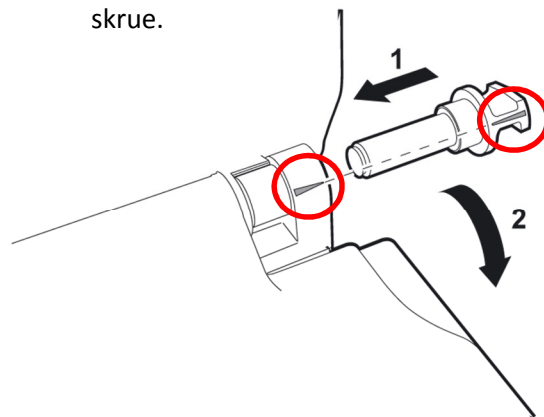
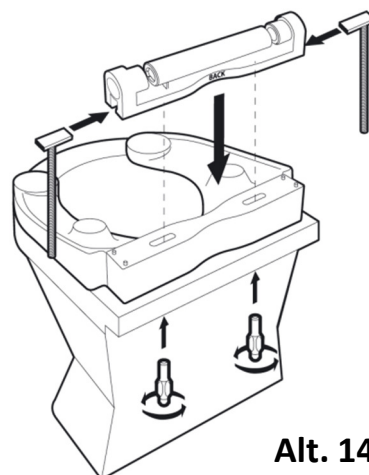
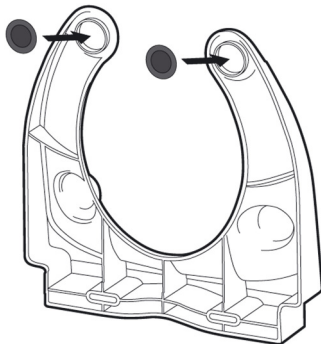
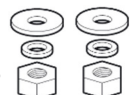
Maintenance instructions: Clean regularly with soap and water, pH 5-9. When used by a new user, the product should be disinfected, autoclaved in max 85 degrees for max 3 minutes. Can alternatively be wiped with 70% isopropanol.

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