# Aquatec® Pico / Pico TS

- EN** Shower stool / Commode chair  
User Manual
- DE** Duschhocker / Toilettenstuhl  
Gebrauchsanweisung
- FR** Tabouret et chaise de douche / Chaise percée avec accoudoirs et seuil  
Mode d'emploi
- IT** Sgabello per la doccia / Sedia da comodo  
Istruzioni per l'uso
- ES** Taburete de ducha / Silla WC  
Manual de instrucciones
- NL** Douchekruk / Toiletstoel  
Gebruiksaanwijzing
- DA** Badetaburet / Toiletstol  
Brugsanvisning
- FI** Suihkujakkara / WC-tuoli  
Käyttöohje
- NO** Dusjkrakk / Toalettstol  
Brukerveiledning
- SV** Duschpallen / Toalettstol  
Bruksanvisning
- PT** Banco de duche / Cadeira de casa de banho  
Manual de instruções
- ET** Duši-/tualetitool  
Kasutusjuhend
- LT** Dušo / tualetu kėdutė  
Veikimo nurodymai
- LV** Dušas krēsls / tualetes krēsls  
Lietošanas instrukcija

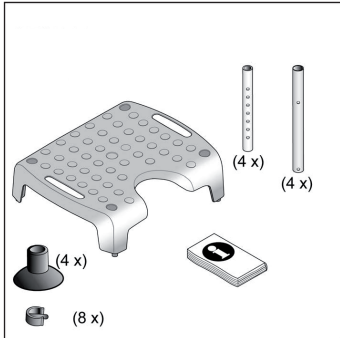
Fig. 1

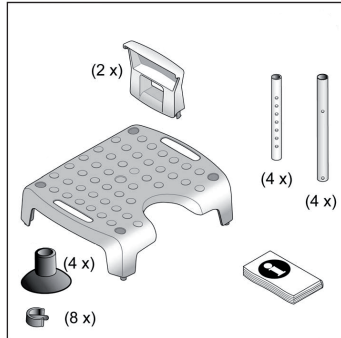
Fig. 2

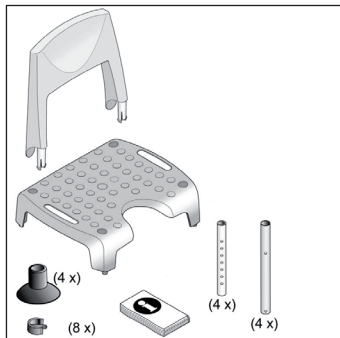
Fig. 3

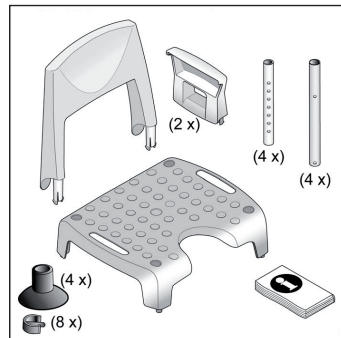
Fig. 4

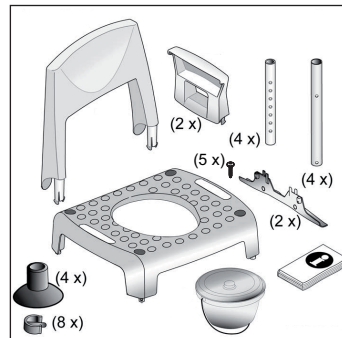
Fig. 5

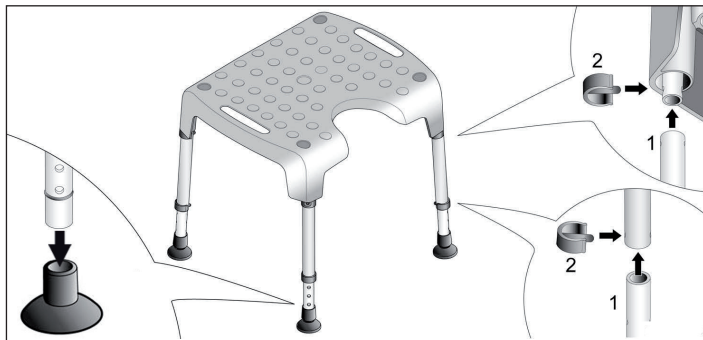
Fig. 6

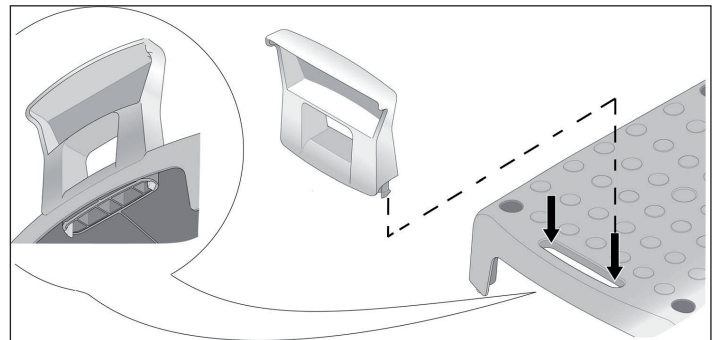
Fig. 7

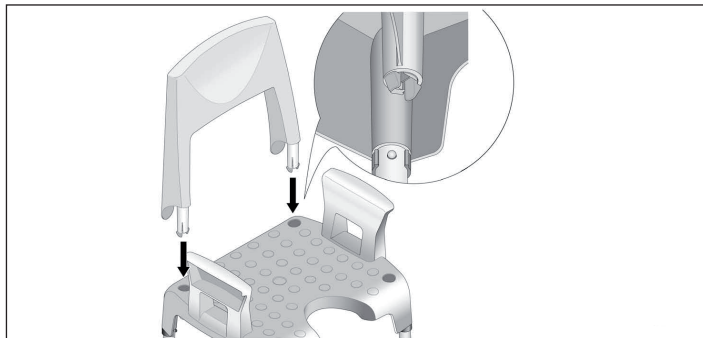
Fig. 8

**AQUATEC** PICO komplett  
with arm- and backrest  
2012-05 160 kg 40286984033804  
Made in Germany (LN)  
(01) 04028698083904 / (10) 12EDA0001

**AQUATEC** PICO TS  
2012-05 160 kg 40286984116999  
Made in Germany (LN)  
(01) 04028698111699 / (10) 12EDA0001

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can!

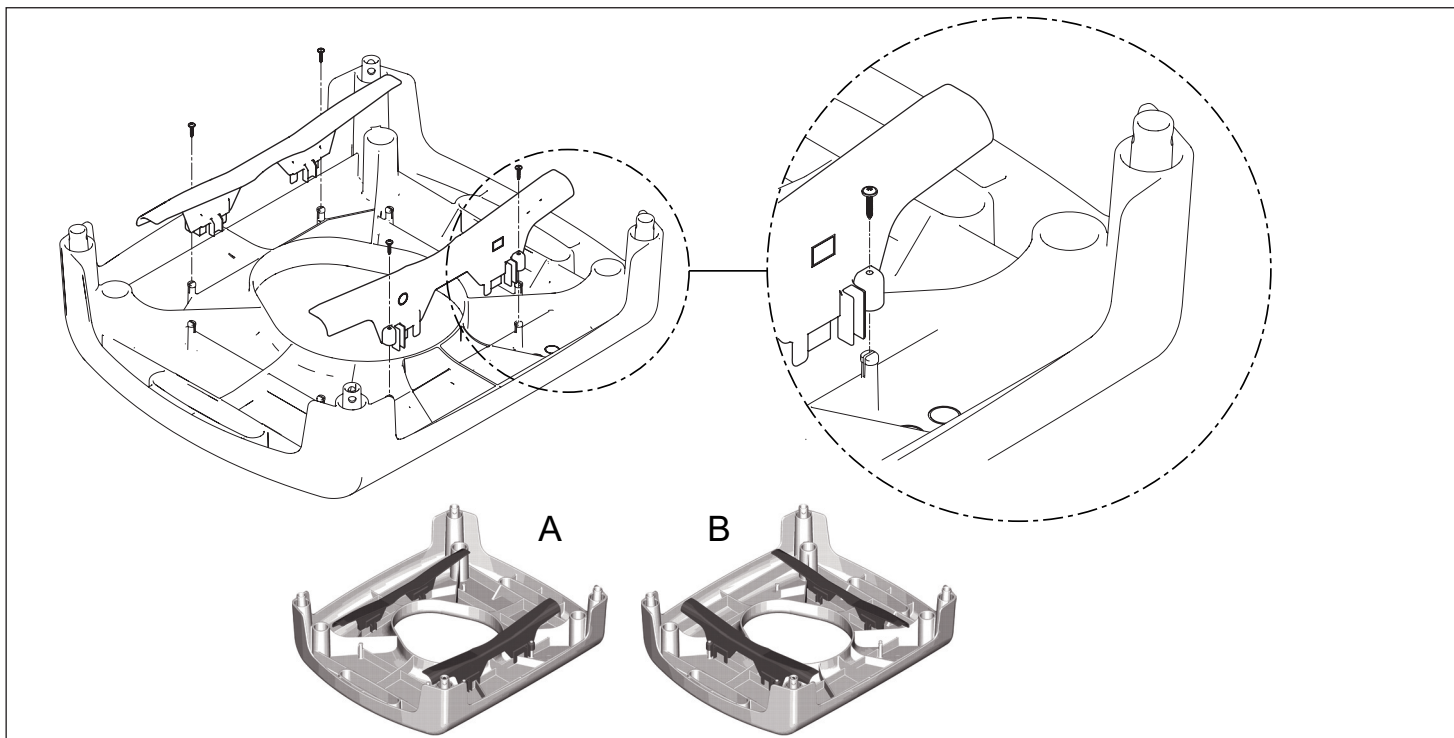
Fig. 9

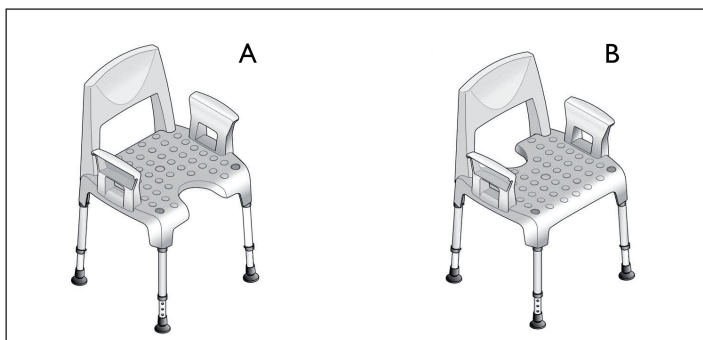
Fig. 10

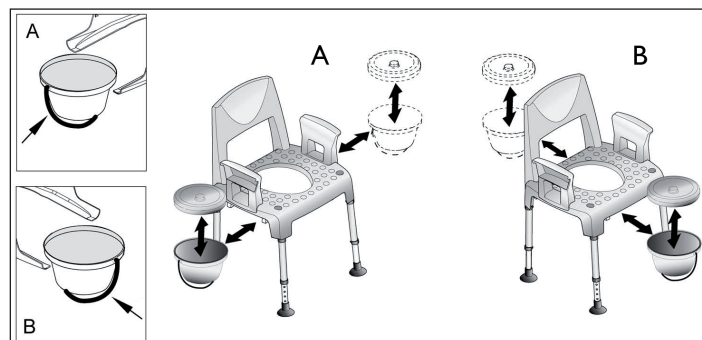
Fig. 11

## Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer

## Intended Use

The Shower stool / Commode chair is to be used solely as an aid for showering or going to the toilet. Any other use is prohibited.

Temperature for use: 10 - 40 °C

Max. user weight: 160 kg

## Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

### WARNING Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

## Scope of delivery

- Fig. 1 - 5. (Depends on model)

In the scope of delivery (Fig. 5) there are five screws included for assembling the two toilet pan holders. Only four screws are required for assembly.

## Assembly

### CAUTION Risk of pinching!

- Take care during assembly.

### WARNING Risk of falling!

- Ensure that all legs are adjusted to the same height.

- Fig. 6 - 9.

## Usage

- Fig. 10, 11.

## Cleaning and disinfection

### IMPORTANT

- Take off any fitted accessories.

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

## ! IMPORTANT

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG002](http://www.nice.org.uk/CG002) and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 85 °C for 3 minutes.

## Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

### WARNING

- Check product regularly for damage and to ensure that it is securely assembled.
- Check the arm and backrest fixings and the clips regularly to ensure they are sitting securely.

## Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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
## Technical Data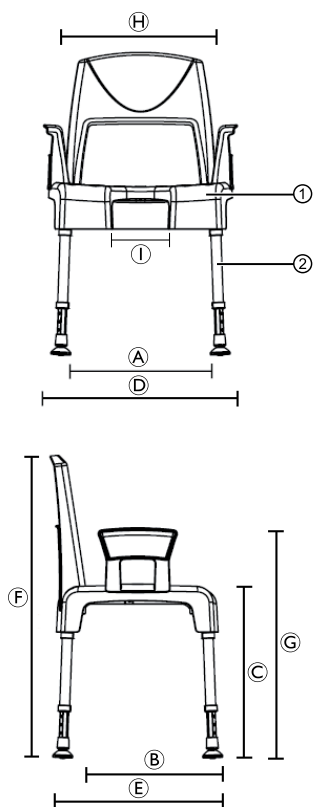

Order numbers	Aquatec® Pico
Shower stool	9.20.001
Shower stool with armrests	9.21.001
Shower chair with backrest	9.23.001
Shower chair complete	9.22.001
Backrest	16469
Armrests (pair)	16468
<b>Measurements</b>	
A	430 mm
B	420 mm
C	425 - 575 mm
D	575 mm
E	520 mm
F	825 - 975 mm
G	605 - 755 mm
H	430 mm
I	180 mm
Depth, hygiene recess	145 mm
Graduation, height adjustment	25 mm

<b>Weights</b>	
Shower stool	3.2 kg
Shower stool with armrests	4.1 kg
Shower chair with backrest	4.0 kg
Shower chair complete	4.8 kg
Max. User Weight	160 kg
Color	White
<b>Materials</b>	
①	PP
②	Aluminium, corrosion-free

### Invacare® Distributors:

#### Belgium & Luxembourg:

Invacare nv, Autobaan 22,  
B-8210 Loppem  
Tel: (32) (0)50 83 10 10  
Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

#### Deutschland:

Invacare GmbH, Alemannenstrasse 10,  
D-88316 Isny  
Tel: (49) (0)75 62 7 00 0  
Fax: (49) (0)75 62 7 00 66  
kontakt@invacare.com / www.invacare.de

#### Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101,  
A-5310 Mondsee  
Tel: (43) 6232 5535 0  
Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

#### Schweiz:

Invacare AG, Benkenstrasse 260,  
CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80  
Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

#### Italia:

Invacare Mecc San s.r.l., Via dei Pini 62,  
I-36016 Thiene  
Tel: (39) 0445 38 00 59  
Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

#### Nederland:

Invacare BV, Celsiusstraat 46,  
NL-6716 BZ Ede  
Tel: (31) (0)318 695 757  
Fax: (31) (0)318 695 758  
nederland@invacare.com / www.invacare.nl

#### Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Cam-  
pus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084  
Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

#### United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed,  
Bridgend CF35 5AQ  
Tel: (44) (0)1656 776200  
Fax: (44) (0)1656 776201  
UK@invacare.com / www.invacare.co.uk

#### España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà,  
E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00  
Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

#### France:

Invacare Poirier SAS, Route de St Roch,  
F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66  
Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

#### Denmark:

Invacare A/S, Sdr. Ringvej 37,  
DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00  
Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

#### Norge:

Invacare AS, Grensesvingen 9, Postboks 6230,  
Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00  
Fax: (47) (0)22 57 95 01  
norway@invacare.com / www.invacare.no  
island@invacare.com

#### Sverige:

Invacare AB, Fagerstagatan 9,  
S-163 91 Spånga  
Tel: (46) (0)8 761 70 90  
Fax: (46) (0)8 761 81 08  
sweden@invacare.com / www.invacare.se

#### Suomi:

Algol-Trehab Oy  
Karapellontie 6, 02610 Espoo  
Puhelin 09 50991  
Faksi 09 5099490  
Sähköposti info.algoltrehab.fi@algol.fi  
Internet www.algoltrehab.fi

#### Portugal:

Invacare Lda, Rua Estrada Velha, 949,  
P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47  
Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

#### USA:

CLARKE HEALTH CARE PRODUCTS INC.  
1003 International Dr. USA, Oakdale, PA 150719226  
Tel: 0724 69 52 122  
Fax: 0724 69 52 922  
info@clarkehealthcare.com / www.clarkehealthcare.com

#### Canada:

INVACARE® CANADA INC., 570 Matheson Blvd. East,  
Unit 8, CDN Mississauga, On. L4Z4G4  
Tel. 0905 8908300  
Fax. 0905 8905244  
www.invacare.ca

#### Eastern Europe, Middle East, Russia & CIS:

European Distributor Organisation:  
Invacare, Kleiststraße 49  
D-32457 Porta Westfalica  
Tel. (49) (0)57 31 754 540  
Fax. (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

Aquatec Operations GmbH  
Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0  
Fax: (49) (0)75 62 7 00 66  
kontakt@invacare.com / www.invacare.de

**AQUATEC**

**Yes, you can.**