GAINSBOROUGH

Specialist Bathing

USER MANUAL
WALK IN BATH s. 3-4

BRUKERVEILEDNING
BADEKAR MED DØR s. 5-6

Distributor in Norway / Distributør i Norge

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User manual

Your bath is simple to use and maintain. However, there are some basic guidelines which should be followed to ensure you get the best out of your unit.

Intended use

To provide assistance when bathing by means of a door employed in the side of the bath.

There is a locking handle employed for the purposes of locking or releasing the door. Open the door by operating the handle mechanism located on the door by pushing it fully forward until it releases the door.

To operate the unit, please follow these simple instructions:

1. Enter the bath, once inside, the door should be closed
2. Make yourself comfortable on the seat and secure the waste plug
3. Commence filling the bath with water at the required temperature. In the event of the bath being overfilled, water may either drain naturally through the overflow, or the plug should be temporarily removed to allow a sufficient amount of water to be drained away.
4. Once the bath is filled to the required level the user can then close the taps and enjoy your soak
5. When finished bathing release the waste and wait for the water to drain
6. Open the door once the bath is completely empty of water, open the door and exit the bath

Helpful hints

- Make sure the bathroom is warm before you decide to have your bath. In this way you will be able to enjoy your bath fully.
- A small towel draped across the shoulders whilst the bath is filling is also a good way to ensure you are fully relaxed.
- We suggest that, as the bath is filling, you wash yourself working from the toes upwards so that as the water rises you are rinsed off automatically.
- Keep a towel handy so that you can dry yourself as the bath empties.

Care and maintenance

Your walk-in bath is a high quality, precision-made unit. However, it does require a small amount of regular care and attention from users to ensure that it will continue to offer you many years of quality operation.

Your bath should be cleaned regularly using a soft, damp cloth in conjunction with either a standard cleaning fluid (e.g. Fairy Liquid[®]) or a non-abrasive cream cleaner.

Weekly cleaning of the door seal to remove any mould or grime is required to ensure that it remains in good condition. A badly maintained seal can lead to problems in the future.

Metal fittings, such as the door hinges and door locking mechanism, may only be cleaned with a soft, damp, soapy cloth.

N.B. On no account should abrasive, bleach, spirit or ammonia based cleaners be used on the bath.

Incorrect use of cleaners can damage the material surface. Surface and material damage that occurs due to incorrect treatment is not covered by the warranty.

IMPORTANT

When not in use the door should be left in the closed position, although not in the fully latched closed position. This will prevent the hinges being strained whilst stopping the seal becoming compressed over time.

ON NO ACCOUNT SHOULD PRESSURE BE PUT ON THE DOOR WHILST IN THE OPEN POSITION.

PLEASE NOTE THERE ARE NO USER SERVICEABLE ITEMS ON THIS UNIT.

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