# Short Instruction

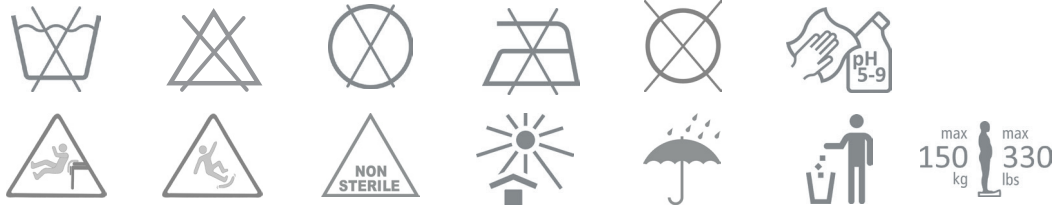
IMM1047\_SI.VER13.2022-05-16

Article no.	Description	Size (mm)
IM99836GT	PediTurn Soft	Ø400
IM99700	PediTurn Hard	Ø380

PediTurn

**immedia**  
by Etac

## GB:

### Material:

#### PediTurn Soft:

Top side: Polyester, TPE  
Inside: Nylon, PE  
Back side: Polyester, TPE  
Connector disc: Polyamide.

#### PediTurn Hard:

Top side: ABS/TPU  
Inside: HDPE  
Back side: ABS/TPU  
Fixing: Brass/steel.

### Washing instructions:

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5-9, or a 70% disinfection solution.

The product should be checked regularly, preferably each time it is used and especially after wash. Check that there is no damage to seams or fabric.

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### Materiale:

#### PediTurn Soft:

Overside: Polyester, TPE  
Indre overflate: Nylon, PE  
Bakside: Polyester, TPE  
Koblingsplate: Polyamid.

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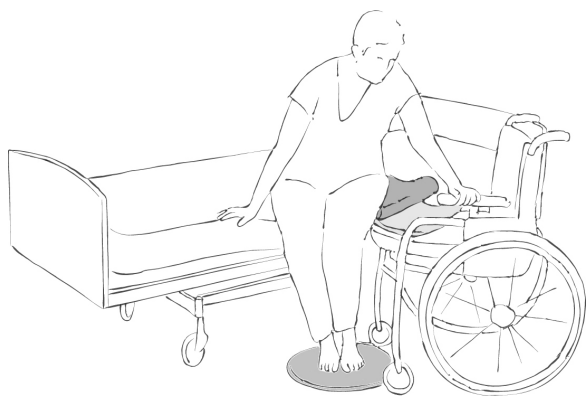
1

# General

## Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

 **This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.**

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac Patient handling, see [www.etac.com](http://www.etac.com).

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

### Symbols in general

 Washing	 Disinfect / Wipe off	 Warning	 Manufacturer
 Do not bleach	 Never leave on the floor	 Production year, month and day	 CE -certification
 Tumbledry	 Risk of sliding down	 Batch no./Lot no.	 Keep dry
 Iron	 Keep out of the sun	 User mass limit = maximum rated load	 Non sterile
 Do not dryclean	 Read the user manual	 The product can be scrapped according to national regulations	 Article number

 Medical device

### Intended use

PediTurn is a slim, lightweight turnplate for floor use in connection with manual handling in the seated position, where the user

must rotate but is not capable of moving his/her legs without assistance.

### Intended environment

Acute care, Long-term care, Home care

# Practical handling

## Positioning

Place PediTurn on the floor. Position the user's feet on the middle of PediTurn so that the user's legs are perpendicular to the device.

## Use

When manually handling a user in the seated position, the user's legs will now follow the user, thus avoiding strain on the user's foot, knee and hip joints (Illustration 1).

It is often beneficial to use SupportBelt, Sling or SmartMove with PediTurn.

# Notice

## Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

Never leave the product on the floor after use.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

Extra care is advised when using PediTurn with spastic users.

Do not take PediTurn apart – For safety reasons, if the components are taken apart, they may not be put together again. In such cases, the product must be scrapped.

If in any doubt - please contact Etac.

## Material and cleaning

### Material:

#### PediTurn Soft

Top side: Polyester, TPE

Inner surface: Nylon

Back side: Polyester, TPE

Connector disc: Polyamide.

#### PediTurn Hard

Top and back side: ABS/TPU

Inside: HDPE

Assembling: Brass, Steel.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

**Never try to repair a product yourself.  
Never use a defective product.**

### Washing instructions:

Can be cleaned with a solvent-free cleaning agent with a pH level between 5-9, or a 70% disinfection solution.

## Products and accessories

Article no.	Product Description	Size (mm)
IM99836GT	PediTurn Soft	Ø400
IM99700	PediTurn Hard	Ø380

## Combinations

Article no.	Product Description	Size (mm)
IM401	Butterfly board	W320xL660
IM403	3B-Board	W230xL770
IM404	E-Board L	W330xL600xH5
IM405	E-Board XL	W330xL750xH6
IM408	E-Board M	W250xL600xH5
IM410	E-Board S	W200xL450xH5
IM409	Dolphin ToiletBoard	L690
IM426	SmartMove w/4 handles	W210xL600
IM428	Sling - long	W160xL1900
IM428K	Sling - short	W160xL1200
IM460PU	OneManSling	W75xL3000
IM8017015	SupportBelt XL	1300-1700
IM8015013	SupportBelt L	1200-1500
IM8013013	SupportBelt M	900-1300
IM8010513	SupportBelt S	650-1050
IM8010509	SupportBelt XS	650-1050
IM8009009	SupportBelt XXS	450-900

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

# Service information

## Inspection

### First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from [www.etac.com](http://www.etac.com) or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

### Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

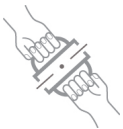
#### Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

#### Stability test:

Try to bend the product verify that it feels firm, stable and solid.

#### Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.

#### Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/moving

### Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Etac Immedia A/S
"Etac Ltd. Unit D4A, Coombswood Business Park East,