# ALVEMA

## *i20*

art.nr. BRU-JTO-ENG-NO rev: 2016081

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english

## INTRODUCTION

Thank you for choosing the ITO pushchair from Alvema. The pushchair is designed for disabled children to be used both indoor and outdoor. Please study the manual carefully and follow all safety instructions. This will ensure that you get the best use from your new chair.

english

## CONTACT

If you have questions or need help with this product, please contact the local agent. To get in touch with the manufacturer see info below:

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english

## SAFETY

- Read the manual carefully before use.
- Please note! The handle should be folded when lifting the pushchair or using on the stairs. Gripping the frame in front of the joint will give better control of the child and the pushchair and therefore be safer (20).
- Do not stand on the footplate, it should only be used for supporting the feet while sitting in the pushchair.

## SAFETY

- Do not leave children unattended in the pushchair.
- A bag attached to the frame adversely affects stability and may cause the chair to tip over.
- Never use defective equipment! Do not attempt repairs yourself.
- For safety reasons, only an approved repairer or your supplier must carry out repairs. Original spare parts are recommended. Failure to observe this requirement will invalidate your guarantee.
- The harness included with the ITO is for positioning purpose only and should not be used as a dynamic restraint belt.

### Guarantee

We give a 2 year warranty on our products for fabrication fault/damage. Exceptions are fabric and wheels where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual. Save your original receipt, you will need it for warranty repair claims.

REMARK! The guideline applying the CE-marking is the medical device directive 93/42/EEC Eurovema reserves the right to change any product without notice.

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english

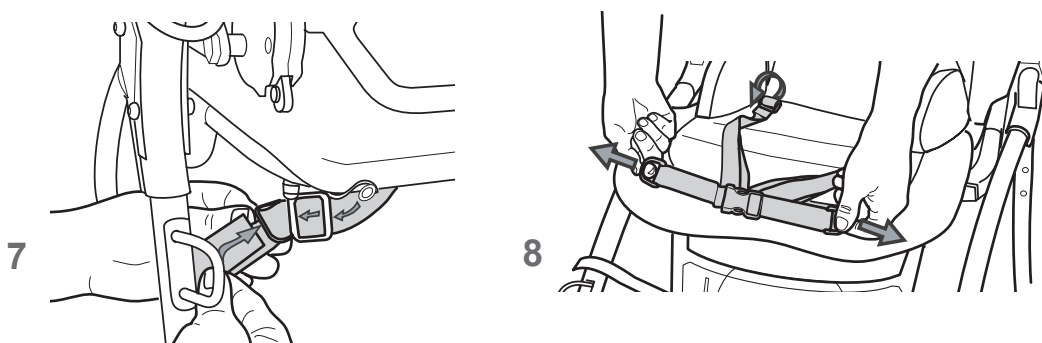
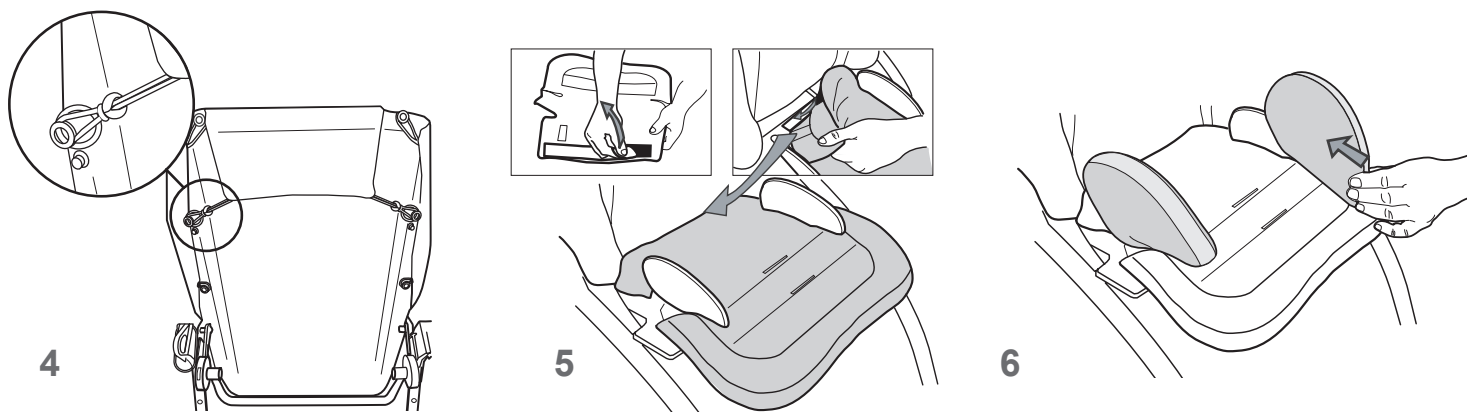
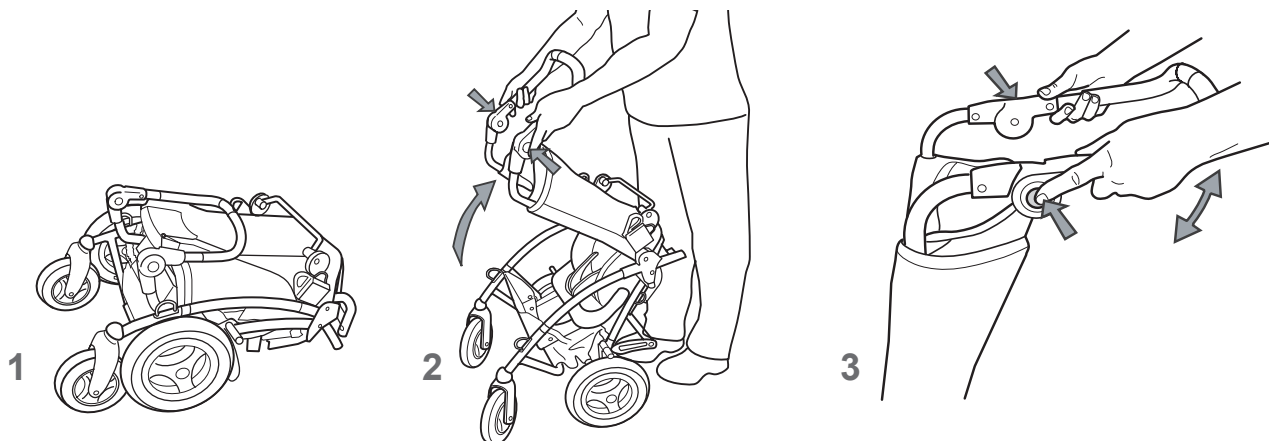
## UNPACKING THE ITO

- Remove the pushchair from the box (1).
- Remove the protective plastic from the wheels before use.
- Unfold the pushchair with one action (2). Check the folding guards and engage them if necessary.
- Press in both buttons and angle the handle up (3).
- Place the cushion on the backrest, and attach the loops around the knobs on the back of the seat (4). Fix the seat cushion to the Velcro on the seat (5).
- Cover the pelvic supports with the pads and lock the Velcro (6).
- Attach the waist-belt straps by threading them through the loops under the seat plate, then thread back.(7).

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english

## FOLDING THE ITO

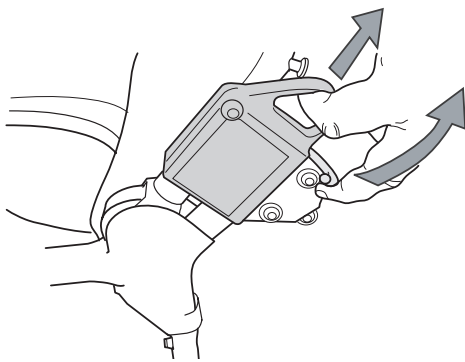
- Fold down the safety catch (9) and pull up the sleeves (10).
- Push the back forwards (11). To minimize the volume of the folded pushchair, the back should be in the most upright position.
- If the back extender is used, the footplate needs to be folded, otherwise this is not necessary.
- Angle the handle down by pressing in both buttons and then turning the handle down (12).

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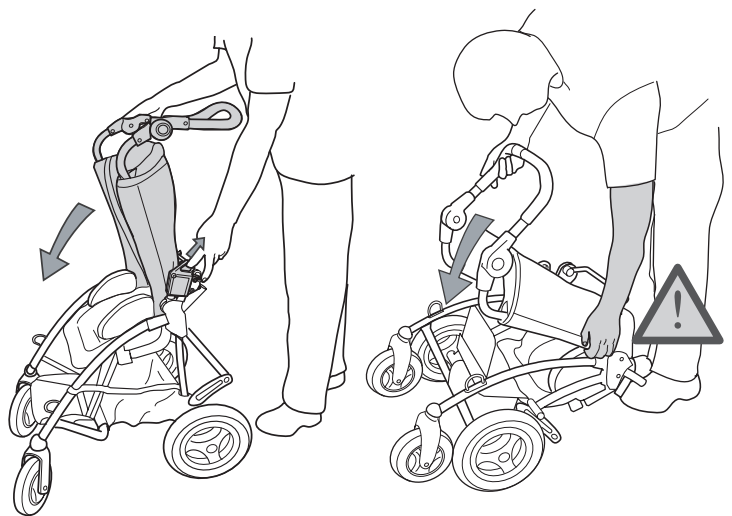
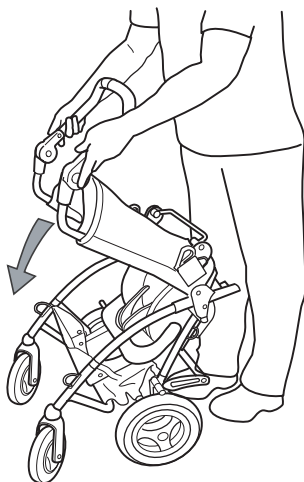
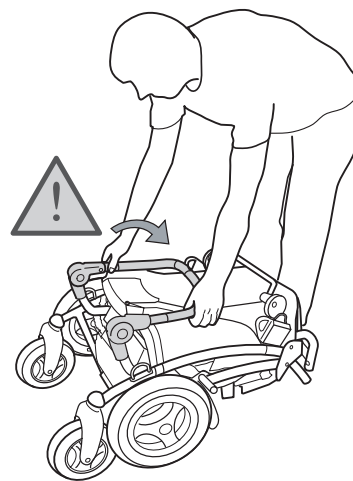
## ADJUSTMENTS

### Footplate adjustment

- Adjust the height of the footplate using a spanner/wrench (13). Release the nuts, set height and re-tighten (14).
- The mesh can be removed for washing by releasing the Velcro on the back of the footplate. Pull the straps out of the slots.

### Width adjustments

- The seat width is adjusted by loosening the two outer screws under the seat (15). Adjust the pelvic supports to the desired position and angle and retighten the screws (16).

### Depth adjustments

- The seat depth is adjusted by loosening the two screws on the tubes under the seat (17). Pull the seat to the desired length and retighten the screws (18).

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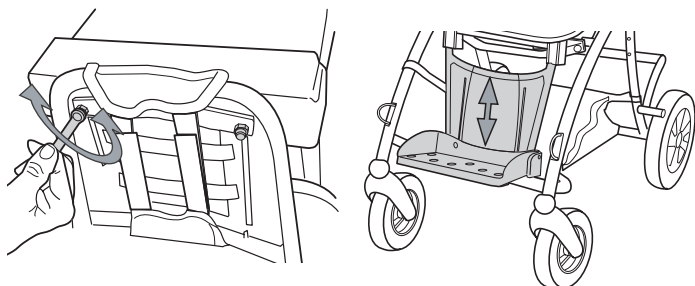
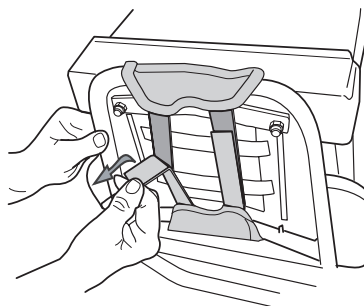
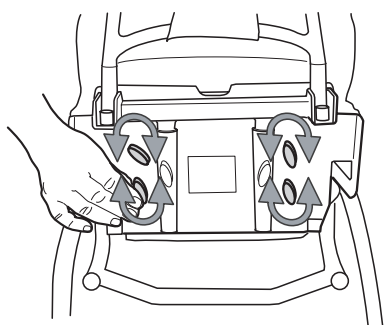
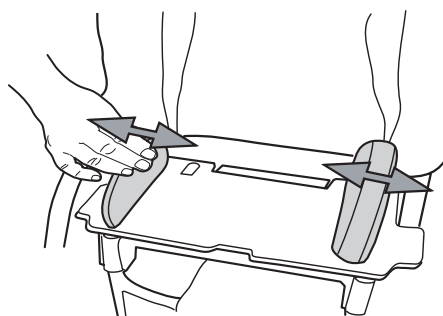
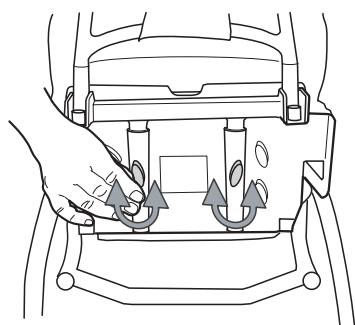
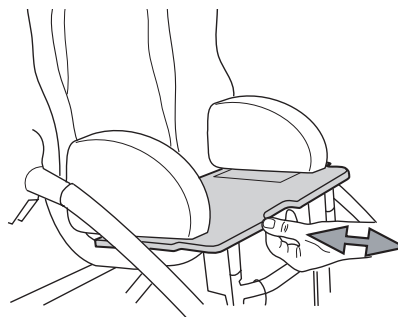
## ADJUSTMENTS

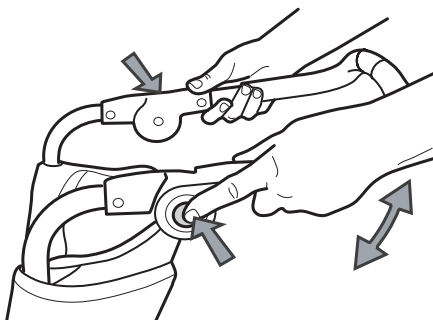
### Push-Handle adjustment

- To adjust the angle of the handle press in both buttons (19). Angle the handle and release the buttons again (20).


*Please note! The handle should be folded when lifting the pushchair or using on the stairs. Gripping the frame in front of the joint will give better control of the child and the pushchair and therefore be safer (21).*

### Tilt adjustment


- The tilt of the seat can be adjusted by pulling the bar upwards (22). Tilt the seat to the desired angle and release the bar (23).

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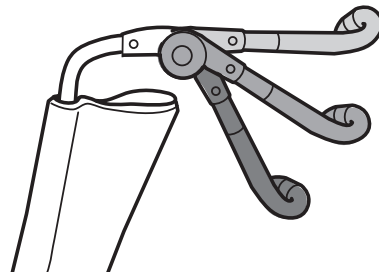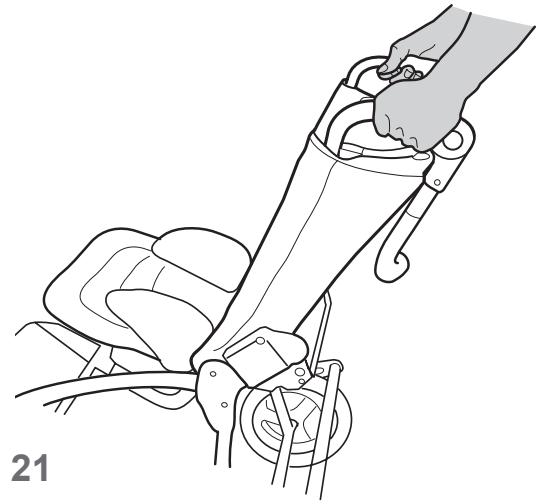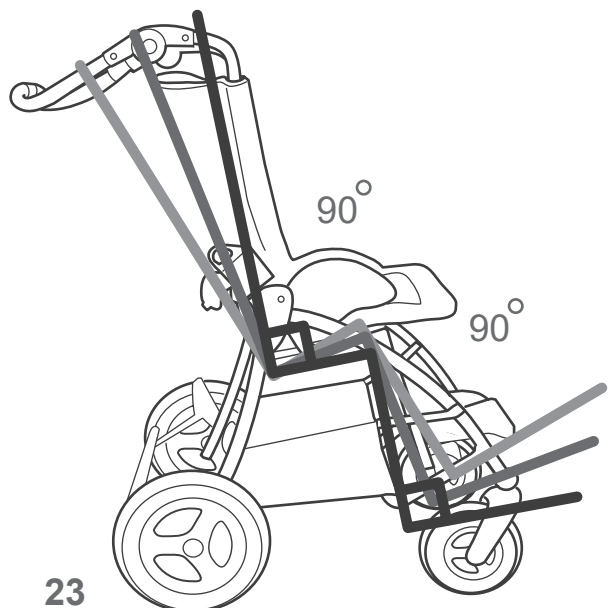
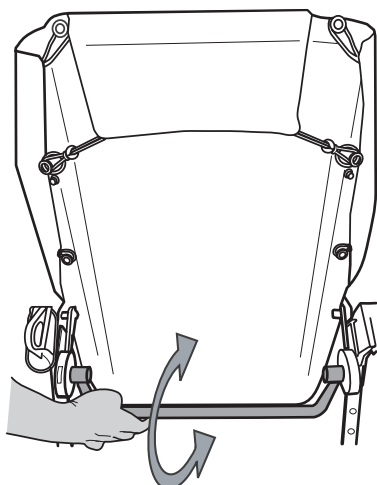
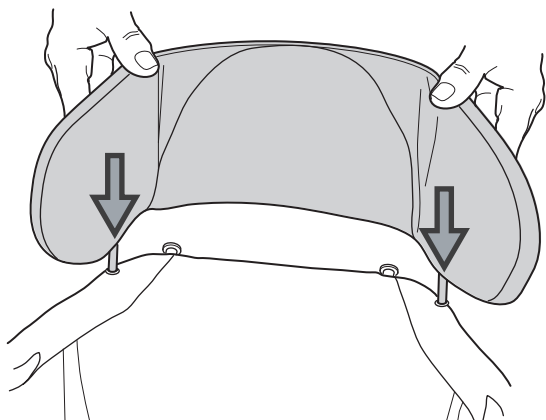
## ASSEMBLY

### Back extender

- Place the extender piece in the tubes on the back piece of the pushchair (24).
- Attach the loops around the knobs on the back of the seat (25)

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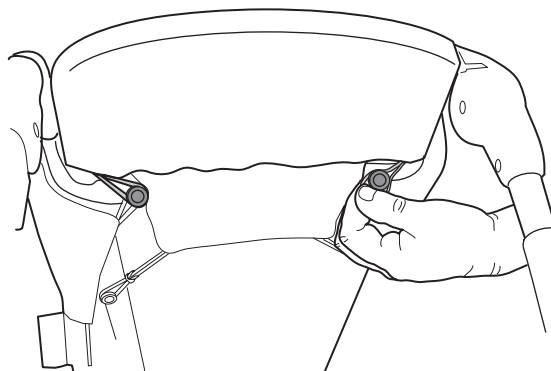
english

## ATTACHING THE CUSHIONS

Slip the pocket at the back cushion over the upper edge of the back support. Pull down the bottom edge of the cushion, through the gap between the seat and back, stretching it so that it connects to the Velcro fastening on the seat base (4). Finally, slip the pockets on the seat cushion over the side supports (6). Fasten the cushion with the Velcro fastening under the seat base (5).

## BRAKE

Brake by pushing down the lever with one foot (12). You release the brake by lifting the lever with one foot (13).

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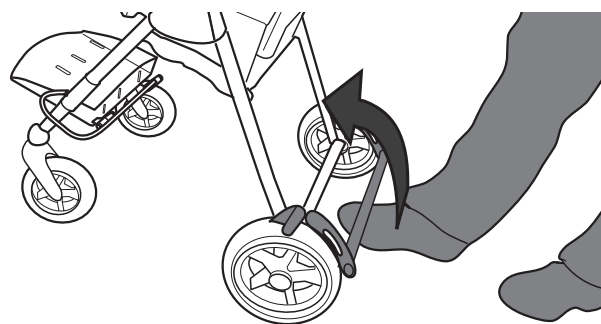
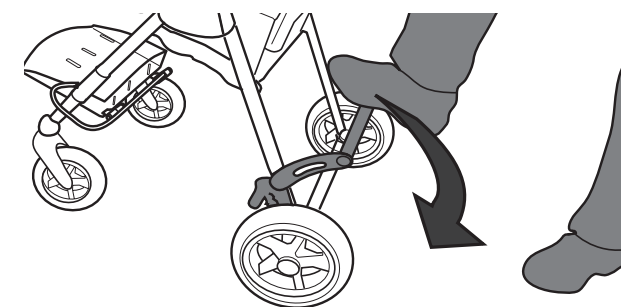
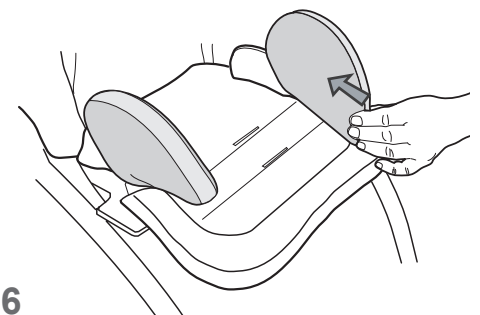
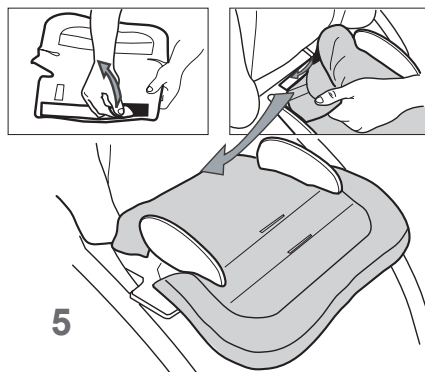
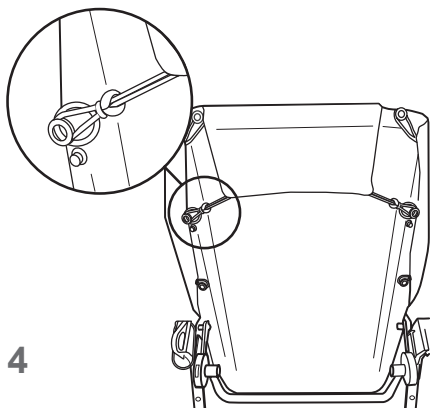
english

## MAINTENANCE AND CARE

The cushion can be washed at 40°C, spun and drip-dried. Do not use fabric softener or detergents with brightening agents. Iron after washing for best maintenance of the finish.

Do not dry clean!

Store the pushchair in a dry place out of bright sunlight.

The frame should be cleaned with a cloth and car shampoo/polish. It is recommended that authorised technical staff thoroughly inspect the aid twice a year. Immediately contact your reseller if a fault should appear.

Make sure tyres are always well inflated.

The solid tyres are maintenance free.

Air-filled tyres: max 2,5 bar.

english

## TRANSPORT

Please, follow these instructions to avoid Injury of the child or/and damage of the Pushchair during transit in motor vehicles

The ITO pushchair is designed to meet the requirements of ANSI/RESNA WC/Vol. 1-Section 19 regarding transportation of a wheelchair in a vehicle.

The ITO is capable of transporting users who are the following sizes:

Small ITO - Maximum Height = 47" (120cm)  
Maximum Weight = 55 lbs. (25kg)

Large ITO - Maximum Height = 57" (145cm)  
Maximum Weight = 88 lbs. (40kg)

The ITO has been dynamically crash tested for use with an approved four point wheelchair tie-down system, such as the "Q'Straint (with lap and shoulder belt.) The ITO showed minimum lateral displacement (2mm) when tested under ANSI/RESNA WC-19 Lateral Stability test using a standard tie-down system. The illustration opposite indicates the four anchor locations for securing the wheelchair tie-down system to the ITO pushchair. Refer to the manufacturer's tie-down instructions for further information.

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english

## TRANSPORT

Please follow the wheelchair tie-down restraint manufacturer's instructions for securing the occupant restraint belt.

The attachment loops for fastening during transportation are factory fitted. The figure (30) shows the positioning of the four attachment loops for transportation. Two on the back legs (A+B), two on the front legs (C+D). The anchor points are "D" rings bolted to the chassis.

Only use the car's approved fastening equipment between the chair and floor. The vest harness and other positioning accessories must only be used to position the child. During transportation in a vehicle the vehicle's belt must always be used.

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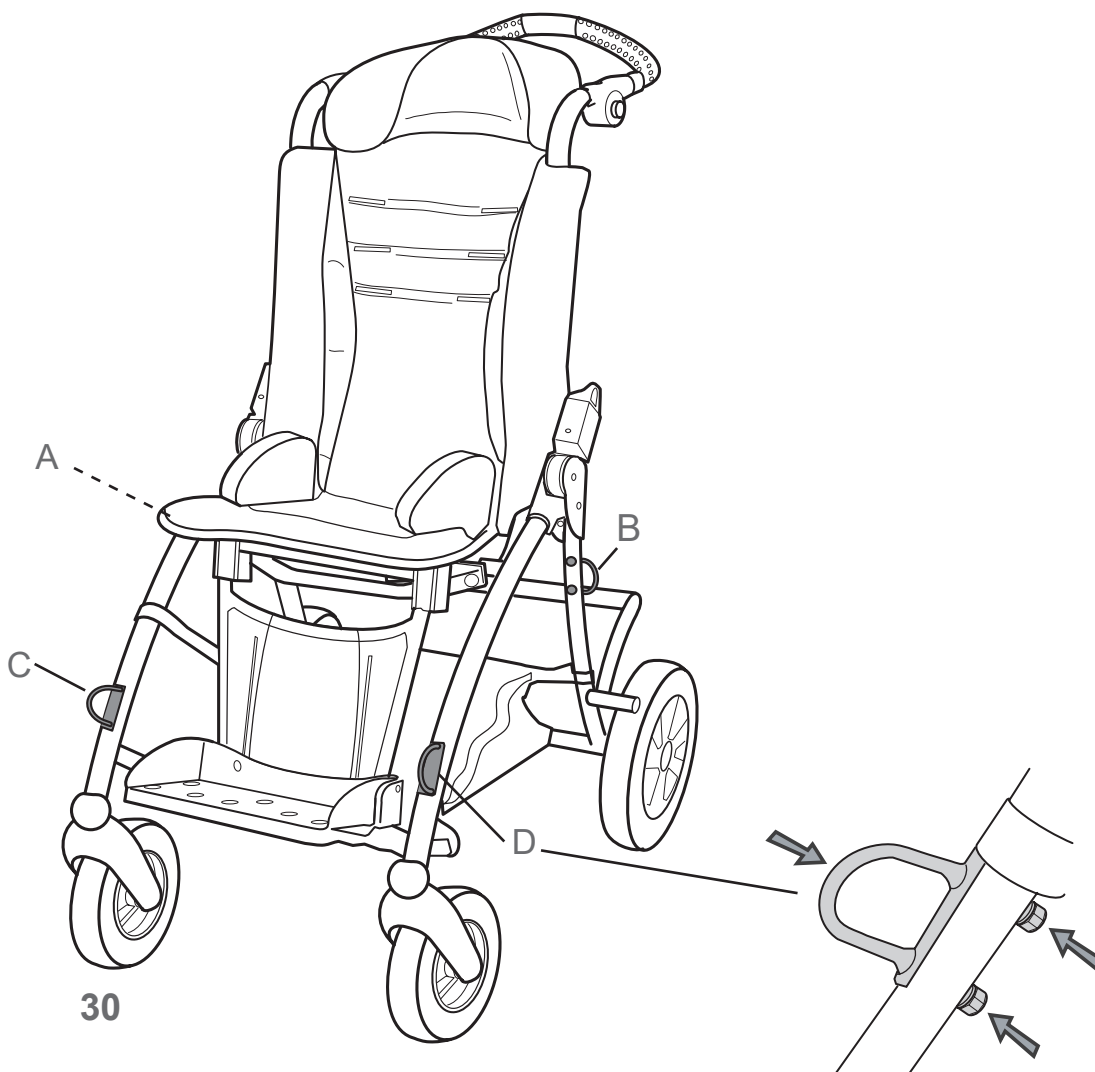
## TRANSPORT

### Warnings:

The ITO pushchair should be used only for forward facing seating in motor vehicles. The ITO should be used only as indicated in the manufacturer's instructions. ITO is crash-tested using a forward-facing position and a pelvic harness. Accessories such as vests and harnesses must only be used for positioning and not as safety belts during vehicular transportation. A clear zone of 65cm (25") should exist in front of the child to prevent the child's head from contacting anything in a crash. Auxiliary equipment attached to the ITO should be effectively secured, or removed and secured in the vehicle during transit, to prevent it from breaking free and causing injury to vehicle occupants in a crash.

The ITO must not be changed, as this may affect performance in the event of a crash.

The basket must always be fitted to the chair. The attachment straps must be fitted between the cross-tubes when the chair is being transported in a car with a child in the chair (31).

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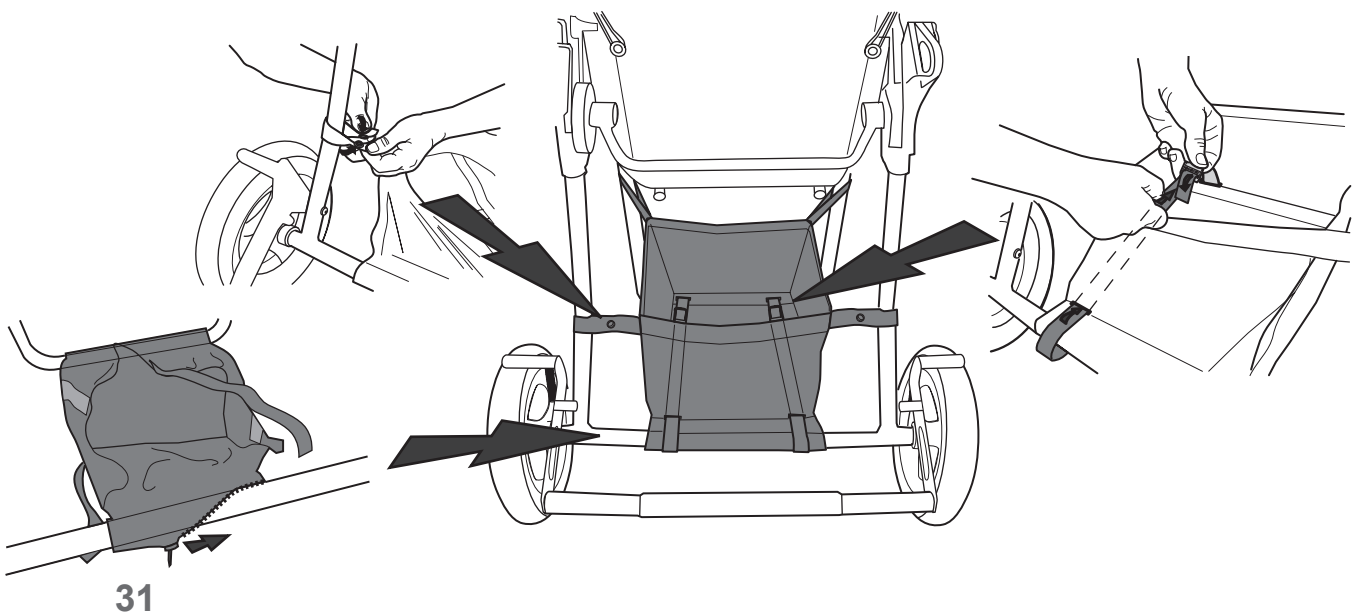
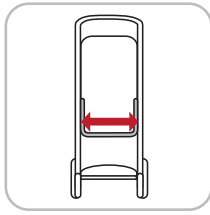
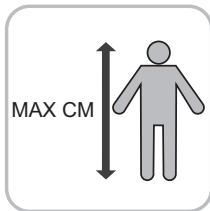
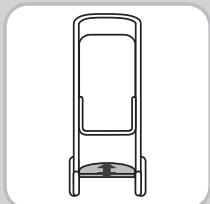
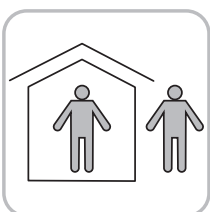
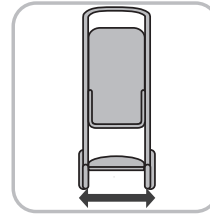
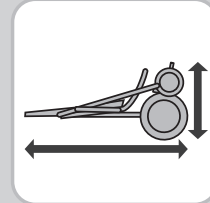
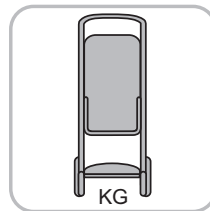
16 - 40 °

**Material:**

During manufacturing only non-toxic paint and powder coating is used. All of the upholstery is non-flammable according to standard ISO EN 1021-1 and 1021-2. Steel and aluminium are used in all tubes. The cushions are washable and tagged with simple cleaning instructions. The material has also been tested for biocompatibility according to standard ISO 10993-5.

**Test:**

Ito is crash tested and approved according to standard ANSI / RESNA WC / Vol. 1 and ISO/ FDIS 7176 and also approved by TÜV according to standard DIN EN 12182 and EN 1888.

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*Eurovema*  
Creates mobility and ergonomics

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Eurovema Mobility AB