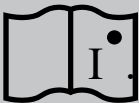
ABILIA

All in One Basic
330700

All In One

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Contents

Chapter 1: Install transmitter

- 25 Transmitter overview
- 26 Mounting a transmitter
- 27 Connect sensors to transmitter

Chapter 2: Install Pager

- 28 Overview of pager and charger
- 29 Install pager
- 30 Pair transmitter to pager and Delete linked transmitter
- 31 Key to pager signals


Chapter 3: Maintenance

- 32 System test and Changing batteries
- 33 Handling and cleaning

Chapter 4: Troubleshooting and other information

- 34 Troubleshooting
- 36 Technical information
- 38 Personalised label in pager
- 39 Important information

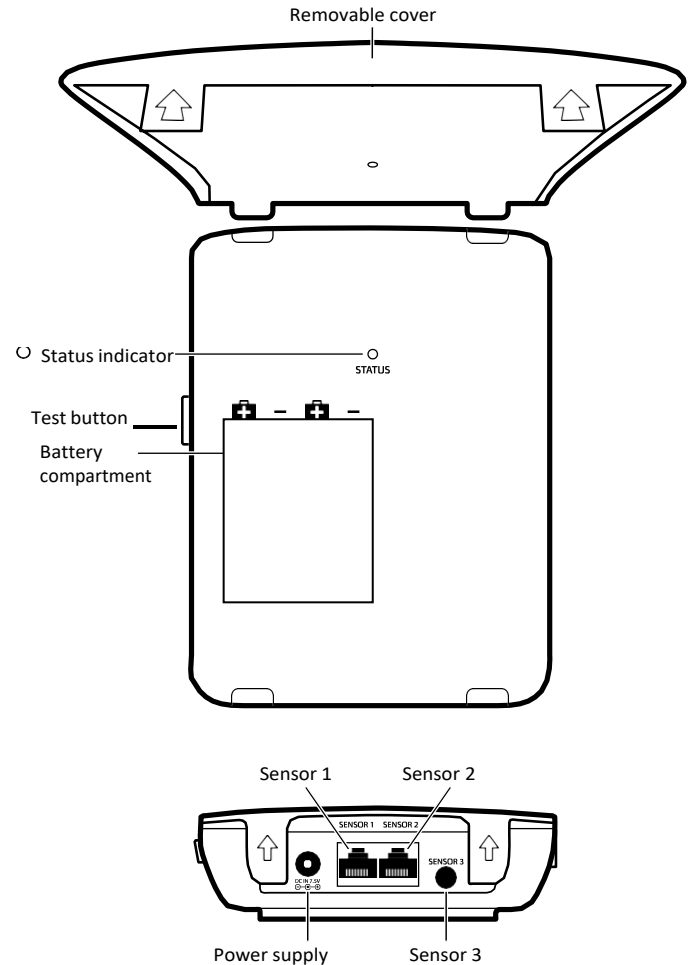Transmitter overview


Mounting a transmitter

How to mount a transmitter

- 1 Pull out the battery tab to start the transmitter.
- 2 Mark and then drill the holes using the drilling template.
- 3 Leave 2 mm distance between screw head and wall when screwing the screws in. Hang the transmitter on the wall and check that it is securely in place.

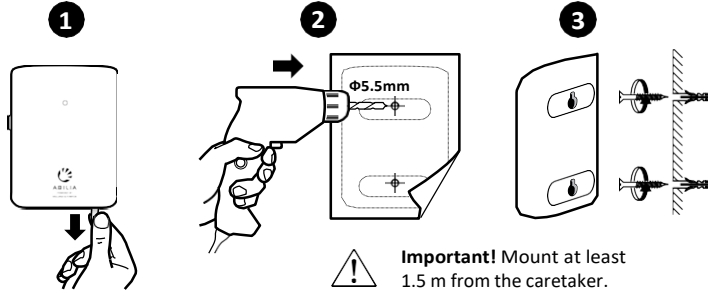Repeat steps 1-3 to mount a new transmitter.

Transmitter status indicator light

The indicator light on the front shows the current status. It can for instance tell you when the transmitter sends an alarm or if an error has occurred.

Note: If an error occurs, check the indicator lights located under the cover.

If the LED is	It means that
Blinking in green	• The system is working normally.
Blinking in yellow	• The transmitter is activated by a sensor and sending an alarm.
Blinking in red	• The transmitter's battery level is low, see p.12.
Blinking in purple	• The batteries are fitted incorrectly/wrong type, see p.12.

Connect sensors to transmitter

You can connect up to 3 different wired sensors to a transmitter, e.g. epilepsy alarm, incontinence alarm and door sensor.

How to connect a sensor

- 1 Connect the sensor on the underside of the transmitter according to the figure below. Select input for SENSOR 1 or SENSOR 2 if it has an RJ11 jack and input for SENSOR 3 if it has a 3.5 mm mono telephone jack. Check that the jack is securely in place and that the cable is secure.

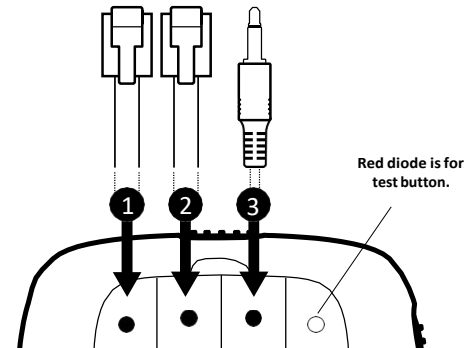
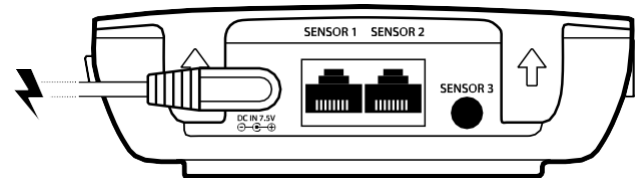
- 2 Verify that the connection works by activating the sensor. The status indicator light will flash yellow to show that an alarm has been sent.

Repeat steps 1-2 to connect a new sensor.



Note: Each sensor is allocated its own LED light on the pager as follows:

- SENSOR 1 - Orange diode
- SENSOR 2 - Green diode
- SENSOR 3 - Yellow diode
- The red diode is reserved for test button.

Note: Detailed information on how to install and use the various sensors can be found in the respective sensor's user instructions.


Overview of pager and charger

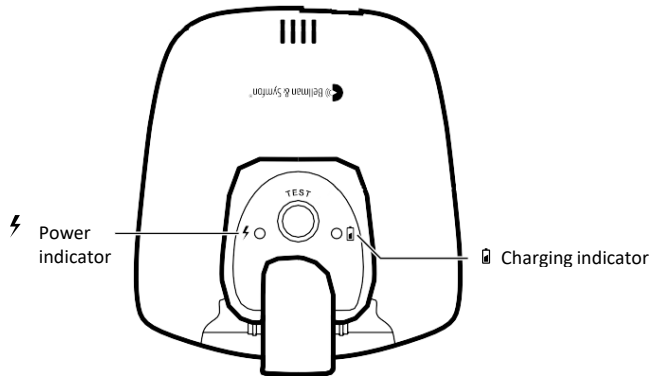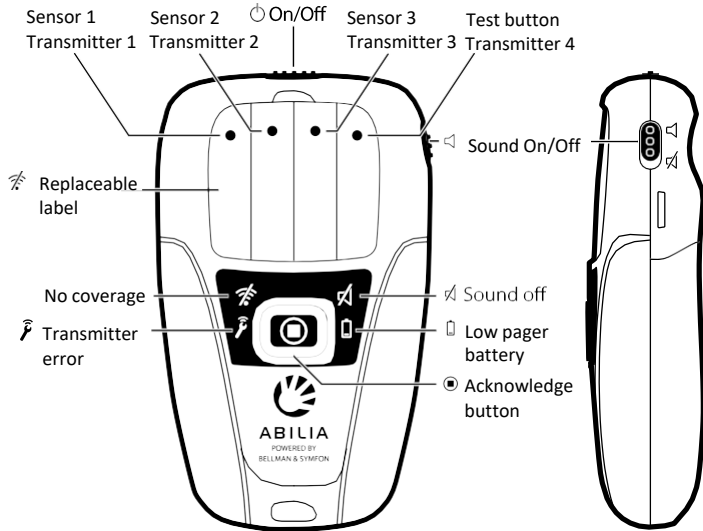Install pager

1. Pull out the battery tab on the pager and charger to start the units.
2. Plug the charger into a mains socket. The power LED ⚡ will light up to show that it is connected. Place the pager in the charger and check that it clicks into place. The charging LED 🔋 will blink/light up to show that it's charging.

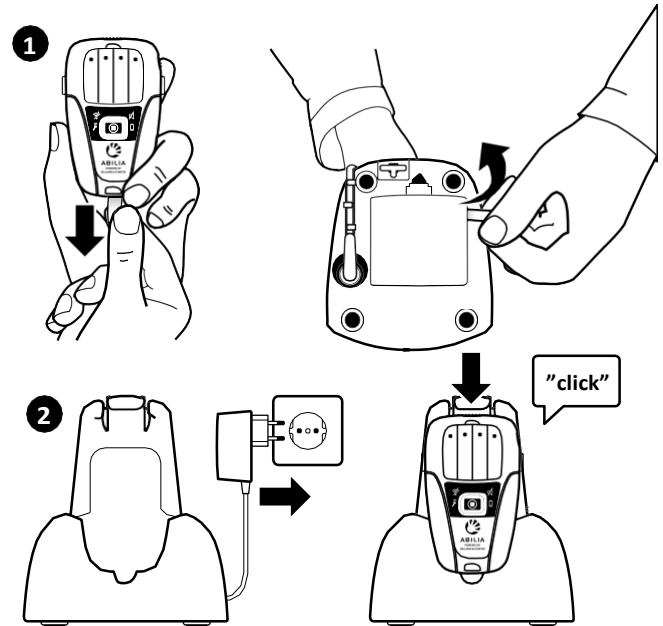Note: Remember to charge the pager for 6 hours before using it for the first time. When fully charged, the charging LED 🔋 will turnoff.


How to use the pager

When a sensor is activated, the pager will sound and vibrate. The diode colour and flashing pattern shows which transmitter/sensor has been activated.

If you want to	Do this
Switch off the pager alarm	• Press the pager Acknowledge button .
Mute the alarm	• Push the pager Sound switch up/down.
Switch the pager on/off	• Push the pager On/Off switch right/left.

Pair transmitter to pager

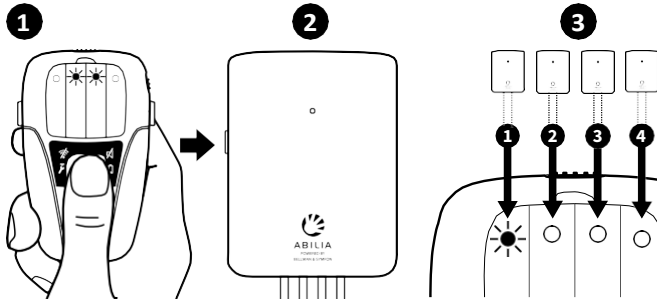
- 1 To link a transmitter, press and hold the pager acknowledge button. Release the button when the two centre diode lights start to flash.
- 2 Press the transmitter test button within 10 seconds to pair the units.
- 3 A diode light on the pager will come on to show that the transmitter is linked:
 - Orange diode - Transmitter 1
 - Yellow diode - Transmitter3
 - Green diode - Transmitter 2
 - Red diode - Transmitter 4

Repeat steps 1-3 to link a new transmitter. You can link up to 4 units.

Delete linked transmitter

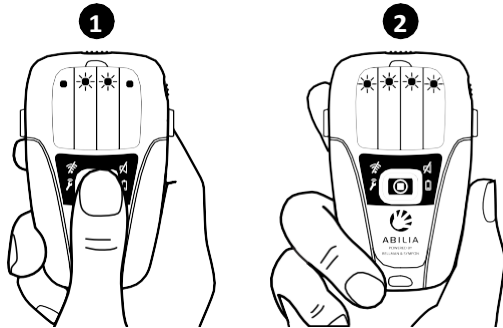- 1 Press and hold the pager OK button until the two centre diodes begin to flash. Press the button three times in quick succession.
- 2 All the diodes will flash twice to show that the linked transmitters have been deleted. This action cannot be cancelled.

Key to pager signals

The pager shows which sensor and transmitter has activated by using various diode colours and flashing patterns.

Note: The pager diode colours and flashing pattern differ depending on whether you have **ONE** or **SEVERAL** transmitters linked to the system.

System with ONE transmitter - Each sensor has a dedicated diode colour

Activated transmitter	Activated sensor	Colour and flashing pattern
Transmitter 1	Sensor 1	Orange
Transmitter 1	Sensor 2	Green " "
Transmitter 1	Sensor 3	Yellow
Transmitter 1	Test button	Red [Solid Bar]

System with SEVERAL transmitters - Each transmitter has a dedicated diode colour - Sensors are identified by flashing pattern

Activated transmitter	Activated sensor	Colour and flashing pattern
Transmitter 1	Sensor 1	Orange [Single Pulse]
Transmitter 1	Sensor 2	Orange [Double Pulse]
Transmitter 1	Sensor 3	Orange [Triple Pulse]
Transmitter 1	Test button	Orange [Solid Bar]
Transmitter 2	Sensor 1	Green [Single Pulse]
Transmitter 2	Sensor 2	Green [Double Pulse]
Transmitter 2	Sensor 3	Green [Triple Pulse]
Transmitter 2	Test button	Green [Solid Bar]
Transmitter 3	Sensor 1	Yellow [Single Pulse]
Transmitter 3	Sensor 2	Yellow [Double Pulse]
Transmitter 3	Sensor 3	Yellow [Triple Pulse]
Transmitter 3	Test button	Yellow [Solid Bar]
Transmitter 4	Sensor 1	Red [Single Pulse]
Transmitter 4	Sensor 2	Red [Double Pulse]
Transmitter 4	Sensor 3	Red [Triple Pulse]
Transmitter 4	Test button	Red [Solid Bar]

System test

It is important that system function is checked at least once a week. Perform a full system test to include activating all sensors and test all transmitter alarm functions.

Important!

- More frequent test intervals can be required if specified by the sensor user manual.
- More frequent test intervals may be required in special circumstances, if the system configuration is modified or if any of the units are dropped.

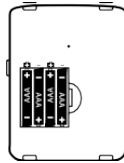
Changing batteries

Check battery performance regularly on all units to ensure that the system is fully functional. Please note that rechargeable NiMH batteries should be replaced approx. every 2 years. Always keep batteries out of reach of children. Dispose of batteries according to your local environmental laws and guidelines.

Replacing batteries in the transmitter

Open the cover of the transmitter. Remove the old batteries and replace with new ones. Instructions are above the battery compartment.

 **Important!** If the transmitter is battery powered, use alkaline AAA batteries only. If the transmitter is mains-powered, use rechargeable NiMH AAA batteries only. NEVER mix NiMH batteries with alkaline batteries as the batteries and charger can overheat.

Replacing batteries in the pager

Open the battery cover on the back of the pager. Remove the old battery and replace with a new one, see the instructions in the battery compartment.

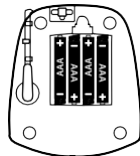
 **Important!** Only rechargeable NiMH AAA batteries shall be used in the pager.

Replacing batteries in the pager charger

Open the battery cover under the pager charger. Remove the old batteries and replace with new ones. Instructions are in the battery compartment.

 **Important!** Only rechargeable NiMH AAA batteries shall be used in the pager charger.

Handling and cleaning

Warning! Failure to follow the instructions for handling and cleaning can damage the products and void the warranty.

Using jacks and outlets

Always check that the jack fits the connector and that it is correctly aligned.  Never force a jack into a connector. If it is difficult to plug in the jack, check that the connector is not blocked in any way and make sure that you are plugging it into the right connector.

Using the products

Do not use or store the products near a source of heat. Only use in dry environments when the temperature is between 15°-35° C. Do not use the pager in the bath or shower. If the pager or transmitter gets wet or is exposed to moisture, it should no longer be regarded as reliable and should be replaced. If you do not plan to use the product for an extended period, remove the batteries.

Cleaning

Before cleaning the product, disconnect it from any power source. Use a soft, lint-free cloth. Avoid any moisture getting into the openings. Never use window cleaner, detergents, sprays, solvents, alcohol, ammonia or products that contain abrasives.

Guarantee and service

If the product is either damaged or out of order, follow the instructions in the user manual. If the product continues to function incorrectly, contact your local dealer for details on guarantee cover and service.

Other important information

Information on how to dispose of the product and other important information on instructions and safety can be found in the section on **Important information** on p. 19.

© and TM 2015 Bellman & Symfon AB. All rights reserved. Bellman and Bellman & Symfon are registered trademarks owned by Bellman & Symfon AB.

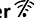
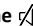
Troubleshooting

Most problems with the system can be quickly solved by following the instructions below. There are more tips and information on troubleshooting on our website.

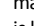
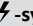
Transmitter

If	Try this
Transmitter status indicator light flashing red	<ul style="list-style-type: none"> The transmitter battery level is low. Replace the batteries if the transmitter is battery powered. Charge the batteries if it is mains powered. Important!  If the transmitter is battery powered, use alkaline AAA batteries only. If the transmitter is mains powered, use rechargeable NiMH AAA batteries only. NEVER mix NiMH batteries with alkaline as the unit can overheat.
Transmitter status indicator light flashing purple	<ul style="list-style-type: none"> Batteries in transmitter are wrong type or fitted incorrectly. See above for correct configuration and fitting of batteries.

Pager

If	Try this
Pager  symbol is lit whilst the pager is vibrating and beeping	<ul style="list-style-type: none"> Pager has lost radio contact with a transmitter. The LED position on the pager will indicate which transmitter is at fault if you have several transmitters. Move to within range of the transmitter.
Pager  symbol is lit whilst the pager is vibrating and beeping	<ul style="list-style-type: none"> A transmitter in the system is indicating an error. The LED position on the pager will indicate which transmitter is at fault if you have several transmitters. See Troubleshooting transmitter on p. 14.
Pager  symbol is white, whilst the pager is vibrating and beeping	<ul style="list-style-type: none"> Pager has less than two hours of battery life left. Charge batteries by putting the pager in its charger. A full charge takes 6 hours.
Pager  symbol is red, whilst the pager is vibrating and beeping	<ul style="list-style-type: none"> Battery in pager is wrong type, incorrectly fitted or flat. Check how the battery is fitted or replace, see Changing batteries on p. 12.
Pager not emitting any sound whilst the  symbol is lit	<ul style="list-style-type: none"> Pager sound switched off. Slide sound switch upwards to restore sound. Switch is on right side of pager (see overview).

Pager charger

If	Try this
Pager cannot be charged	<ul style="list-style-type: none"> Check that the pager is correctly placed in the charger. The LED by the  -symbol will blink/light up to show that it's charging. Check that the charger is connected to mains power. The LED by the  -symbol is lit to show that it is connected. If the LED blinks, it is using the backup battery.
Reserve battery in charger not working and charger  -symbol off	<ul style="list-style-type: none"> Check batteries are correctly fitted and charged. Charge or replace batteries, see Changing batteries on p. 12.

Technical information

All In One Transmitter

Dimensions and weight h140mm x w105mm x d30mm,210g incl.batteries

Mains electricity Input: 100 - 240 V ~ 50/60 Hz 0.3 A
Output: 7.5 V 1000 mA
Use the supplied adaptor model SSA-10W-12 EU 075100 by SIL

Batteries With mains electricity: 4 x 1.2 V AAA rechargeable NiMH batteries. Use the supplied rechargeable batteries model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co. Ltd., or model 5703 1000 mAh by Varta. Without mains electricity: 4 x 1.5V AAA alkaline batteries.

Operating time 365 days

Electricity consumption 0.3 mA in basic configuration

Connections 1 x 7.5V powercord
2 x RJ-11 inputs
1 x 3.5 mm tele jack

Activation Via test button
Via sensors connected to the 3 sensor inputs

Radio frequency 869.2375 MHz
Range Up to 500 m line of sight. The range is reduced by walls and large objects that obstruct radio signals. The range can also be affected by other radio transmitters such as TVs, PCs, mobile phones or tablets.

Environment For indoor use only Temperature range: 15° - 35° C
Relative air humidity: 5 to 95%, non-condensating Shall be installed at least 1.5m from the caretaker

Regulatory Fulfils WEEE, CE, RoHS and R&TTE

All In One Pager

Dimensions & weight h 91 mm x b 55 mm x d 23 mm, 60 g incl. battery
Battery power 1 x 1.2 V AAA rechargeable NiMH battery. Use the supplied rechargeable battery model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co. Ltd.

Operating time 24 h

Charging time Up to 6 h

Electricity consumption 20 mA

Activation Via radio

Radio frequency 869.2375 MHz

Range Up to 500 m line of sight. The range is reduced by walls and large objects that obstruct radio signals. The range can also be affected by other radio transmitters such as TVs, PCs, mobile phones or tablets.

Environment For indoor use only temperature range: 15° - 35° C

Regulatory Relative air humidity: 5 to 95%, non-condensating
Fulfils WEEE, CE, RoHS and R&TTE

All In One pager charger

Dimensions & weight h110mm x w93mm x d110mm,195g incl. batteries

Mains power Input: 100-240V~50/60Hz 0.5A,
Output: 7.5V \equiv 1600mA. Use the supplied adaptor model SFP0751600P made by Helms-Man.

Battery power 4 x 1.2V AAA rechargeable NiMH batteries. Use the supplied rechargeable batteries model 80AAAHC 800 mAh by GP.

Power consumption 200 mA

Connections 7.5V power cord
3.5 mm tele jack for bed shaker

Activation Via the test button that activates a bed shaker

Environment For indoor use only. Temperature range: 15° - 35° C
Relative humidity: 5% - 95%, non-condensing

Regulatory Fulfils WEEE, CE, RoHS and R&TTE

CAUTION! Explosion hazard if battery is replaced by an incorrect type.
Dispose of used batteries according to the recycling instructions.

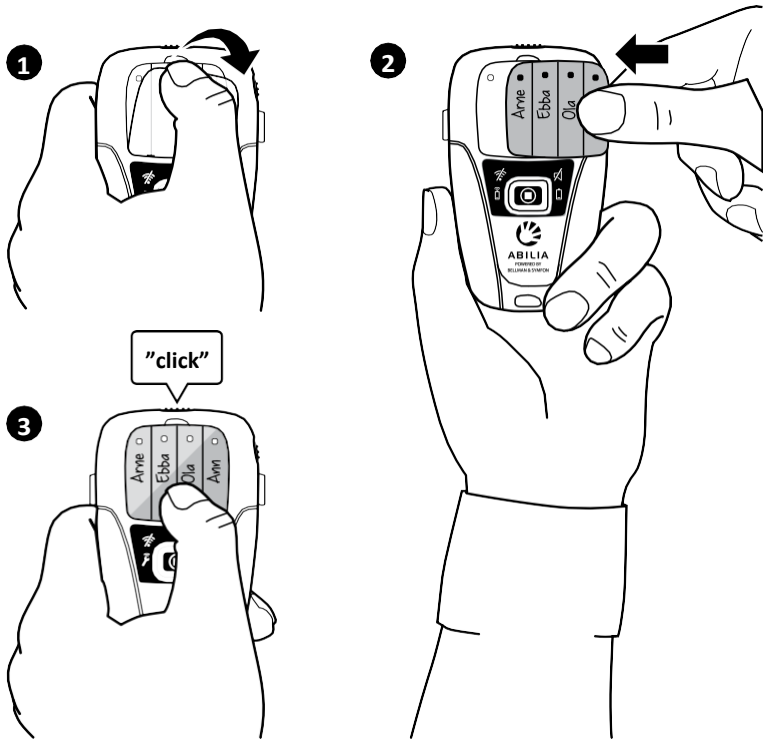
Personalised label in pager

A personalised label makes it easier to identify alarms on your pager. Depending on configuration, you can write the name of residents or the sensors linked. An extra label is included in the package.

Replacing labels in the pager

- 1 Remove the transparent plastic cover protecting the panel with coloured diodes. Avoid damaging the plastic.
- 2 Replace the label.
- 3 Snap the plastic cover back into place.

Note: You can download and editable PDF version on our website.

Important information

This section contains important information about safety, handling, disposal, recycling and warranty.

EN

Read the manual and all safety information before using All In One. Keep the manual for future use.

Warning!

- There is a potential risk of fire, electrical shock, or damage to the product if the safety instructions are not followed.
- Do not use or store this product near any heat sources such as radiators, naked flames, ovens or any other heat output.
- Protect the cables from any potential source of damage.
- Use only the power supply and accessories specified in this manual.
- Disconnect the power supply from the mains during thunderstorms or if the product is not in use for a long period.
- If the product is dropped always test it before reusing.
- Please note that the deliberate or accidental switching off the volume or power switch on the Pager can result in a missed alarm.
- Be aware that alarms can be missed if batteries run out, connectors become disconnected, or mobile systems are out of operation. Only refer damaged or faulty products to qualified service personnel.
- Do not dismantle the product; there is a risk of electric shock. Tampering with or dismantling the product will void warranty.
- The product is intended to be used as a care alerting system but should not be used as the only security in life-threatening situations. This product does not replace the need for personal supervision of patients/users.
- The product has no power switch. In order to disconnect the product, the charger and power supply must be disconnected from the wall outlet. Make sure that these are easily accessible.
- The product is designed for indoors use only.
- Do not expose the product to moisture.
- Protect the product from shocks during storage and transport.

Model, type and classification

Information is available at the back of the All In One Pager and on the charger for the Pager.

Recycling of old products

The symbol means that according to local laws and regulations it should not be disposed of with household waste. When the product is no longer of use, hand it into a recycling centre for electrical waste and electronic equipment. By recycling used products you will help to conserve natural resources and protect health and the environment.

Recycling of used batteries

This symbol means that the product contains batteries that according to local laws and regulations must not be disposed of with household waste. Take used batteries to a battery disposal facility or a recycling centre.

The European Union - declaration of conformity


CE 0359 Hereby, Bellman & Symfon Europe AB, declares that this product is in compliance with the essential requirements and other relevant provisions of R&TTE 1999/5/EC, LVD 2006/95/EC, MDD 93/42/EEC and EMC 2004/108/EC directives. The declaration of conformity may be consulted at Bellman & Symfon Europe AB, Södra Långebergsgatan 30, 421 32 Västra Frölunda, Sweden.

My notes

Use the illustration below to note details of your installation of the All In One system.

Note. Examples of notes include a reminder of which sensors you have linked to the transmitter, which transmitter is linked to the pager and contact details for assistants.

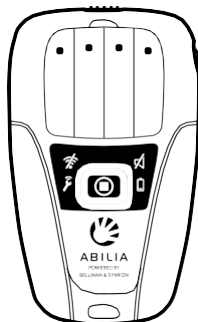If you need more pages, go to our website for a PDF version under **My notes**.

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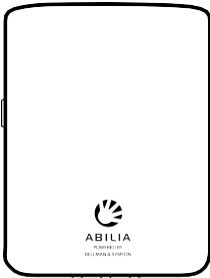
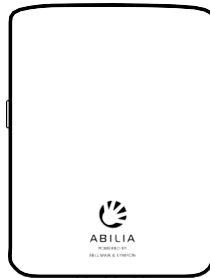
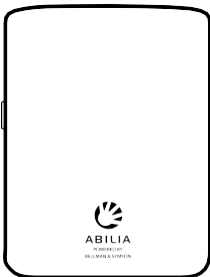
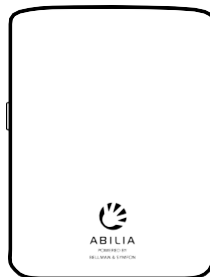
Mains electricity Input: 100 – 240 V ~ 50/60 Hz 0.3A

Output: 7.5 V = 1000 mA

Use the supplied adaptor model SSA-10W-12EU075100by SIL

Battery power With mains electricity:

4 x 1.2 VAAA rechargeable NiMH batteries.

Use the supplied rechargeable batteries model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co., Ltd. or model 5703 1000 mAh by Varta.

Without mains electricity: 4 x 1.5V AAA alkaline batteries

BE2340 Pager

Battery power 1 x 1.2 V AAA rechargeable NiMH battery.

Use the supplied rechargeable battery model HFR-AAA900 900 mAh by Shenzhen Highpower Technology Co., Ltd.

Receiver category Pager is a category 2 receiver

BE1250 Pager charger

Mains electricity Input: 100 – 240 V ~ 50/60 Hz 0.5A

Output: 7.5 V = 1600 mA

Use the supplied adaptor model SFP0751600P made by Helms-Man

Battery power 4 x 1.2 V AAA rechargeable NiMH battery

Use the supplied rechargeable batteries model 80AAAHC 800 mAh by GP

CAUTION: Explosion hazard if battery is replaced by an incorrect type.
Dispose of used batteries according to the instructions in the leaflet.

**POWERED BY
BELLMAN & SYMFON**