# **myGaze® EyeMouse Power Brukermanual**

**Version 1.2**

Mai 2017

**visual  
interaction**

## Innhold

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**7**

## 7. Vedlegg

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**DO NOT CLICK "ACCEPT" UNLESS (1) YOU ARE AUTHORIZED TO AGREE TO THE TERMS AND CONDITIONS OF THIS LICENSE AGREEMENT ON BEHALF OF YOURSELF AND/OR THE COMPANY OR OTHER LEGAL ENTITY YOU WORK FOR ("YOUR COMPANY"); AND (2) YOU INTEND TO ENTER INTO THIS LEGALLY BINDING AGREEMENT ON BEHALF OF YOURSELF AND/OR YOUR COMPANY.**

**IMPORTANT – PLEASE READ CAREFULLY:** This license agreement ("Agreement") is an agreement between you (either an individual or Your Company, "Licensee") and Visual Interaction GmbH, Behlertstr. 3a, Haus B2, 14467 Potsdam, Germany ("VI"). The "Licensed Materials" provided to Licensee subject to this Agreement include the software programs VI has granted Licensee access to (the "Software") and any "on-line", electronic or written documentation associated with the Software, or any portion thereof (the "Documentation"), as well as any updates or upgrades to the Software and Documentation, if any, or any portion thereof, provided to Licensee at VI's sole discretion. The application of conflicting general terms and conditions of Licensee shall be excluded. This applies irrespective of whether or not such terms and conditions have been expressly rejected by VI or whether VI, having knowledge of such conflicting terms and conditions, has accepted or effects contractual performance without reservation.

By clicking "Accept", you agree to abide by the following provisions. This Agreement is displayed for you to read and accept prior to using the Licensed Materials.

**If you do not agree with these provisions, do not install, download or use the Licensed Materials. If you have already paid for the Licensed Materials, please return them for a full refund to your supplier.**

#### **1) \_\_\_\_\_ License.**

Subject to the terms of this Agreement, VI hereby grants and Licensee

accepts a non-exclusive permanent license without the right to sublicense for the use of the Licensed Materials only for Licensee's business operations and only by one (1) concurrent user. Licensee may use the Licensed Materials as necessary for trouble shooting provided that, despite a written request by Licensee, VI has not offered to rectify the defect within a reasonable period of time and on reasonable terms. Licensee may make one (1) copy of the Software in machine readable form for backup purposes only; every notice on the original will be replicated on the copy. Installation of the Software is Licensee's sole responsibility. The use of all third party software provided as part of or with the Software ("Third Party Software") is exclusively governed by the terms and conditions of the respective third party license agreements executed directly between Licensee and the third party, which will be displayed for you to read and accept prior to using the Licensed Materials and/or which you can find in the system folder "Licenses". The Licensed Materials may be protected by technical means as explained in the user manual, if any. Usage data of the Software, which do not include any personal data, are automatically transmitted in order to support VI's efforts to continuously improve, update and further develop VI software. Licensee may sell or give away the Software to a third party on a permanent basis provided that (i) such third party agrees vis-à-vis VI to comply with the terms of this Agreement defining the scope of the licensed use, (ii) any Software purchased by Licensee embedded into a device will be handed over by Licensee to the third party only in its original physical fixation on the device (including any updates or upgrades, if any), (iii) Licensee ceases any own use of the Software, (iv) Licensee deletes all own copies of the Software, unless handed over to the third party, including any backup copy, and (v) Licensee provides to VI a written confirmation documenting compliance with the aforementioned requirements upon request. Licensee is not entitled to rent, lease or otherwise make available the Software to third parties on a non-permanent commercial basis (including as part of any software as a service or application service provider offering), except with the prior written consent of VI.

## **2) Rights in Licensed Materials.**

With the exception of Third Party Software and the limited license rights granted to Licensee under this Agreement, title to and ownership in the Licensed Materials and all proprietary rights with respect to the Licensed Materials remain exclusively with VI and/or its licensors.

**3) Confidentiality.**

Licensed Materials are proprietary to VI and constitute VI trade and business secrets. Licensee shall maintain Licensed Materials in confidence and prevent their disclosure using at least the same degree of care it uses for its own trade and business secrets, but in no event less than a reasonable degree of care. Licensee shall not disclose Licensed Materials or any part thereof to anyone for any purpose, other than to its employees and sub-contractors, if any, for the purpose of exercising the rights expressly granted under this Agreement, provided they have in writing agreed to confidentiality obligations at least equivalent to the obligations stated herein. The foregoing does not apply to information that (i) is or becomes generally known or available to the public without any breach of the confidentiality obligation by Licensee, (ii) was already known to Licensee prior to the disclosure by VI, or (iii) was rightfully acquired by Licensee from a third party without a breach of a confidentiality obligation towards VI. In case of a dispute, Licensee has the burden of proof that the Licensed Materials and/or any portion thereof fall under one of these exceptions. Should Licensee be legally compelled to disclose any Licensed Materials to a third party, such as pursuant to a mandatory order by a court or authority or any comparable action, Licensee shall, to the extent permitted under applicable law, inform VI without undue delay and undertake all possible measures to safeguard secrecy.

**4) No Reverse Engineering.**

Licensee shall not, and shall not allow any third party to, decompile, disassemble or otherwise reverse engineer or by any means whatsoever attempt to reconstruct or discover any source code or underlying ideas, algorithms, file formats or programming or interoperability interfaces of the Software or of any files contained or generated using the Software.

**5) Warranty.**

a) If Licensed Materials show a defect during the limitation period specified in Sec. 5 d), which already existed at the time of transfer of risk, VI will, at its own option and discretion, either provide Licensee with a corrected version of such defective Licensed Material or replace the Licensed Material with a non-defective version ("Repeat Performance"). Where considered reasonable, VI may also render Repeat Performance by revealing adequate and reasonable ways to work around the effects of the defect. In case of defects in title

[Rechtsmängel], VI may render Repeat Performance also by procuring for Licensee the right to continue to use the Licensed Materials or by replacing or amending the Licensed Materials (without limiting their quality or functionality) so that they no longer infringe third party rights.

b) Licensed Materials are defective, if they do not conform to VI's specification.

c) Licensee's claims for defects are excluded:

- 1) for insignificant divergences from the written agreement between VI and Licensee, if any, or VI's specification;
- 2) for insignificant impairments of usability;
- 3) for damages caused after the transfer of risk by external influences such as fire, water, currency surge, etc., improper installation, operation, use or maintenance, use in fields of application and environmental conditions other than those expressly specified by VI, use in combination with other products not approved by VI for this purpose, excessive stress or normal wear and tear;
- 4) for defects of the Software, which cannot be reproduced;
- 5) for defects resulting from Licensee's or a third party's modifications of Licensed Materials;
- 6) in as far as Licensee has not notified VI of apparent defects within five (5) business days of delivery and of hidden defects within five (5) business days of discovery.

d) The limitation period for warranty claims is twenty four (24) months. This does not apply for claims for damages in cases of injury to life, body or health, fraudulently concealed defects, intent, gross negligence and if a guarantee as to the condition of the Licensed Materials [Beschaffenheitsgarantie] has been given; these claims shall become time-barred within the limitation periods defined by applicable law or, in case a guarantee has been given, within the limitation period stipulated in such guarantee, if any. Repeat Performance does not restart the limitation period; the remainder of the original limitation period shall run from return of the corrected version of the Licensed Material. The same shall apply in case of a replacement of the Licensed Materials.

e) Licensee has to always first provide VI the opportunity for Repeat Performance within a reasonable period of time. Repeat Performance does not constitute acceptance of a legal obligation.

- f) If the Repeat Performance fails even within an additional period of time of reasonable length to be granted by Licensee, Licensee shall, notwithstanding its claims under Sections 6 and 7, if any, be entitled to rescind the Agreement or to claim a reasonable price reduction in accordance with applicable law.
- g) If the analysis of an alleged defect shows that it is not covered by the above warranty, VI may charge for the failure analysis and correction of the defect, if any, at VI's then applicable rates.

## **6) Liability Limitations.**

- a) VI shall be liable without limitation for damages caused by willful intent or gross negligence.
- b) VI shall further only be liable for damages caused by slight negligence [einfache Fahrlässigkeit] if such are due to a material breach of duty, which endangers the achievement of the objective of the Agreement, or to a failure to comply with duties, the very discharge of which is an essential prerequisite for the proper performance of the Agreement and on the observance of which Licensee may rely.
- c) In cases of Section 6 b), the liability is limited to the damage, which is typical for contracts like this Agreement and which could have been foreseen.
- d) For damages caused by the gross negligence of an agent [Erfüllungsgehilfe] or an employee of VI, who is not a statutory representative [gesetzlicher Vertreter], officer or executive of VI, VI's liability is also limited to the damage, which is typical for contracts like this Agreement and which could have been foreseen.
- e) In cases of Sections 6 b) and d), VI's liability is limited to a maximum amount of EURO 500.000, respectively EURO 100.000 for financial losses.
- f) Licensee's claims for damages caused by slight negligence or caused by the gross negligence of an agent [Erfüllungsgehilfe] or an employee of VI, who is not a statutory representative [gesetzlicher Vertreter], officer or executive of VI, which are not based on defects and thus not subject to time-barring in accordance with Sec. 5 d) are time-barred at the latest two (2) years from the point in time Licensee became aware of the damage and regardless of Licensee's awareness at the latest three (3) years after the damaging event.
- g) With the exception of claims for damages in the event of (i) an injury to life, body or health, (ii) liability under the Product Liability Act

[Produkthaftungsgesetz], (iii) defects after having given a guarantee for the condition of the Licensed Materials [Beschaffheitsgarantie] (in which case the liability terms and limitation period under the guarantee, if any, shall apply), and (iv) fraudulently concealed defects, the above limitations of liability shall apply to all claims for damages, irrespective of their legal basis, including but not limited to all claims based on breach of contract or tort.

h) The above limitations of liability also apply in case of Licensee's claims for damages against VI's employees, statutory representatives [gesetzliche Vertreter] or agents [Erfüllungsgehilfen].

### **7) VI Indemnity.**

Subject to VI being liable according to Section 6 and excluding claims for which Licensee is obligated to defend or indemnify VI under Section 8, VI will defend or settle any claim brought against Licensee to the extent such claim is based on a claim that Licensed Materials, used within the scope of the license granted in this Agreement, infringe any valid copyright and will pay the cost of any final settlement or judgment attributable to such claim, provided (i) Licensee has given prompt notice to VI of such claim, (ii) Licensee has not recognized an infringement of the third party's copyright, and (iii) the defense and the settlement negotiations are reserved for VI to the largest extent possible under applicable law with Licensee's full cooperation and support. If Licensee discontinues the use of Licensed Materials for damage reduction or other important reasons, Licensee is obligated to point out to the third party that the discontinuation of use does not constitute an acknowledgement of a copyright infringement. VI shall have no obligation to defend (or any other liability) to the extent any claim involves a Software release other than the current, unaltered release made available to Licensee, if such would have avoided infringement, or use of the Software in combination with non-VI programs or data, unless the infringement would also incur without such combination.

### **8) Licensee Indemnity.**

Licensee will defend and indemnify VI, and hold it harmless from all costs, including attorney's fees, arising from any claim that may be made against VI by any third party as a result of Licensee's use of Licensed Materials, excluding claims for which VI is obligated to defend or indemnify Licensee under Section 7.

**9) Export Restriction.**

Licensee will not remove or export from Germany or from the country Licensed Materials were originally shipped to by VI or re-export from anywhere any part of the Licensed Materials or any direct product of the Software except in compliance with all applicable export laws and regulations, including without limitation, those of the U.S. Department of Commerce.

**10) Non-Waiver; Severability; Non-Assignment.**

The delay or failure of either party to exercise any right provided in this Agreement shall not be deemed a waiver. If any provision of this Agreement is held invalid, all others shall remain in force. Except as expressly set forth in this Agreement, Licensee may not, in whole or in part, assign or otherwise transfer this Agreement or any of its rights or obligations hereunder.

**11) Entire Agreement; Written Form Requirement.**

Unless otherwise specified herein, VI's General Terms and Conditions for the Supply of Products and Services available under Terms & Conditions at [www.mygaze.com/shop/](http://www.mygaze.com/shop/) or attached hereto apply. Any supplementary agreements or modifications hereto must be made in writing. This also applies to any waiver of this requirement of written form.

**12) Notices.**

All notices under the Agreement must be in writing and shall be delivered by hand or by overnight courier.

**13) Applicable Law and Jurisdiction.**

German law applies with the exception of its conflict of laws rules. The application of the United Nations Convention on Contracts for the International Sale of Goods (CISG) is expressly excluded. The courts of Berlin, Germany, shall have exclusive jurisdiction for any action brought under or in connection with this Agreement.

© 2017 Visual Interaction GmbH  
Potsdam, Germany

## 7.2 About Visual Interaction

Visual Interaction (VI) makes gaze-based interaction, multimodal user interfaces and focused analysis of visual interaction commonplace and affordable. Based on leading technology from SMI, a leader in eye tracking for 20 years, Visual Interaction brings to market myGaze, an easy to use and cost efficient stationary gaze tracking solution, specifically customized for the need of interactive solutions. For more information, see [www.myGaze.com](http://www.myGaze.com).

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Copyright © 2017 Visual Interaction (VI). Opphavsrett til andre produktnavn tilhører deres respektive eiere.