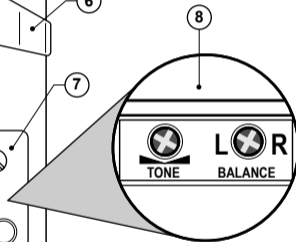
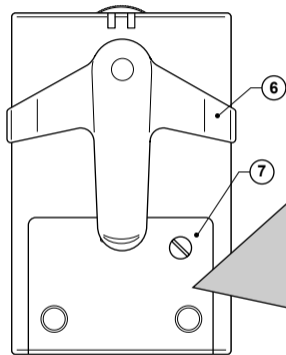
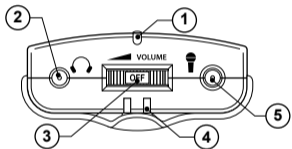
PA 100

Bedienungsanleitung (DE)
Operating Instructions (GB)
Mode d'emploi (FR)
Istruzioni d'uso (IT)
Bedieningshandleiding (NL)
Manual de instrucciones (ES)
Manual de instruções (PT)
Bruksanvisning (SE)
Bruksanvisning (NO)
Käyttöohje (FI)

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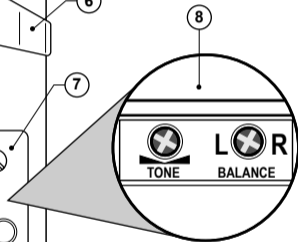
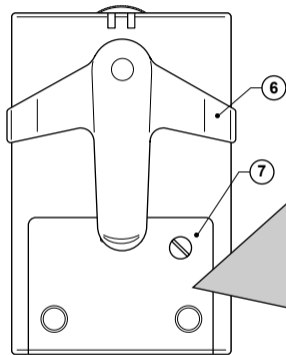
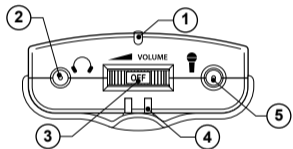
- 1 Power-on LED
- 2 Jack for headset / induction neck-loop (available as an accessory)
- 3 On/Off / Volume setting
- 4 Fixing for lanyard
- 5 Microphone jack

Packaging contents

- 1 PA 100 with lanyard attached
- 1 Mini earphone with 3.5 mm stereo audio plug
- 1 Microphone

- 6 Belt clip
- 7 Battery comp. (unscrew before opening)
- 8 Tone and balance controls (inside battery compartment)

GB

- 1 Microphone extension cable (3 m) with holder
- 1 Belt clip
- 1 User guide

WARNING!

This device can generate very high volume levels and is not suitable for users with no hearing impairment.

Commissioning/Use

- 1 Remove the belt clip by slightly pulling the bracket apart on the outer side and lifting it.
- 2 Loosen the screw, open the battery compartment and insert the two batteries as depicted in the compartment.
- 3 Close the battery compartment and replace the screw.
- 4 Remount the belt clip. Make sure the locking pins on the sides properly lock into place in the notches on the casing.

- 5 Insert the plug mount microphone into the microphone jack . Use the extension cable if necessary.
- 6 Insert the earphone lead into the phone jack .
- 7 Switch on the device using the volume wheel and continue to turn the wheel to increase the volume on the earphone. The power LED turns on green when the unit is in use.
- 8 For switching off the device turn the volume wheel into position "OFF". The power LED goes off.

GB

Adjustments

Open the battery compartment (see above).

There you find adjustments for tone control  and balance control .

Use a small screwdriver to rotate the control.

Note

- The battery life will be approximately 1 month when using alkaline cells if the unit is used every day for an average duration of 3 hours.
- An induction loop is available as an accessory. It is connected to the earphone jack.

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The symbol on this product indicates that electrical and electronic apparatus and batteries must be disposed of separately from domestic waste at suitable collection points provided by the public waste authorities. Batteries can also often be disposed of at the point of sale. Packaging materials must be disposed of according to local regulations.

Safety instructions

- Never open the device. Repairs to this device may only be performed by qualified customer service personnel.
- Avoid smoke, dust, vibrations, chemicals, moisture, heat or direct sunlight. The device is not water-proof so it must be kept dry.
- Make sure your PA 100 is sufficiently cooled.

Maintenance

- Please clean your equipment's surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation,




GB

Acts of God or other external influence are not covered by the terms of guarantee. In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer. If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet. Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

Helpline

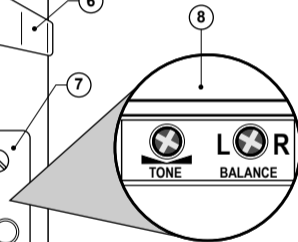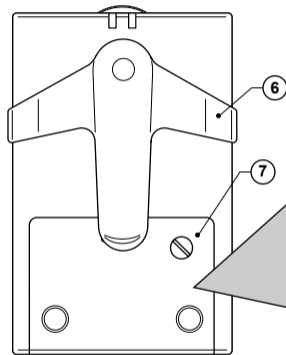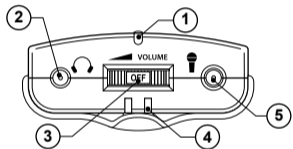If you need some help and you're in the UK, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays. If you've a claim on your 2-year guarantee, please contact your supplier.

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Audioline GmbH