## Gebrauchsanweisung / User instruction / Bruksanvisning SWIVEL 1-1136 / 1-1137 FLEXIBLE 1-1130 /1-1131

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**EN - General:** For the described product conforms to the EC Directive 93/42 is approved as medical devices class 1. Even with change of the user the manual is part of the article and must be made available to the user. Reprints, translations and copying in any form, in whole or in part, is not allowed without the express written permission of the manufacturer Nordic activity AB / Sweden. The copyright lies with the manufacturer. This statement is not covered by automatically updating. For the current status information please contact us. Further development in the design or constructions for the design in this statement are possible without prior notice.

Indications: The product was developed for people with reduced mobility and need for support and assistance.

**Security / using:** The bath seat SWIVEL / bath seat Flexible must rest safely on all rubber pads on the bathtub edge. Slipping risk are a fact when sitting down / standing up in bathtubs or wet floors, particularly if bath additives or detergents were used. Make sure before use that the bathing seat rests stable and securely on the bathtub. When put the SWIVEL on the tub the stabilizers must be screwed onto the tub walls to avoid slipping. If you do not have enough staying ability by your own, please let an assistant be present during the use.

**Delivery:** This item is delivered mounted in a carton and after correct installation ready for use.

**Cleaning:** Cleaning and disinfection should be done periodically (at user change always necessary). For cleaning use a mild soap lotion without abrasive additives. Perform the disinfection with conventional solutions.

**Maintenance:** The product is maintenance free. Defective component - please contact your dealer or service for replacement.

**Warranty:** Warranty is 24 months and extends to defects in the product which indicate faults in in workmanship or material defects.

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