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I tvivl?: +45 8693 8585

## Washing- and tumble instructions

Washing temperature: **40 - 85°C**  
Tumble dry: **max 60°C**  
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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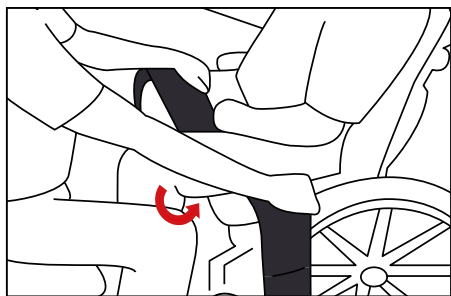
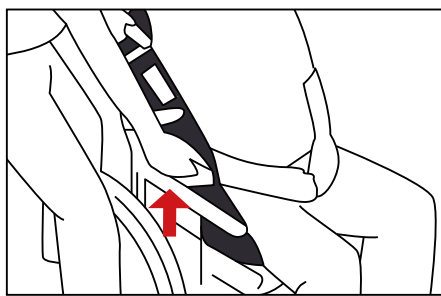
UK The user sits in the wheelchair and needs to move further back into the chair. Lock the wheelchair and place Master Grip under the user's thigh

The helper positions himself/herself to the one side slightly behind the wheelchair and takes hold of Master Grip close to the user's hip.

The helper now pulls the user diagonally backwards by using the transfer of weight. The same procedure is repeated for the opposite side.

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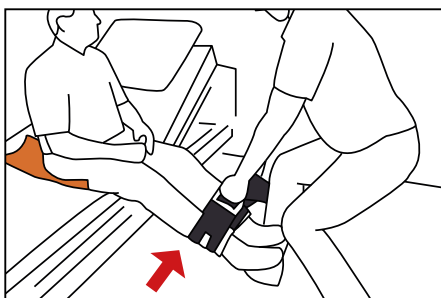
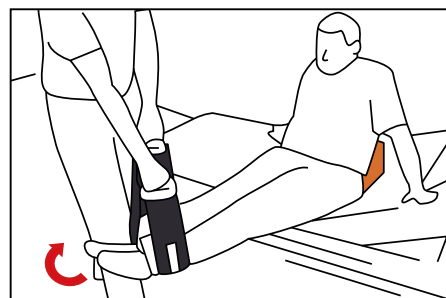
UK Remove the rail at the end of the bed. The user sits on the edge of the bed where a nylon Master Grip is placed under the user's pelvic region. The helper places Master Grip around the user's ankles.

The bed is lowered all the way down and the helper bends his/her knees and takes hold of Master Grip with outstretched arms.

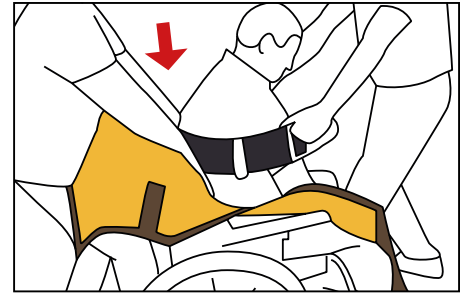
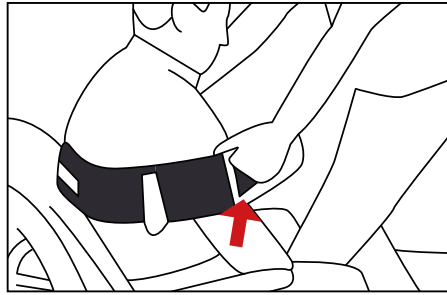
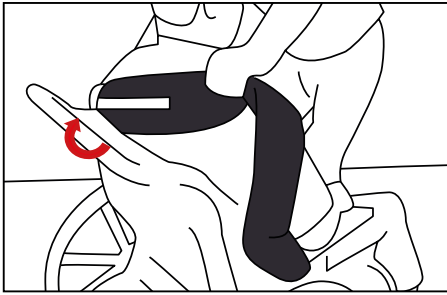
The user supports with the arms. The helper stands up and by walking in a circular movement around the bed can pull the user's legs up into the bed.

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UK The user sits in the wheelchair. The helper places Master Grip around the user's shoulders

The helper takes hold of the Master Grip handle and by transferring the weight, the helper leans backwards and thus pulls the user's shoulders away from the back of the wheelchair

There is now room to place the sheet behind the user's back

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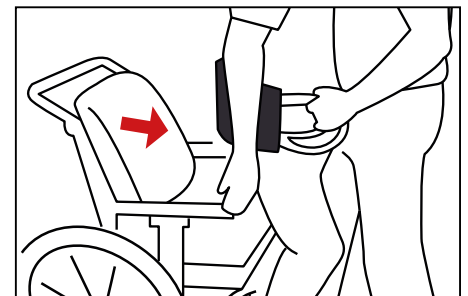
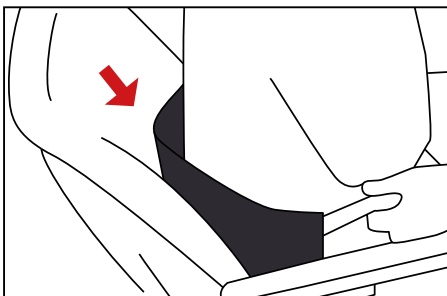
UK The helper stands in front of the user. The user leans forwards while the helper inserts Mini Master behind the user's back

The helper bends his/her own legs and supports up against the user's one leg

The helper pulls the Mini Master handle and finds a good balance together with the user. By using weight transference from the helper, the user can stand up

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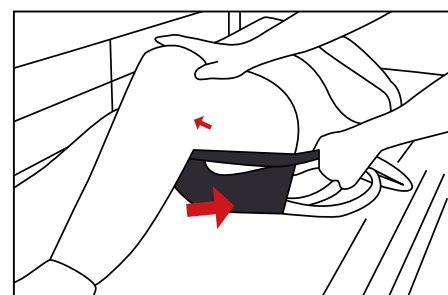
**UK** Place a Slide Master Nylon under the user's head and shoulders and bend the user's legs. Guide Mini Master in under the user's bottom.

The helpers stand on each side of the user and take hold of Mini Master's handle by using weight transference, they pull the user higher up in the bed while the user pushes away with his/her feet

Bend the user's one leg. Fold Mini Master in under the user from the one side the user has bent his/her leg. Stretch out the user's leg again. Bend the opposite leg. The helper takes hold of Mini Master and pulls it under the user

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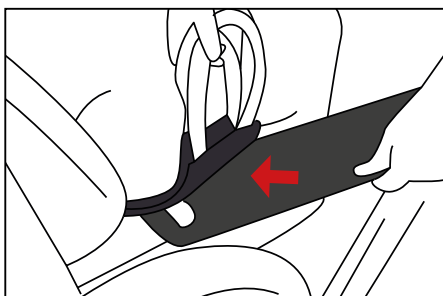
**UK** Place the wheelchair as close as possible to the bed. The bed should preferably be slightly lower than the wheelchair. Fold Mini Master together and guide it in under the user's bottom to the side from which the transfer will be done

Guide the Master Glide sliding board in under Mini Master

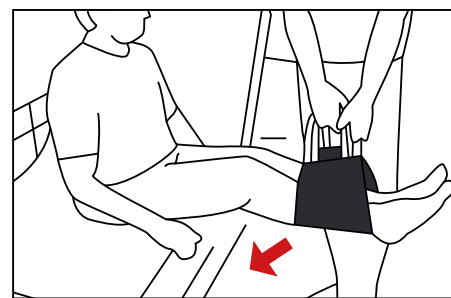
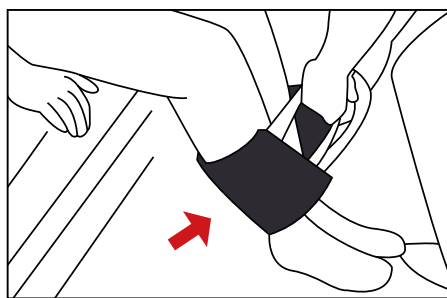
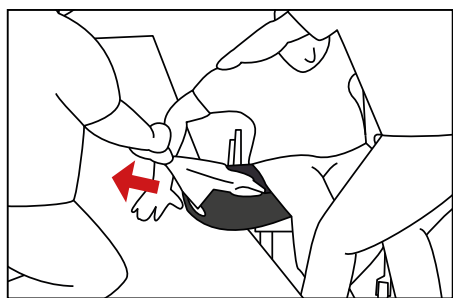
The user leans his/her weight against the Mini Master and the Mini Master Glide sliding board

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**UK** The helpers stand on each side of the bed and the one pulls in Mini Master while the other supports the user during the transfer

Elevate the headboard of the bed. Guide Mini Master around the user's legs

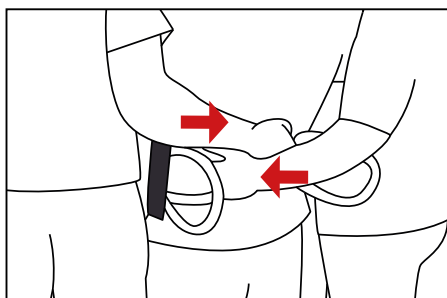
The helper kneels down and takes hold of Mini Master's handle. The helper stands up and can now turn the user's leg in towards the bed. If the user is sitting on a Turn Master, the user will turn easily into the bed

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**UK** When the user stands up, Mini Master is placed against the user's mid-section


Helpers cross their arms and take hold of Mini Master's handle behind the user's back. In this way, Mini Master can be used as a support when walking

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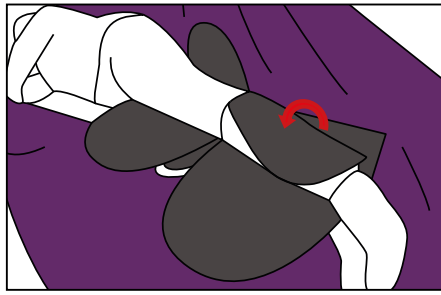Benyt Master Comfort Rygkile/-og mavekile for at give bruger optimal støtte.

UK Master Heavy Leg is folded at the bottom in the middle and guided between the two legs

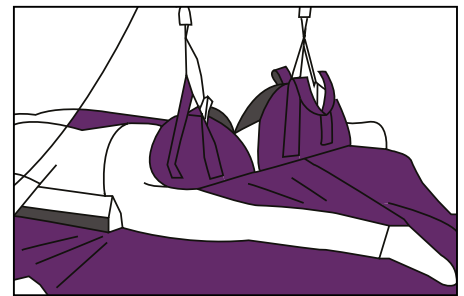
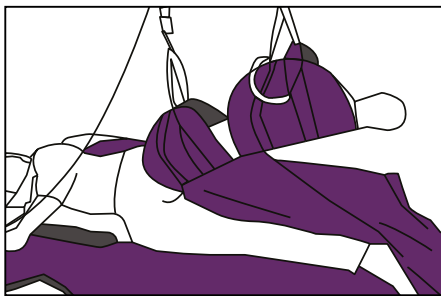
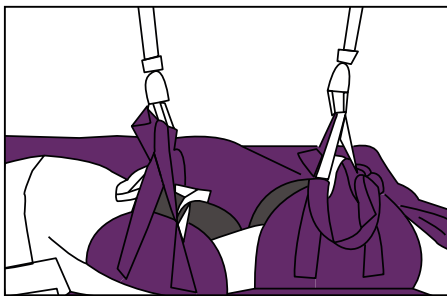
The user is turned manually or with Master Turner. The inner support lock is closed with Velcro tape around the calf

Use Master Comfort back wedge and stomach wedge to give the user optimal support

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UK Take the lift off Master Turner and place it on Master Heavy Leg. Connect the purple long strap closest to the hip and the grey/white strap

Lift the leg up to the required height. The large support surface makes it comfortable

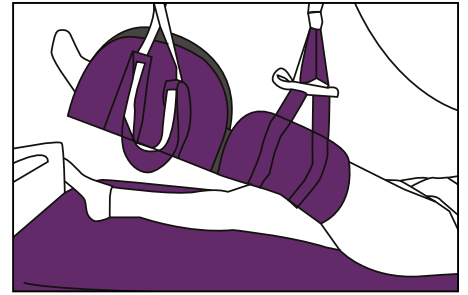
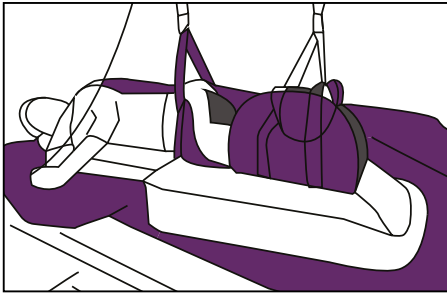
Legs with contractures are support as Master Heavy Leg can be bent by the knee

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UK Is also used to get heavy legs up onto a Master Comfort Washing wedge

Master Heavy Leg is placed the same way as for lying on the side. The user is now guided back onto lying on the back before the lift is placed on Master Heavy Leg

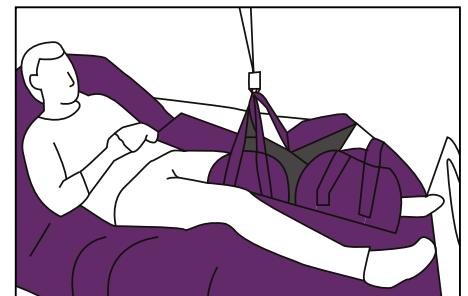
Place the purple long strap closest to the hip and the grey/white strap closest to the foot onto the lift. The leg can now be lifted

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og ben sænkes.

UK Master Heavy Leg is fitted onto the leg. Remember to close the support lock around the calf. The user may take hold of the guard rail/remote control. If Master Turner is used, this is placed across the bed

The user leans back and turns easily into the bed

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UK

- "Extends" the helper's arms
- Promotes good working postures for helper
- Is soft and flexible
- Has several grasp-friendly handles
- Shapes itself to the user's body and provides high comfort

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UK

- Due to different types of surfaces, it can be used for users with and without clothing
- Has several grasp-friendly handles
- Easy to place and remove

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UK

- Incredibly easy and gentle way to handle heavy legs
- Distributes the weight evenly, so pressure between the leg and the product does not cause shear
- The product is smooth on the outer side which makes it easy to place and remove
- Many setting options in regard to where the lift is attached – adapts the product to the user with challenging contractures
- Possible to bend the knee while the leg is being lifted
- Practical help during execution of personal hygiene, dressing or mobilizing in bed
- Lifts the leg where there is support under the thigh, calf or both parts

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# Tilkøb / Acquisitions / Akquisitionen

**MC 001-1103** [ HMI: 34306 ]  
Nylon Master Tube-shaped - **Medium**

Størrelse / Dimensions / Größe:  
L 80 x B/W 80

**MC 001-2540** [ HMI: 49370 ]  
Master Turner Two In One Soft - **Large**

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
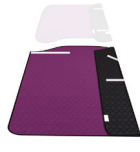
**MC 001-2500** [ HMI: 49873 ]  
Master Turner DUO Two In One Soft Bund / Bottom  
/ Unterteil - **Large**

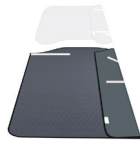Størrelse / Dimensions / Größe:  
L 160 x B/W 145 cm

**MC 001-2501** [ HMI: 49872 ]  
Master Turner DUO Two In One Soft Top / Top /  
Oberteil - **Large**

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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