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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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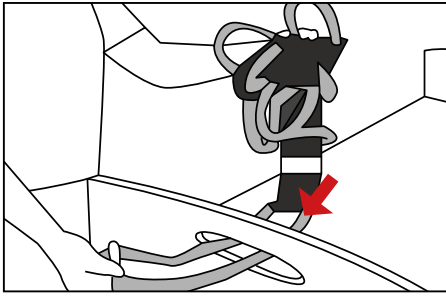
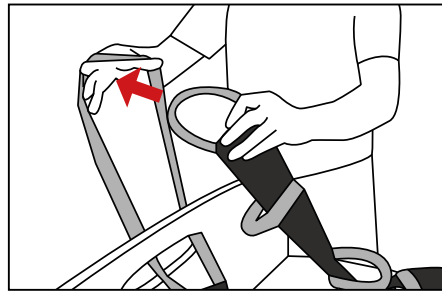
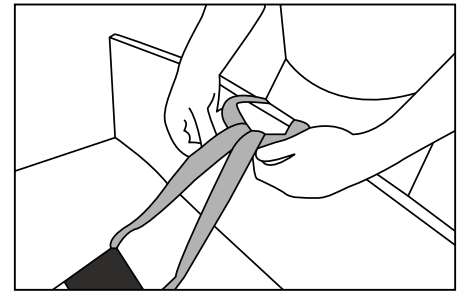
UK Pass the lower part of the Master Bed Grip Soft through the headboard

While holding the lower eye, pass the Master Bed Grip Soft, with its handles, through this

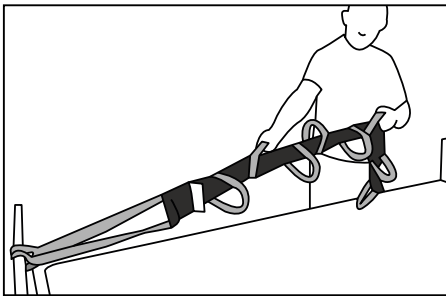
Pull until the Master Bed Grip Soft is fully tightened around the entire headboard

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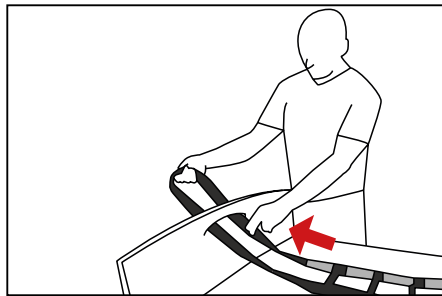
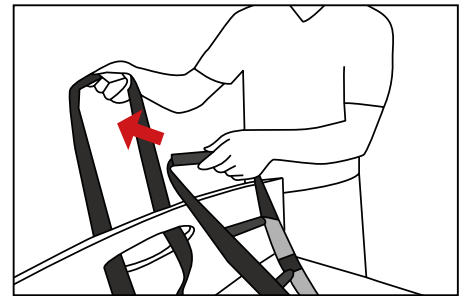
UK Master Bed Grip Soft is now ready for use

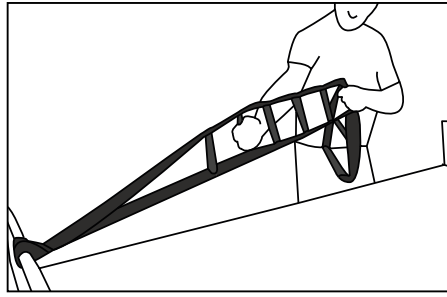
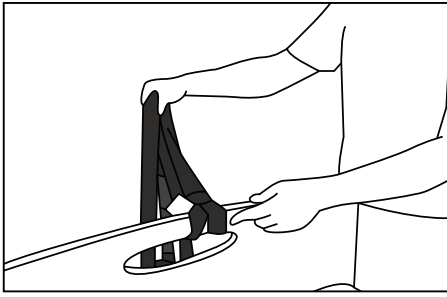
Pass the lower part of the Master Bed Grip Hard through the headboard

While holding the lower eye, pass the Master Bed Grip Hard, with its handles, through this

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UK

Pull until the Master Bed Grip Hard is fully tightened around the entire headboard

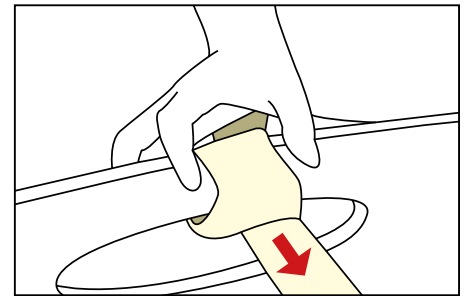
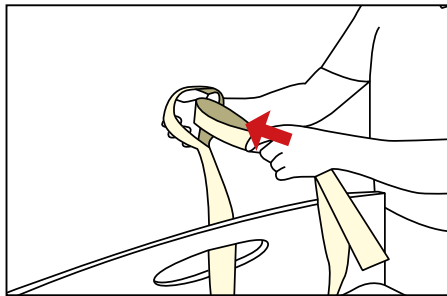
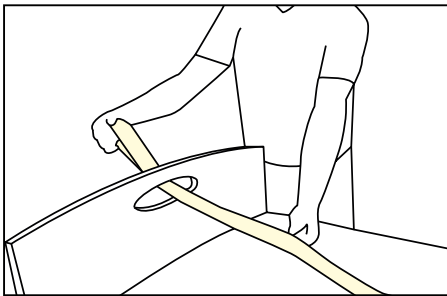
Master Bed Grip Hard is now ready for use

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UK

Pass the lower part of the Master Bed Grip Single through the headboard

While holding the lower eye, pass the Master Bed Grip Single through this

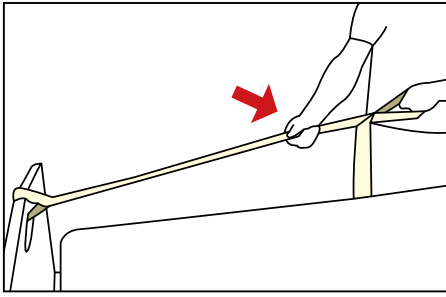
Pull until the Master Bed Grip Single is fully tightened around the entire headboard

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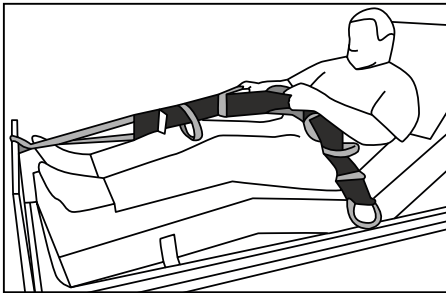
UK

Master Bed Grip Single is now ready for use

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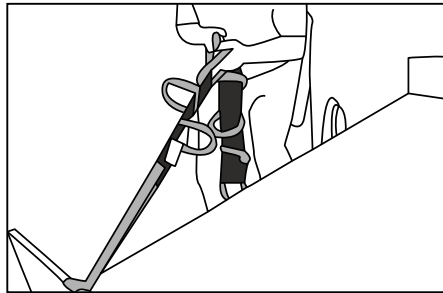
UK

Master Bed Grip is ideal when the user has to sit up in bed

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Master Bed Grip is also used to transfer users between chair and bed. For this, we also recommend our Master Glide sliding board for a smooth transfer

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Fordele / Advantages / Vorteile

Master Turner Soft & Frotté / Terry Cloth / Frottee - Medium, Large & X-Large

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UK

- "Extends" the assistant's arms
- Promotes good posture for the assistant
- Promotes mobilization of the user
- Wide range of application

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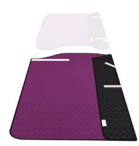
Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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