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Master Turner® Pro / Low Air / X / Turn 2 Lift / Lift Uniq

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○ **MC 001-1770** [HMI: 109944]
Master Turner Uniq Soft - **Large**

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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the top piece

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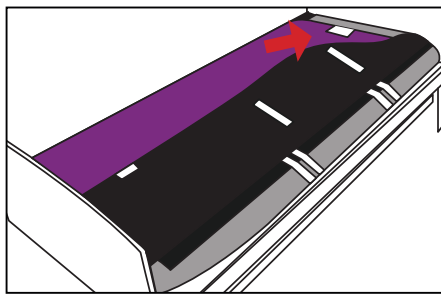
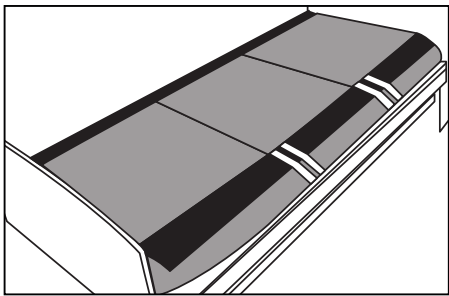
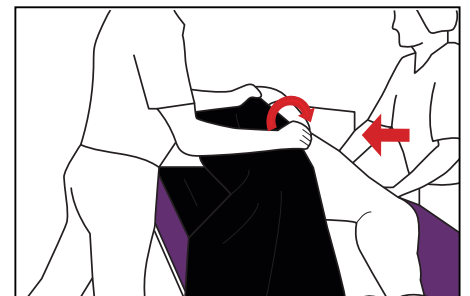
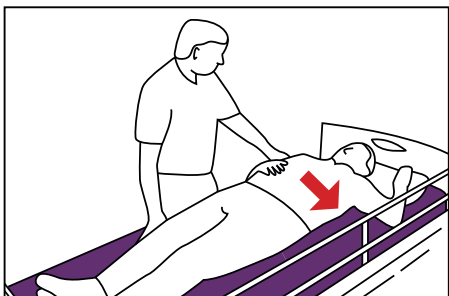
UK Before the Master Turner is placed, a mattresscover is mounted on the mattress. Shown here as an Easy cover

Place the Master Turner with the smooth side down on the mattress cover. The label "This side up and against the headrest" is toward the headboard

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UK Cross the user's legs. Lead the user's hand to touch their opposite shoulder. Slide the pillow and shoulder toward the side that you need to turn the user

Person 1 pulls and person 2 pushes the user toward the desired bedside

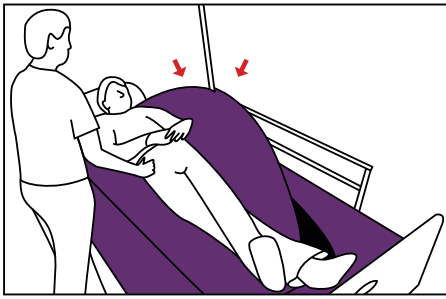
Person 1 pulls the turn sheet and helps position the user on their side

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
UK Cross the user's legs and pull their pillow and shoulder toward the side that you need to turn the user. Attach the turn sheet's straps to one side of the lift

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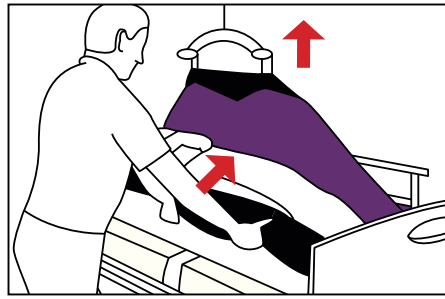
Lift the user slightly above the surface beneath. The helper then positions the user toward the bed guard. Use the Master Comfort Stomach Wedge for additional relief and support during turn

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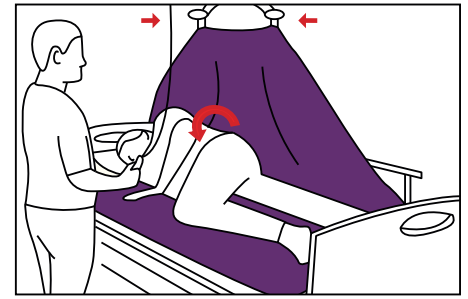
Raise the lift. Turn the user over onto a lateral position

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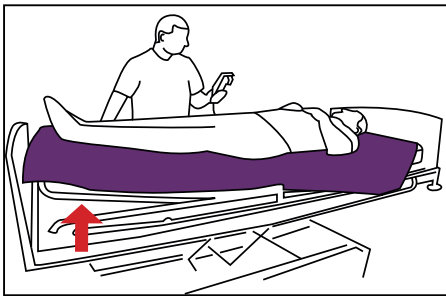
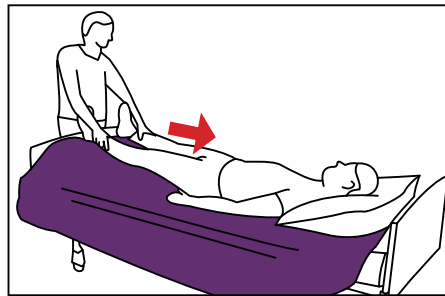
Person 1 stands at the foot of the bed and makes a weight transfer of the user's ankle/calf. The user then slides toward the head of the bed

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
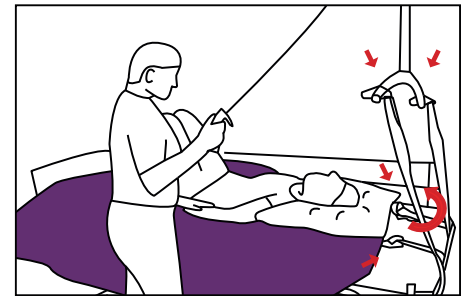
Alternatively, dismount the bed's headboard. Attach the Master Turner pull straps to the turn sheet's straps at the head of the bed, pulled under the bedframe and hooked onto the lift. Raise the lift to pull the user into an upright position in bed

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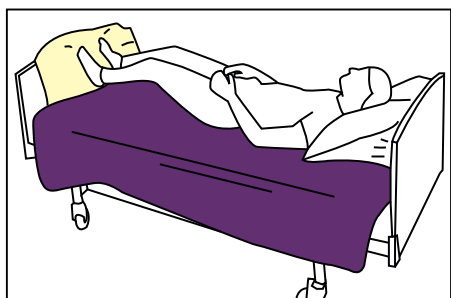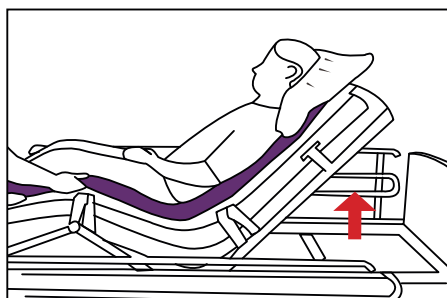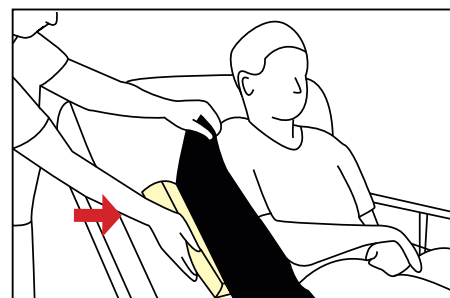
UK Activate the adjustable bed under the user's knees. Place the Master Comfort Washing Wedge at the foot of the bed

Elevate the head of the bed

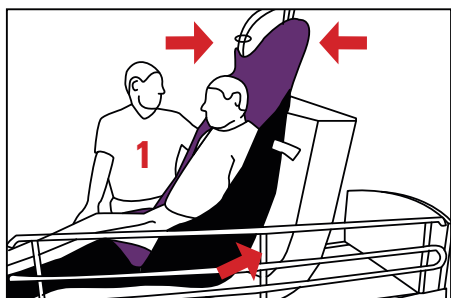
Place the Master Comfort Support Wedges under the turn sheet on either side of the user

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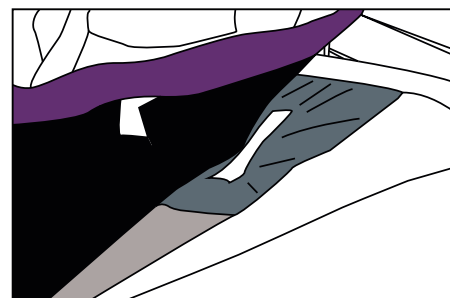
UK When this method of transfer is used, person 1 must ensure that the sliding flap's opening is facing the helper. Elevate the adjustable bed under the user's knees. Elevate the head of the bed. Attach the turn sheet's straps to the main section of the lift and raise until it fits snugly behind the user's back

Position the user's legs over the edge of the bed

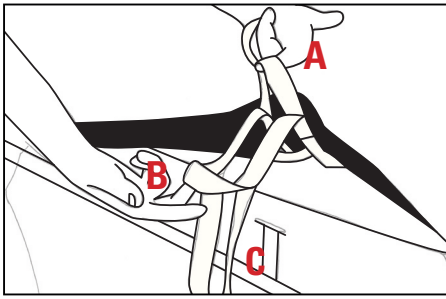
The user can then slide into the desired seating position. In order to avoid sliding forward, the mattress covers sliding flap can be pulled back to make the anti-slip visible

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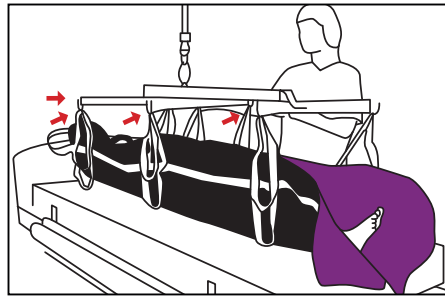
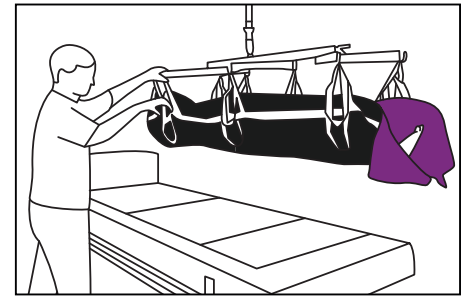
UK For not heavy users choose handle A. For middle heavy users choose handle B and for heavy users choose handle C

Place the four handles on each side of Master Turner® Turn2Lift on the lifter beam. To ensure that the user lies flat in bed, adjust the weight balance on the gravity regulating wheel of the lifter

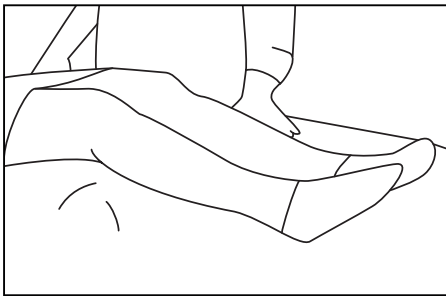
When lifting the user from the underlie adjust the weight balance on the gravity regulating wheel. The user can now be moved

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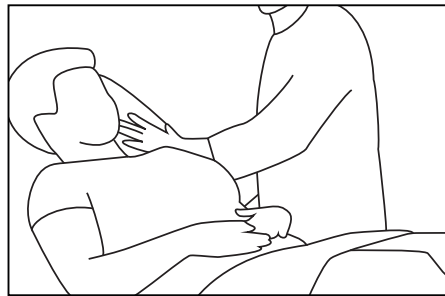
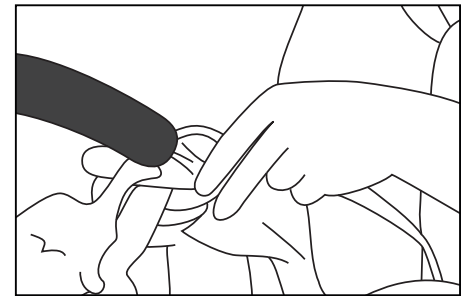
UK Elevate the adjustable bed under the user's knees and, if necessary, raise the foot of the bed to prevent the user sliding down

Raise the adjustable bed under the user's hips to raise the head of the bed at least 45 degrees

Fix the top and leg straps to the lifter beam. Use the inner top straps and outer leg straps to start with. If you need to change the angle of the hips, you can then adjust the straps

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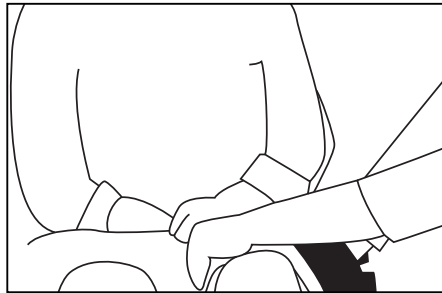
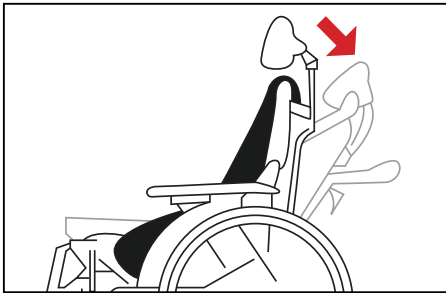
UK Lower the bed so the user can be lifted clear of the mattress. If the user's feet are still resting against the mattress, you can lower the foot of the bed

You can now transfer the user. If using a wheelchair, make sure that the back is positioned as far down as possible to secure a comfortable position for the user. If necessary, you can use a piece of Master Stop between the Master Turner and the surface of the seat

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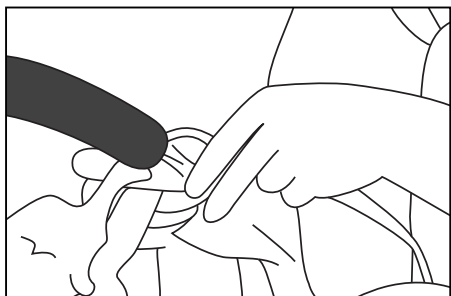
UK Here we show how to lower the back of the wheelchair: approx. 45° for optimal positioning

Once the user is in place, the Master Turner Lift can be folded down next to the user. Fold away from the user, so they are folded round below, around the sliding fabric. This provides good support and is easy to take out again if you need to lift the user again

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UK Fix the top and leg straps to the lifter beam. Use the inner top straps and outer leg straps to start with. If you need to change the angle of the hips, you can then adjust the straps

Before placing the user in the bed, raise the adjustable bed under the user's hips and set the head of the bed to 45 degrees. It is a good idea to place a Master Comfort Vaskekile at the foot of the bed to keep the user back at the head of the bed. Once the user is in position, remove the top and leg straps. Smooth out any bumps etc. so the user is lying comfortably

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UK

- Specially designed for alternating pressure mattresses
- Thin padding prevents overheating
- Can be used with all types of floor or ceiling lifts
- Handles for easy handling
- Allows users with very limited mobility to be mobilised
- Reduces the risk of pressure ulcers/shear
- Easy to transfer users further up in bed
- Reduces reaching distance and bent backs for helpers

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UK

- Specially designed for Low Air mattresses
- Allows the air from the mattress free passage
- Thin padding prevents overheating
- Can be used with all types of floor or ceiling lifts
- Handles for easy handling
- Reduces the risk of pressure ulcers/shear
- Reduces reaching distance and bent backs for helpers

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
MC 004-3011 [HMI: 73522]
Madrascover / Mattress cover / Matratzenbezug
Exchange 3 - Large

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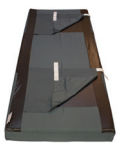Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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