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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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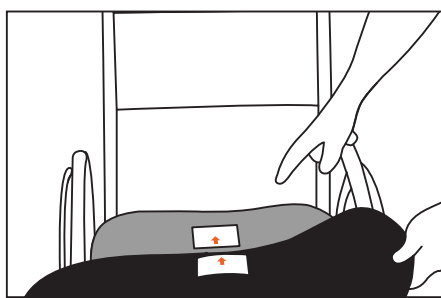
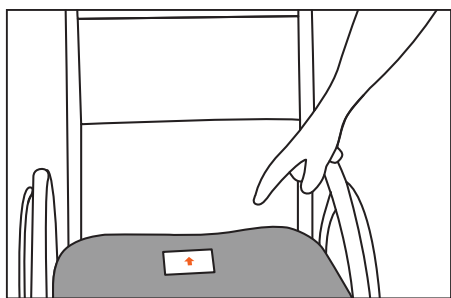
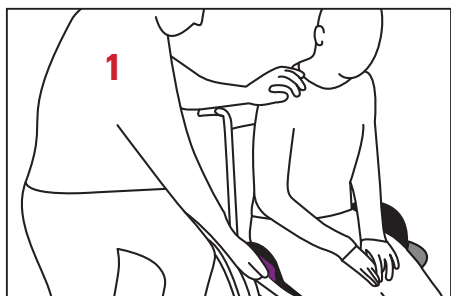
UK Position the Sit Move Bottom base with the grey surface facing up and with the label, 'This side up and against the back/headrest' against the back of the chair

Place the Sit Move Top on top. The smooth underside with the handle should face downwards

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

UK The user can slide from side to side using his/her own strength or with assistance from 1. This creates mobility without friction and can prevent pressure injuries. The correct positioning of Sit Move prevents the user from sliding forward

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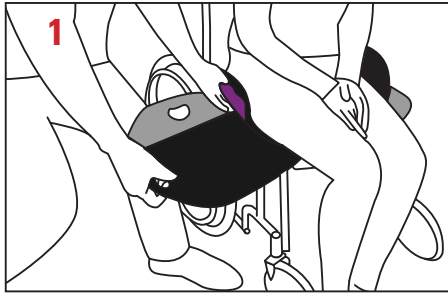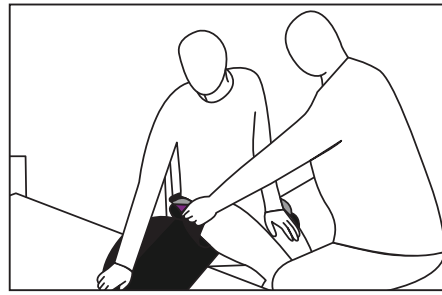UK Position the Master Glide sliding board by guiding it between the sliding surfaces on the Sit Move. 1 grips the handle of the Sit Move and rolls the user's hip slightly over to the opposite side. Use the Master Glide Flexi between movement points

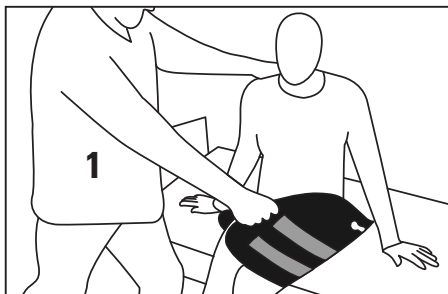
Position the wheelchair as illustrated. Lock the wheels on the wheelchair

1 places him/herself behind the user and clutches the handle of the Sit Move. The user is moved and slides on top of the Master Glide sliding board with the upper section of the Sit Move from the wheelchair to the bed

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
UK 1 clutches the handle of the Master Glide sliding board, bends it upwards and pulls it out from under the user

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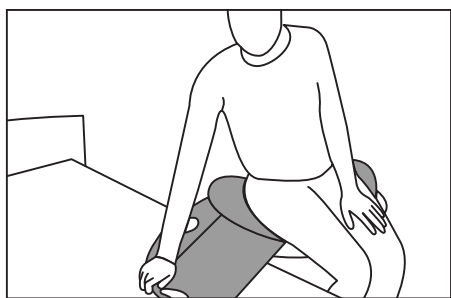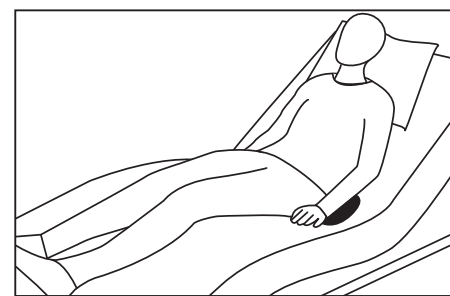
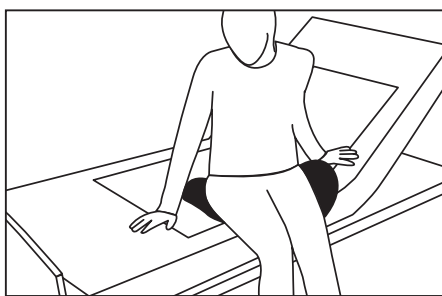
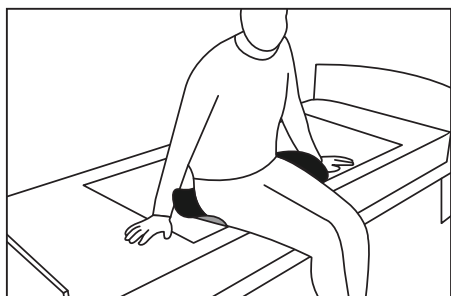
UK The user lifts the handle of the Sit Move and leans his/her hip to the side opposite to that, to which s/he wishes to move. The Master Glide sliding board is then guided in between the Sit Move Bottom and the Top

The user makes sure that the wheelchair and the bed are at the same height and that all brakes are locked. The wheelchair should be positioned as illustrated. The user now moves with the Sit Move and slides on to the Master Glide sliding board, before bending it and pulling it out after the transfer is completed

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Se illustration

UK The Silk Master, Silk Master Pro or Quick On sliding sheet is fitted to the mattress. The user sits on the Sit Move at the edge of the bed

The user turns his/her hip and, using the Sit Move, slides onto the bed on the smooth surface of the sliding sheet

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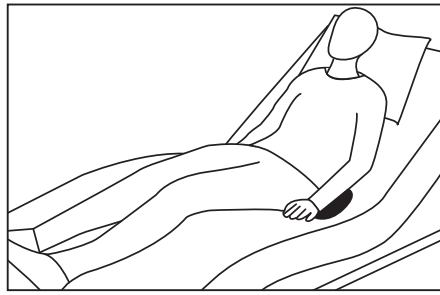
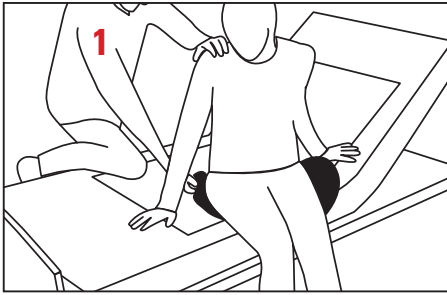
Se illustration

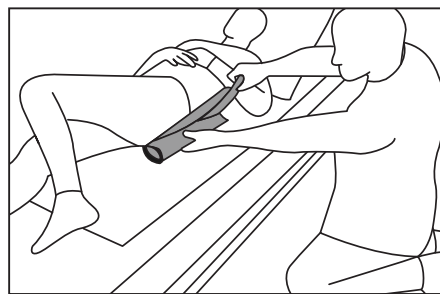
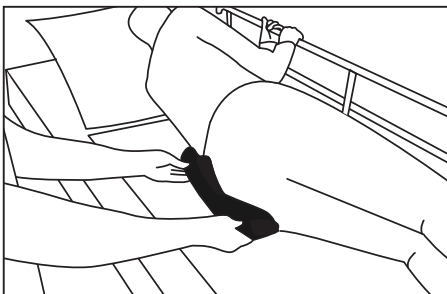
UK 1 leans in and clutches the handle of the Sit Move and helps the user to move his/her posterior towards the smooth surface

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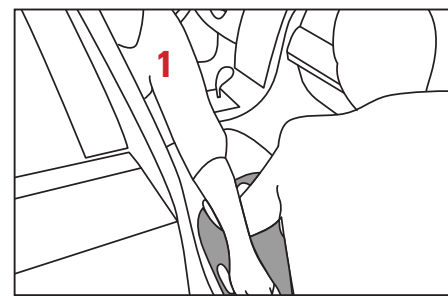
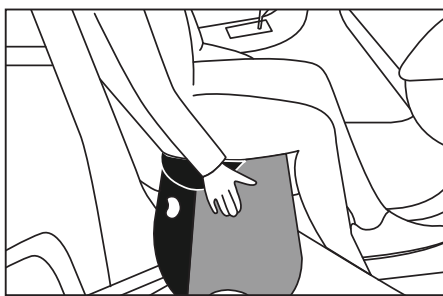
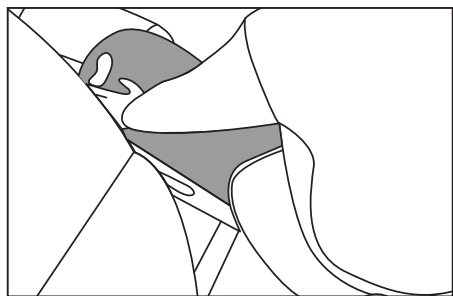
UK The user rolls on to his/her side and clutches the guardrail. The helper now folds up the Sit Move under the user

The user rolls to the opposite side and lifts his/her hips. The Sit Move can now be pulled out from under the user

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UK Use the Master Glide sliding board by pushing it under one of the user's hips and in between the Bottom and the Top of the Sit Move

The user is moved and slides on the Master Glide sliding board with the upper section of the Sit Move from the wheelchair to the car seat. If necessary, use an extra Sit Move Top on the car seat

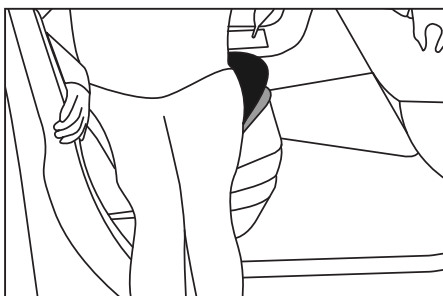
1 can pull the handle of the Sit Move. At the same time they can use a Master Belt Multi Velour or Master Belt Multi Non-Slip activation belt round the user's waist for a better-controlled transfer

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Se illustration

UK A self-reliant user can use the Sit Move to turn in and out of the car seat.

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UK

- Makes transfer using a Master Glide easy
- Easy to change sitting posture
- Pressure under the sitting surface is moved easily from side to side without sliding forwards
- Easy to use for both user and helper

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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