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Master Turner® DUO Large, Low Air & X-Large

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Størrelse / Dimensions / Größe:
L 100 x B/W 145 cm

○ **MC 001-2500** [HMI: 49873]
Two In One Soft - Bund / Bottom / Unterteil - **Large**

Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

○ **MC 001-2501** [HMI: 49872]
Two In One Soft - Top / Top / Oberteil - **Large**

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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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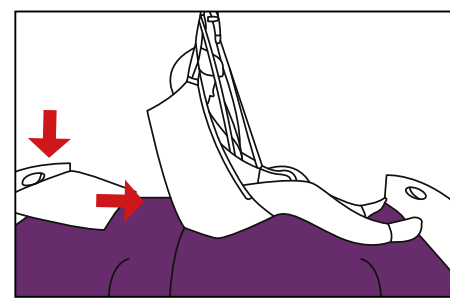
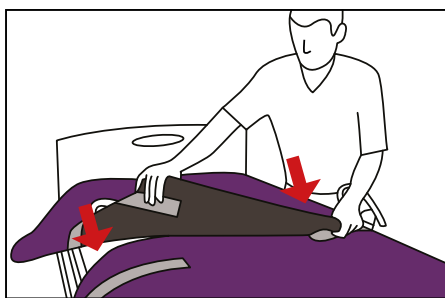
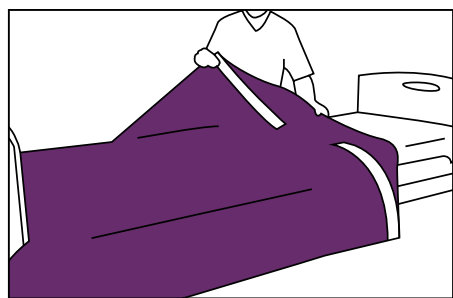
UK Position the bottom piece with the sliding part against the cover. The two soft straps must be facing upward toward the middle of the bed. Place the top piece with the sliding part against the cover

Remove the washing straps and save for later use. Attach the straps of the top piece to the bottom piece. Be careful not to work over the middle of the bed and that there are no unintended folds in the system. Pull up the top and bottom pieces until they are touching the headboard

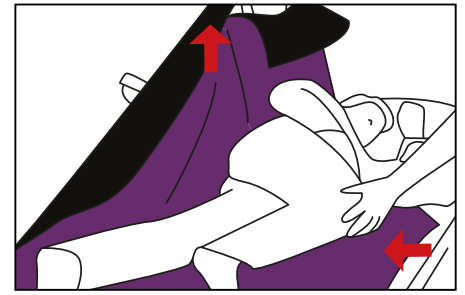
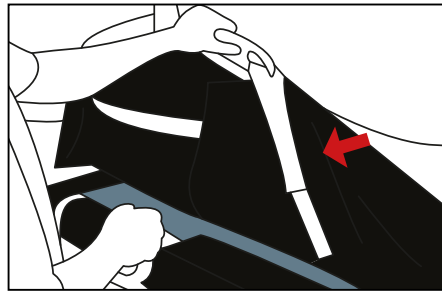
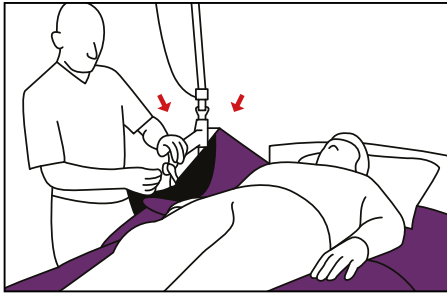
Position the user so their lower back is positioned over the system in the middle of the turning sheet. See illustration

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UK Cross the user's legs if possible. Fasten the straps on the one side of the top- and bottom piece to the lift

Tighten the hip guard so that the top and bottom pieces straps are equally tight. Raise the bed guard on the side where the straps have been fitted

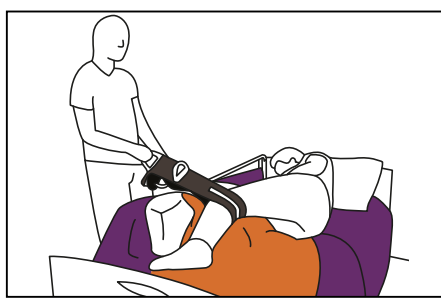
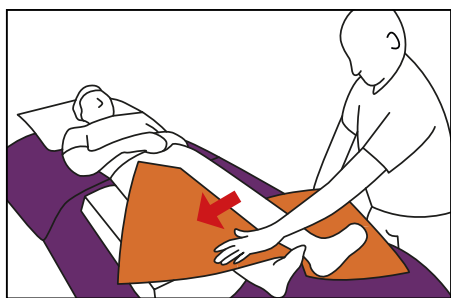
Lift until the user's shoulder blades and hips are elevated. Now position the user toward the bed guard

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UK Before the user is moved into a lateral position, position the Master Comfort Washing Wedge from the user's hip and down on the side you wish to turn the user. Fold a Nylon Master slide piece across the washing wedge and in between the user's legs

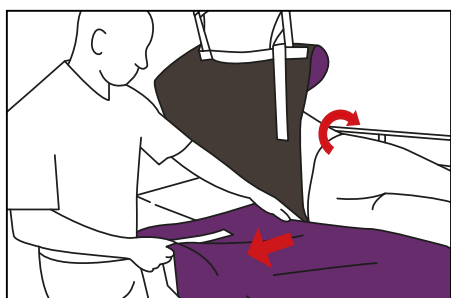
Turn the user as outlined in 2. Extend the pull arm by positioning the Master Grip under the hollow of the knee. Use the bed guard to keep the washing wedge in place and pull the Master Grip until the leg has slid up on the washing wedge. Remove the Nylon Master slide piece

Remove the slide piece once the user is in a stable position. Detach the bottom piece from the top piece. Then, keep the user in a lateral position using the lift, washing wedge and top piece

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UK If the user is heavy or uncooperative, the top piece can be fastened to a lift to help keep the user in a lateral position

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Fordele / Advantages / Vorteile

Master Turner DUO Soft, Frotté / Terry Cloth / Frottee - Large, Low Air & X-Large

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UK

- The system can keep the user in a lateral position during personal care of the lower part of the body
- The top and bottom piece can be purchased individually and combined with all top and bottom pieces
- Thin padding prevents overheating
- Can be used with all types of floor or overhead lifts
- Handles for proper handling
- Reduces the reach distance and bending over for the assistant
- Prevents pressure ulcers

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Master Turner DUO Two In One Soft, Two In One Frotté / Terry Cloth / Frottee - Large & X-Large

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UK

- Helpers can now keep the user in that position without excess strain or discomfort
- Used for personal care
- Used for dressing users in bed
- Allows users with very limited mobility to be mobilised
- Reduces the risk of pressure ulcers/shear
- Easy to transfer users further up in bed
- Easy to position users into an upright position (with or without lift)
- Incontinence membrane prevents liquid from penetrating into the mattress

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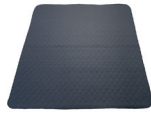
Størrelse / Dimensions / Größe:
L 200 x B/W 80 x H 13 cm

MC 004-3011 [HMI: 73522]
Madrascover / Mattress cover / Matratzenbezug
Exchange 3 - Large

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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