### JAY® TC Back

EN

Owner's Manual

Important Consumer Information.

**NOTE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:** This manual must be given to the user of this product.

**USER:** Before using this product, read this entire manual and save for future reference.

**WARRANTY REGISTRATION:** To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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06

<b>ENGLISH</b>	<b>6</b>
INFORMATION	6
COMPONENTS	7
WARNINGS	8
INSTALLATION / OPERATION / MAINTENANCE	9
DISPOSAL & RECYCLING	11
WARRANTY	12
CONTACT DETAILS AND NOTES	13

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**SUNRISE MEDICAL LISTENS:**

Thank you for choosing a JAY TC System. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Sunrise supplier. Please feel free to write or call us at the address and telephone number below:

**SHIPPING:**

Check all parts are free from shipping damage. In case of damage DO NOT use. Contact Sunrise Medical or your supplier for further instructions.

Sunrise Medical  
 Thorns Road  
 Brierley Hill  
 West Midlands  
 DY5 2LD  
 England  
 Phone: 0845 605 66 88  
 Fax: 0845 605 66 89  
 www.SunriseMedical.co.uk

 **WARNING!**

The products described in this manual were designed to be fitted and installed exclusively by:

- An authorised Sunrise Medical supplier.
- A qualified and competent healthcare professional who has undergone appropriate certified training by either Sunrise Medical or their approved supplier.

**THE JAY TC BACK INTENDED USE:**

The JAY TC System is designed to provide the important benefits of a proper back support for a wide variety of users. As well as providing forced air ventilation and mild heating through the TC unit. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable, and easy to use.

 **WARNING!**

Any application other than that described in this Owner's Manual is not safe. It may result in injury and constitutes a misuse of this product.

Word	Definition
 <b>DANGER!</b>	Advice to the user of potential risk of serious injury or death if the advice is not followed.
 <b>WARNING!</b>	Advice to the user of a risk of injury if the advice is not followed.
 <b>CAUTION!</b>	Advice to user that potential damage to equipment may occur if the advice is not followed.
<b>NOTE:</b>	General advice or best practice.
	Reference To Additional Documentation

**Maximum User Weight.**

136 kg (300 lbs).

**Intended Lifetime.**

The intended lifetime of this products is 2 years.

**Transit Approved.**

 Please reference the JAY J3 Back Owner's Manual for set-up and adjustment of the back. The following instructions pertain to the installation and adjustment of the JAY TC Back System

 **WARNING!**

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

 **WARNING!**


Do not use the TC Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

 **WARNING!**

Carelessness causes fire.

 **WARNING!** 

Please read this manual before usage.

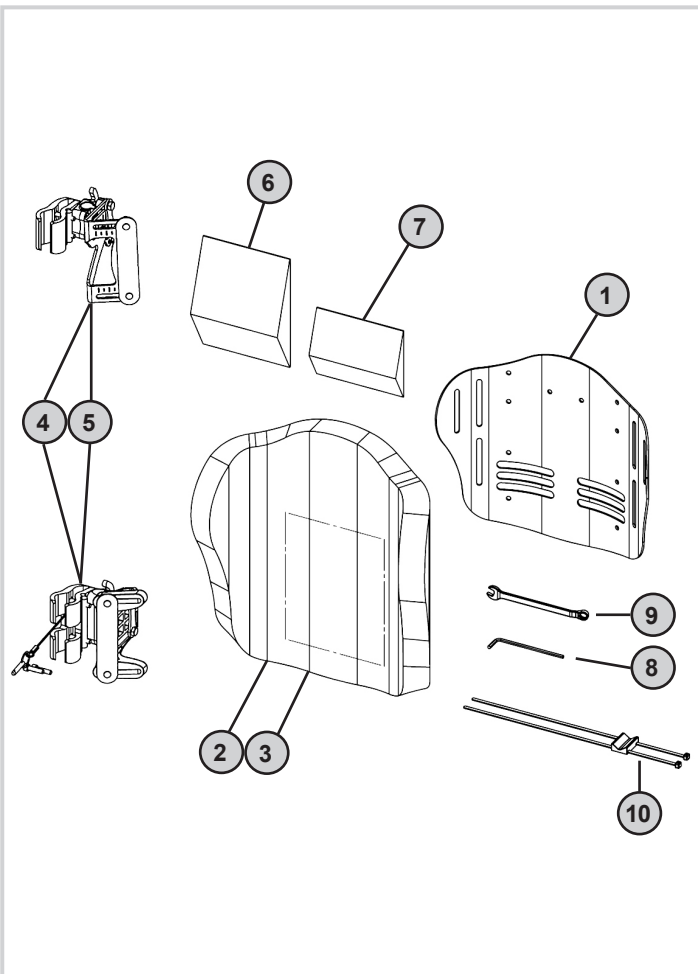NUMBER	DESCRIPTION
1	Backrest Shell
2	Foam
3	Cover
4 & 5	Left & Right Hardware
6	Large Lumbar
7	Small Lumbar
8	Hex Key
9	10 mm Wrench
10	Switch Box Mount
11	Switch Box
12	Battery
13	Charger Stand
14	Charger

Fig.1.0

**NOTE:**

This owner's manual is a supplement to the JAY J3 Back.

Owner's Manual.

The JAY J3 Back owner's manual should be referenced in conjunction with this supplement prior to the installation, set-up, or use of this product.

**NOTE:**

This supplement contains important instructions that must be passed on to the user of this product. Please do not remove this supplement before delivery to the end user.

**SUPPLIER:** This supplement must be given to the user of this product.

**USER:** Before using this product, read this supplement, and the JAY J3 Back manual. Save these instructions for future reference.

**USER WARNING :**

Attention backrest users, do not operate this wheelchair without first reading this owner's manual. If you do not understand the instructions and warnings of this owner's manual please contact your authorized Dealer before operating this back system. Failure to do so may result in damage and/or injury.

**DEALER & TECHNICIAN WARNING :**

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading this owner's manual. If you do not understand the instructions and warnings of this owner's manual please contact the Sunrise Technical Service Department before operating and/or servicing this back system. Failure to do so may result in damage and/or injury.

**GENERAL WARNINGS:**

- **BATTERY:** This battery is constructed with lithium-ion cells, and has a protective circuit module (PCM) to prevent overcharge, over discharge, over current, and short circuits. The PCM prevents risk of fire when used as intended. However, care should be taken when handling the battery.
- **TEMPERATURE RANGE:** Maximum surface temperature of the back with the heat on is 40C (104F). It has a built in thermostat that automatically shuts off the system if it exceeds this limit.
- **CAUTION - Using TC Unit Heat Function:** Persons who are unable to feel pain to the skin must exercise care when using the back heater. Do not place anything on the back that insulates against heat, such as a blanket or towel because this may cause the back heater to overheat and turn off.
- **FANS/LUMBAR:** Warning if using J3 spine align components insure that they do not block the path of the TC unit fans. As this can reduce the effectiveness of the ventilation feature of the unit.
- **ACCESSORIES:** Warning due to the size and positioning of the TC unit and its components, some accessories (not limited to laterals, headrest, etc. will not work in conjunction with the TC unit.
- **REPLACEMENT PARTS:** Use only Sunrise Medical certified replacement parts.
- **BACKREST COVER:** Warning always use the TC unit in conjunction with the backrest upholstery. Do not sit directly on top of the TC unit.
- **CHARGING CYCLE:** The TC unit battery has a charging cycle life of 500 cycles.
- **BATTERY DISPOSAL:** Please refer to the disposal and recycling information in this owner's manual for proper techniques to dispose of the TC unit battery.

**SAFETY INFORMATION AND PRECAUTIONS ON HOW TO TRANSPORT AND SHIP THE BATTERY:****WARNING!**

- Lithium-ion cells are used in the Gentherm battery. Transport and shipping of the battery are, therefore, subject to all applicable statutory requirements, which must be strictly observed. For instance, a defective battery must never be transported by plane.
- If your battery is defective, take it personally to your specialist dealer as shipping lithium-ion batteries by post or other carriers is subject to strict rules and regulations. Once again, we recommend that you contact your specialist dealer.
- As transport regulations may change annually, we strongly recommend that you consult your travel operator, airline or shipping line before you set out on your trip in order to learn about the current applicable regulations. A defective battery must not be taken on a plane or put in your luggage.

## A. LUMBAR, HEADREST, LATERALS

The TC Back can be used with all existing headrests and laterals, as long as no interference with the battery or battery pouch is encountered.

The use of lumbar inserts is OK, as long as the inserts don't interfere with the path of fans.

## B. SWITCH BOX MOUNTING OPTIONS

1. There are multiple ways to secure the switch box to the wheelchair. Choose the position and method best suited for your individual preference. (Either on left or right side of your WC.)
2. The switch box can be mounted anywhere you desire by simply applying the supplied hook and loop Velcro to the side of the switch box and your wheelchair armrest, frame, or other desired location, (Fig.2.0).
3. The supplied Switch box mount (shown in white for clarity) can be used to attach the switch box to either a vertical or horizontal tube with a diameter of 19 mm to 28 mm (.75" to 1.12"), using the supplied double sided tape and zip ties, (Fig.3.0 - 4.0).
4. The switch box can be mounted to a flat surface using zip ties also, (Fig.5.0).
5. The switch box can be permanently mounted to the wheelchair using the supplied double sided tape. Caution – This method will be near permanent and is only advised if none of the other methods are practical.

## C. BATTERY CHARGING (REMOVAL / REPLACEMENT)

1. The battery should be charged prior to first use. Simply plug the charger first into a 230 volt wall outlet, then into the battery. Charging is taking place when the light on the charger turns red. Charging is complete when the light turns green.
2. The battery has a charge level indicator built into it. Push and hold the red button on the side of the battery to determine current level of charge. Green indicates a fully charged battery, yellow indicates charge is nearing it's end, and red indicates it's time to charge.
3. Open the flap on battery pouch (on the back of cover), and insert the battery into the holder, orient the battery terminal per the label on pouch. Be sure the battery plug fully inserts into the receptacle. Pull the battery pouch flap firmly over the top of the battery and secure by engaging the Velcro® hook and loop.

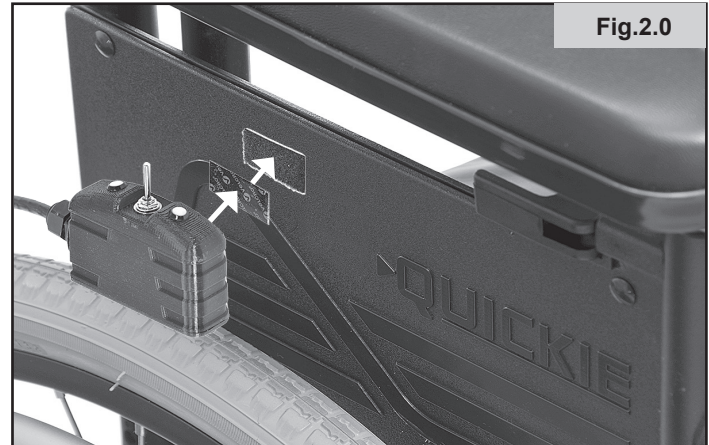
Fig.2.0

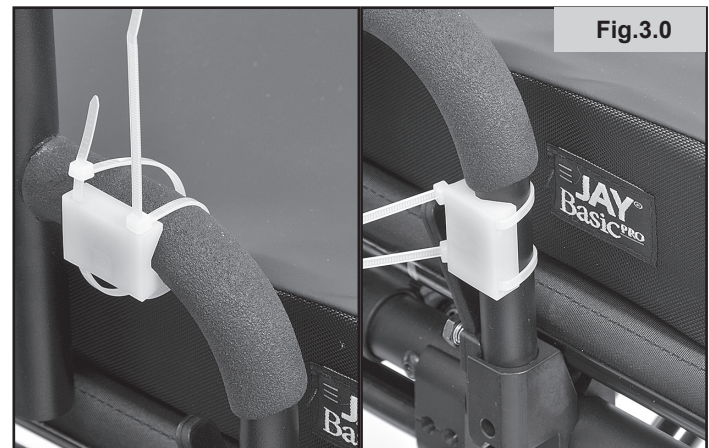
Fig.3.0

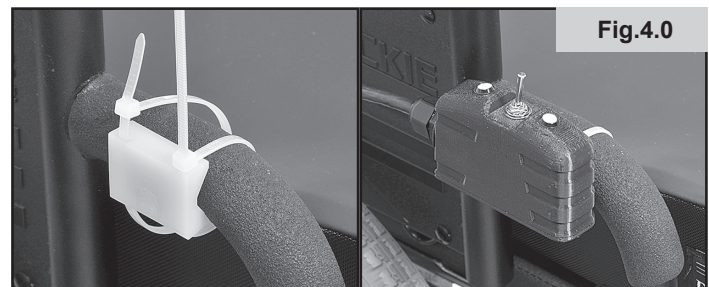
Fig.4.0

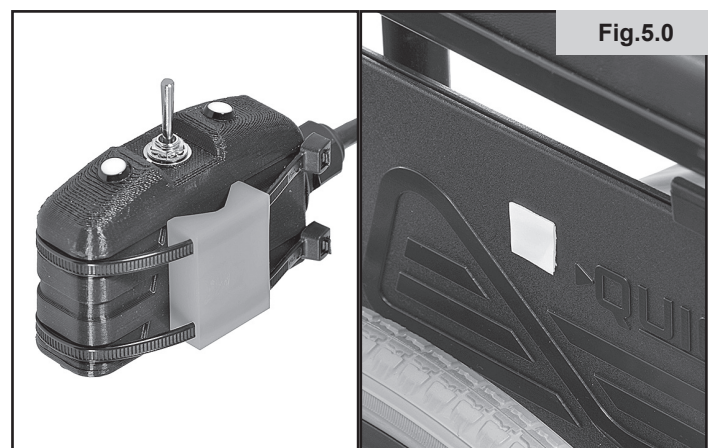
Fig.5.0

**D. OPERATION**

1. Once your switch Box is properly mounted to your wheelchair, simply plug the connector at the end of the switch box cable into the connector coming out of the battery box. The TC back unit is now fully functional.
2. Push the switch forward to turn on the heat. The LED in front of the switch will illuminate RED, indicating the heat is on.
3. Pull the switch back to turn on the ventilation fans. The LED behind the switch will illuminate BLUE, indicating the fans are operating.

**E. EASY CLEANING**

**1. Back cover**

- a. Unplug the switch box from the backrest unit.
- b. Detach the backrest from the wheelchair.
- c. Remove the battery pack from the cover by detaching the Velcro®.
- d. Remove the cover from back shell by detaching the Velcro®.
- e. Unzip the zipper on the bottom of the cover.
- f. Remove the foam and TC unit, then turn the cover inside out.
- g. Zip the cover closed to avoid snagging.
- h. Machine wash in warm water (60°C).
- i. Drip and tumble dry in on Low Heat.

**2. TC Unit**

- a. The TC Unit should be wiped clean with damp cloth on a regular basis to prevent buildup of dirt and grime.
- b. The TC unit should not be removed from the backrest foam as it is permanently attached.
- c. Ensure that the TC Unit is free from damage or obstructions before refitting the cover.


LAUNDRY CARE SYMBOLS	
	Machine wash (60°C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	No chemical cleaning

**NOTE:** Do not dry clean covers or use industrial washers or dryers to clean covers. Do not steam autoclave. Do not bleach. Laundry care symbol definitions are shown.

**DISPOSAL AND RECYCLING**

- When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local authority.
- This wheelchair is manufactured using a variety of materials.
- Your product should not be disposed of as ordinary household waste.
- You should dispose of your wheelchair properly, according to local laws and regulations.
- Most materials that are used in the construction of this product are fully recyclable.
- The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.
- Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations.

**NOTE:** No component of this product was made with Natural Rubber.

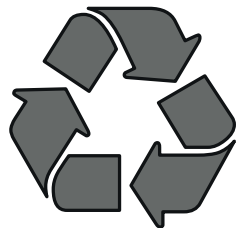**GUARANTEE****THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.**

Sunrise Medical\* provides a guarantee, as set out in the warranty conditions, for products to its customers covering the following.

**Warranty conditions:**

1. Should a part or parts of the product require repair or replacement as a result of a manufacturing and/or material fault within 24 months, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects.
2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the product outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The product must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the product in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise, if a repair or replacement of a product or a part is required for the following reasons:
  - a. Normal wear and tear, which include but is not limited to the following parts where fitted; batteries, armrest pads, upholstery, tyres, brakes shoes, ferrules, etc.
  - b. Any overloading of the product, please check the EC label for maximum user weight.
  - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
  - d. Accessories have been used which are not specified as original accessories.
  - e. The product or part having been damaged by neglect, accident or improper use.
  - f. Changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
  - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical"

\* Means the Sunrise Medical facility from which the product was purchased.

**SUNRISE MEDICAL LISTENS**

Thank you for choosing a JAY TC System. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Sunrise supplier.

Please feel free to write or call us at the address and telephone number below:

Sunrise Medical  
 Thorns Road  
 Brierley Hill  
 West Midlands  
 DY5 2LD  
 England  
 Phone: 0845 605 66 88  
 Fax: 0845 605 66 89  
 www.SunriseMedical.co.uk

**FOR ANSWERS TO YOUR QUESTIONS**

Your authorized supplier knows your wheelchair best, and can answer most of your questions about chair safety, use and maintenance. For future reference, fill in the following:

Supplier:
Address:
Telephone:
Serial Number:
Purchase Date:

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S