# Control Medi Easy

User manual

Art. Nr: 429240-1

# Content

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10.2	Kassering .....	20
11.	Sammanställning snabbguide Gewa Control Medi Easy.....	21
GB:	English Manual Control Medi Easy .....	23
1.	Introduction .....	23
1.1	Product description.....	23
1.2	Intended purpose.....	23
1.3	General information about Control Medi .....	24
1.4	Symbols .....	24
1.5	Inserting Batteries.....	25
1.6	Change the batteries .....	25
2.	Important information.....	26
2.1	Safety.....	26
2.2	Safety instructions for operation .....	26
2.3	Serious incidents .....	26
2.4	Conformance with requirements for medical devices .....	26
2.5	Product liability .....	26
3.	Settings .....	27
3.1	Alerts when programming.....	27
3.2	Cancel Programming.....	27
3.3	Locking/Unlocking the Settings Lock .....	27
3.4	Recording IR Channels from another Transmitter.....	28
3.5	Adding a GewaLink channel .....	29
3.6	Adding a 4096 code.....	30
3.7	Feedback when pressing buttons .....	31
3.8	Setting an acceptance delay.....	31
3.9	Direct control .....	31
3.10	Setting the channel group.....	33
4.	Button Labelling.....	34
5.	Control Medi Holders.....	34
6.	Cleaning.....	35
7.	Default settings in your Control Medi .....	35
7.1	Resetting to Default Settings .....	35
8.	Technical data.....	36
8.1	Technical data .....	36
8.2	Accessories included with your Control Medi Easy.....	36
8.3	Other accessories available for Control Medi.....	36
9.	Backups and reinstallation.....	37
10.	Storage and disposal.....	37
10.1	Storage.....	37
10.2	Disposal .....	37

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## **1. Introduction**

### **1.1 Product description**

Control Medi Easy is a programmable IR transmitter, which replaces ordinary remote controls for TV, DVD, stereo etc. It has totally four different programming positions on one level. Simply press the buttons to send signals. You can also connect an external control switch that can directly control one button on the transmitter.

When you press a button, confirmation is given as vibration, sound and light.

The Control Medi is a part of the “GewaLink system” and therefore can be used together with all of these products.

### **1.2 Intended purpose**

Control Medi is a simple to use remote control intended to support people who are unable to use a standard remote control due to motor, cognitive or visual impairments. The device can help the user control electronic equipment at home, including TV sets, lighting, and entrance and garage door openers. The device is intended for indoor and outdoor use.

### 1.3 General information about Control Medi

To be able to use your Control Medi there are certain things that you should know:

**⚠ WARNING:** Make sure that the user always can get help through a backup system if the product does not operate correctly.

**⚠ WARNING:** Before you use the product, make sure that all connected appliances and electronic equipment are correctly installed, used as intended and safe to use.

Control Medi is delivered preset with GewaLink channels, meaning that it is possible to use the transmitter immediately after the batteries has been inserted. See section *Inserting Batteries*.

To enter the programming mode, press first Button **4** and then **simultaneously** one of the *Function* buttons for 2 seconds. Different *Function* buttons are chosen depending on what is to be programmed.

If you press the wrong button or makes a mistake when programming you can always press Button **4** for two seconds. It works as a *Reset* button.

It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power. If a button on your Control Medi or control switch is pressed for more than 1 minute, Control Medi will stop transmitting to save power.

The settings can be protected by using the Settings Lock.

This picture shows the various terms and functions which are described in the text.

Fig 1

### 1.4 Symbols

The following symbols appear in the text throughout this manual:

- X** and **Y** Any button on Control Medi
- Y** Any button on another transmitter to be programmed
- 1...4** Button 1-4
- 4**<sup>2 sec</sup> Button 4 held pressed for approximately 2 seconds
- 4** + **1**<sup>2 sec</sup> Button 4 and 1 held pressed for approximately 2 seconds

## 1.5 Inserting Batteries

Unscrew the battery lid screws on the back of your Control Medi using a screwdriver and insert three AAA batteries (alkaline LR03 1.5 V). Insert the batteries according to the guidelines at the bottom of the battery compartment.

**NOTE:** Use non-rechargeable alkaline batteries for Control Medi Easy.

## 1.6 Change the batteries

When the batteries need replacing, the LED indicator blinks red and one long and three short beeps are repeated twice.

## 2. Important information

Before you use the product, read and understand the contents of this user manual.

### 2.1 Safety

#### Safety definitions

 **WARNING:** Risk of injury or death if the instructions are not followed.

 **CAUTION:** Risk of damage to the product if the instructions are not followed.

**NOTE:** Information that is necessary in a given situation.

### 2.2 Safety instructions for operation

 **WARNING:** Read the following warning instructions before you use the product.

- Make sure that the user always can get help through a backup system if the product does not operate correctly.
- Before you use the product, make sure that all connected appliances and electronic equipment are correctly installed, used as intended and safe to use.
- The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

### 2.3 Serious incidents

If a serious incident related to the product occurs, this must be reported to the manufacturer and the competent authority of your country.

An incident is regarded serious if it directly or indirectly led, might have led or might lead to either of these situations:

- The death of a patient, user or other person.
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health.
- A serious public health threat.

### 2.4 Conformance with requirements for medical devices

Control Medi is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

### 2.5 Product liability

The product has been developed and risk-assessed in accordance with ISO 14971. The user manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. If the product is used in other ways than the intended use, it may involve risks for which Abilia AB does not take responsibility. The estimated life of the product applies when used in accordance with its intended use and maintenance instructions.

Abilia AB is not liable for any loss, damage or consequential loss if the product has been used or installed in any way other than as specified in this document. Any form of damage or tamp.

## 3. Settings

Below is described how to program your Control Medi for different setting options.

To enter the programming mode, you should first press Button **4** and then simultaneously press the button **X** for 2 seconds for your desired setting option (see below). When you release the buttons a “Setting” alert is heard, and the indication lamp on the button for the setting option flashes. An “Error” alert will sound if you select an invalid setting option.

Press simultaneously	Setting option
<b>4</b> + <b>1</b> 2 sec	Function
<b>4</b> + <b>2</b> 2 sec	System
<b>4</b> + <b>3</b> 2 sec	Reset

This is followed by different button presses depending on what is to be set, see the separate descriptions below.

It is important not to wait too long between the different steps when programming. After a certain time your Control Medi automatically leaves the programming mode to save power.

If your programming is successful, a “Success” alert will sound. If something has gone wrong or takes too long, an “Error” alert will sound.

### 3.1 Alerts when programming

Whilst you are programming, your Control Medi gives feedback with flashes and different alerts:

“Setting” -  (three increasing tones).

“Success” -  (three positive tones).

“Error” -  (eight negative tones).

### 3.2 Cancel Programming

To cancel the programming, you can press Button **4** and keep it pressed for two seconds. It works as a *Reset* button.

### 3.3 Locking/Unlocking the Settings Lock

In order to protect the programmed settings in your Control Medi, Settings Lock can be activated.

When the Setting Lock is enabled, the settings are protected. If you try to change the settings without first unlocking, the “Error” alert is given.

Do as follows to lock/unlock the Settings Lock:

1. First press **4** and then simultaneously press **1** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **3**.
3. Press **1** to unlock the Settings Lock, or **2** to lock.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Locking/Unlocking the Settings Lock

Press **4** + **1** 2 sec, **3**, then press your desired option **1** or **2**.

**1** = Unlock. **2** = Lock.

Default setting: Unlocked

### 3.4 Recording IR Channels from another Transmitter

Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the rear of your Control Medi, see picture.

1. First press **4** and then simultaneously press **1** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **1**.
3. Press the button **X** where the channel is to be recorded. The indicator light for the selected button glows steadily and the “Press” alert sounds.
4. Press and hold the corresponding button **Y** on the original transmitter pressed. Release the button when the indicator light for the selected button goes off and the “Release” alert sounds. This may take up to 10 seconds. If the indicator light glows again and the “Press” alert sounds, you should press the button one or more times until the “Success” alert sounds.

If the “Error” alert sounds, the recording has failed. Try again from step 3.

5. If the recording of the first button was successful, repeat steps 3 and 4 until all buttons are programmed.
6. Finish by pressing **4** for 2 seconds.
7. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
8. Now test your recorded functions on the device that is to be controlled.

Original transmitter

#### Recording IR Channels

Press **4** + **1**<sup>2 sec</sup>, **1**, select button **X**, select button **Y**, then continue or finish with **4**<sup>2 sec</sup>

### 3.5 Adding a GewaLink channel

Your Control Medi Easy comes with GewaLink channels 16-19. If you want to use an other GewaLink channel on a certain button, you can insert a chosen GewaLink channel like this:

1. First press **4** and then simultaneously press **1** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **2** and then press **1**.
3. Select the button **X** where the GewaLink channel is to be saved.
4. Select your **Channel range**,
  - 1** = GewaLink 00 – 15.
  - 2** = GewaLink 16 – 31.
  - 3** = GewaLink 32 – 47.
  - 4** = GewaLink 48 – 63.
5. Select your **Channel group** according to *Table 2* below.
6. Press the **Button** that corresponds to your desired GewaLink channel in the selected channel group according to *Table 2* below.
7. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

**Table 2: Channel group and button to add GewaLink channel**

	<b>Channel range 1</b> : GewaLink 00-15				<b>Channel range 2</b> : GewaLink 16-31			
Button	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>
<b>1</b>	00	04	08	12	16	20	24	28
<b>2</b>	01	05	09	13	17	21	25	29
<b>3</b>	02	06	10	14	18	22	26	30
<b>4</b>	03	07	11	15	19	23	27	31
	<b>Channel range 3</b> : GewaLink 32-47				<b>Channel range 4</b> : GewaLink 48-63			
Button	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>	Ch. group <b>1</b>	Ch. group <b>2</b>	Ch. group <b>3</b>	Ch. group <b>4</b>
<b>1</b>	32	36	40	44	48	52	56	60
<b>2</b>	33	37	41	45	49	53	57	61
<b>3</b>	34	38	42	46	50	54	58	62
<b>4</b>	35	39	43	47	51	55	59	63

#### Adding a GewaLink channel

Press **4** + **1** <sup>2 sec</sup>, **2**, **1**, choose the button **X** choose the **Channel range**, choose the **Channel group**, then choose GewaLink channel.

See *Table 2* for your choice of Channel groups and GewaLink channels.

Example: You want to add GewaLink channel 28 onto a certain button **X**.

Press **4** + **1** <sup>2 sec</sup>, **2**, **1**, **X**, **2**, **4**, **1**.

### 3.6 Adding a 4096 code

A coded channel is normally used for functions that require strict authorization such as apartment or house doors. The code has 4096 different combinations.


If you have to program a new coded channel there are two methods of doing so. You can set the 4096 code on any button.

**Method one:** record the code from another transmitter, as described under *Recording IR channels from another Transmitter*.

**For method two** a binary code is entered. Make up the binary code yourself or else use a code that is already set in an IR receiver or IR transmitter.

There are two types of IR receivers: an older type of receiver where you set the code with a 12-pole switch, or a newer type where the IR receiver is programmable. The principle is the same when it comes to creating the binary code. *Fig 3* shows a 12-pole switch. Switches 1, 4, 6 and 12 are in the “On” position, the others are in the “Off” position.

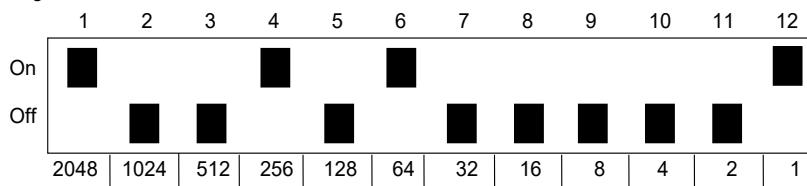Fig 3

The following example describes how the binary code is created according to method two:

1. First press **4** and then simultaneously press **1** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **2** and then press **2** again.
3. Select the button where the 4096 code should be saved.
4. Enter the code by pressing on **1** or **2**.  
**1** = switch in Off position and **2** = switch in On position.  
 Press twelve times in turn: **2**, **1**, **1**, **2**, **1**, **2**, **1**, **1**, **1**, **1**, **1**, **2**, (Fig 3).
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.
6. The code is now set on your selected button. NOTE! The code above is just an example and should be different for different users. Make a note of the chosen code for future usage.

#### Adding a 4096 code

Press **4** + **1** <sup>2 sec</sup>, **2**, **2**, choose the button **X**, then press **1** or **2** 12 times.

**1** = Off position. **2** = On position.

### 3.7 Feedback when pressing buttons

Your Control Medi always gives feedback in the form of light signals when you press a button. You can also get feedback as sound alerts and vibration. It is possible to get feedback from all three ways simultaneously, or you can set up what will be on or off (light is always on).

Do as follows to set the levels of sound and vibration.

1. First press **4** and then simultaneously press **2** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 1 flashes.
2. Press **1**.
- 3a. Sound level:  
Press **1** to lower the volume, press **2** to increase the volume. Repeat pressing until the desired volume is set.  
*and/or:*
- 3b. Vibration level:  
Press **3** to lower the vibration, press **4** to increase the vibration. Repeat pressing until the desired vibration is set.
4. Finish by pressing **4** for 2 seconds.
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Set level for sound and vibration

Press **4** + **2**<sup>2 sec</sup>, **1**, choose the setting, then **4**<sup>2sec</sup>.

**1** = Lower volume. **2** = Increase volume. **3** = Lower vibration. **4** = Increase vibration.

*Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)*

### 3.8 Setting an acceptance delay

An acceptance delay can be set for pressing the buttons. Acceptance delay is the time a button has to be pressed for before a function is activated. An acceptance delay can be set from 1 to 4. 1 is no delay at all (off) and 4 is the maximum delay (1 sec.).

1. First press **4** and then simultaneously press **2** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press **2**.
3. Press your desired value for the acceptance delay, **1** is off, and **4** is max.
4. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Setting an acceptance delay

Press **4** + **2**<sup>2 sec</sup>, **2**, then choose the setting.

**1** = 0 sec. **2** = 0.3 sec. **3** = 0.6 sec. **4** = 1 sec.

*Default setting: Acceptance delay = OFF (0 sec).*

### 3.9 Direct control

Your Control Medi can directly control a button with an external control switch. You can choose which button that should be activated when you press the control switch.

It is also possible to direct control two buttons some seconds apart. The time can be from 2 to 8 seconds. (Direct Control of two buttons applies from ver. 1.0.6).

A control switch with 3.5mm plug will fit in to the side of your Control Medi.

Do as follows to activate/deactivate Direct control:

1. First press **4** and then simultaneously press **2** for 2 seconds. When you release the buttons the "Setting" alert sounds and the indication lamp on button 2 flashes.

2. Press **3**.

3a. Activate direct control.

Press **2**, and then press the button **X** that you want to be activated when pressing the control switch.

or:

3b. Activate direct control of two buttons.

Press **3**, and then press the first button **X** that you want to be activated when pressing the control switch. Press the second button **Y** to be activated. Select time before the second button should be activated with **1** - **4**.

3c. Press **1** to turn off direct control.

4. If the action is successful, the "Success" alert sounds. If something goes wrong or takes too long, the "Error" alert sounds.

#### Activate/deactivate Direct control

Activate direct control: Press **4** + **2** <sup>2 sec</sup>, **3**, **2**, choose the button **X**.

Activate direct control of two buttons: **4** + **2** <sup>2 sec</sup>, **3**, **3**, choose the first button **X**, choose the second button **Y**. Select time **1** - **4**.

**1** = 2 sec, **2** = 4 sec, **3** = 6 sec, **4** = 8 sec

Deactivate direct control: Press **4** + **2** <sup>2 sec</sup>, **3**, **1**.

*Default setting: OFF (none)*

### 3.10 Setting the channel group

Your Control Medi has the option to set different channel groups for the transmitter to operate within. When the change of channel group is applied, programmed channels will disappear.

Normally, the channel group does not need to be changed. If several GewaLink transmitters are to be used in the same area and you do NOT want to control the same equipment, it may be necessary to use different channel groups.

Table 3 shows the different settings.

**Table 3: Channel groups**

	Channel range, GewaLink channels			
	00-15 ①	16-31 ②	32-47 ③	48-63 ④
Channel group ①	00-03	16-19	32-35	48-51
Channel group ②	04-07	20-23	36-39	52-55
Channel group ③	08-11	24-27	40-43	56-59
Channel group ④	12-15	28-31	44-47	60-63 <sup>1</sup>

<sup>1</sup> Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

Do as follows to set the **Channel group**:

1. First press ④ and then simultaneously press ② for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 2 flashes.
2. Press ④.
3. Choose the Channel range ① to ④ according to table 3 above.
4. Choose the **Channel group** ① to ④ according to table 3 above.
5. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Set channel group

④ + ②<sup>2 sec</sup>, ④, choose Channel range, choose **Channel group**.

See *Table 3* for choice of channel range and channel group.

*Default setting: GewaLink channel 16-19*

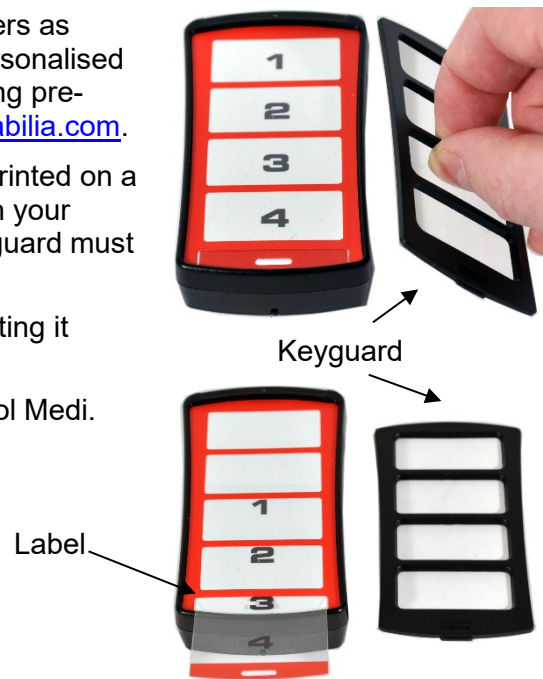
## 4. Button Labelling

The buttons on your Control Medi are marked with numbers as standard. The numbered label can be replaced with a personalised design. It is possible to create your own button labels using pre-designed templates with symbols and text, refer to [www.abilia.com](http://www.abilia.com).

The document can also be downloaded. The labeling is printed on a clear sheet or white paper and inserted through the slit on your Control Medi. Before the button label is inserted, the keyguard must be removed.

Remove the keyguard by gripping the middle, and then lifting it straight up, see photo.

An A4 sheet for making notes is included with your Control Medi.

## 5. Control Medi Holders

Your Control Medi can be equipped with a clip (accessory) which can be attached to a belt or pocket.

As an optional accessory, there is also a holder that fits to your Control Medi. This holder can be assembled for mounting on to a table or attaching to a pipe. The holder can also be mounted on a mobile stand.

Another option is to hang your Control Medi around the neck. A neck strap is supplied with your Control Medi.

Do as follows to attach the neck strap:

1. Unscrew the battery lid.
2. Place the thin loop of the neck strap in the groove around the screw mount.
3. Screw back the battery lid back down.
4. Adjust the length of the neck strap.

**WARNING:** The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap!

## 6. Cleaning

Your Control Medi can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

## 7. Default settings in your Control Medi

On delivery your Control Medi Easy will be set with certain default settings as follows:

Acceptance delay is set to value OFF

Feedback sound level is set to value 1 (0 – 5)

Feedback vibration level is set to value 1 (0 – 5)

GewaLink channels 16-19

### 7.1 Resetting to Default Settings

For Control Medi it is possible to reset one button or reset the entire Control Medi to Default settings. If the settings lock is activated, the device must first be unlocked. See Locking/Unlocking the Settings Lock

Do the following to restore:

1. First press **4** and then simultaneously press **3** for 2 seconds. When you release the buttons the “Setting” alert sounds and the indication lamp on button 3 flashes.
- 2a. Reset **one button**:  
Press **2**.  
Choose the button **X** to be restored.  
*or:*
- 2b. Reset to **factory settings** (all buttons):  
Press **1**.  
Press **1**, **2**, **3** in sequence.
3. If the action is successful, the “Success” alert sounds. If something goes wrong or takes too long, the “Error” alert sounds.

#### Resetting to default settings

To reset one button:

Press **4** + **3** <sup>2 sec</sup>, **2**, choose button **X**.

To reset to factory settings:

Press **4** + **3** <sup>2 sec</sup>, **1**, **1**, **2**, **3**.

## 8. Technical data

### 8.1 Technical data

Power consumption at transmitting:	Approx. 40mA (GewaLink channel)
Input control switch type:	3.5mm jack plug 2-pole
Battery life (button use only):	Approx. 2 years (100 seconds transmitting per day with GewaLink channel)
Programming locations:	4 places
Operating force buttons:	< 160g
Range:	40-50 m (GewaLink channels)
EMC-standard emission: " immunity:	EN 301 489-3 V1.4.1:2002, EN(IEC) 61000-6-2:2005, EN(IEC) 61000-6-3:2007, selected parts from EN 300 220-2 V2.3.1:2010
Computer Connection:	Via 429252 Control Tool USB interface
Measurements (L x W x H):	100 x 55 x 22 mm (incl. rubber feet)
Weight (incl. batteries):	105g
Operating temperature (excl. batteries):	-20°C to +45°C

### 8.2 Accessories included with your Control Medi Easy

Art.no	Name	Description	Qty
429256	Control Medi Labelling 4	For labeling of buttons	1 pcs
429254	Control Medi Labelling 2	For labeling of buttons	1 pcs
-	Note sheet	A4-sheet for noting your programmed channels	1 pcs
429258	Control Medi Keyguard 4	Keyguard for 4 buttons	1 pcs
429259	Control Medi Keyguard 2	Keyguard for 2 buttons	1 pcs
890410	Neck strap		1 pcs
8375	Battery	LR03 (AAA) 1,5 V	3 pcs

### 8.3 Other accessories available for Control Medi

Art.no	Name	Description
429249	Control Medi Holder 6 mm Hole	For assembly of Control Medi to different systems with a 6 mm hole
429250	Control Medi Holder 10 mm Pipe	For assembly of Control Medi to 10 mm pipe system
429251	Control Medi Belt Clip	
429252	Control Tool USB interface	Adapter for connecting a Control Medi to PC

## 9. Backups and reinstallation

Your Control Medi can be connected to a computer in order to make a backup of the settings and recordings of your transmitter. The backup copy can be used to reinstall the settings and recordings onto your transmitter. Use the Control Tool USB Interface accessory (sold separately). The software can be downloaded from Abilia's website. [www.abilia.com](http://www.abilia.com). See instructions on the website.

## 10. Storage and disposal

### 10.1 Storage

Keep the product indoors at room temperature and away from moist.

### 10.2 Disposal

The Control Medi contains electronics components that must be recycled. Dispose the product at an applicable disposal location.

# 11. Summary and Quick guide Gewa Control Medi Easy

## Locking/Unlocking the Settings Lock

Press **4** + **1** <sup>2 sec</sup>, **3**, then press your desired option **1** or **2**.

**1** = Unlock. **2** = Lock.

*Default setting: Unlocked*

## Recording IR Channels

Press **4** + **1** <sup>2 sec</sup>, **1**, select button **X**, select button **Y**, then continue or finish with **4** <sup>2 sec</sup>

## Adding a 4096 code

Press **4** + **1** <sup>2 sec</sup>, **2**, **2**, choose the button **X**, then press **1** or **2** 12 times.

**1** = Off position. **2** = On position.

## Set level for sound and vibration

Press **4** + **2** <sup>2 sec</sup>, **1**, choose the setting, then **4** <sup>2sec</sup>.

**1** = Lower volume. **2** = Increase volume. **3** = Lower vibration. **4** = Increase vibration.

*Default setting: Sound alert = 1 (0 – 5) Vibration = 1 (0 – 5)*

## Setting an acceptance delay

Press **4** + **2** <sup>2 sec</sup>, **2**, then choose the setting.

**1** = 0 sec. **2** = 0.3 sec. **3** = 0.6 sec. **4** = 1 sec.

*Default setting: Acceptance delay = OFF (0 sec).*

## Activate/deactivate Direct control

Activate direct control: Press **4** + **2** <sup>2 sec</sup>, **3**, **2**, choose the button **X**.

Activate direct control of two buttons: **4** + **2** <sup>2 sec</sup>, **3**, **3**, choose the first button **X**, choose the second button **Y**. Select time **1** - **4**.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit12 Buckingham Business Park, Swavesey,