Piko Button & Finger Button

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EN

Description

Pikobutton Ability Switches are Medical Devices Class I designed to be very durable and easy to use. Their intended use is to control environmental control systems, communication etc. non-hazardous, not life supporting devices. The activation force of buttons is between 75 g -250 g which allows different kind of users with different abilities to use the buttons.

Warnings

- Do not use to control movement of mobility devices or any life supporting or potentially hazardous devices.
- Do not exceed electrical specification 12VDC 50mA
- Do not wet non water-resistant devices
- Not MR compatible 

PikoButton

Single button switch has a 3,5 mm mono jack plug.

With a M4-hole in the middle of the bottom, it can be easily mounted to different surfaces. Adapters are available to attach Pikobuttons to Fisso and Manfrotto mounting arms.

Activation force: 75 g / 125 g

Plug: 3,5 mm mono jack

PikoButton is also available as a water-resistant version.

Protection class, standard Pikobutton: IP30

Protection class, water resistant Pikobutton: IP67

Size: 13 x 30 mm / 17 x 50 mm

Activation force: 125g

Cleaning

Standard PikoButtons: Wipe with damp cloth and mild detergent / alcohol

Water resistant PikoButtons: Clean under running water using mild detergent or wipe with alcohol

FingerButton

FingerButton can be adjusted around one or two fingers with Velcro.

FingerButton has one switch.

Protection class: IP20

Activation force: 75 g

Plug: 3,5 mm mono jack

Cleaning

Fingerbutton: Wet cleaning is not possible

Technical information

Class I Medical Device according EU MDR 2017/745.

See separate Declaration of Conformity.

Max load: 12V/50 mA

Max/min temp: -30 to +80°C

Cable length: 1,5 m

Recycling

Recycling as electrical equipment

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Haltija Group Oy Koivuhaantie 2-4 D
Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingham Business Park, Swavesey,