Andromeda IR-REC8 BED

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Introduction

Product description

Andromeda IR-REC8 BED is an IR receiver with which it is possible to control the functions of a motorized bed wirelessly via a Gewa IR transmitter. The receiver has 8 relay outputs, which can control a maximum of 8 functions of a bed; the head end, the foot end, knee part and the whole bed up/down.

The receiver is delivered pre-programmed for use with, for example, the IR transmitter Gewa Control Omni, which has pre-made pages for bed control. When necessary, the receiver can be reprogrammed.

For the receiver there are finished custom split cables adapted to be connected to beds with the following control units:

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Depending on the control unit for the bed, the appropriate split cable should be used. The split cable is attached to a terminal and can be replaced easily.

Intended purpose

Andromeda IR-REC8 BED is intended to support people with motor, cognitive or visual impairments to remotely control electronic equipment at home. It can receive IR and GEWA radio signals from remote controls and use relay outputs to control electronic equipment. The device is intended for indoor use.

Important information to the installation engineer

NOTE: Incorrect installation or use of an electric adjustable bed that is wirelessly controlled by an IR transmitter can cause the device to not work as expected.

- Use only non-latching (momentary, monostable) control.
- Do not use macro functions on the IR transmitter.
- If the user experience difficulties at a certain bed angle, for instance difficulties in breathing, the minimum/maximum angle should be limited.
- Keep good ventilation for the power adapter in use to prevent it from overheating. A 10-15 cm clearance must also be kept from heat sources.
- If the equipment is not used for long period of time, disconnect the power adapter to avoid being damaged by voltage peaks or lightning strike.
- Do not remove the case of the power adapter. The openings should be protected from foreign objects or dripping liquids.
- Do not put the power adapter in places with high moisture or near the water.
- Disconnect the power adapter from the AC power before cleaning. Do not use any liquid or aerosol cleaner. Only use a damp cloth to wipe it.
- For other information about the power adapter, refer to www.meanwell.com for details.
- The installing engineer shall give the following information to the user and assisting personnel.

Important information to the user and assisting personnel

NOTE: Incorrect installation or use of an electric adjustable bed that is wirelessly controlled by an IR transmitter can cause the device to not work as expected.

- For best functionality of IR communication, it needs to be clear sight between the IR transmitter and IR receiver.
- The IR receiver does not work on power failure.

Symbols on the equipment

	<p>Recycling of Electrical and Electronic Equipment.</p>
	<p>TUV approval mark.</p>
	<p>UL approval mark.</p>
	<p>Efficiency standard mark.</p>
	<p>CE-mark. Conformité Européenne.</p>
	<p>FCC Declaration of Conformity. Federal Communications Commission.</p>
	<p>Double insulation. No earth wire.</p>
	<p>Indoor use only.</p>


Get started with Andromeda IR-REC8 BED

Connect the IR receiver

Connect the split cable to the control unit of the bed, and then connect the receiver and the manual control handle to the split cable.


Hang the receiver on the wheel frame of the bed or on the headboard. Put the receiver in the direction with the best reception conditions (not towards a wall). On the inside of the attachment component there is a Velcro to fasten the IR receiver in the right place.

Make sure no cables are squeezed. Fasten the excess cables with the straps. Check that the cables are long enough when the bed is in its highest position.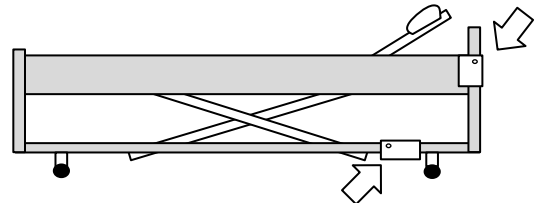

Connect an external detector

If necessary, an external detector can be connected, for example, if the detector on the IR receiver cannot be mounted so that the detector is not obstructed. The detector is the actual eye that receives the IR signal. The colors of the wires are to be connected as shown in the figure when connecting an external detector. Make sure that the wires are correctly connected.

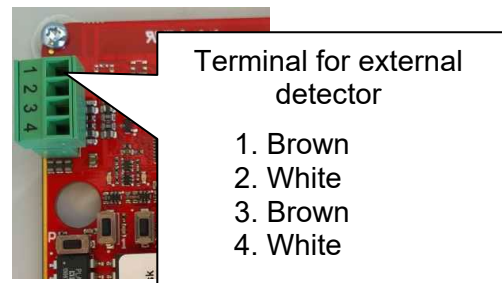
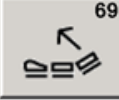
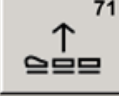
It is possible to connect up to 5 external detectors by connecting them in parallel to the terminal.

NOTE: The range will be reduced if the detector eye is covered or obstructed in some way, for example by curtains or furniture.

Function table

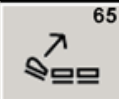The table below shows how the Andromeda IR-REC8 BED is set when delivered.

Function	Programming button	GewaLink channel	Premade page in Control Omni
Head down	1	64	
Head up	2	65	
Knee down	3	66	
Knee up	4	67	
Foot down	5	68	
Foot up	6	69	
Bed down	7	70	
Bed up	8	71	

Program Andromeda IR-REC8 BED

Channels 64-71 is the default setting of the receiver, see the *function table* above. The receiver is set to work with an IR transmitter with 8 functions. If another transmitter or other channels will be used, the receiver must be reprogrammed.

The receiver can be programmed with GewaLink channels 0-127 (channel 62 has no function) or the coded channel (4096). There is one programming button for each relay.

A Gewa IR transmitter is required to program the receiver.

Simple programming

Andromeda IR-REC8 BED is designed to be easy to program and has a simple and an advanced programming mode. Follow the instructions below for programming the most common uses.

Simple programming of a relay

1. Press the selected button on the IR transmitter and, at the same time, press the programming button for the selected relay on the receiver.
2. Wait until the indicator lamp flashes three times.
3. Release the programming button of the receiver and the button on the IR transmitter.
4. Check the function by pressing the selected button on the IR transmitter.

Simple programming of a relay with double IR channels

The Andromeda IR-REC8 BED can be programmed so that a relay can be activated by two different GewaLink channels. This can be used, for example, to have a button “down” which lowers the foot and head simultaneously. In this case, program the two relays with a second GewaLink channel that controls this function.

1. Press the selected button on the IR transmitter for the channel that is to be programmed as the first channel, and at the same time, press the programming button for the selected relay on the receiver.
2. Wait until the indicator lamp flashes three times.
3. Keep pressing the programming button for the selected relay. Press the button on the IR transmitter for the channel that is to be programmed as the second channel.
4. Wait until the indicator lamp flashes three times.
5. Release the programming button of the receiver and the button on the IR transmitter.
6. Check the function by pressing the selected buttons on the IR transmitter.


Advanced programming

The Andromeda IR-REC8 BED is designed to be easy to program and has a simple and an advanced programming mode. Follow the instructions below for advanced programming.

Programming channels

Do as follows to re-program:

1. Press programming button *P*.
2. Press programming button 1.
3. Press the programming button for the desired relay. The indicator lamp for the selected relay is lit.
4. Press the button on the IR transmitter for the channel that is to be programmed until the indicator lamp flashes three times.
5. Repeat step 1-4 to program the other relays in the same way.
6. Check the functions by pressing the selected buttons on the IR transmitter.

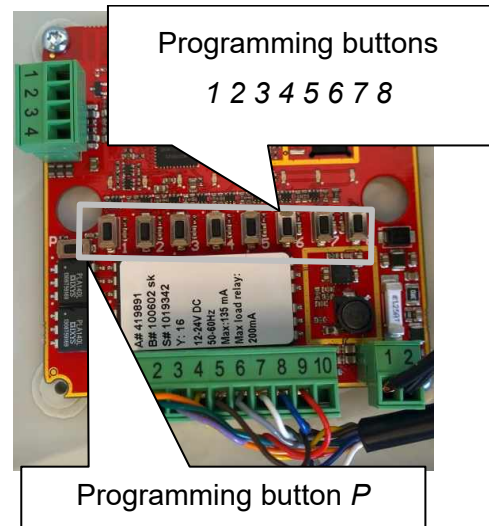If you are not satisfied with the selected channel (button on the IR transmitter) or function, simply repeat the procedure.

NOTE: The Andromeda IR-REC8 BED can only be used with monostable function.

Programming a relay with a second GewaLink channel

Andromeda IR-REC8 BED can be programmed so that a relay can be activated by two different GewaLink channels. This can be used for example to have a button “down” which lower the foot and head simultaneously. Then program the two relays with a second GewaLink channel that controls this function.

Do as follows to program a relay with an second GewaLink channel:

1. Press programming button *P*.
2. Press programming button 2.
3. Press the programming button for the desired relay. The indicator lamp for the selected relay is lit.
4. Press the button on the IR transmitter for the channel that is to be programmed as the second channel until the indicator lamp flashes three times.
5. Repeat step 1-4 to program the other relays in the same way.
6. Check the functions by pressing the selected buttons on the IR transmitter.

Programming a new Channel Group

The Andromeda IR-REC8 BED default setting of the receiver is channels 64-71, see the *function table* above. If another Channel Group is to be used, this can be programmed by setting the GewaLink channel with which the Channel Group shall begin with. Relay 1 gets the set GewaLink channel and Relay 2-8 gets the following 7 GewaLink channels.

Example: If GewaLink channel 16 has been selected, the receiver will have GewaLink channels 16-23 on relay 1-8.

Do as follows to program a new Channel Group:

1. Press programming button *P*.
2. Press programming button 3.
3. Press the button on the IR transmitter for the channel that is to be programmed as the first channel of the Channel Group until the indicator lamp flashes three times.
4. Check the functions by pressing the corresponding buttons on the IR transmitter.

Setting maximum activation time for relays

The Andromeda IR-REC8 BED has a built in safety function where the relay has a maximum activation time of 3 seconds. This time can be changed to 1.5 seconds. The safety function allows the relay to operate only for the set time even if the button on the transmitter is kept pressed for a longer time.

Do as follows to program maximum activation time:

1. Press programming button *P*.
2. Press programming button 4.
3. Press programming button 1 for 3 seconds
or
Press programming button 2 for 1.5 seconds
The indication lamp flashes three times.
4. Check the function by pressing a button on the IR transmitter and keep it pressed to see that the desired maximum activation time is obtained.

Default setting

The Andromeda IR-REC8 BED comes preset with monostable functioning and GewaLink channels 64-71 on relay 1-8.

Resetting to default setting

To restore the receiver to the default setting, press and hold programming button *P* for 5 seconds until the indicator lamp flashes, and then press programming button *P* again within 2 seconds when the indicator lamp remains lit.

Resetting a single relay

To restore a selected relay to the default setting, press and hold the programming button *P* for 5 seconds until the indicator lamp flashes. Then, press the programming button for the relay to be reset within 2 seconds when the indicator lamp remains lit.

Cleaning

The receivers outer casing can be cleaned with a damp cloth dipped in water with a little detergent.

Product specifications

Receiver Andromeda IR-REC8 BED

Voltage range:	12-24 V DC
Power consumption, idle.	0,15 W
Power consumption, max:	1,6 W
Max load per relay:	200 mA/30 V AC, 350 mA/60 V DC
Programming channels:	GewaLink channels 0-61, 63-127 and 4096 code. Delivered with GewaLink channels 64-71.
Ingress protection:	IP 54
Temperature range:	0° to +40°C
Operating range:	Indoors. Up to 2000 m above sea level.
Storage and transport:	-20° to +60°C, relative humidity 80% non-condensing.
Estimated lifetime:	5 years

Power Adapter MEAN WELL GEM18I AC-DC Adaptor

Input:	Voltage range:	80-264 V AC
	Frequency range:	47 – 63 Hz
Output:	DC voltage:	12 V
	Rated current:	1.5 A (1.35A at 100 V AC)

Relay configuration 10-pin terminal

1	Output relay 1
2	Output relay 2
3	Output relay 3
4	Output relay 4
5	Output relay 5
6	Output relay 6
7	Output relay 7
8	Output relay 8
9	Common input
10	No function

Accessories/Included parts

Art. no.	Name
419891	Gewa Andromeda IR-REC8 BED
419892	Gewa Andromeda IR-REC8 BED (LINAК 10 PIN MODULAR)
419893	Gewa Andromeda IR-REC8 BED (LINAК 8 PIN DIN)
419894	Gewa Andromeda IR-REC8 BED (DEWERT 5 PIN DIN)
419895	Gewa Andromeda IR-REC8 BED (DEWERT 13 PIN DIN)
419896	Cable kit (LINAК 10 PIN MODULAR)
419897	Cable kit (LINAК 8 PIN DIN)
419898	Cable kit (DEWERT 5 PIN DIN)
419899	Cable kit (DEWERT 13 PIN DIN)
419889	Power Adapter Gewa Andromeda IR-REC8 BED

Storage and disposal

Storage

Keep the product indoors at room temperature and away from moist.

Recycling Waste Electrical and Electronic Equipment

The product contains electronic components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Important information

Before you use the product, read and understand the contents of this user manual.

Safety definitions

 **WARNING:** Risk of injury or death if the instructions are not followed.

 **CAUTION:** Risk of damage to the product if the instructions are not followed.

NOTE: Information that is necessary in a given situation.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Conformance with requirements for medical devices

Andromeda IR-REC8 BED is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingham Business Park, Swavesey,