# Control 18

IR-sändare / IR transmitter / IR-sender

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Allvarliga incidenter .....	25
Överensstämmelse med krav för medicintekniska produkter .....	25
Produktansvar.....	25
<b>EN: User manual Control 18.....</b>	<b>26</b>
<b>Introduction .....</b>	<b>26</b>
Intended purpose.....	26
About this document.....	26
<b>Overview of Control 18 .....</b>	<b>27</b>
Explanation of symbols.....	28
<b>Get started with Control 18 .....</b>	<b>29</b>
Insert batteries.....	29
Mount Control 18.....	29
<b>Use Control 18.....</b>	<b>30</b>
Levels in Control 18.....	30
Lock or unlock the settings lock.....	31
Open or close a level.....	31
Record IR channels from another transmitter .....	32
Reset a button or a level.....	33
Reset to factory setting.....	33
Lock for transport.....	34
<b>Customise Control 18 .....</b>	<b>35</b>
Label buttons .....	35
Add a 4096-code .....	35
Copy a button .....	36
Set beep volume.....	37
Setting an acceptance delay .....	37
Create an automatic sequence (Macro) .....	38
Direct choice of level .....	38
Set the channel range .....	39
<b>Summary and quick guide Control 18.....</b>	<b>41</b>
<b>Maintenance.....</b>	<b>43</b>
Cleaning.....	43
<b>Storage and disposal .....</b>	<b>43</b>
Storage .....	43
Disposal.....	43
<b>Product specifications.....</b>	<b>44</b>
Accessories included with your Control 18.....	44
Other accessories available for Control 18 .....	44
<b>Important information .....</b>	<b>45</b>

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# EN: User manual Control 18

## Introduction

Control 18 is a programmable IR transmitter that replaces ordinary remote controls for TV, DVD, stereo etc. It has in all 81 different programming positions on five levels. Simply press the buttons to send signals, and change level by pressing the *Level*-button.

The Control 18 is a part of the “Gewa Smart House” concept and therefore can be used together with all of these products.

## Intended purpose

Control 18 is a simple to use remote control intended to support people who are unable to use a standard remote control due to motor, cognitive or visual impairments. The device can help the user control electronic equipment at home, including TV sets, lighting, and entrance and garage door openers. The device is intended for indoor and outdoor use.

## About this document

 Read section *Important information* before using the product.

This user manual describes how to set up and use the product. The manuals intended for technicians and support persons who will configure the product and educate users on how to use it.

## Overview of Control 18

This picture shows the various terms and functions which are described in the text.

Fig. 1

1. Indication lights indicating different levels, Level 1-5
2. *Functions*-buttons  
0-9, \*, #, A, B, C, D
3. *Code*-button
4. Indication lamp when programming
5. *Level*-button /  
*Function*-button
6. Programmingsbutton/Reset

To be able to use your Control 18 there are certain things that you should know:

- The Control 18 is delivered preset with GewaLink channels, meaning that it is possible to use the transmitter immediately after the batteries has been inserted. See section *Inserting Batteries*.
- To enter the programming mode, press the Programming button using a paper clip, for example, at the same time as you choose a *Function*-button. Different *Function*-buttons are chosen depending on what is to be programmed.
- If you press the wrong button or make a mistake when programming you can always press the Programming button for two seconds. It works as a *Reset*-button.
- It is important not to wait too long between the different steps when programming. After a certain time Control 18 automatically leaves the programming state to save power.
- If a button on your Control 18 is pressed for more than 25 seconds, your Control 18 will stop transmitting to save power.
- The settings can be protected by using the Settings Lock.
- You can only change the settings of levels that are unlocked.

On delivery your Control 18 will be set with certain default settings as follows:

- Acceptance delay is set to value 0.
- Feedback beep volume is set to value 4 (0 – 9).
- Only level 2 is open, other levels are closed.
- GewaLink channels 00-79 are on level 1-5.
- A random code on the -button.

## Explanation of symbols

The following symbols appear in the text throughout this manual:

	Programming button
	Any button on your Control 18
	Any button on another transmitter to be programmed
	Button 0-9, *, # on Control 18
	Button A B C D on Control 18
	<i>Code</i> -button
	<i>Level</i> -button
 <sup>2s</sup>	<i>Level</i> -button held pressed for approximately 2 seconds

# Get started with Control 18


## Insert batteries

Unscrew the battery lid screw on the back of Control 18 using a screwdriver and insert four alkaline LR03 (AAA) 1.5 V batteries. Insert the batteries according to the marking at the bottom of the battery compartment. Pay attention to the polarity. It is important to use long life alkaline batteries that are protected against leaking battery acid. When you have inserted the new batteries, **press P**.

**NOTE:** During battery replacement: Do not press any button. The transmitter may lose recorded channels.

## Battery warning

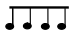Control 18 has a built-in battery warning  indicating low battery voltage. A sound signal with short beeps is heard at an interval of 30 seconds. After a while it starts warning every ten minutes. Replace the batteries as soon as possible.

## Mount Control 18

**⚠ WARNING:** The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

Rubber feet are mounted on the underside so that your Control 18 can lie still on flat surfaces. In some cases Control 18 needs to be permanently mounted. As an additional accessory, there is a holder, which can be secured to Control 18. This holder can in its turn be mounted on a table with a 'G' clamp or attached to a tube with a 'U' clamp. The holder can also be mounted on a mobile stand.

Another option is to hang your Control 18 around the neck. A neck strap is supplied with your Control 18.

# Use Control 18

## Levels in Control 18

Your Control 18 has five levels in total with 17 programmable buttons. On delivery the buttons are factory-programmed with GewaLink channels 00-79 on level 1-5. You can freely use one, several or all five levels. Five indication lamps indicate the current level (see Fig 1).

Your Control 18 is delivered with only level 2 open, other levels are closed. To open further levels, see section *Opening/closing a Level*.

When only one level is open, the *Level*-button  automatically contains one channel which is always is the first channel on the next level. In this way, it is possible to control 18 functions even if only one level is open. See <sup>1</sup> in Table 1.

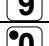

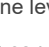
When several levels are active you can change to another level by pressing the *Level*-button . Each time you press the button the next level is chosen.

Table 1 shows the channels and codes for Control 18 on delivery.

**Table 1: Default setting GewaLink channels**

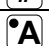GewaLink channels position					
Button on Control 18	Level 1	Level 2	Level 3	Level 4	Level 5
	00	16	32	48	64
	01	17	33	49	65
	02	18	34	50	66
	03	19	35	51	67
	04	20	36	52	68
	05	21	37	53	69
	06	22	38	54	70
	07	23	39	55	71
	08	24	40	56	72
	09	25	41	57	73
	10	26	42	58	74
	11	27	43	59	75
	12	28	44	60	76
	13	29	45	61	77
	14	30	46	<sup>2</sup> 62	78
	15	31	47	63	79
<sup>1</sup> 	16	32	48	64	00
	<b>Code</b>				

<sup>1</sup> If only one level is open the *Level*-button automatically becomes one channel. It is the first channel on the next level.

<sup>2</sup> Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

## Lock or unlock the settings lock

To protect channels/codes from being erased by accident when new codes are programmed it is possible to lock each level separately. All levels are unlocked on delivery.

A locked level is indicated by a beep  when you try to program it.

Do as follows to lock or unlock the settings lock:

1. Press **P** and  at the same time. The indication lamp for the *Level*-button and the current level are now lit.
2. Change to the level to be locked/unlocked by pressing .
3. Press button  to lock or button  to unlock.  
The indication lamp for the levels is automatically moved to the next level, which thereby can be locked/unlocked immediately.
4. Press <sup>2s</sup> for approximately 2 seconds until the indication lamp for the *Level*-button flashes green and goes out.

### Lock or unlock the settings lock

Press **P** + , choose level with , then press your desired option  or  then <sup>2s</sup>.

 = Lock.  = Unlock.

*Default setting: Unlocked.*

## Open or close a level

Your Control 18 has five different levels, which means that each button can have five different functions. You can select the number of levels that are open and hence, how many functions each button should have. You should close levels that are not to be used.

1. Press **P** and  at the same time. The indication lamp for the *Level*-button and the current level are now lit.
2. Press  a number of times until the level to be opened or closed is reached.
3. Press  to open or  to close the chosen level.  
The level indication lamp is automatically moved to the next level, which thereby directly can be opened or closed.
4. Press <sup>2s</sup> for approximately 2 seconds until the indication lamp for the *Level*-button flashes green and goes out.

You can also open all levels at the same time, then press .

### Open or close a level

Press **P** + , choose level with , then press the desired option  or , then <sup>2s</sup>.

 = Open.  = Close.

*Default setting: Level 2 open.*

## Record IR channels from another transmitter

Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the rear of your Control 18, see picture.

1. Choose the level where the channels are to be recorded with . Only one level at a time can be programmed.
2. Press **P** and **1** at the same time. The indication lamp on button 1 and the present level now flashes.  
To check that the transmitters are in the right position, keep any button pressed on the original transmitter. The indication lamp for the *Level*-button now flashes green. Move the original transmitter to the left or to the right until the indication lamp goes out. Then move the original transmitter backwards until the indication lamp goes out (one meter for certain transmitters). Place the original transmitter centrally between these positions. Certain types of transmitters only emit a short flash each time the button is pressed. In these cases, press several times.
3. Press and release the button **X** on Control 18, which is to be programmed. The indication lamp for the *Level*-button now emits a steady orange light.
4. Keep the corresponding button **Y** on the original transmitter pressed. Release the button when the indication lamp for the *Level*-button goes green. It might take up to 10 seconds.  
If the indication lamp goes orange again press the button one or more times until the light flashes green and goes out.  
If the indication lamp flashes red the programming has failed. Try once more from step 2.  
If a beep is heard the level is locked and no programming can be made. See section *Locking/Unlocking the Settings Lock* how to unlock a level.
5. If the programming of the first button was successful, repeat steps 3 and 4 until all buttons are programmed. Finish by pressing .
6. The light for the *Level*-button flashes green to indicate that the programming is complete.
7. Now test the programmed functions on the device which is to be controlled.

Original transmitter

### Recording IR channels

Press **P** + **1**, select button **X**, select button **Y**, then continue or finish with .

**NOTE:** When just one level is used, the *Level*-button automatically becomes the next levels first button, see *Table 1*. It means that it is possible to record an IR channel on the *Level*-button, but only when just one level is being used.

**TIP:** Control 18 can record codes from most IR transmitters on the market with a few exceptions. You may have problems with a few or all of the functions for a certain IR transmitter. In these cases it is possible to try another recording method. Instead, do as follows in step 2; Press **P** and **1** at the same time but keep them pressed for approximately 2 seconds until the indication lamp for the *Level*-button flashes green and goes out. Control 18 is now in another recording mode. Otherwise use the same recording procedure as above.

## Reset a button or a level

Control 18 makes it possible to reset either one button or all buttons on a level. When this is done the button or buttons return to earlier chosen channel setting. Other settings are not affected. The level where the button/s is/are placed must be unlocked before resetting is possible. Read the section *Locking/Unlocking the settings lock*.

Do as follows to reset:

1. Unlock the level where the button/s is to be reset. Use the *Level*-button to choose the level for which you want to perform the reset operation.
2. Press **P** and **3** at the same time. The indication lamp for button 3 and the current level are now lit.
3. Resetting a single button.  
Press the button **X** to reset.
4. Resetting all buttons for a single level.  
Press in turn the buttons, **---**, **1**, **2**, **3**, **---**.
5. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

## Reset to factory setting

Control 18 makes it possible to reset all buttons on levels 1-5 at the same time. All buttons are returned to the factory settings which means that they are replaced by the GewaLink channels 00-79. Other settings will also return to default settings, see above.

1. Press **P** and **3** at the same time. The indication lamp on button 3 and the current level are now lit.
2. Press in turn the buttons **!**, **1**, **2**, **3**, **---**.
3. The indication lamps for levels 1-5 flashes red in turn. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

### Resetting to default setting

To reset one button:

Choose level, **P** + **3**, choose button **X**.

To reset one level:

Choose level, **P** + **3**, **---**, **1**, **2**, **3**, **---**.

To reset to factory settings:

Press **P** + **3**, **!**, **1**, **2**, **3**, **---**.

## Lock for transport

When moving the Control 18 between different places it is possible to lock the buttons to prevent Control 18 from starting by accident and avoid unnecessary power consumption.

Do as follows to lock or unlock:

- Press **P** and **0** at the same time to lock. The indication lamp for the *Level*-button flashes.
- To unlock just press **P**.

### Lock for transport

Locking for transport: **P** + **0**

Unlock: **P**

# Customise Control 18

## Label buttons

The buttons on your Control 18 are marked with numbers as standard. The numbered label can be replaced with a personalised design. On Abilia's website it is possible to create your own button labels using pre-designed templates with symbols and text.

The document can also be downloaded. The labeling is printed on a clear sheet or white paper and inserted through the slit on your Control 18. Before the button label is inserted, the keyguard must be removed.

A plastic leg at each corner holds the keyguard. Put a knife blade between the keyguard and Control 18. Remove the keyguard carefully and smoothly, otherwise the plastic legs may break.

Two A4 sheet for making notes is included with your Control 18.

## Add a 4096-code

Control 18 is delivered with a special coded channel (4096) on the *Code*-button . The code is selected randomly. This coded channel is normally used for functions that require strict authorization such as apartment or house doors. The code has 4096 different combinations and has always the same code irrespective of which level you use. If you have to program a new coded channel there are three methods of doing so.

**Method one:** record the code from another transmitter, as described under *Recording IR channels from another Transmitter*. By this method it's possible to program a coded channel on any button independently of level.

**Method two:** Enter a numerical value between 0 and 4095. The indicated code will always be associated with the *Code*-button . It is then possible to move the code from this button to other buttons (See under *Copying a button*). Then you can create a new coded channel if you wish. Do as follows:

1. Press  and  at the same time. The indicator lamp on the *Code*-button and the current level are now lit.
2. Enter a number from 0 to 4095.
3. Finish with .
4. The indication lamp for the *Level*-button flashes green to show that the operation is complete. Make a note of the chosen code for future usage.


### Add a 4096 code, method two

Press  + , press the desired code, press .

*Default setting: Random code.*

**Method three:** Enter a binary code. The binary code will always be associated with the *Code*-button  and it is the same for all levels. Make up your own binary code or use the code already programmed in an IR receiver or transmitter.

There are two types of IR receivers: an older type of receiver where you set the code with a 12-pole switch, or a newer type where the IR receiver is programmable. The principle is the same when it comes to creating the binary code. *Fig 3* shows a 12-pole switch. Switches 1, 4, 6 and 12 are in the “On” position, the others are in the “Off” position.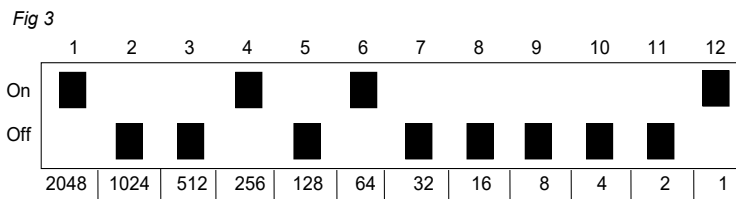

The following example describes how the binary code is created according to method three:

1. Press **P** and **!** at the same time. The indication lamp for the *Code*-button and the current level are now lit.
2. Enter the code by pressing on **---** or **!**.  
**---** = switch in Off position and **!** = switch in On position.  
 Press twelve times in turn; **!**, **---**, **---**, **!**, **---**, **!**, **---**, **---**, **---**, **---**, **---**, **!**, (*Fig 3*).
3. When you have pressed twelve times the light for the *Level*-button flashes green to show that the operation is finished.

The code is now programmed on the **!**-button. Make a note of the chosen code for future usage.

#### Adding a 4096 code, method three

Press **P** + **!**, press **---** or **!** 12 times.

**---** = Off position. **!** = On position.

*Default setting: Random code.*

## Copy a button

Using your Control 18, it is possible to copy a buttons' function to another button.

**NOTE:** The code/channel remains in its original place.

1. Choose level and press the button **X** with the code to be copied.
2. Choose the level to which you want to copy the code.
3. Press **P** and **8** at the same time. The indication lamp for button 8 and the current level are now lit.
4. Press the button **Y** to which you want to copy the code.
5. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

#### Copy a button

Copying code/channel from button **X** to button **Y**:

Choose level, choose **X**, choose level, **P** + **8**, choose **Y**.

## Set beep volume

Control 18 has a beep signal which sounds when a button is pressed. Set the beep volume in 10 stages (1 – 9).

**0** = no sound at all and **9** = maximum sound level.

Do as follows:

1. Press **P** and **4** at the same time. The indication lamp for button 4 and the current level are now lit.
2. Press a button from **0** - **9** to choose beep volume.
3. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

### Setting beep volume

Press **P** + **4**, choose the setting.

**0** = no sound at all and **9** = maximum sound level.

*Default setting: Beep volume = 4 (0 – 9).*

## Setting an acceptance delay

Control 18 makes it possible to set an acceptance delay. By acceptance delay is meant the time a button has to be pressed before a result is reached. The acceptance delay can be set in 10 steps (0 – 9).

**0** = no delay time and **9** = maximum delay time (0.9 second).

Do as follows:

1. Press **P** and **2** at the same time. The indication lamp for button 2 and the current level are now lit.
2. Press a button from **0** - **9** to choose delay time.
3. The indication lamp for the *Level*-button flashes green to show that the operation is complete.

### Setting an acceptance delay

Press **P** + **2**, choose setting.

**0** = no delay time and **9** = maximum delay time (0.9 second).

*Default setting: Acceptance delay = OFF (0 sec).*

## Create an automatic sequence (Macro)

Your Control 18 can send sequences of codes. When you press **one** button, Control 18 automatically carries out what is equivalent to pressing several buttons. One available application for instance is muting the TV at the same time as you answer the telephone.

Your Control 18 can also be programmed to change levels and it remembers how long a button should transmit. The transmission of a sequence can be terminated at any time by pressing another optional button. When a sequence is programmed your Control 18 transmits at the same time what is already on the button. Max. 25 seconds and 48 button presses per sequence can be programmed.

**NOTE:** The button used to store the sequence must not be part of the sequence.

1. Press **P** and **A** at the same time.
2. Press the buttons which should be used for the sequence.
3. Choose the level where you want the sequence to be.
4. Press **P** and **B** at the same time. The indication lamp for level B and the current level are now lit.
5. Choose the button **X** where you want the sequence to be stored.
6. The indication lamp for the *Level*-button flashes green to indicate that the operation is complete.

### Create an automatic sequence (Macro)

Press **P** + **A**, create the sequence, choose level, **P** + **B**, choose button **X**.

*Max 25 seconds per press, 48 presses.*

## Direct choice of level

Control 18 makes it possible to create quick buttons which means that you can reach a level by pressing a selected button. If you are at level 1 for instance and press button 2 you will directly reach level 2. And by pressing button 3 you will directly reach level 3 etc. When you are at level 2, 3, 4 or 5 you can press a new quick button to get back to level 1. You need to create the quick buttons for these levels in advance. See Fig 4.

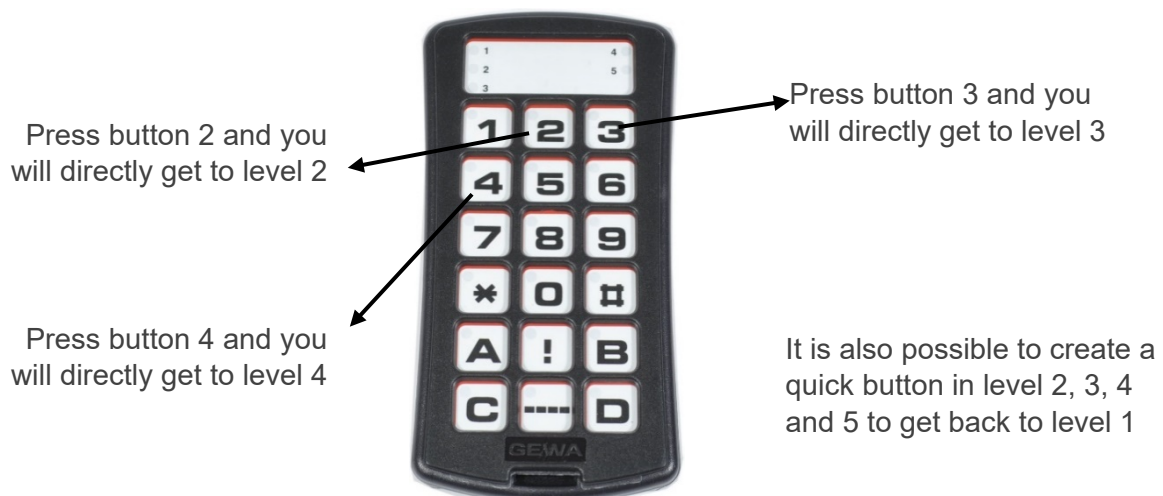
Fig 4: Level 1

Do as follows to create quick buttons:

1. Choose the level you want to reach with a quick button.
2. Press **P** and **A** at the same time.
3. Choose the level where the quick button is to be stored.
4. Press **P** and **B** at the same time. The indication lamp for button B and the current level are now lit.
5. Press the button **X** which is to be used as a quick button.
6. The indication lamp for the *Level*-button flashes green to indicate that the operation is complete.

### Direct choice of level

Create a quick button:

Choose level to be reached through direct choice, **P** + **A**, choose level where the quick button is to be stored, **P** + **B**, choose button **X** which is going to be a quick button.

### Set the channel range

Your Control 18 has the option to set different channel ranges for the transmitter to operate within. When the change of channel range is applied, programmed channels will disappear and a new random code is added on the *Code*-button **!**.

Normally, the channel range does not need to be changed. If several GewaLink transmitters are to be used in the same area and you do NOT want to control the same equipment, it may be necessary to use different channel ranges.

Table 2 shows the different settings.

**Table 2: Channel range**

	Channel range, GewaLink channels	
	00-79 <b>1</b>	64-127 <b>2</b>
Level 1	00-15	64-79
Level 2	16-31	80-95
Level 3	32-47	96-111
Level 4	<sup>1</sup> 48-63	112-127
Level 5	64-79	<sup>1</sup> 62-62

<sup>1</sup> Channel 62 is a stop code and cannot control any function. On the other hand you can store channels/codes on this button which in turn can control a function.

Do as follows to set the channel range:

1. Press **P** and **---** at the same time. The indication lamp for the *Level*-button and the current level are now lit.
2. Press **!**.
3. Choose channel range **1** or **2** according to table 2 above.
4. The lights for levels 1-5 flashes red in turn. Then the light for the *Level*-button flashes green to indicate that the operation is complete.

### Set the channel range

Press **P** + **---**, **!**, choose the channel range.

**1** = 00-79, **2** = 64-127.

*Default setting: GewaLink channel 00-79.*

## Summary and quick guide Control 18

### Locking/Unlocking the settings lock

Press **P** + **---**, choose level with **---**, then press your desired option **3** or **4** then **---**<sup>2s</sup>.

**3** = Lock. **4** = Unlock.

*Default setting: Unlocked.*

### Opening/Closing a level

Press **P** + **---**, choose level with **---**, then press the desired option **1** or **2**, then **---**<sup>2s</sup>.

**1** = Open. **2** = Close.

*Default setting: Level 2 open.*

### Recording IR channels

Press **P** + **1**, select button **X**, select button **Y**, then continue or finish with **---**.

### Resetting to default setting

To reset one button:

Choose level, **P** + **3**, choose button **X**.

To reset one level:

Choose level, **P** + **3**, **---**, **1**, **2**, **3**, **---**.

### To reset to factory settings

Press **P** + **3**, **!**, **1**, **2**, **3**, **---**.

### Locking for transport

Locking for transport: **P** + **0**.

Unlock: **P**.

### Adding a 4096 code, method two

Press **P** + **!**, press the desired code, press **---**.

### Adding a 4096 code, method three

Press **P** + **!**, press **---** or **!** 12 times.

**---** = Off position. **!** = On position.

*Default setting: Random code.*

### Copying a button

Copying code/channel from button **X** to button **Y**:

Choose level, choose **X**, choose level, **P** + **8**, choose **Y**.

### Setting beep volume

Press **P** + **4**, choose the setting.

**0** = no sound at all and **9** = maximum sound level.

*Default setting: Beep Volume = 4 (0 – 9).*

### Setting an acceptance delay

Press **P** + **2**, choose setting.

**0** = no delay time and **9** = maximum delay time (0.9 second).

*Default setting: Acceptance delay = OFF (0 sec).*

### Creating an automatic sequence (Macro)

Press **P** + **A**, create the sequence, choose level, **P** + **B**, choose button **X**.

*Max 25 seconds per press, 48 presses.*

### Direct choice of level

Create a quick button:

Choose level to be reached through direct choice, **P** + **A**, choose level where the quick button is to be stored, **P** + **B**, choose button **X** which is going to be a quick button.

### Setting the channel range

Press **P** + **---**, **!**, choose the channel range.

**1** = 00-79, **2** = 64-127.

*Default setting: GewaLink channel 00-79.*

# Maintenance

## Cleaning

Your Control 18 can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

## Storage and disposal

### Storage

Keep the product indoors at room temperature and away from moist.

### Disposal

The product contains electrical components that must be recycled. Dispose the product at an applicable disposal location.

## Product specifications

Power consumption at rest	Approx. 12 $\mu$ A
Power consumption at transmitting	Approx. 35 mA (GewaLink channel)
Battery life (button use only)	Approx. 2.5 years (100 seconds transmitting per day with GewaLink channel)
Programming locations	81 places
Operating force buttons	< 160g
Range	20-30m (GewaLink channels)
EMC-standard emission	EN 61000-6-4:2001, EN 60601-1-1-2:2001
” immunity	EN 61000-6-2:2001, EN 60601-1-1-2:2001
Measurements (L x W x H)	134 x 70 x 28 mm (incl. rubber feet)
Weight (incl. batteries)	180g
Operating temperature (excl. batteries)	-20°C to +45°C
Estimated lifetime	5 years

## Accessories included with your Control 18

Art.no	Name	Description	Qty
425776	Keyboard Label	For labeling of buttons	1 pcs
-	Note sheet	A4-sheet for noting your programmed channels	2 psc
425720	Control 18 Keyguard	Keyguard for 18 buttons	1 pcs
890410	Neck strap		1 pcs
8375	Battery	LR03 1.5V	4 pcs

## Other accessories available for Control 18

Art.no	Name	Description
8105	Holder	To connect Control 18 to a mounting system

# Important information

Before you use the product, read and understand the contents of this user manual.

## Safety definitions

 **WARNING:** Risk of injury or death if the instructions are not followed.

 **CAUTION:** Risk of damage to the product if the instructions are not followed.

**NOTE:** Information that is necessary in a given situation.

**TIP:** Additional information that can improve the use of the product.

## Safety instructions before using Control 18

 **WARNING:** Read the following warning instructions before you use the product.

- The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

## Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

## Conformance with requirements for medical devices

Control 18 is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

## Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingway Business Park, Swavesey,