AIRGO S2 ELRULLSTOL

BRUKSANVISNING

Tack!

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IATA Dangerous Goods Regulations

60th Edition (English)

2.3 Dangerous Goods Carried by Passengers or Crew

2.3.2.4 Wheelchairs/Mobility Aids with Lithium Batteries

Lithium ion battery powered wheelchairs or other similar mobility aids for use by passengers whose mobility is restricted by either a disability, their health or age, or a temporary mobility problem (e.g. broken leg), subject to the following conditions:

- a) the batteries must be of a type which meets the requirements of each test in the UN Manual of Tests and Criteria, Part III, subsection 38.3;
- b) the operator must verify:
 1. the battery terminals are protected from short circuits, e.g. by being enclosed within a battery container;
 2. the battery is securely attached to the wheelchair or mobility aid (see 9.3.14.5 and Figure 9.3.C);
 3. electrical circuits have been inhibited.
- c) mobility aids must be secured against movement in the cargo hold and must be carried in a manner so that they are protected from being damaged by the movement of baggage, mail or other cargo;
- d) where a battery powered wheelchair or other similar mobility aid is specifically designed to allow its battery (ies) to be removed by the user (e.g. collapsible):
 1. the battery(ies) must be removed. The wheelchair/mobility aid may then be carried as checked baggage without restriction;
 2. the battery(ies) must be protected from short circuit by insulating the terminals (e.g. by taping over exposed terminals);
 3. the removed battery(ies) must be protected from damage (e.g.) by placing each battery in a protective pouch. The battery(ies) must be carried in the passenger cabin;
 4. removal of the battery(ies) from the device must be performed by following the instructions of the manufacturer or device owner;
 5. the battery must not exceed 300 Wh, or for a device that is fitted with two batteries required for operation, each battery must not exceed 160 Wh;
 6. a maximum of one spare battery not exceeding 300 Wh or two spares each not exceeding 160 Wh may be carried.
- e) the pilot in command must be informed of the location of the mobility aid with an installed battery or the location of the lithium battery when removed and carried in the cabin;
- f) it is recommended that passengers make advance arrangements with each operator.

Confirmation letter

This confirmation is in response to questions raised by flight operators regarding the classification of the electrical wheelchairs from Airgo powered by lithium ion batteries.

Airgo battery-powered wheelchairs used by passengers whose mobility is restricted due to disability, health or age, or temporary disabilities may be transported on the flight in checked luggage.

Classification:

Airgo wheelchairs are powered by lithium ion batteries and must be consigned under the entries **UN 3481** (Lithium ion batteries contained in equipment).

Transport of Lithium Ion Batteries:

This document confirms that Airgo wheelchairs meet all the demands made by IATA for transport by air of lithium batteries contained in equipment (UN 3481) as set out in the IATA Dangerous Goods Regulations (DGR) and the IATA lithium Battery Shipping Guidelines (LBSG).

Facts about Airgo electrical wheelchairs:

Lithium batteries class:	UN 3480 (Lithium ion batteries)
Electric potential:	24 volt
Watt-hour rating:	240 Wh
UN-tested battery:	YES (UN38.3)
Battery type:	Dry
Protected against short circuit:	YES
Isolated power circuits:	YES
Firmly mounted on equipment:	YES
Battery weight:	1,8 kg
Total weight of wheelchair excl battery:	25 kg
Wheelchair measures:	40 x 60 x 70 cm

Special rules for Mobility Aids:

Lithium-ion battery powered wheelchairs or other similar mobility aids for use by passengers whose mobility is restricted by either a disability, their health or age, or a temporary mobility problem (e.g. broken leg), subject to special rules according to **IATA Dangerous Goods Regulations 2.3.2.4** allowing the passenger to bring one battery not exceed 300 Wh and one spare battery not exceeding 300 Wh.

Declaration of LiON Batteries

The undersigned, representing the following company:

Unix Furniture AB
Orgnr: 5591110-2206
Päronvägen 1
SE-312 61 Mellbystrand

Herewith declares that the product:

LiON battery

Which is applicable to Airgo (brandname) power wheelchairs model Airgo S2 which is a LiON battery from Dongguan Bobes New Energy Technology Co LTd which is certified by IATA.

Mellbystrand, Sweden, 2021-09-15

A handwritten signature in blue ink, appearing to read 'Björn Bernsfelt', is written over a light blue horizontal line.

Björn Bernsfelt
CEO

This document confirms that the Lithium batteries included in the Airgo wheelchairs meet all the demands made by IATA for transport by air of lithium batteries regards to the test UN38.3.

Unix Furniture AB
Päronvägen 1
SE-312 61 Mellbystrand

info@airgo.se
www.airgo.se

+46 706 949 220

TEST REPORT UN38.3, Sixth Edition Recommendations on transport of dangerous goods, manual of test and criteria, Section 38.3 - Lithium metal and lithium ion Batteries	
Report Reference No.....:	19PNS080145 01001
Date of issue	2019-09-07
Total number of pages	13 pages
Testing Laboratory 检测单位	GUANGDONG UTL CO., LTD. 广东联鼎检测科技有限公司
Address	Lianding Testing Building, No.18 Center Road of Yayuan Industrial Zone, Nancheng District, Dongguan, Guangdong, China. 东莞市南城街道雅园工业区中心路18号联鼎检测大厦
Tested by (name + signature).....:	Sunny Yan/Project Engineer 鄢倩倩/项目工程师 
Reviewed by (name + signature).....:	Ivy Bi/Senior Engineer 毕春微/资深工程师 
Approved by (name + signature).....:	Free Zhao/Safety Manager 赵庆孩/安规部经理 
Applicant's name..... 申请者:	Guangzhou Siekom Rehabilitation Equipment Manufacturing Co., Ltd. 广州赛通康复设备制造有限公司
Address	1 Room 302, No.1 Building, No.63 Punan Road, Huangpu District, Guangzhou, China 广州市黄埔区浦南路63号一号厂房自编302号房
Manufacturer's name 生产厂	Dongguan Bobes New Energy Technology Co., Ltd. 东莞市博贝斯新能源科技有限公司
Address	2nd Floor, Building A2, 8th Courtyard, Shanxia Industrial Zone, Hengli Town, Dongguan City, China 东莞市横沥镇山厦工业区8号院A2栋二楼
Phone number/联系方式	+86-769-83737389
Email/邮件地址	1014861942@qq.com
Website/网址	https://bobeisi.1688.com
Test specification/测试规范	
Standard.....:	ST/SG/AC.10/11/Rev.6/Amend.1/Section 38.3
Test procedure	N/A
Non-standard test method.....:	N/A
Test item description/样品名称.....:	
	Li-ion Battery Pack / 锂离子电池组
Trade Mark/商标.....:	N/A
Model/Type reference/型号	18650 7S4P
Ratings/规格	24V, 10Ah, 240Wh

List of Attachments (including a total number of pages in each attachment):

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- Photos documentation (2 pages)
- 产品图片 (2页)

Summary of testing:

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Tests performed (name of test and test clause):

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The sample's status is good.

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The conditions of the batteries of samples No. b1# to b4# are at first cycle, in fully charged states.

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The conditions of the batteries of samples No. b5# to b8# are after twenty-fifth cycles ending in fully charged states.

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The conditions of the cells of samples No. c1# to c5# are at first cycle at 50% of the design rated capacity.

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The conditions of the cells of samples No. c6# to c10# are after twenty-fifth cycles ending at 50% of the design rated capacity.

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The conditions of the cells of samples No. c11# to c20# are at first cycle, in fully discharged states.

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The conditions of the cells of samples No. c21# to c30# are after twenty-fifth cycles ending in fully discharged states.

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The test results: Pass

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GUANGDONG UTL CO., LTD.

Liangding Testing Building, No.18 Center Road of Yayuan Industrial Zone, Nancheng District, Dongguan, Guangdong., China

Tel: +86-769-3893 3228 E-mail: sales@gdutl.com Website: http://www.gdutl.com

Test item particulars	
样品信息:	
Cell type	INR18650E 26
电芯型号	
Nominal Voltage of cell.....	3.7V
电芯额定电压	
Rated Capacity of cell.....	2600mAh
电芯额定容量	
Appearance.....	Black
颜色	黑色
Number of cell.....	28pcs (7S4P)
电芯数量	
Dimension(mm)	85.0mm(max) × 138.0mm(max) × 173.0mm(max)
尺寸	
Test case verdicts	
测试判定	
Test case does not apply to the test object	N/A
判定不适用于测试对象	
Test item does meet the requirement	P(Pass)
测试符合规定	
Test item does not meet the requirement.....	F(Fail)
测试不符合规定	
Testing 测试	
Date of receipt of test item	2019-08-18
接样日期	
Date(s) of performance of test	2019-08-19 ~ 2019-09-04
测试周期	
General remarks 备注	
This report shall not be reproduced, except in full, without the written approval of the testing laboratory. 除非全部复制, 未经本实验室书面批准不得部分复制。	
The test results presented in this report relate only to the item tested. 本报告的测试结果仅对送检样品负责。	
"(see remark #)" refers to a remark appended to the report. “(见注#)”指报告的备注。	
Throughout this report a point is used as the decimal separator. 本报告中以点代替小数点。	
According to the Standard, a single-cell battery (Battery Pack) is considered a "Cell" (Battery Cell) and shall be tested according to the testing requirements for "Cell". This testing included the samples of Battery Pack and Battery Cell as aforementioned. For testing details, please refer to Table of Test Conclusion and individual test record. 按照标准要求, 单电芯电池(电池包)被视作“电芯”(电池芯), 以“电芯”的要求进行测试, 本测试项目样品包含如前所述电池包和电池芯。有关测试详情, 请查阅测试结论表格及各单项测试记录。	

GUANGDONG UTL CO., LTD.

Lianding Testing Building, No.18 Center Road of Yayuan Industrial Zone, Nancheng District, Dongguan, Guangdong., China

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