# High-low:xo User Guide

Power version  
High-low:xo indoor: EN 12182  
High-low:xo outdoor: EN 12182 & 12183

2021-06-rev.1.0

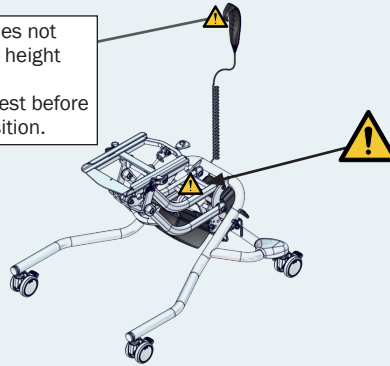
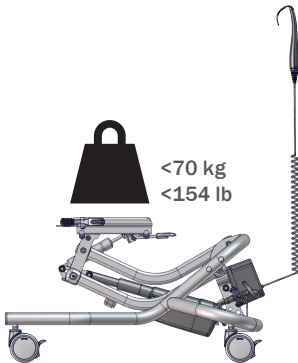
[www.etac.com](http://www.etac.com)

|   |       |                                  |                    |
|---|-------|----------------------------------|--------------------|
|    | 3     | ENGLISH<br>DEUTSCH               | 46-50<br>51-57     |
|    | 4     | NEDERLANDS<br>DANSK              | 58-62<br>63-67     |
|    | 5     | NORSK<br>SVENSKA                 | 68-72<br>73-77     |
|    | 6     | SUOMI<br>ESPAÑOL                 | 78-82<br>83-88     |
|    | 7-9   | FRANÇAIS<br>ITALIANO             | 89-93<br>94-98     |
|    | 10-13 | PORTUGUÊS<br>PORTUGUÊS DO BRASIL | 99-103<br>104-108  |
|    | 14-15 | РОССИЯ<br>POLSKI                 | 109-115<br>116-120 |
|    | 16-20 | ČESKY<br>中國                      | 121-125<br>126-132 |
|    | 21    | 日本<br>ΕΛΛΗΝΙΚΗ                   | 133-140<br>141-146 |
|    | 22-23 |                                  |                    |
|   | 24    |                                  |                    |
|  | 25-28 |                                  |                    |
|  | 29    |                                  |                    |
|  | 30-31 |                                  |                    |
|  | 32-36 |                                  |                    |
| mm (inch)   | 37-43 |                                  |                    |
|  | 44-45 |                                  |                    |



**High-low:xo - POWER**  05




The frame with power does not have a safety stop at the height adjustment function. Please remove the foot rest before adjusting into lowest position.

**SIZE 1****SIZE 2****SIZE 3**

**High-low:xo - POWER VERSION**  05

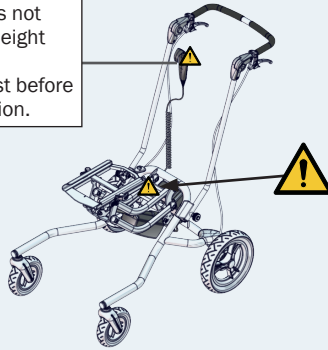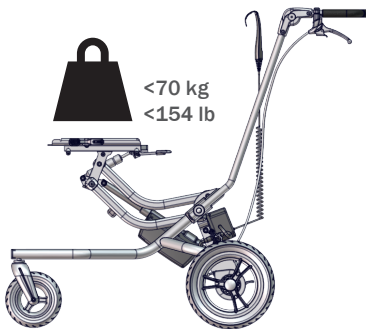
The frame with power does not have a safety stop at the height adjustment function. Please remove the foot rest before adjusting into lowest position.

**SIZE 1****SIZE 2**

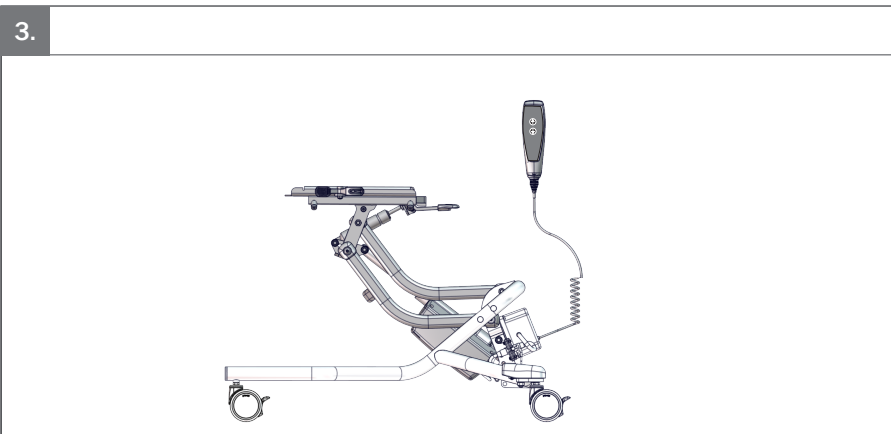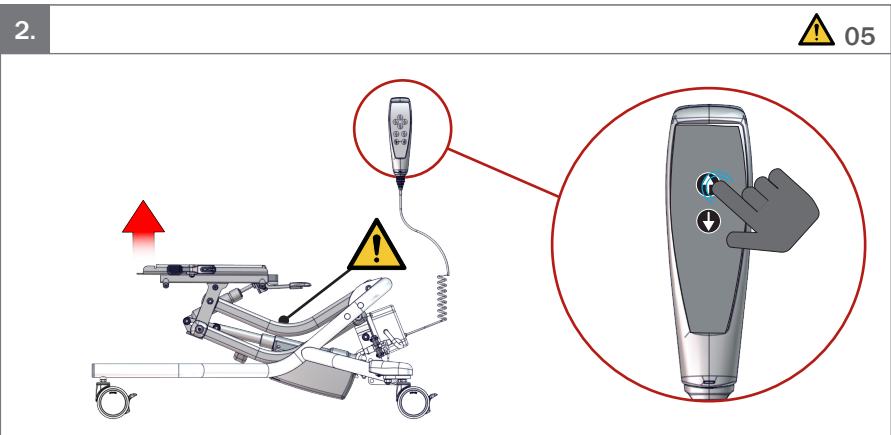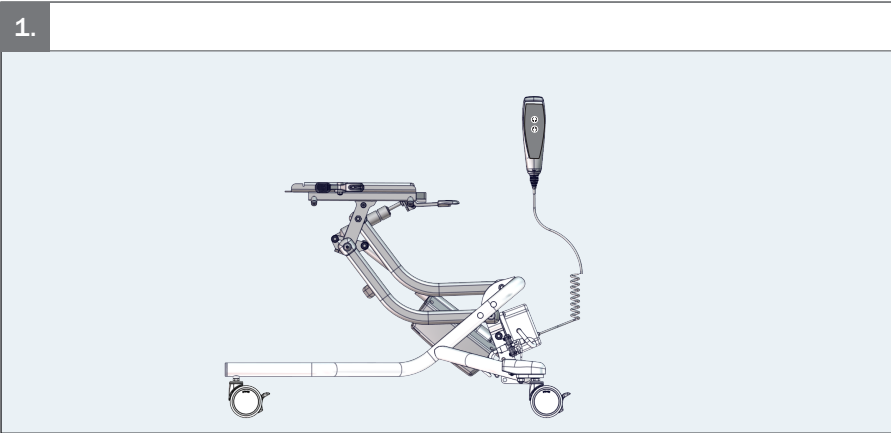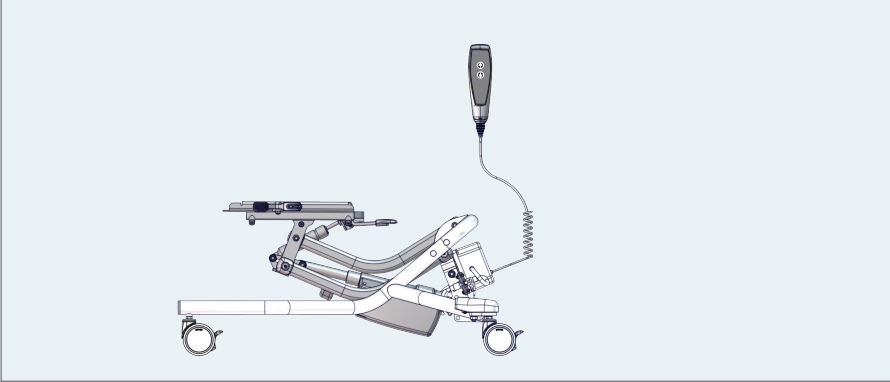
1.


2.

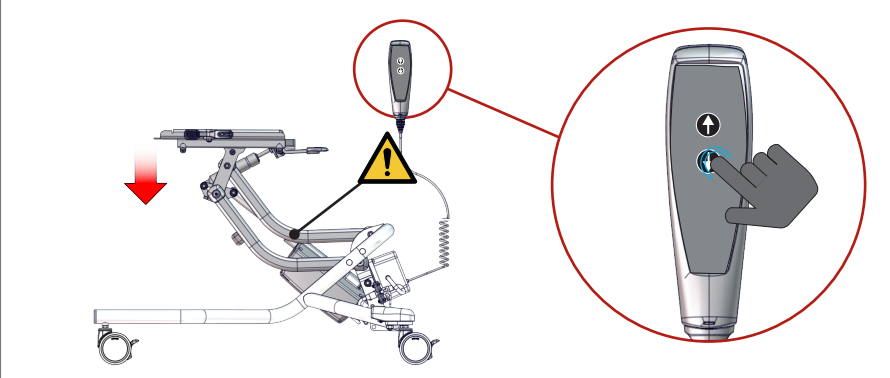 05

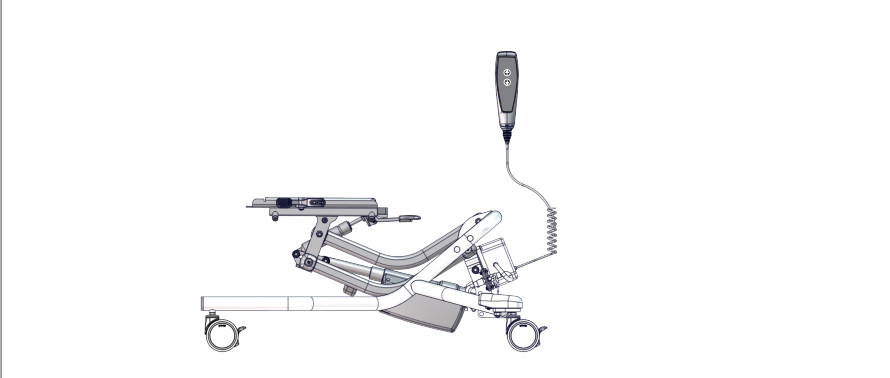
3.


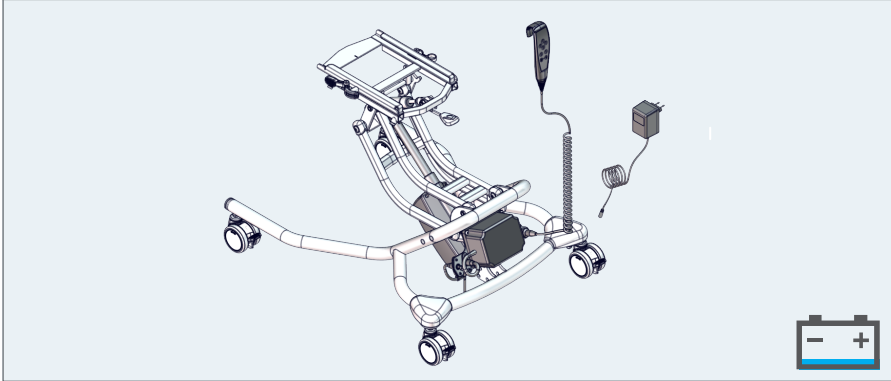
1.

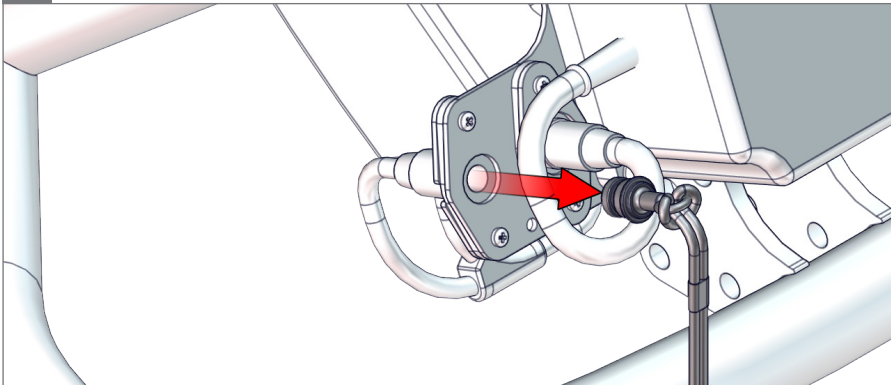
2.

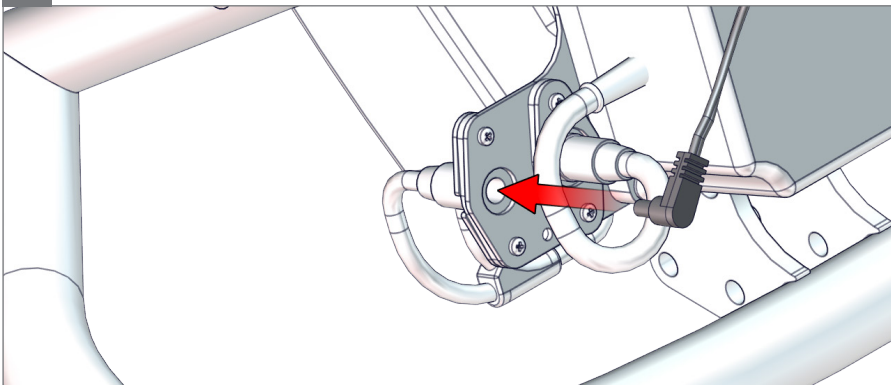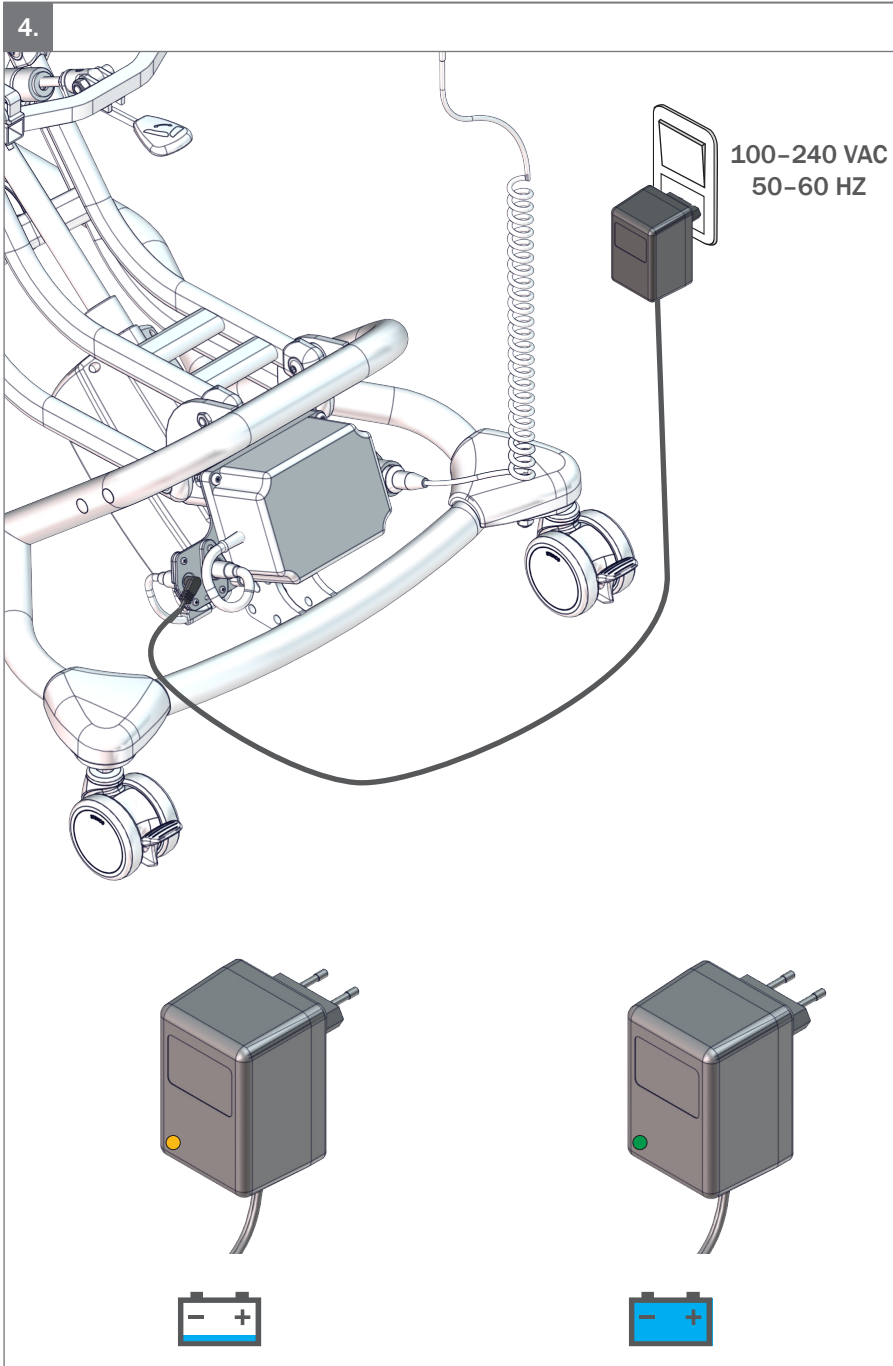3.



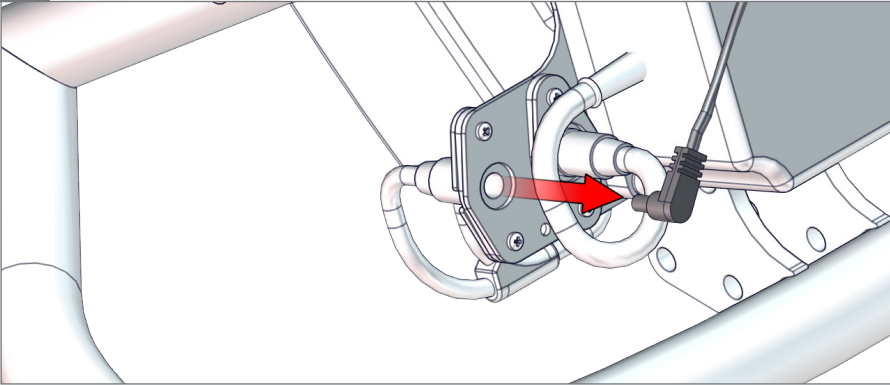
5.

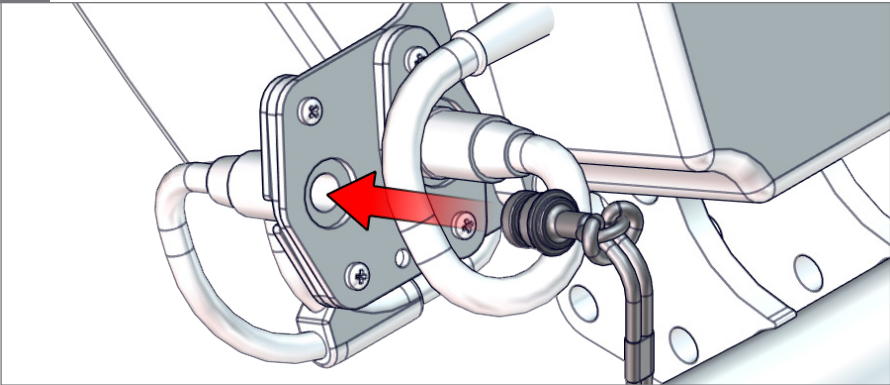6.

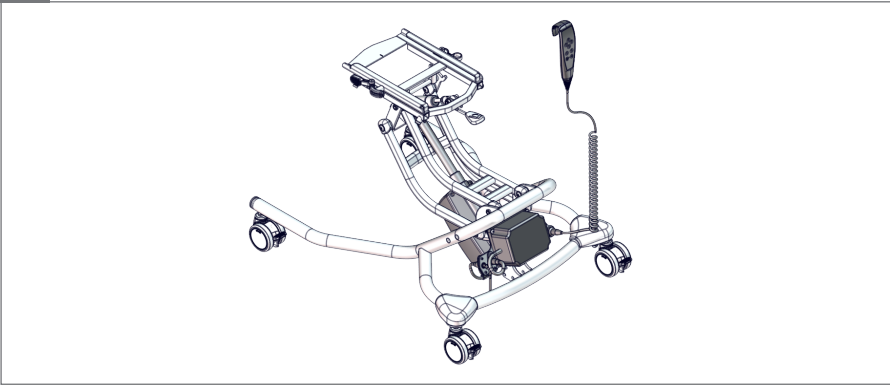
7.


1.

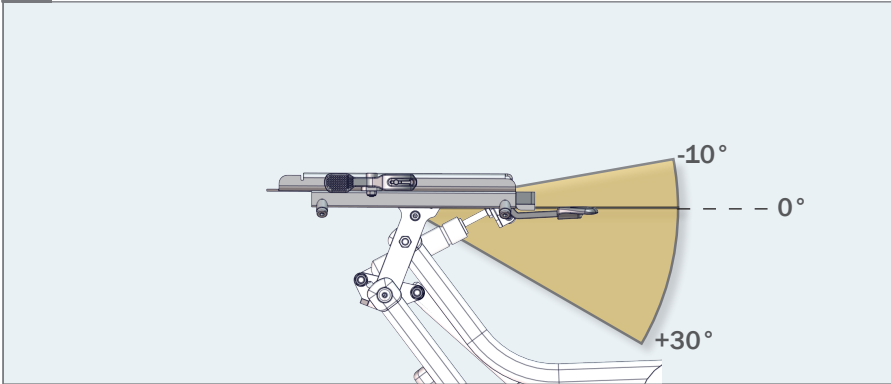
SIZE 1 AND 2

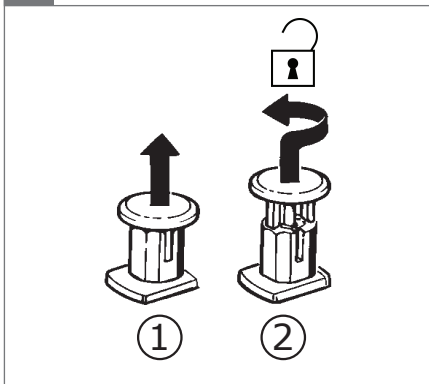
2.

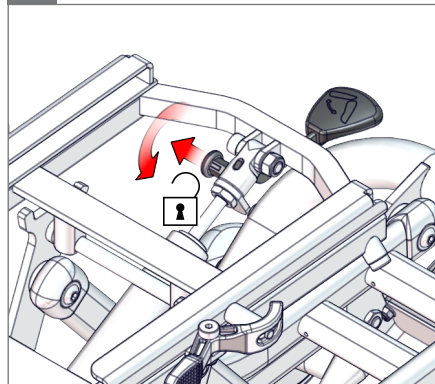3.

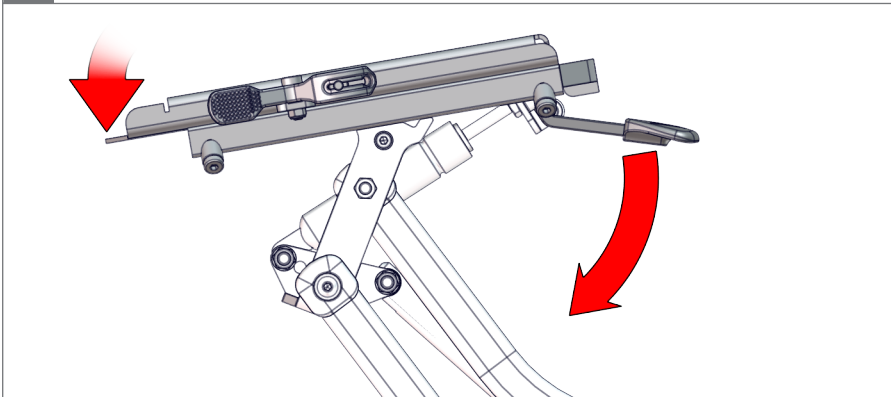
4.


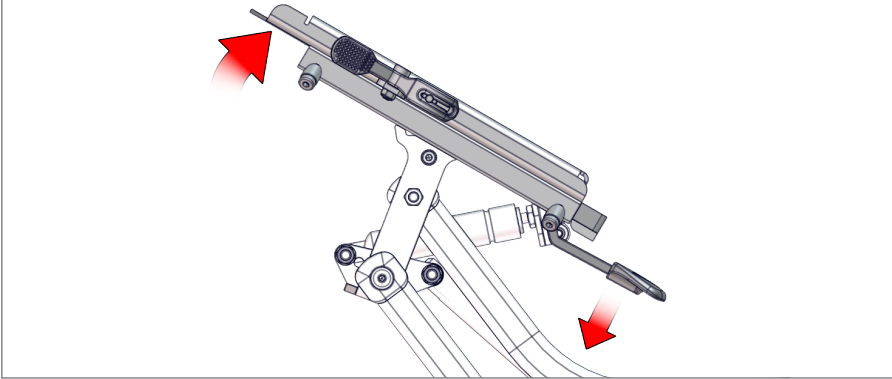
5.

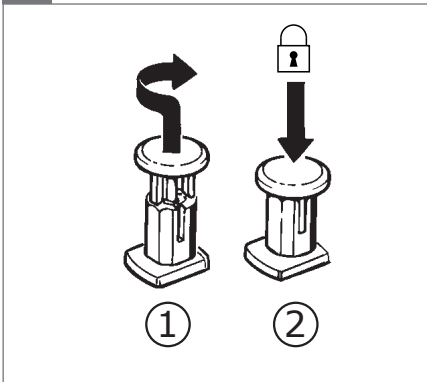6.

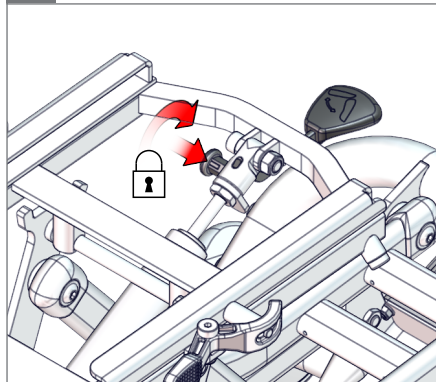
7.


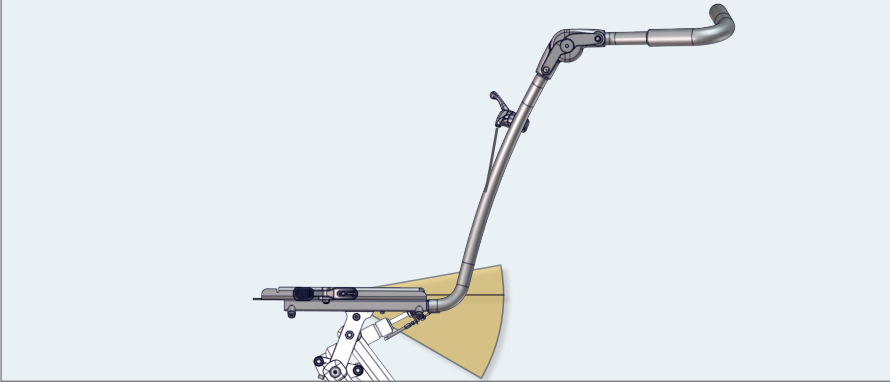
1.

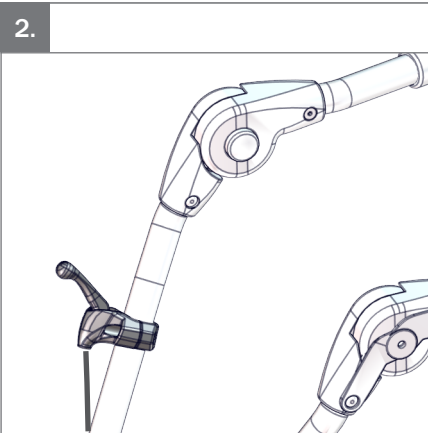
2.

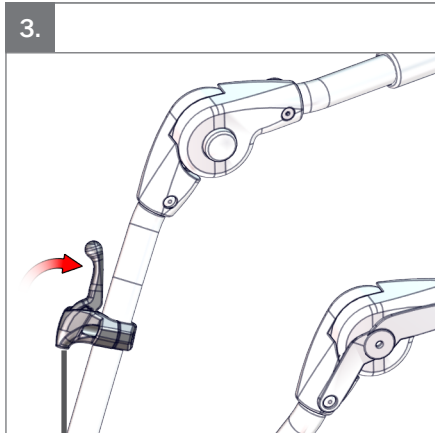3.

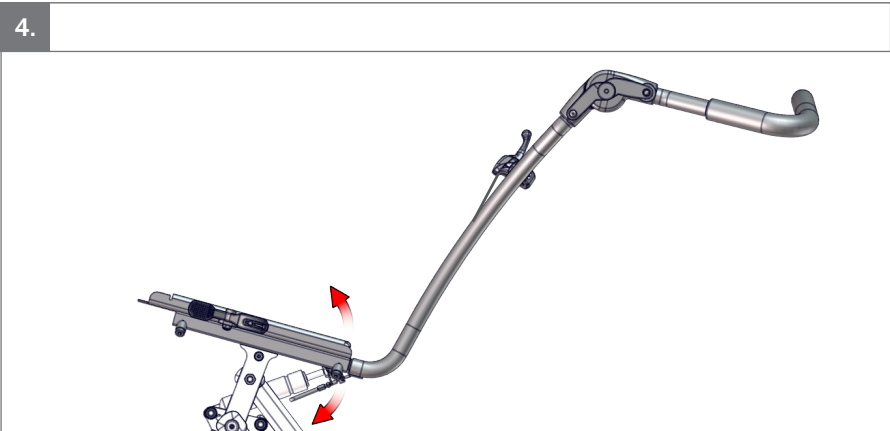
4.


1.

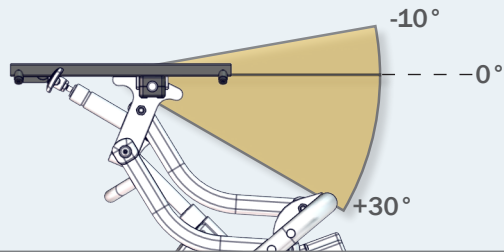
SIZE 3

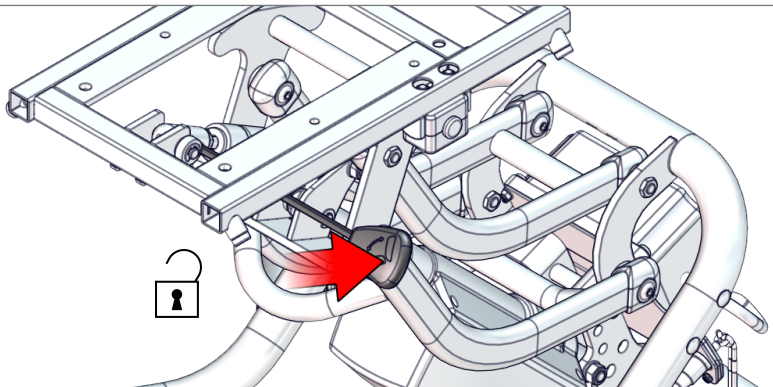2.

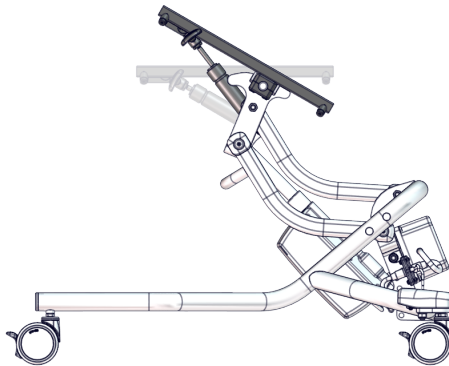
3.

RB2<sup>®</sup>


1.


### GAS SPRING ADJUSTMENT

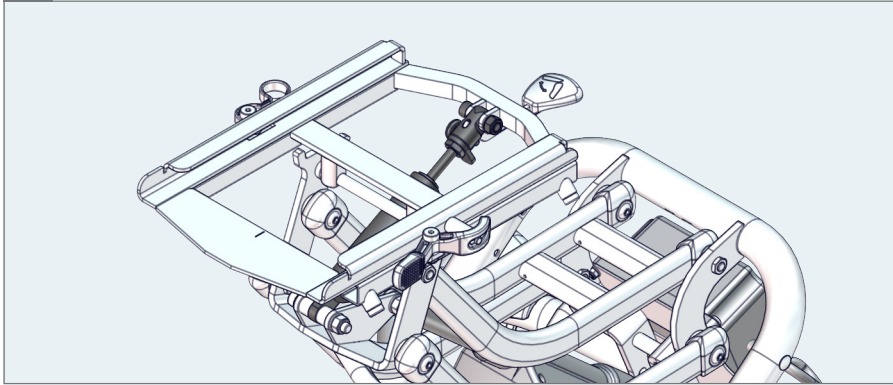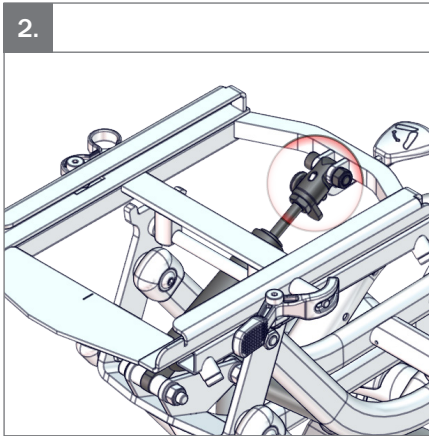
2.

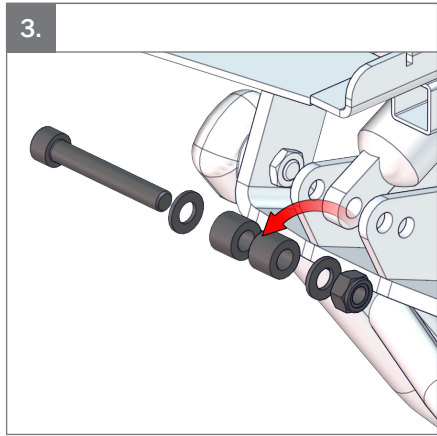3.

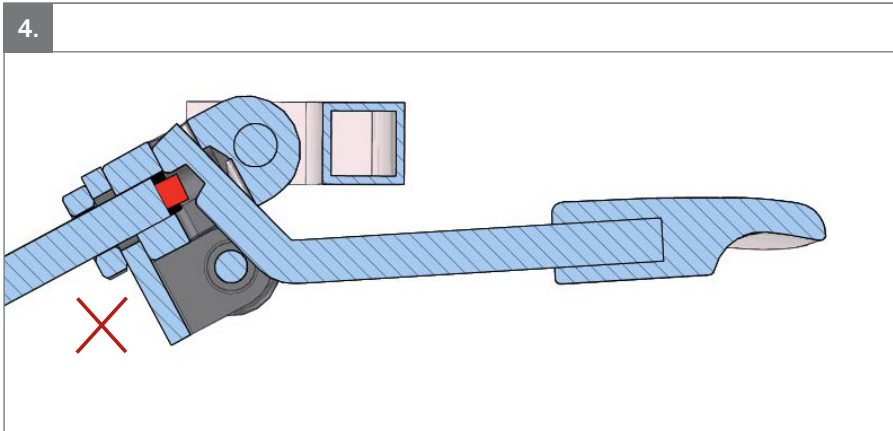
4.


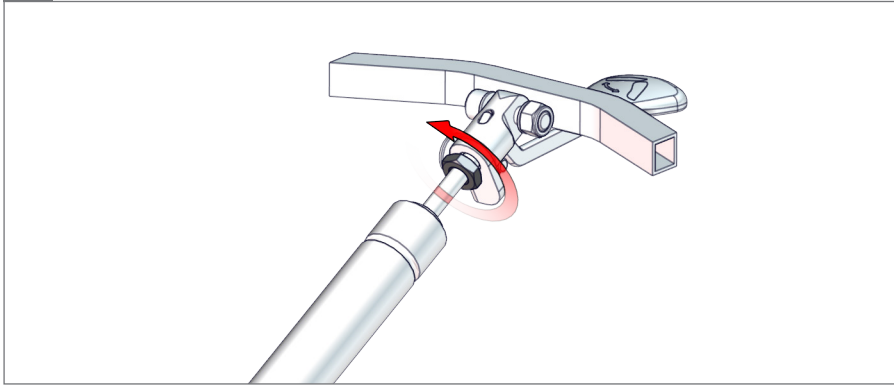
5.

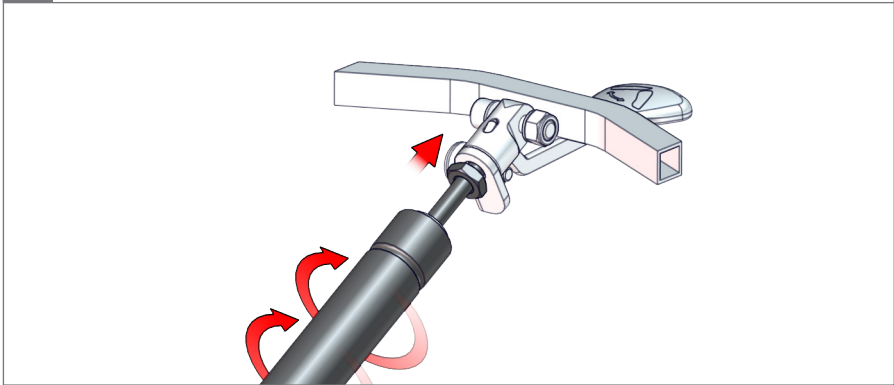6.

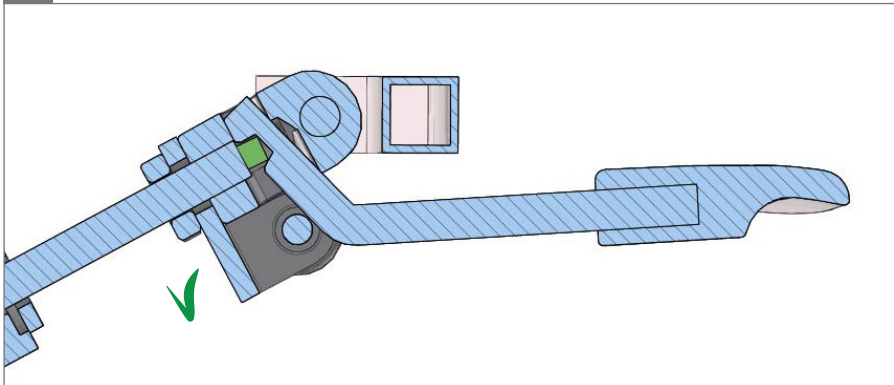
7.


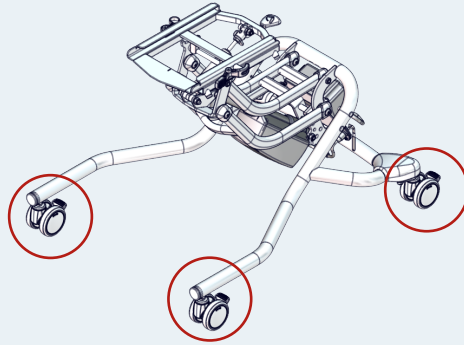
1.

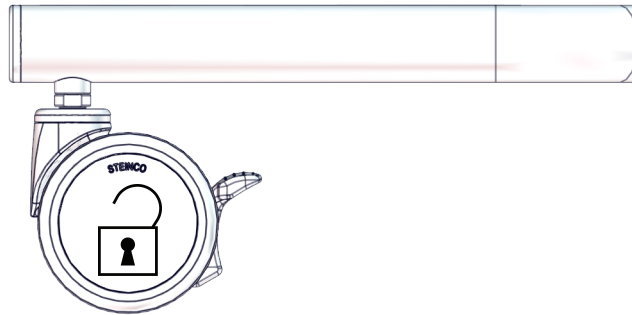2.

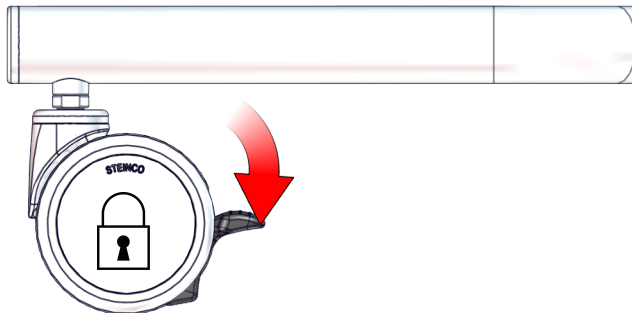
3.


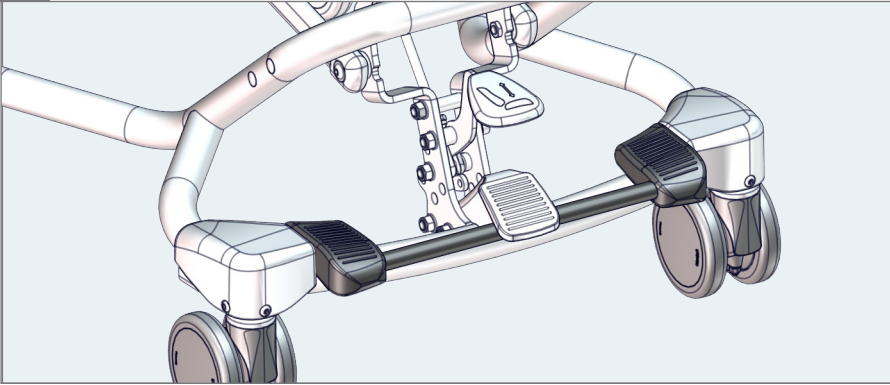
1.

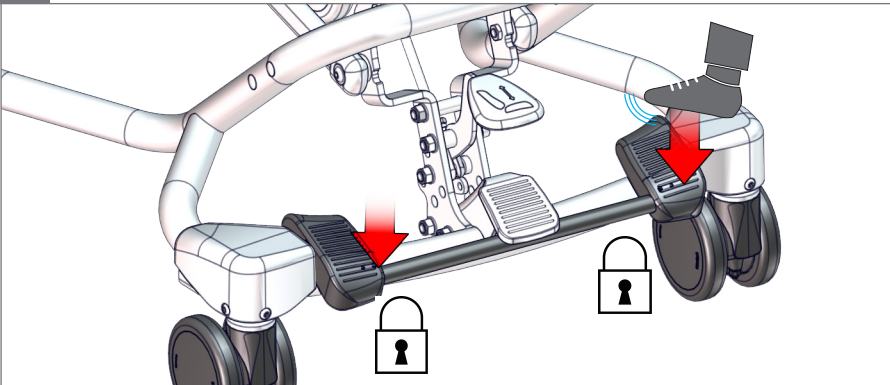2.

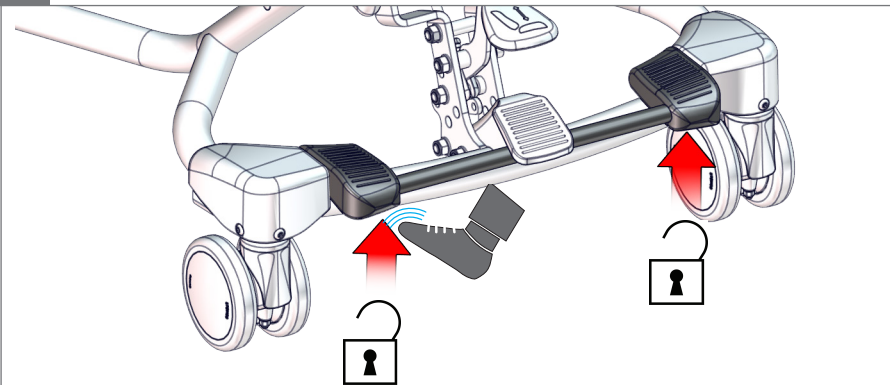
3.


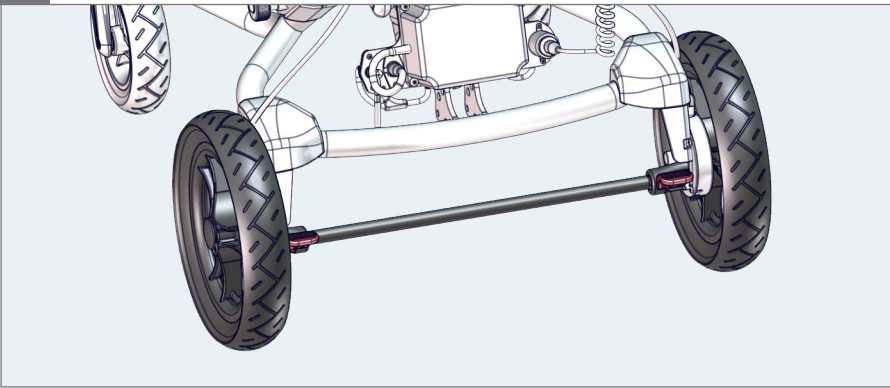
1.

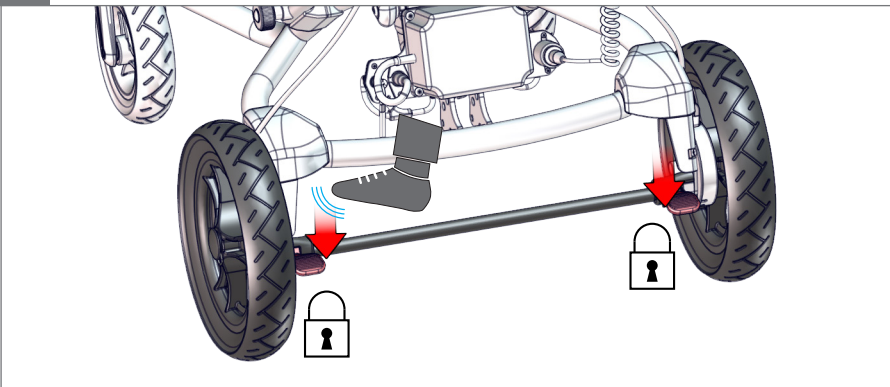2.

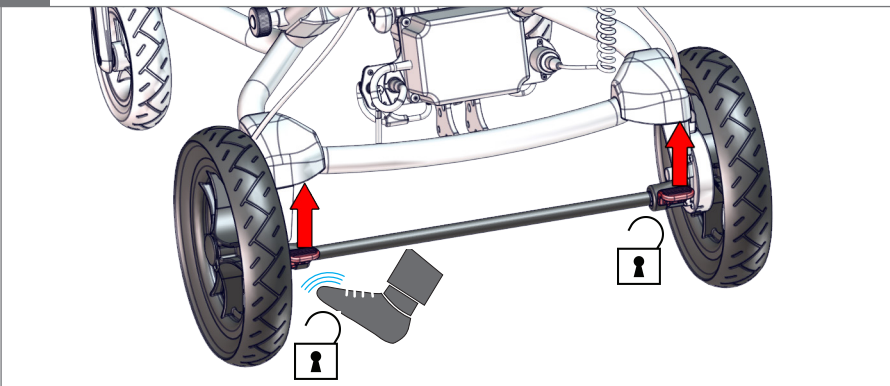
3.


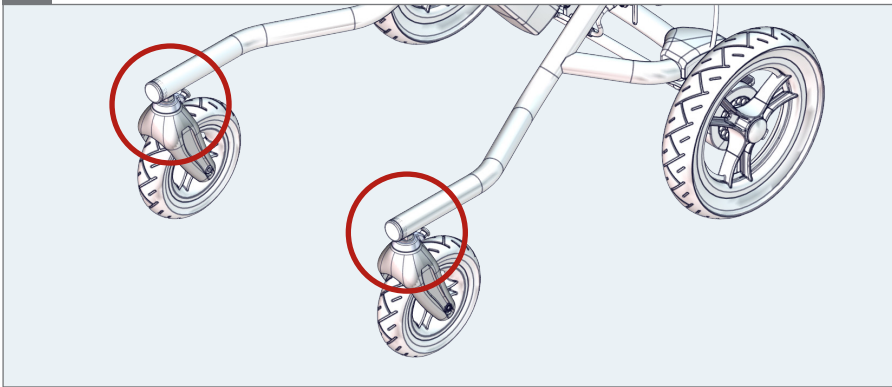
1.

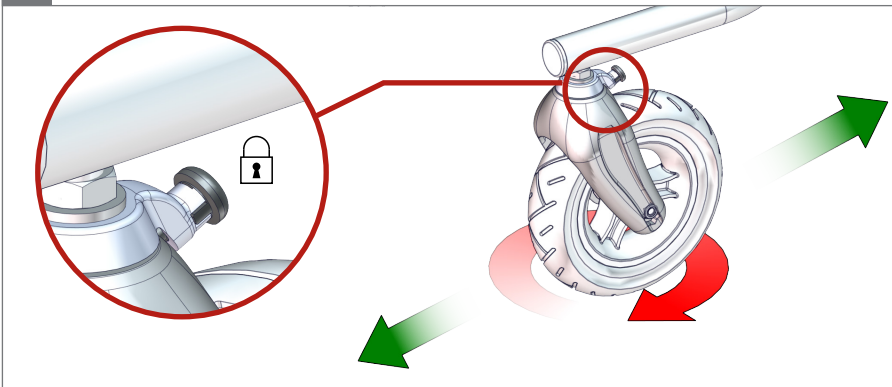2.

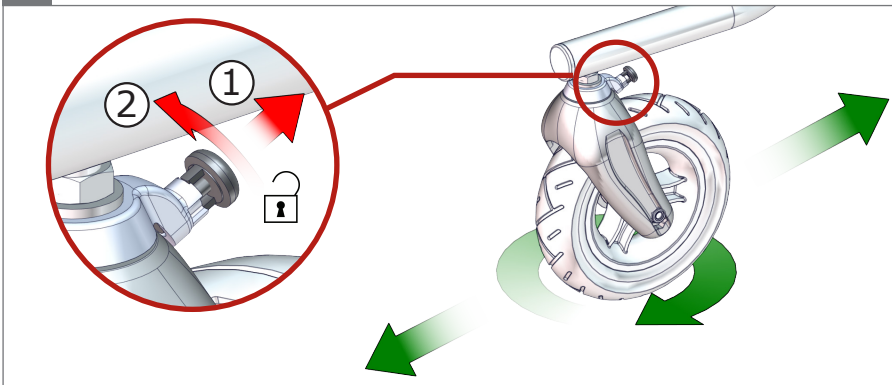
3.


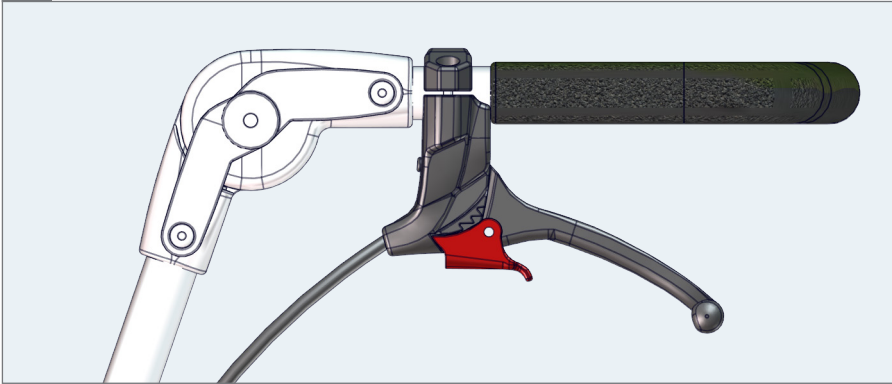
1.


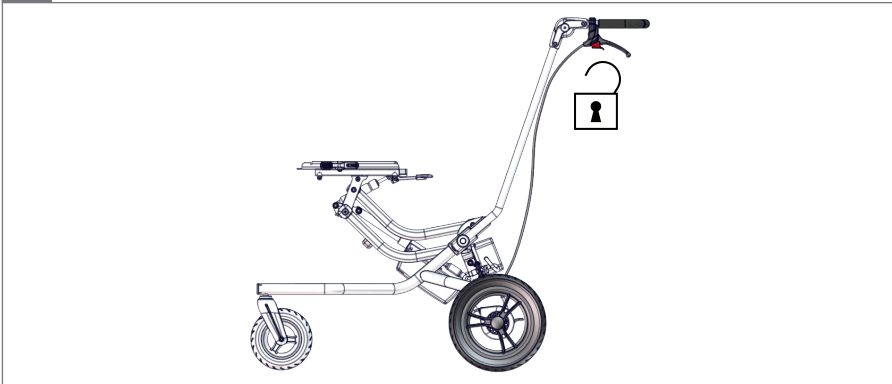
2.

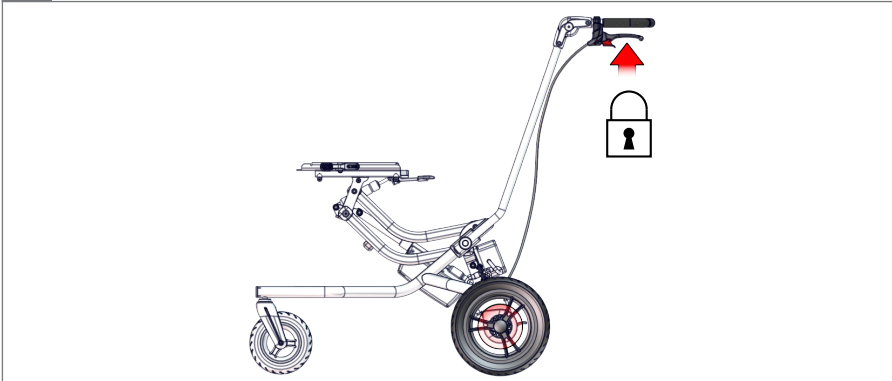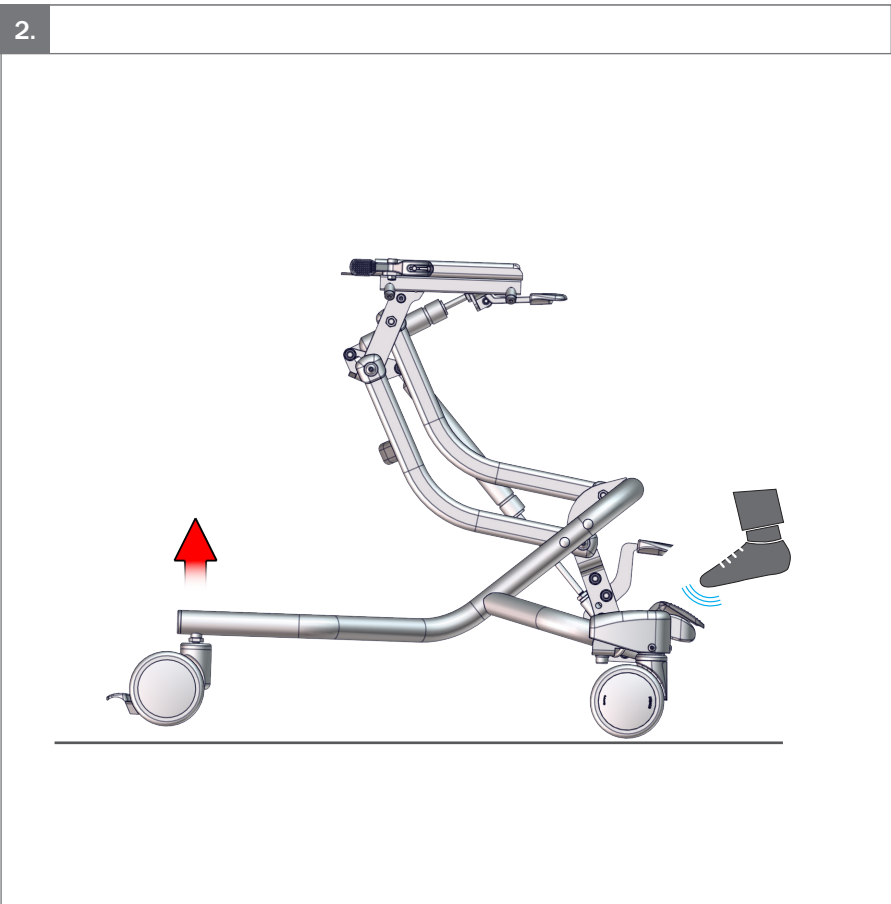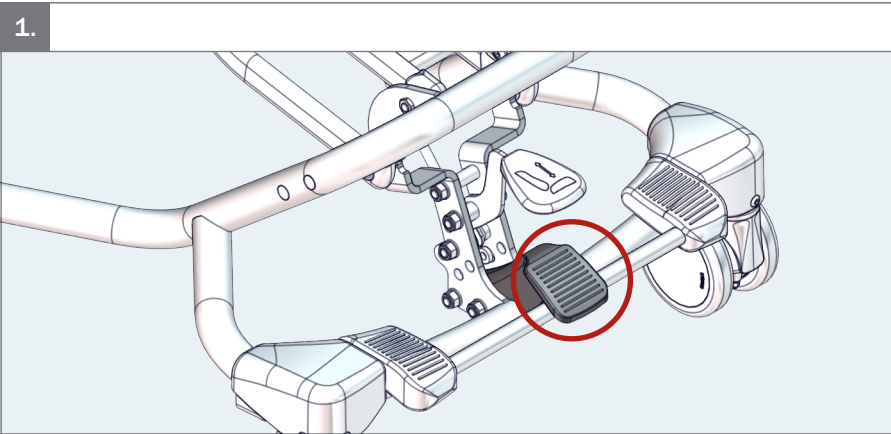
3.

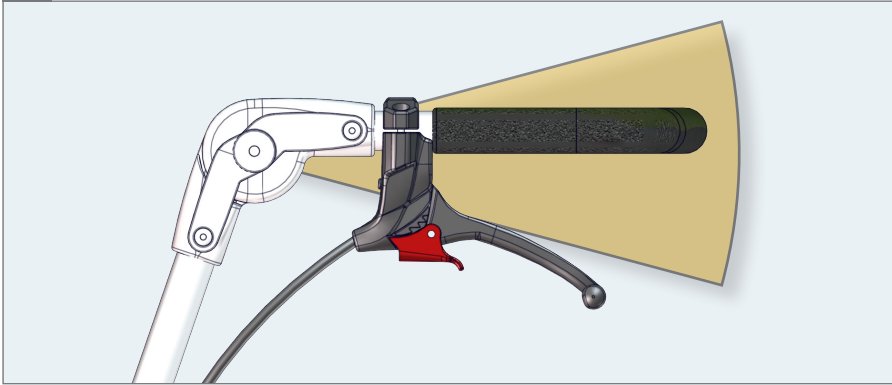
1.

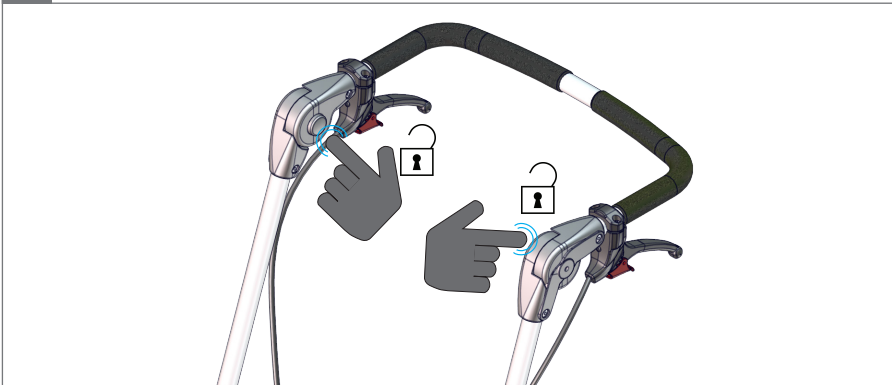2.

3.

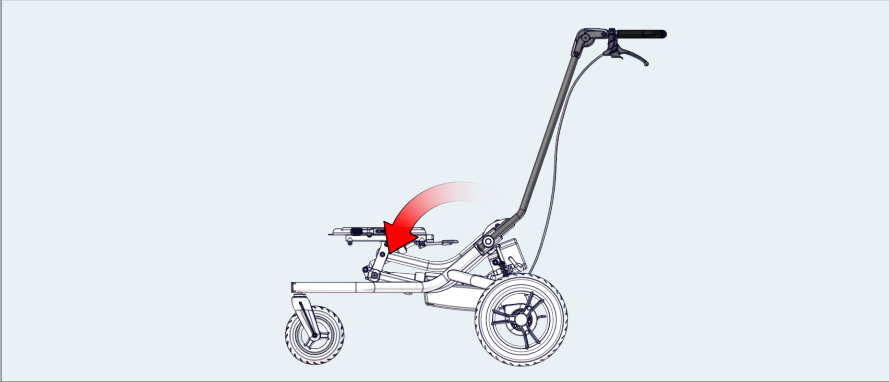
4.



1.

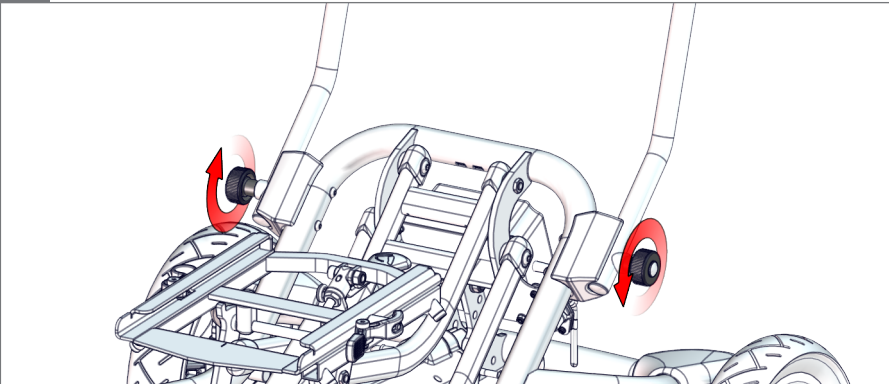2.

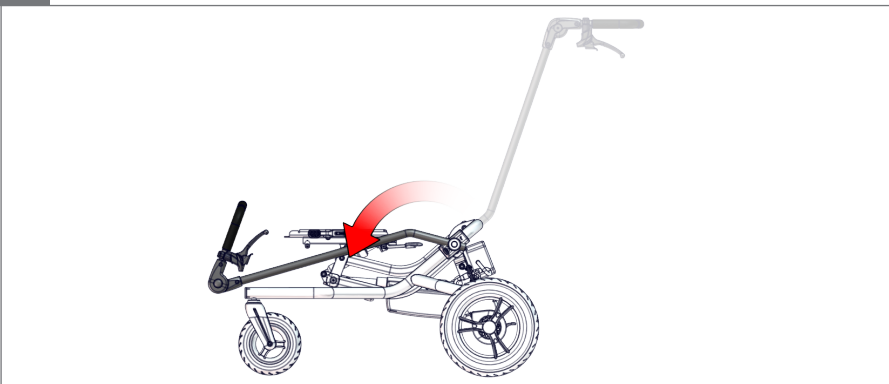
3.

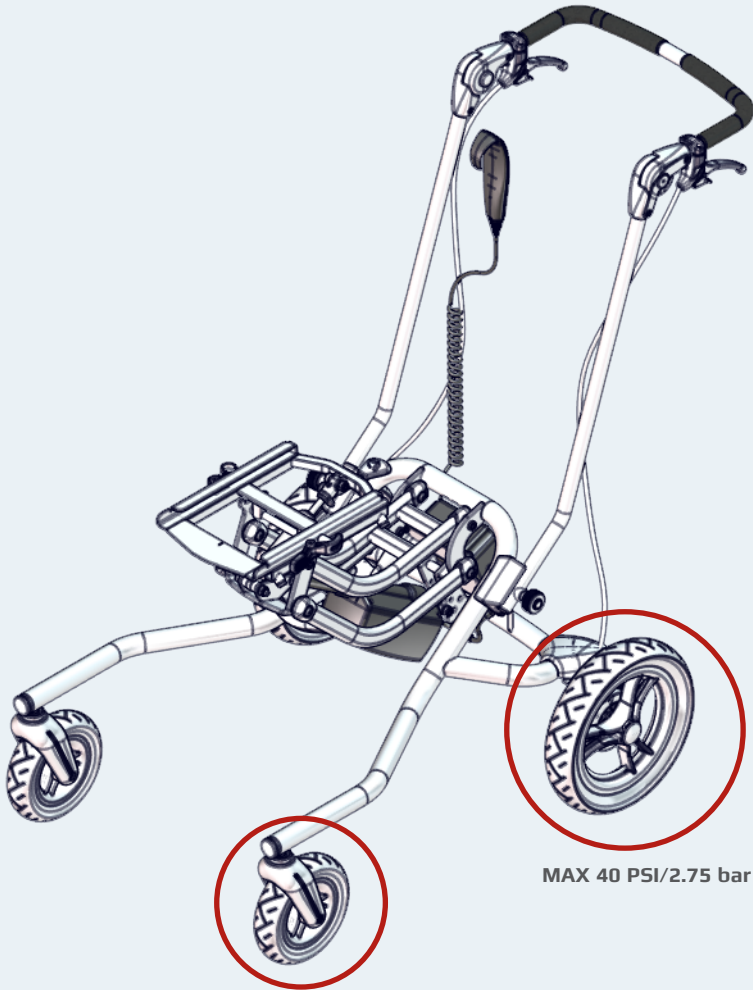1.


MAX 36 PSI/2.5 bar

MAX 40 PSI/2.75 bar


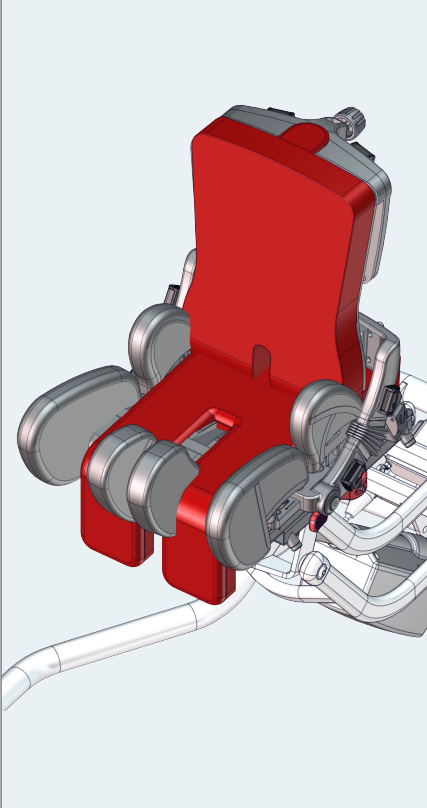
- 1a. Size 1 with x:panda 1 - 2  
Size 2 with x:panda 3  
Size 1 with Panda Futura 1 - 4

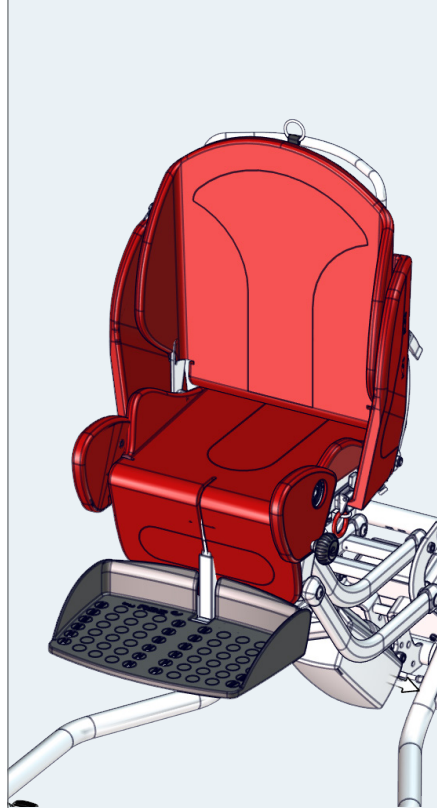
- 1b. Size 1 with Stingray 1  
Size 2 with Stingray 2

2a.

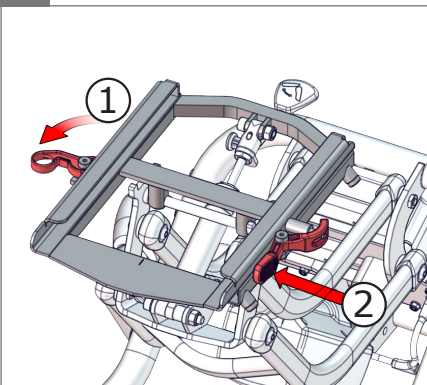
2b.

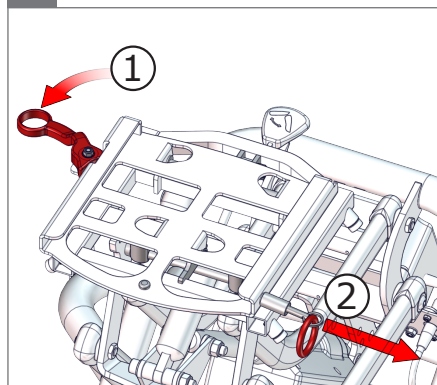
3.

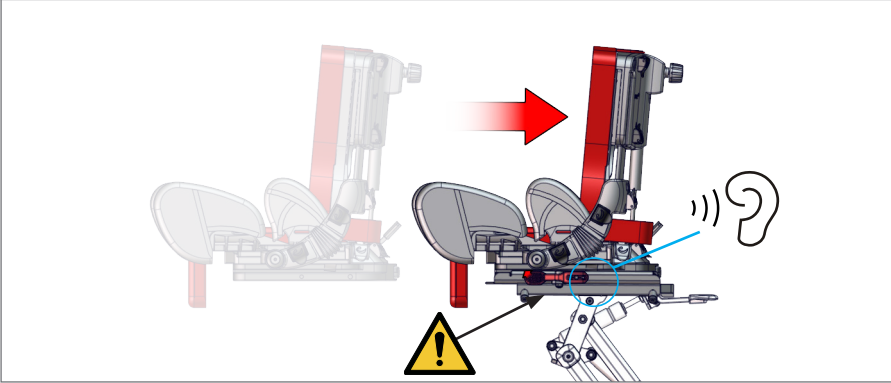
05

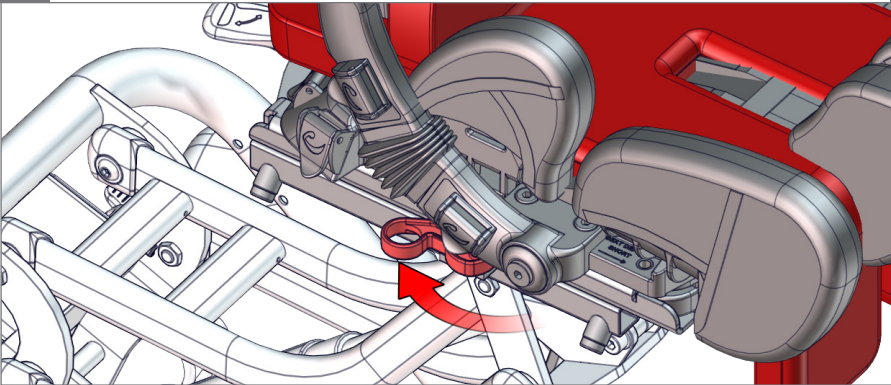4.

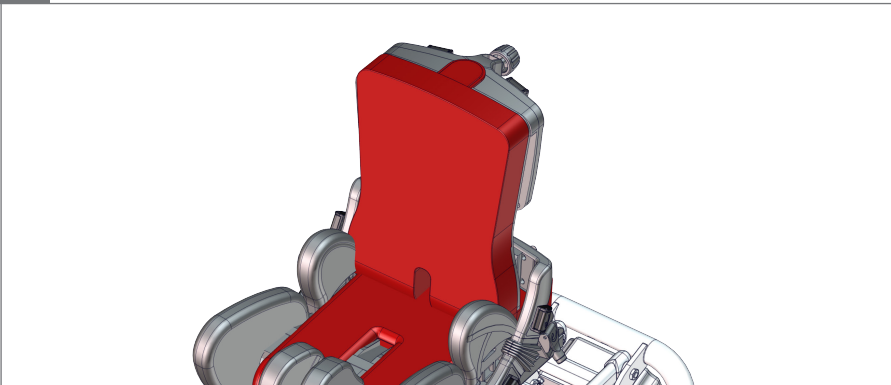
5.

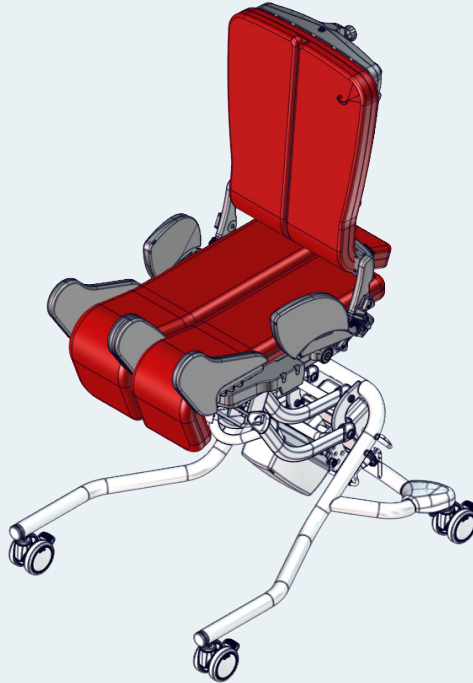
1. Size 3 with x:panda 4  
Size 3 with Panda Futura 5

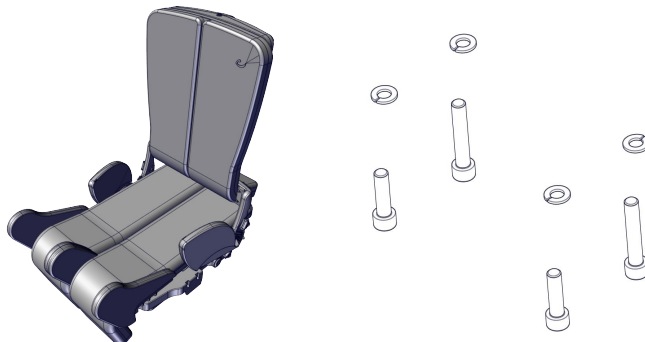
2.


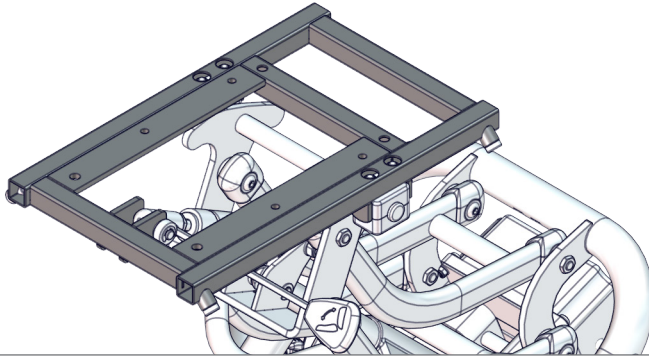
3.


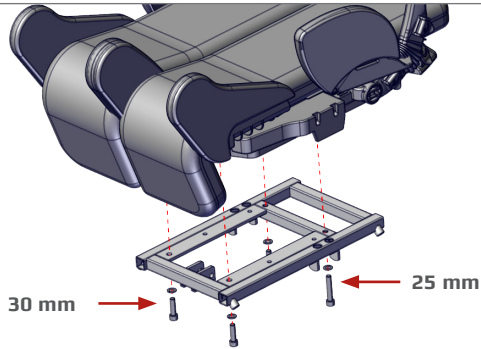
4.

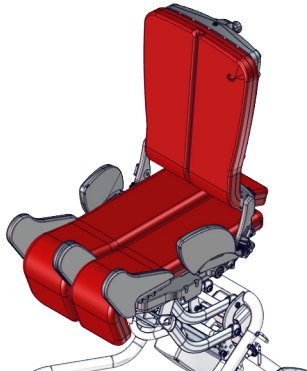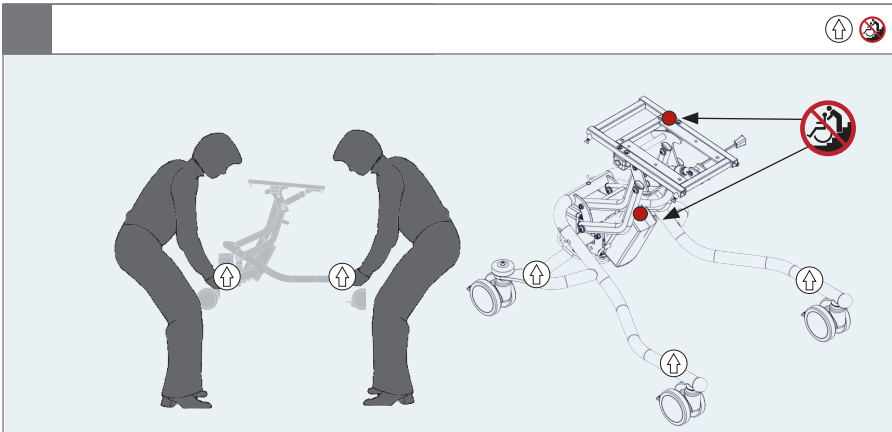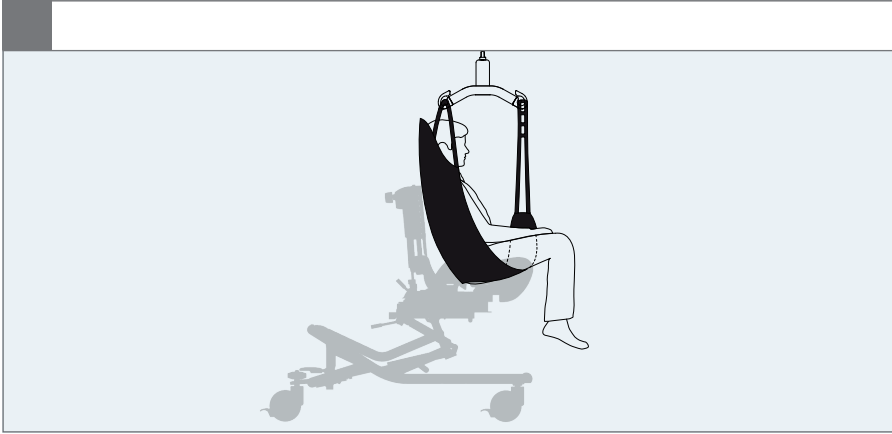
5.

High-low:xo
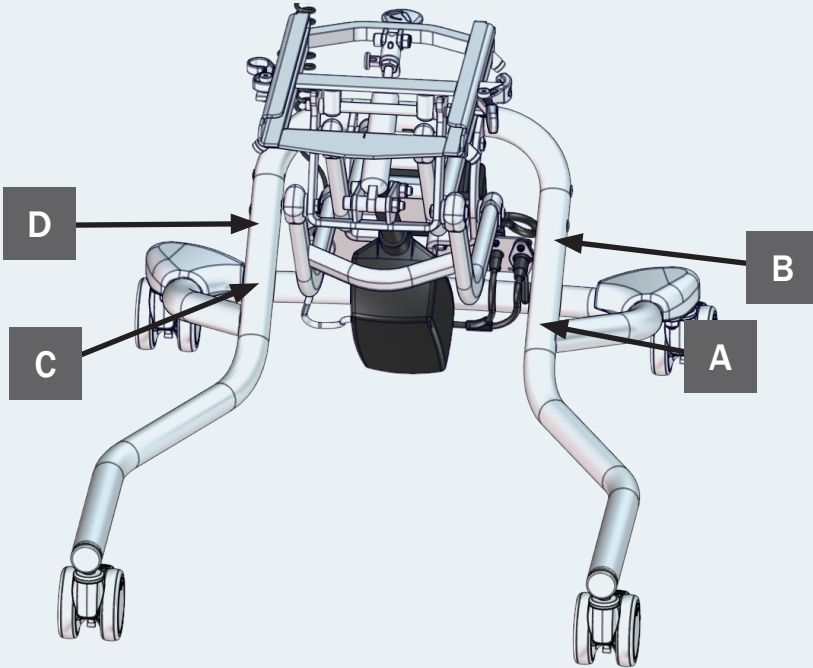
|          |                       |               |            |                     |
|----------|-----------------------|---------------|------------|---------------------|
| <b>C</b> | Storage and Transport | 50°C<br>-10°C | 80%<br>20% | 1060 hPa<br>700 hPa |
|          | Operating             | 40°C<br>5°C   | 80%<br>20% | 1060 hPa<br>700 hPa |

DUTY CYCLE: 10% MAX 2 min/18 min.


|          |  |                        |
|----------|--|------------------------|
| <b>A</b> | Product: xxxx - Size x   | Max load: xxx kg       |
|          | Art. no.: xxxxxxxxxxxx   | Mass device: xxx kg    |
|          | DMR ver.: XXXX   | Max rated load: xxx kg |
|          | <br><small>(xxx)xxxxxxxxxxxxxxxxxxxxxx(00)xxxxxxxxxx(00)xxxxxxxx</small> |                        |

|          |       |  |
|----------|-------|--|
| <b>D</b> | IP 24 |  |
|----------|-------|--|

|          |                         |            |  |
|----------|-------------------------|------------|--|
| <b>B</b> | <b>R82</b> <sup>®</sup> |            |  |
|          | by Etac                 |            |  |
|          | R82 A/S Parallelvej 3   |            |  |
|          | DK-8751 Gedved          |            |  |
|          | etac.com                |            |  |
|          | Made in XXXXXXXX        | YYYY-MM-DD |  |

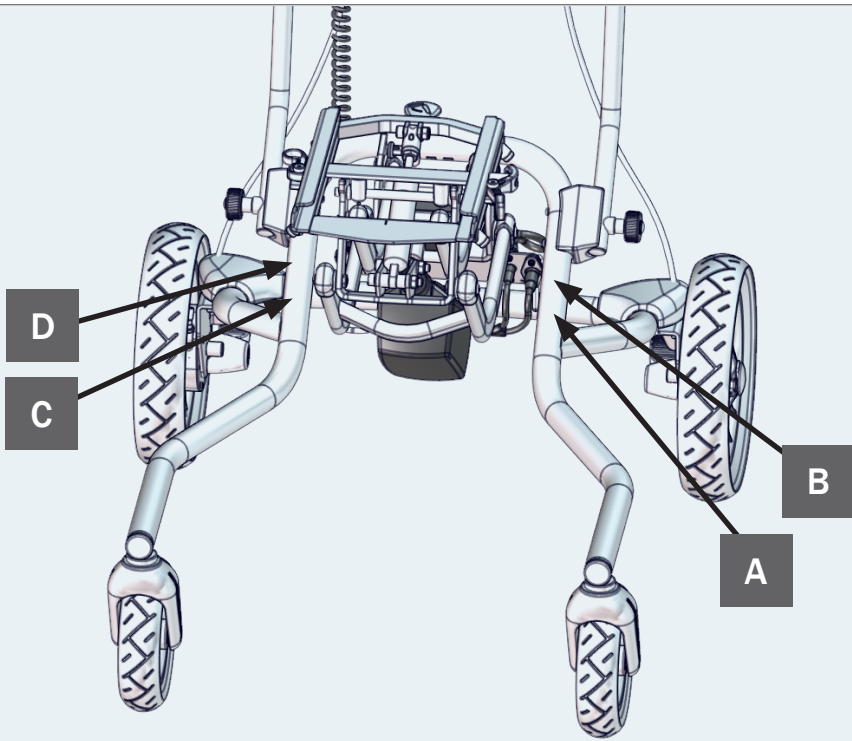High-low:xo

|          |                       |               |            |                     |
|----------|-----------------------|---------------|------------|---------------------|
| <b>C</b> | Storage and Transport | 50°C<br>-10°C | 80%<br>20% | 1060 hPa<br>700 hPa |
|          | Operating             | 40°C<br>5°C   | 80%<br>20% | 1060 hPa<br>700 hPa |

DUTY CYCLE: 10% MAX 2 min./18 min.

**A**

Product: xxxx - Size x  
 Art. no.: xxxxxxxxxx Max load: xxx kg  
 XXXXXXX Max user mass: xxx kg  
 DMR ver.: XXXX Mass device: xxx kg  
 Max rated load: xxx kg

(xxx)xxxxxxxxxxxxxxxxxxxxxxx(00)xxxxxxxx(00)xxxxxxxx

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
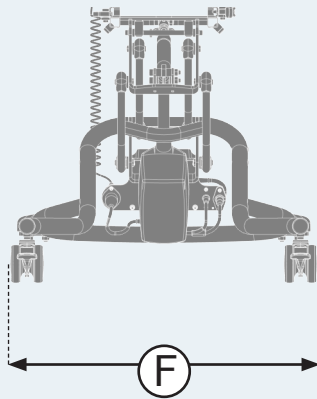
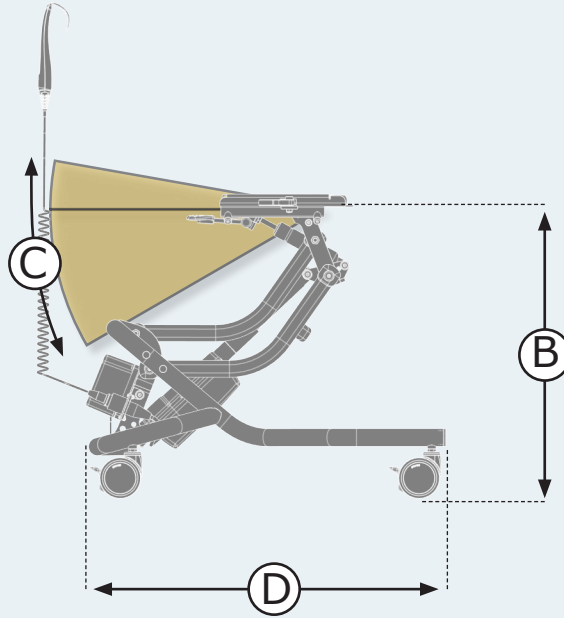
High-low:xo SIZE 1

High-low: xo SIZE 2-3
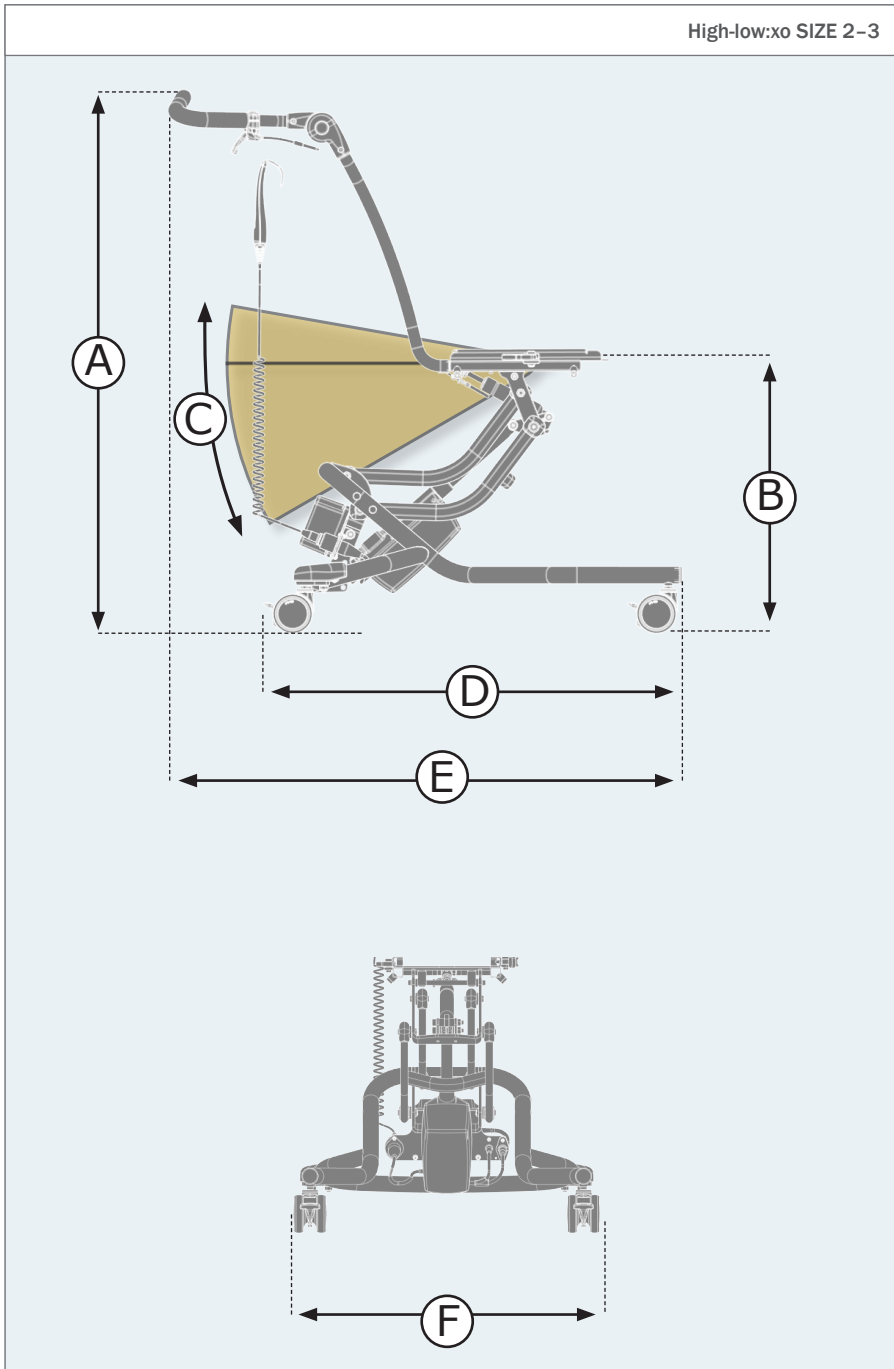
| Indoor   | 1   | 2   | 3  |
|--|---|---|--|
| Overall height (A), to pushbrace with 75 mm wheels                       | 1100<br>(43 $\frac{3}{8}$ " )                     | 1100<br>(43 $\frac{3}{8}$ " )                     | 1100<br>(43 $\frac{3}{8}$ " )                      |
| Overall height (A), to pushbrace with 100 mm wheels                      | 1130<br>(44 $\frac{1}{2}$ " )                     | 1130<br>(44 $\frac{1}{2}$ " )                     | 1130<br>(44 $\frac{1}{2}$ " )                      |
| Overall height (A), to pushbrace with 125 mm wheels                      | 1170<br>(46 $\frac{1}{8}$ " )                     | 1170<br>(46 $\frac{1}{8}$ " )                     | 1170<br>(46 $\frac{1}{8}$ " )                      |
| Seat surface height at front edge (B) (adaptor plate) with 75 mm wheels  | 170-580<br>(6 $\frac{3}{4}$ -22 $\frac{3}{4}$ " ) | 210-580<br>(8 $\frac{1}{4}$ -22 $\frac{3}{4}$ " ) | 270-530<br>(10 $\frac{3}{4}$ -20 $\frac{3}{4}$ " ) |
| Seat surface height at front edge (B) (adaptor plate) with 100 mm wheels | 200-610<br>(7 $\frac{3}{4}$ -24" )                | 240-610<br>(9 $\frac{1}{2}$ -24" )                | 300-560<br>(11 $\frac{3}{4}$ -22" )                |
| Seat surface height at front edge (B) (adaptor plate) with 125 mm wheels | 240-650<br>(9 $\frac{1}{2}$ -25 $\frac{1}{2}$ " ) | 280-650<br>(11-25 $\frac{1}{2}$ " )               | 350-600<br>(13 $\frac{1}{2}$ -23 $\frac{1}{2}$ " ) |
| Seat plane angle (C)   | (-10)-30°   | (-10)-30°   | (-10)-30°  |
| Length (D)*  | 660 (27 $\frac{3}{4}$ " )                         | 795 (31 $\frac{1}{4}$ " )                         | 795 (31 $\frac{1}{4}$ " )                          |
| Overall length (E)   | 670 (26 $\frac{3}{8}$ " )                         | 830 (32 $\frac{3}{4}$ " )                         | 830 (32 $\frac{3}{4}$ " )                          |
| Overall width (F)  | 560 (21" )  | 620 (24 $\frac{3}{8}$ " )                         | 620 (24 $\frac{3}{8}$ " )                          |
| Castors diameter   | 75/100/125 (3/4/5" )                              | 75/100/125 (3/4/5" )                              | 75/100/125 (3/4/5" )                               |
| Minimum turning radius   | 400 (15 $\frac{3}{4}$ " )                         | 475 (18 $\frac{3}{4}$ " )                         | 475 (18 $\frac{3}{4}$ " )                          |
| Max load   | 70 kg (154 lb)                                    | 75 kg (165 lb)                                    | 95 kg (209 lb)                                     |
| Mass device*   | 16 kg (35 lb)                                     | 17 kg (38 lb)                                     | 18 kg (42 lb)                                      |
| Max rated load*  | 88 kg (194 lb)                                    | 94 kg (207 lb)                                    | 116 kg (256 lb)                                    |
| Static stability Sideways, downhill and uphill                           | 10°   | 10°   | 10°  |

**EN:** \*Measurements: The High-low:xo measurements above are based on the use of 75 mm castors with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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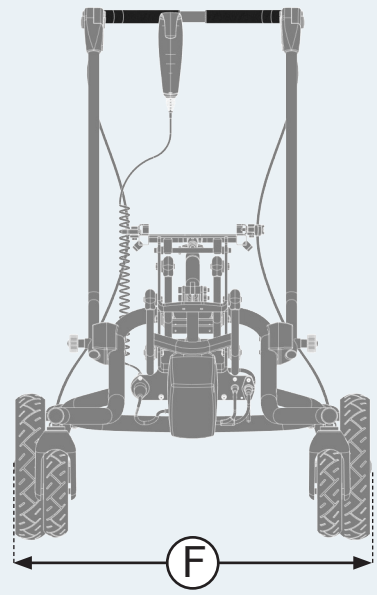
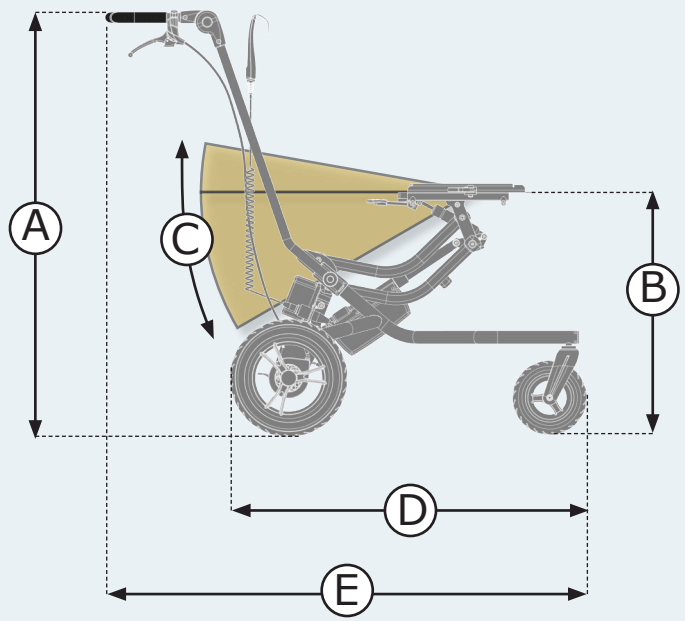
High-low: xo SIZE 1-2 OUTDOOR

| Outdoor                                  | 1                  | 2                  |
|--|--------------------|--------------------|
| Overall height (A), to pushbrace         | 1045 (41¼")        | 1045 (41¼")        |
| Seat surface height at front edge (B)    | 300–720 (11¾–28¾") | 345–720 (13½–28¾") |
| Seat plane angle (C)                     | (-10)–30°          | (-10)–30°          |
| Length (D)*                              | 797 (31¼")         | 800 (31½")         |
| Overall length (E)                       | 1070 (42½")        | 1170 (46½")        |
| Overall length with leg rest             | 1115 (43⅞")        | 1190 (46⅞")        |
| Overall width (F)                        | 660 (26")          | 700 (27½")         |
| Footrest to adaptor plate                | 50-290 (2-11½")    | 50-290 (2-11½")    |
| Leg to seat surface angle                | (-30)–30°          | (-30)–30°          |
| Front wheel                              | 6¾                 | 6¾                 |
| Rear wheel                               | 12½                | 12½                |
| Folded height                            | 460 (18")          | 460 (18")          |
| Minimum turning radius                   | 875 (34½")         | 930 (36½")         |
| Horizotal location of axle               | 200–210 (7¾–8¾")   | 255–265 (10–10½")  |
| Max load                                 | 70 kg (154 lb)     | 75 kg (165 lb)     |
| Min. user mass in transportation         | 22 kg (49 lb)      | 22 kg (49 lb)      |
| Max. user mass in transportation         | 57 kg (125 lb)     | 57 kg (125 lb)     |
| Mass device*                             | 25 kg (55 lb)      | 26 kg (57 lb)      |
| Max. rated load*                         | 97 kg (214 lb)     | 103 kg (227 lb)    |
| Mass of the heaviest part                | 21 kg (46 lb)      | 22 kg (49 lb)      |
| Pivot width                              | 1180 (46½")        | 1250 (49¼")        |
| Maximum slope (for use of parking brake) | 7°                 | 7°                 |
| Static stability                         | 10°                | 10°                |
| Sideways, downhill and uphill            |                    |                    |

**EN:** \*Measurements: The High-low:xo Outdoor measurements above are based on the use of standard solid wheels with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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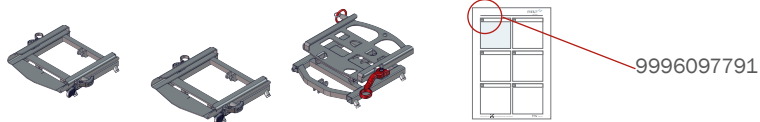
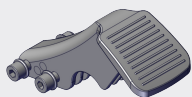
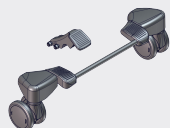
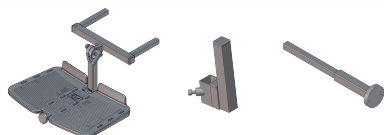
## ENGLISH

**General**

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

**Intended purpose**

The High-low:xo is a frame for a modular seating system for children and adults with disabilities

**Indications**

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

The outdoor version of the device is approved for transportation for children and adults in the weight range of 22 kg to specified max user weight for transportation.

**Application areas**

The product is for indoor and outdoor use

**Contra indications**

None

**Precautions**

None

**Enclosed tools:**

- 4, 5 and 6 mm Allen key
- 10 and 13 mm spanner

**Declaration of Conformity**

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- High-low:xo indoor: EN 12182
- High-low:xo outdoor: EN 12182 & EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

**Disposal**

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

**Accessories and spare parts**

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

**R82 Warranty**

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the [etac.com/support](http://etac.com/support).

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

**Presales information**

- [www.etac.com](http://www.etac.com)

### Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

For information about product safety notices and product recalls, local dealer information are available at the website [etac.com](http://etac.com)

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Do not let children and youngsters play with the handset
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.
- Always use the product in a moderate pace. Do not run with the product
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: [etac.com](http://etac.com)

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- Do not place more than one user in the product
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

### Product

- Removable parts without use of tool: Foot support and seat. The removal will have adverse effect on the device
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Ensure the stability of the product is maintained before placing the user in it
- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- If the frame has to be used in the lowest position, Remove the footrest before adjustment.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- When mounting a Panda Futura seat size 4 on any High-low:xo frame, you must mount the anti-tip device that comes as an accessory to the product
- The power unit does not have a safety stop
- Do only use the power unit within specified working limits
- Do not attempt to repair any part of the power unit.
- The user must not occupy the product when it is lifted from the ground
- Ease of access to and maneuverability in, motor vehicles can be significantly affected by size of device and turning radius, and the smaller device and/or device with a shorter turning radius will generally provide greater ease of vehicle access and maneuverability to a forward facing position. In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"
- The device conforms to ISO 7176-8, covering requirements and test methods for static, impact and fatigue strengths
- Unoccupied device is suitable for land and air transport
- Details covering EMC can be found in the appendix "EMC details" supplied with the product

**Safety product**

Due to variant and settings the wheelchair may become larger than the recommended dimensions.

Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

**Table 1**

| Recommended dimensions     | Ready for use (mm) |
|----------------------------|--------------------|
| Overall length - maximum   | 1200               |
| Overall width - maximum    | 700                |
| Overall height - maximum   | 1200               |
| Turning diameter - maximum | 1000               |
| Pivot width - maximum      | 1300               |
| Ground clearance - minimum | 30                 |

**Product safety**

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

**Table 2**

| Wheelchairs transportable by train - limits                |   |
|--|---|
| Overall length - maximum                                   | 1200 mm + 50 mm for feet  |
| Overall width - maximum                                    | 700 mm + 50 mm on each side for hands   |
| Overall height - maximum                                   | 1375 mm (including 95th percentile male occupant)   |
| Turning diameter - maximum                                 | 1500 mm   |
| Smallest wheels  | Accommodate a gap of 75 mm horizontal and 50 mm vertical  |
| Weight - maximum   | 200 kg for manual wheelchair and occupant (including baggage)   |
| Obstacle height that can be overcome - maximum             | 50 mm   |
| Ground clearance - minimum                                 | 60 mm under foot rest with upward slope angle of 10° on top for going forward                                 |
| Safe slope on which the wheelchair remain stable - maximum | 6° dynamic stability in all directions<br>9° static stability in all directions, including with brake applied |

### Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact local dealer for assistance, if needed, in setting up, using or maintenance of the device.

#### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

#### Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Be aware of any MAX mark indications before adjustment
- Check that all cables connecting the power system are in good shape

#### Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use

#### Monthly

- If equipped with airfilled tyres, check the air pressure
- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

#### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year

#### Washing machine

Before washing remove the following items and clean them separately:

- Power Unit, Seat and Accessories
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

#### After washing:

- Install the power unit, seat and accessories.
- Check that the handset functionalities works correct

#### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

#### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months if the product is used very often - daily use for more than 7 hours), every time the product is reissued for use and after a long term storage (more than 4 months). The inspection should be carried out by a technically competent person who has been trained in the

use of the product.

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime

#### Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Frame is adjustable in height
- Powered versions are able to retain power after charging the battery
- On outdoor versions, the push-brace can be folded over the frame
- If equipped with airfilled tyres, check the air pressure
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

#### Materials

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

#### Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

#### Recharging the battery:

- Remove the plug from the power system and plug in the charging lead
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

**Technical Data****Battery - BA18**

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

**Battery charger - CH01**

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA
- FRIWO Gerätebau GmbH, Model FW7218M/24

**Actuator - LA28/LA32**

- Input voltage: 24V
- Noise level:  $\leq 45$  dB(A)
- IPX5
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

**Handset - HB 30**

- IPX6

**Detachable parts**

- Handset - HB 30
- Battery - BA 18

**Applied part**

- Handset - HB 30

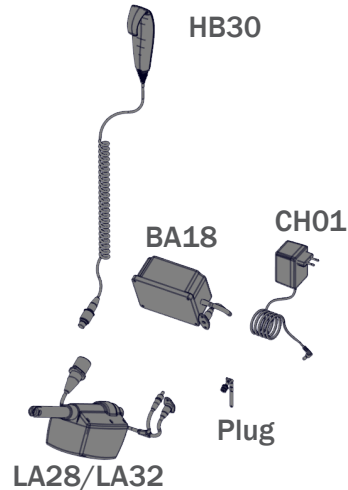
**Environmental conditions**

- Operating temperature: +5 °C to +40 °C
- Storage temperature: -10 °C to +50 °C
- Relative humidity: 20% to 80%
- Atmospheric pressure: 700 to 1060 hPa

**IP-Rating table - First Digit: Solids**

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects

|      |   |
|------|---|
| IPOX | No protection against contact and ingress of objects  |
| IP1X | Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.   |
| IP2X | Fingers or similar objects.   |
| IPX3 | Tools, thick wires, etc.  |
| IPX4 | Most wires, screws, etc.  |
| IPX5 | Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact. |
| IP6X | No ingress of dust; complete protection against contact.  |

**Ambient and surface temperature**

The table below illustrates how long time it takes for the surfaces of the device to reach operating temperature from -10 °C and +50 °C.

| Surface temperature |    |        |         |
|---------------------|----|--------|---------|
| -10 °C              | to | +5 °C  | 108 min |
| +50 °C              | to | +40 °C | 129 min |

**Second Digit: Liquids**

Protection of the equipment inside the enclosure against harmful ingress of water.

|      |   |
|------|---|
| IPX0 | No protection   |
| IPX1 | Protection against vertically falling drops of water e.g. condensation                                |
| IPX2 | Protection against direct sprays of water up to 15 ° from the vertical                                |
| IPX3 | Protected against direct sprays of water up to 60 ° from the vertical                                 |
| IPX4 | Protection against water sprayed from all directions - limited ingress permitted                      |
| IPX5 | Protected against low pressure jets of water from all directions - limited ingress                    |
| IPX6 | Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted |
| IPX7 | Protected against the effect of immersion between 15 cm and 1 m                                       |
| IPX8 | Protects against long periods of immersion under pressure   |

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© 2021 R82 A/S All rights reserved The R82 logo and the icons are
registered trademarks of R82 A/S