710 mm (27 3/4")

<60 kg (132 lb)




Kudu User Guide

EN 12183
ISO 7176-19

2021-01-rev 20

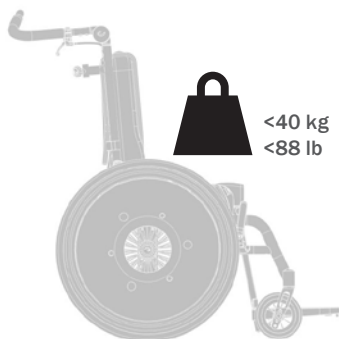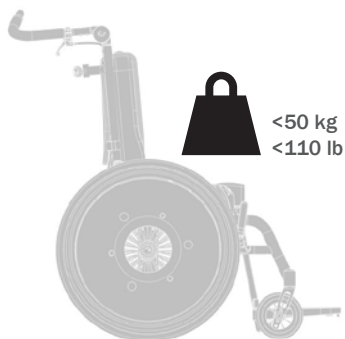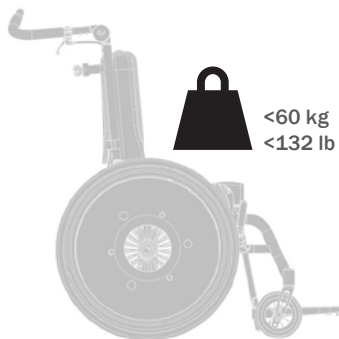	3		29
	4-5		30
	6-7		31
	8-9		32-36
	10	mm (inch)	37-38
	11		39-40
	12 - 14	ENGLISH	41-44
	15	DEUTSCH	45-48
	16	NEDERLANDS	49-52
	17 - 18	DANSK	53-56
	19	NORSK	57-60
	20	SVENSKA	61-64
	21 - 22	SUOMI	65-68
	23 - 24	ESPAÑOL	69-72
	25 - 26	FRANÇAIS	73-76
	27-28	ITALIANO	77-80
		PORTUGUÊS	81-84
		PORTUGUÊS DO BRASIL	85-88
		РОССИЯ	89-92
		POLSKI	93-96
		ČESKY	97-100
		中國	101-104
		日本	105-108
		ΕΛΛΗΝΙΚΗ	109-112

**KUDU 1****KUDU 2****KUDU 3+4**

1.

2.

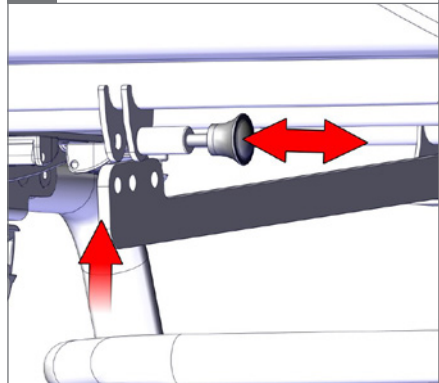
3.


05

4.

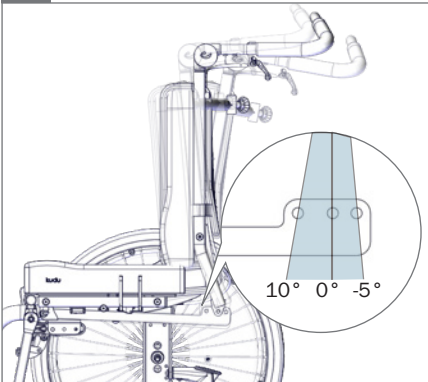5.

6.

1.

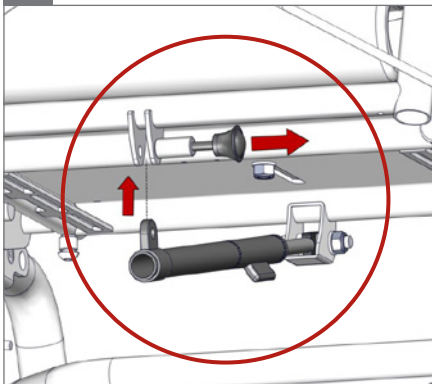
2.


3.

4.




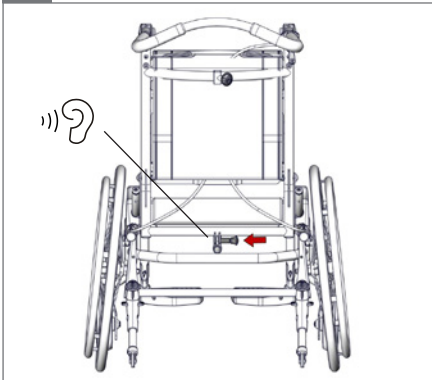
5.

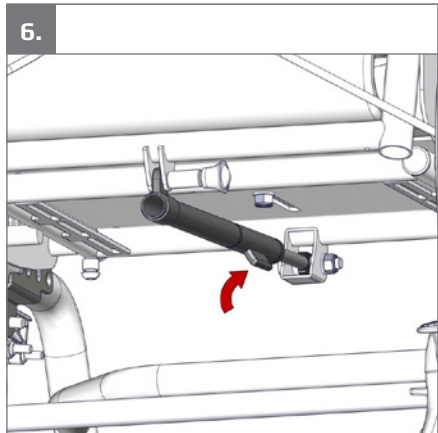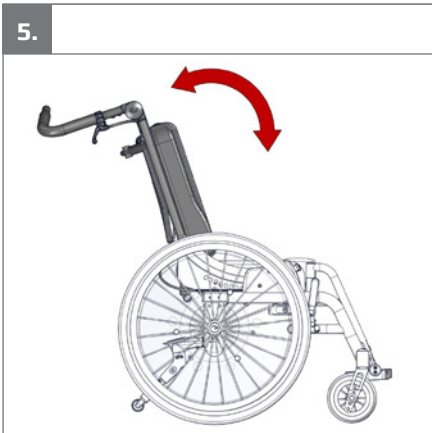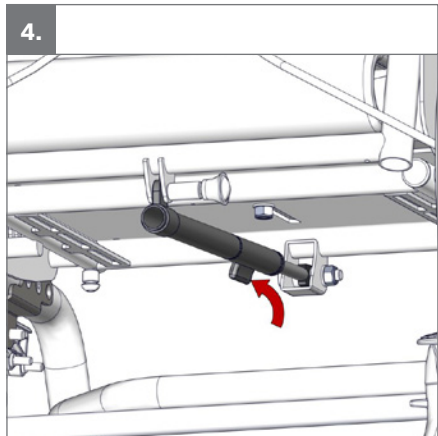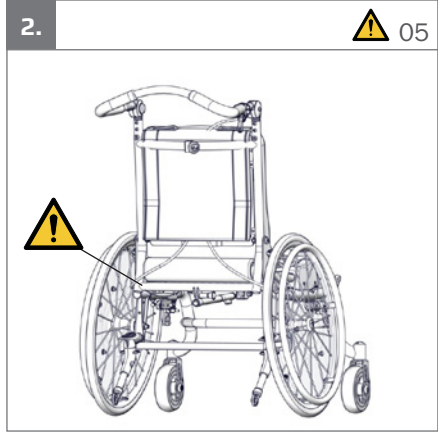
6.

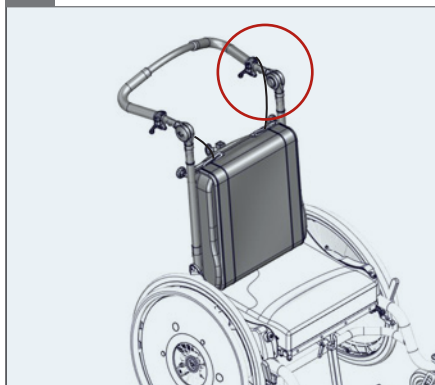
1.


2.

 05

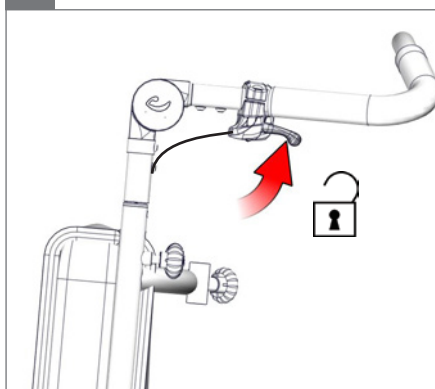
3.



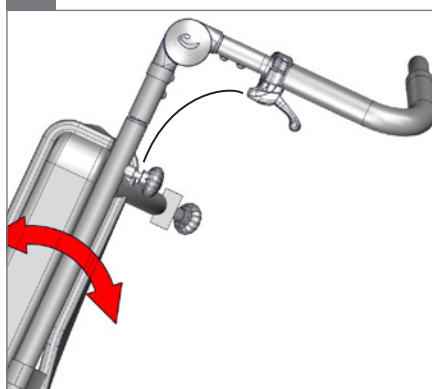
4.

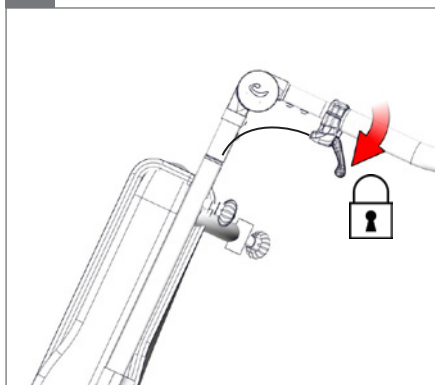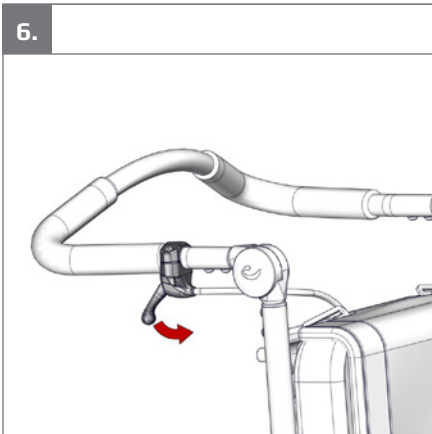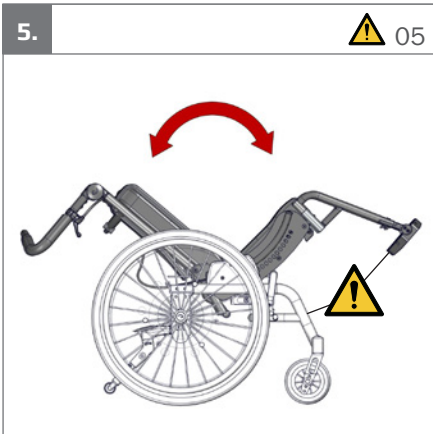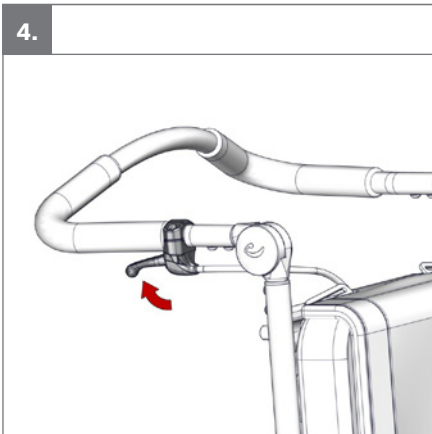
5.


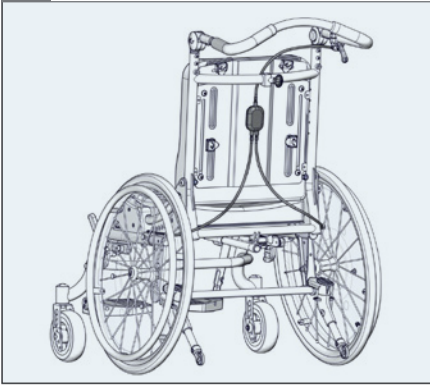
1.

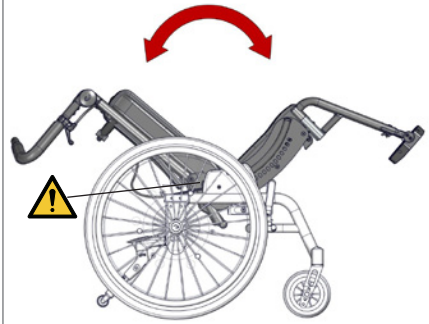
2.

 05

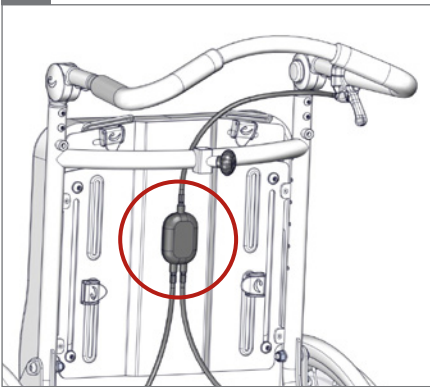
3.

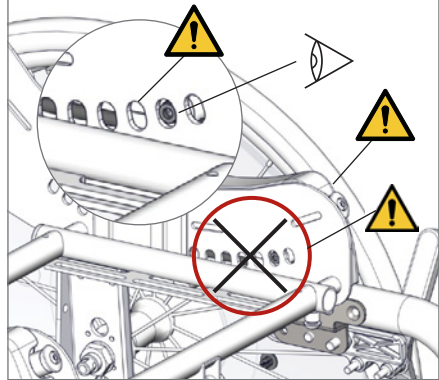
4.

 01, 05

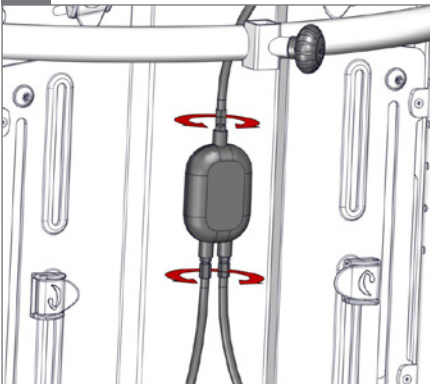5.

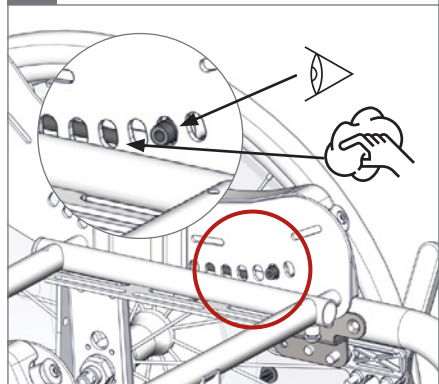6.


1.

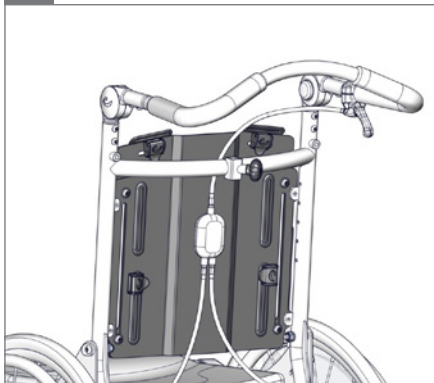
2.


3.

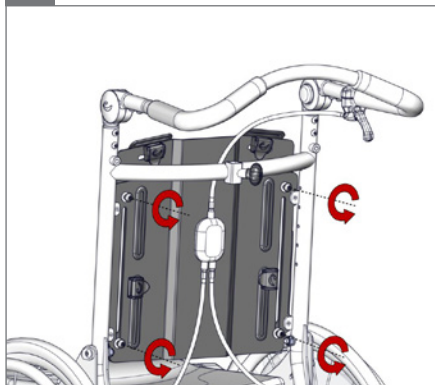○ 5 mm


4.

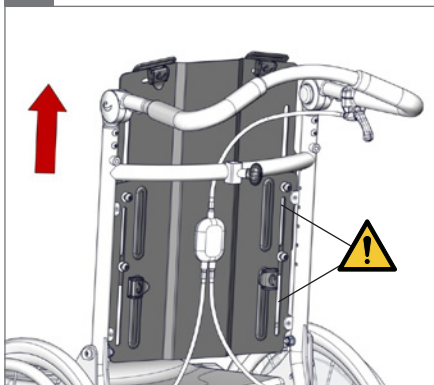⚠ 01

5.

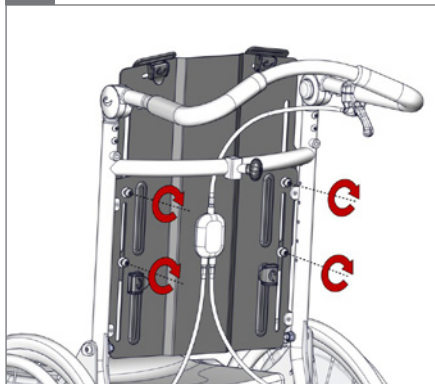○ 5 mm


6.

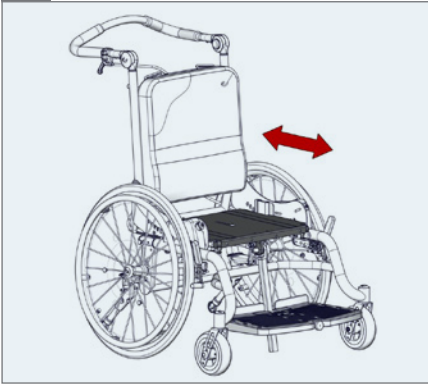
1.

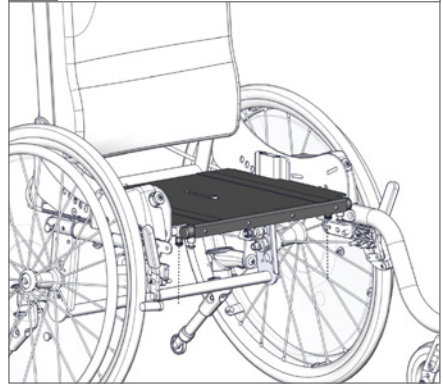
2.


3.

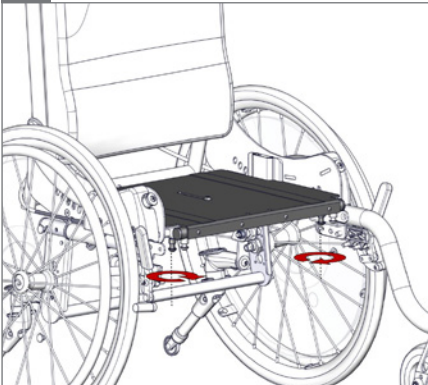○ 5 mm


4.

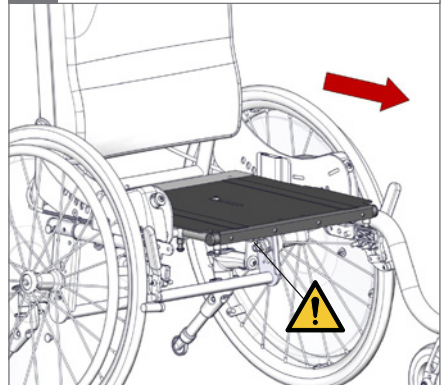⚠ 05

5.

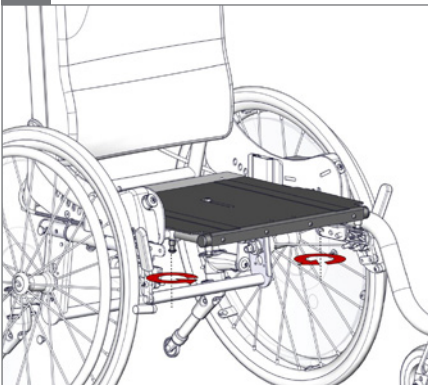○ 5 mm - 2,5 Nm

6.

1.

2.

3.

05

4.

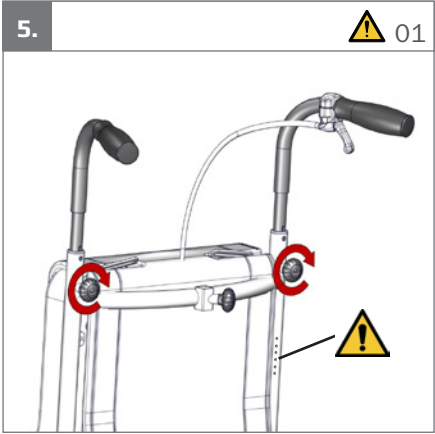
5.

6.



1.

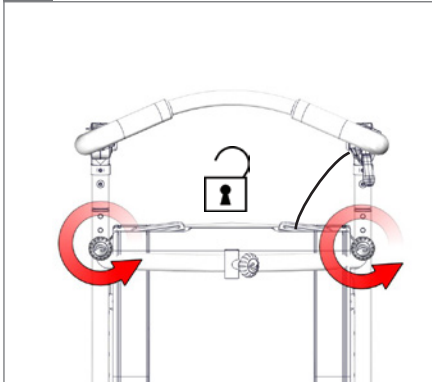
2.

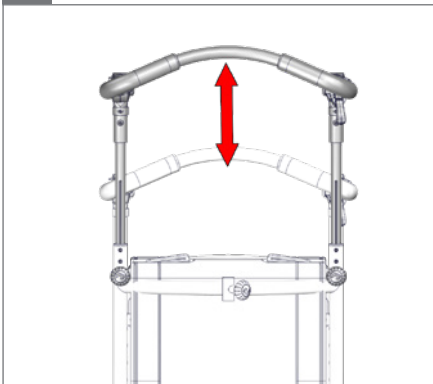3.

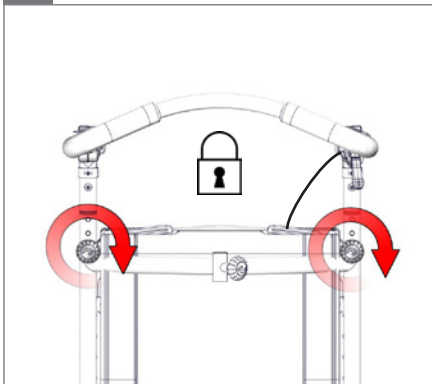
4.


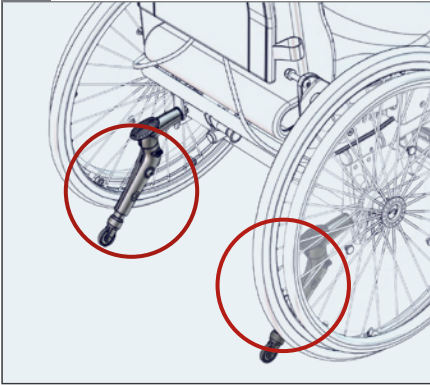
1.

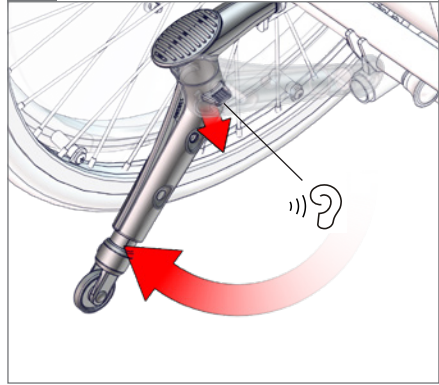2.

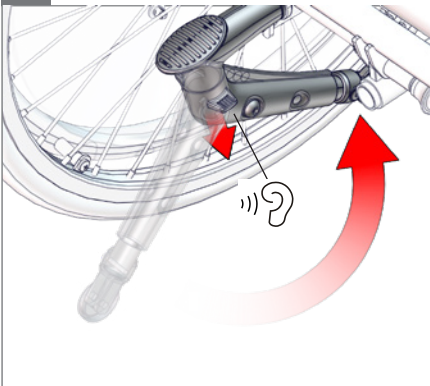
3.


4.

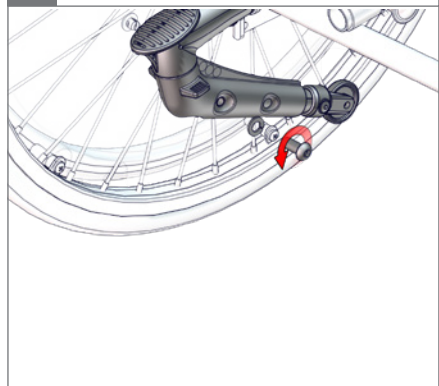⊕ 5 mm

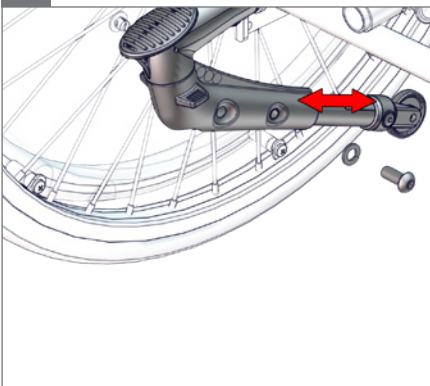
5.


6.

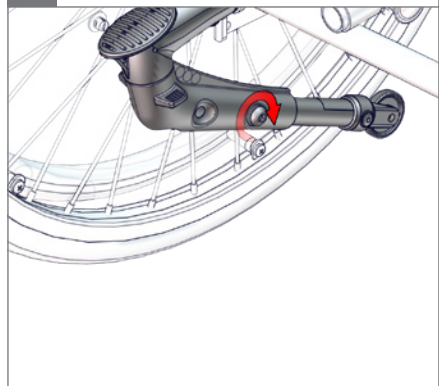⊕ 5 mm


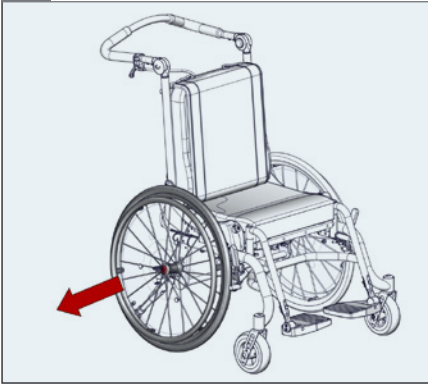
1.

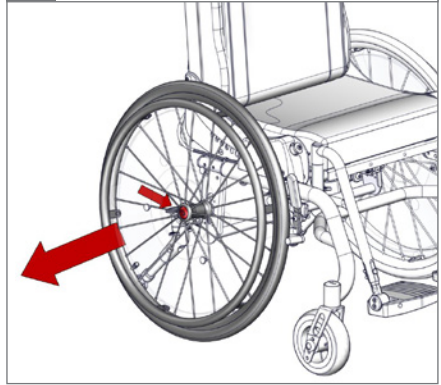
2.

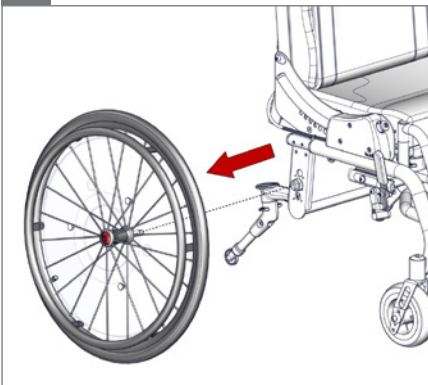
3.

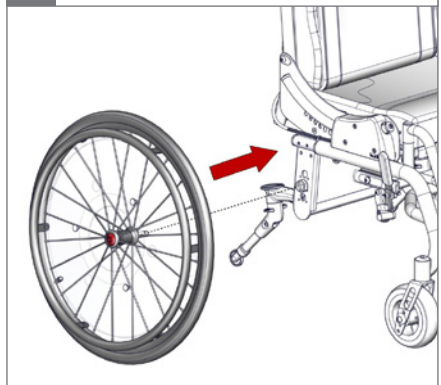
4.

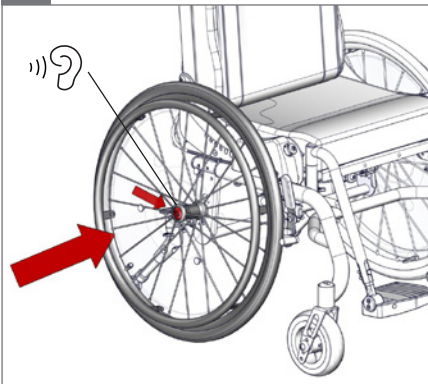5.

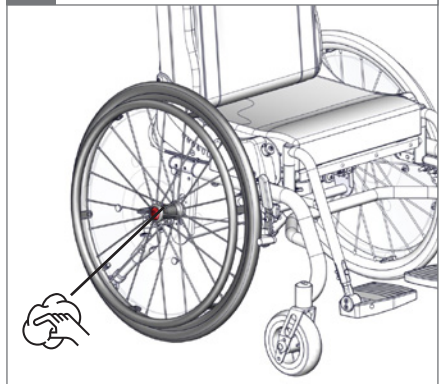6.


1.

2.

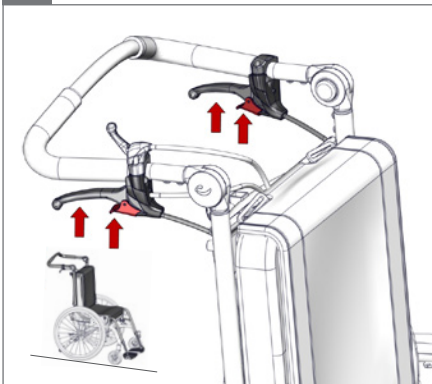
3.


4.

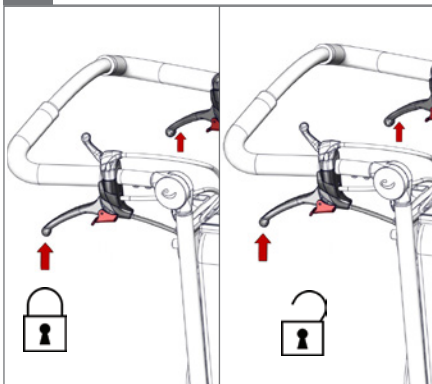Drum brake


5.






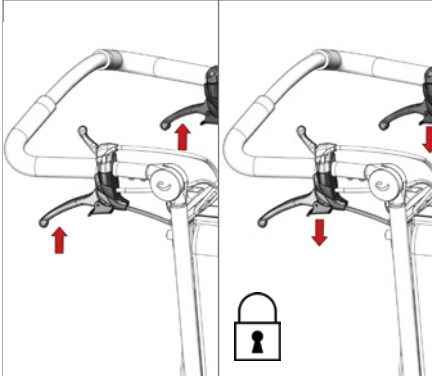
Decelerating drum brake

6.

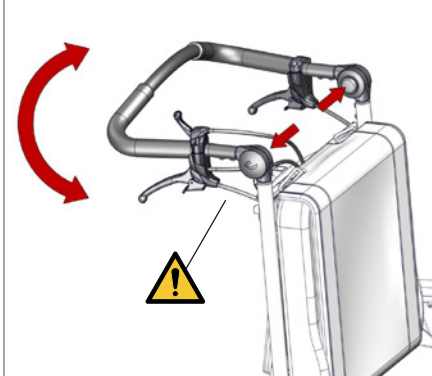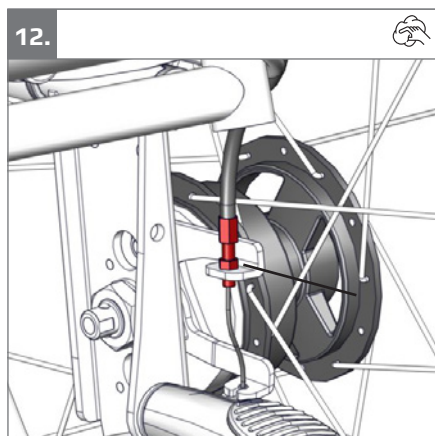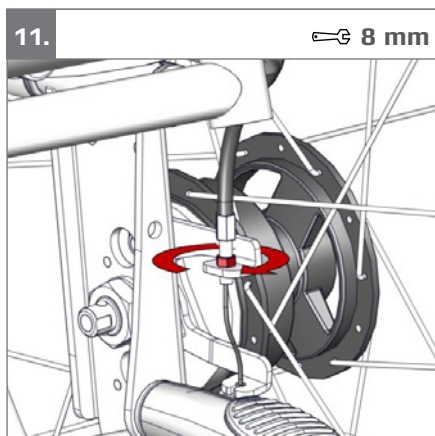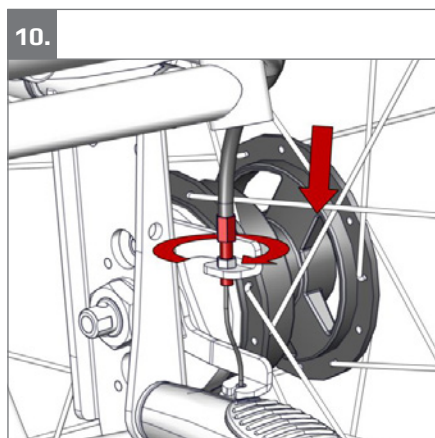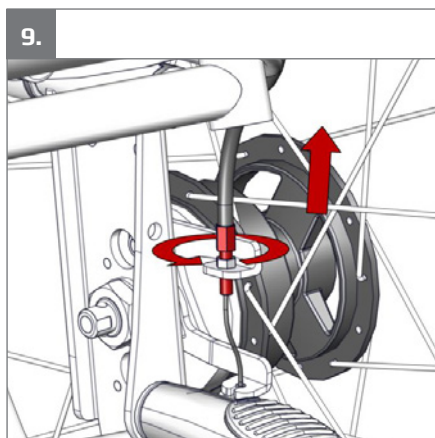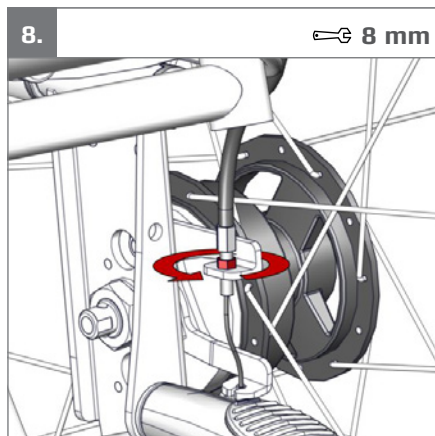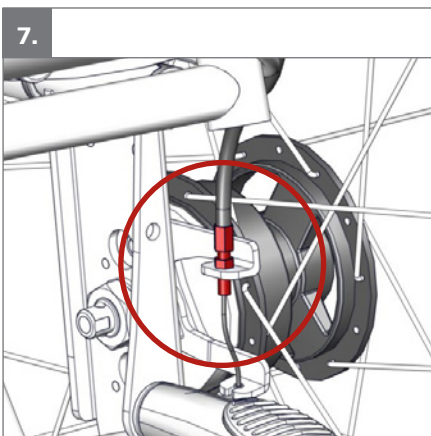
05


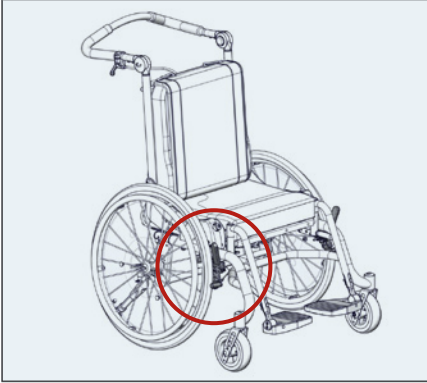
1.

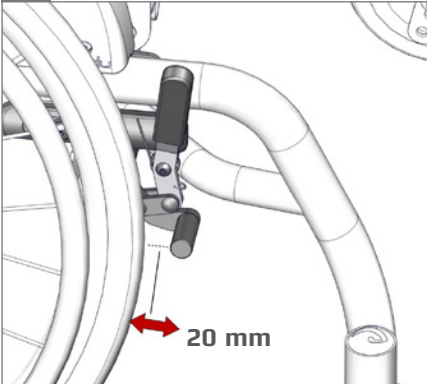
2.

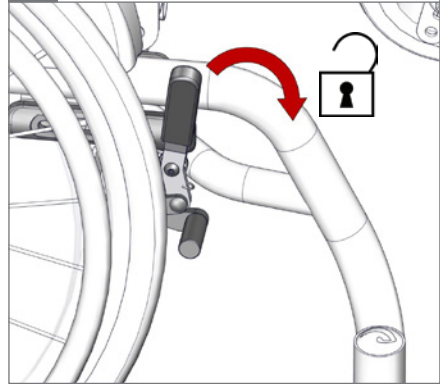
3.

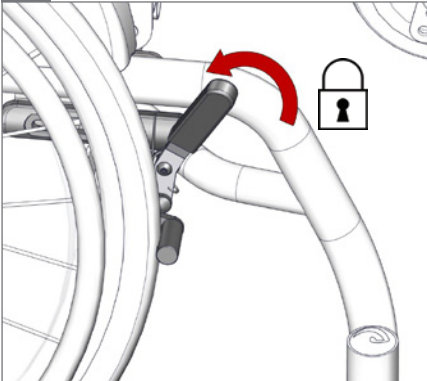4.

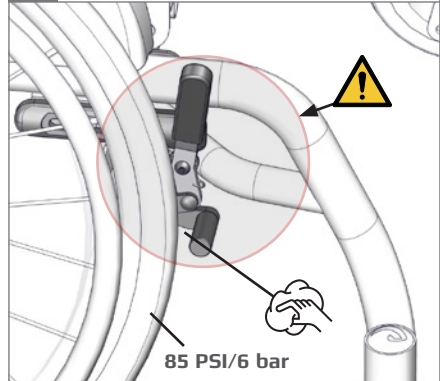
5.


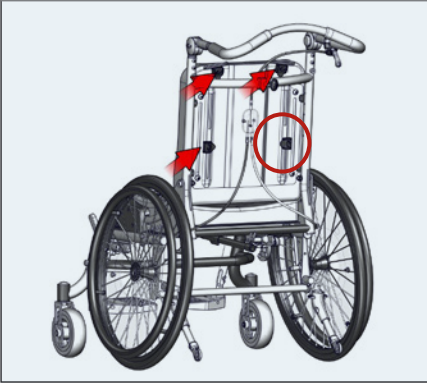
1.

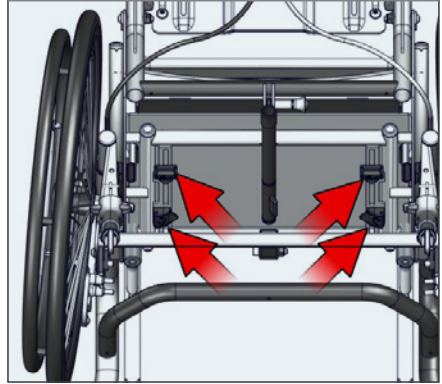2.

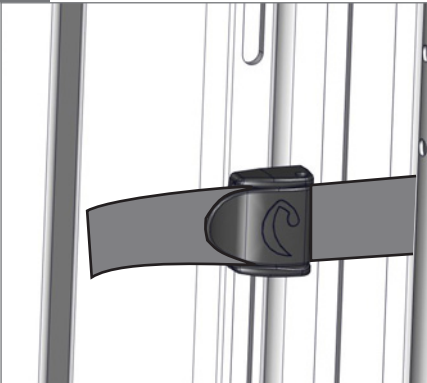3.


4.

⊙ 3 mm






5.




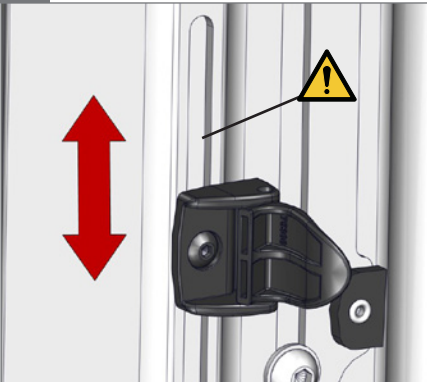
⚠ 01

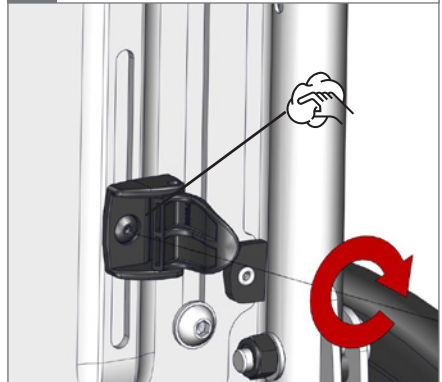6.

⊙ 3 mm - 2,5 Nm 

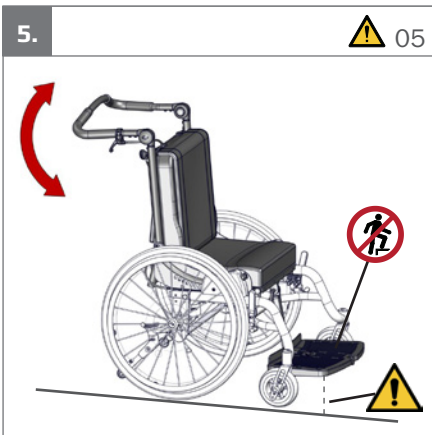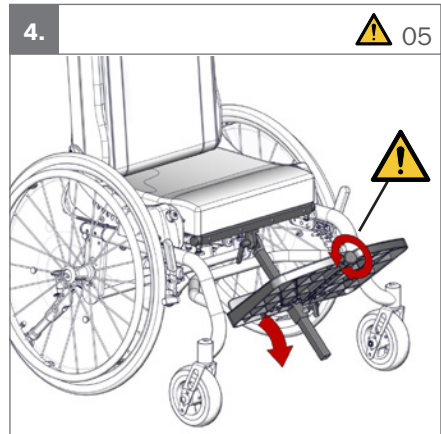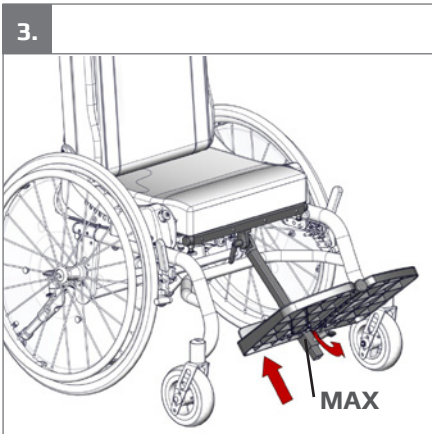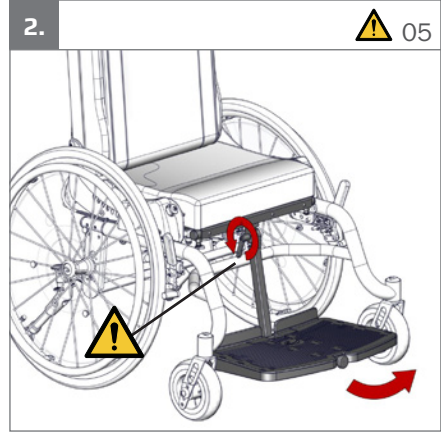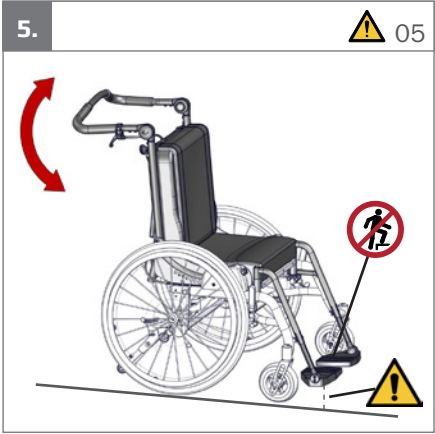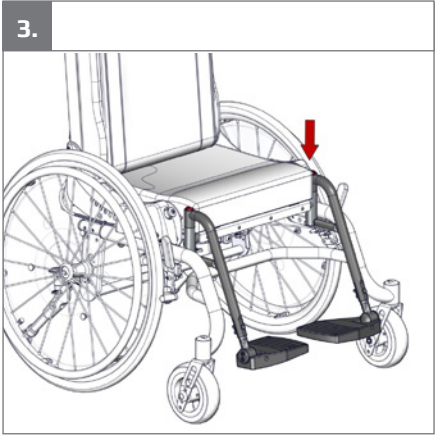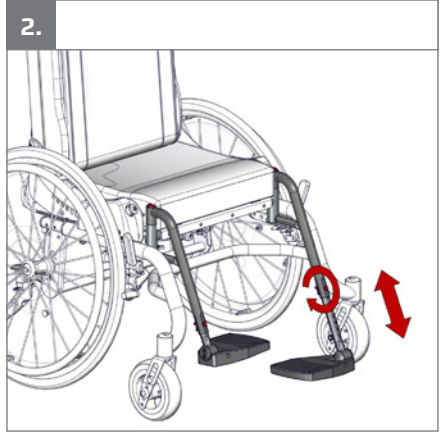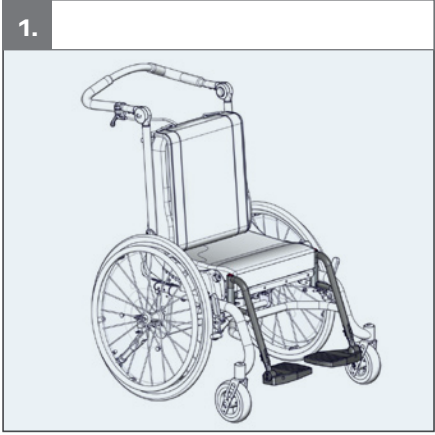1.

2.

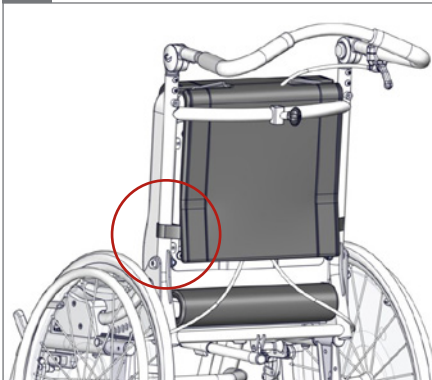
3.

4.

5.

6.

1.

2.

3.

4.

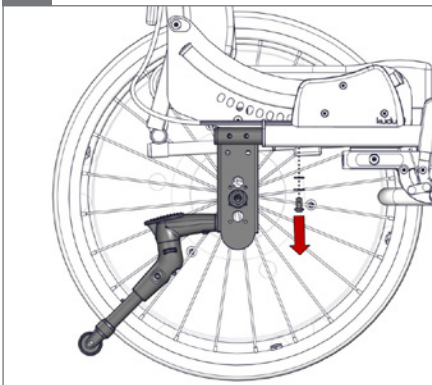
1.


2.

○ 5 mm

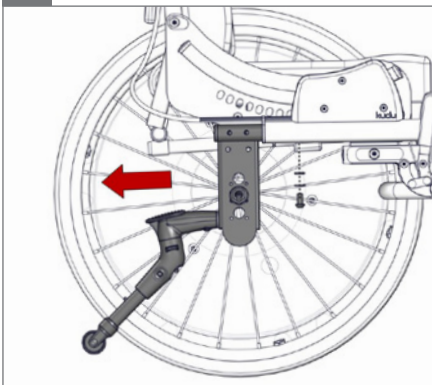
3.

4.

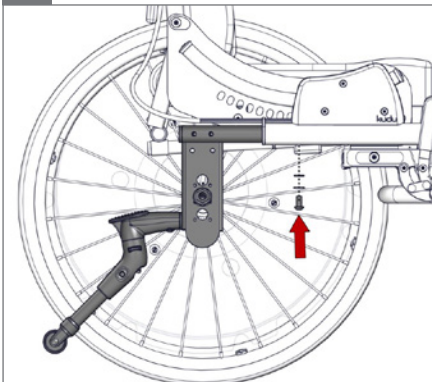○ 5 mm - 2,5 Nm

5.

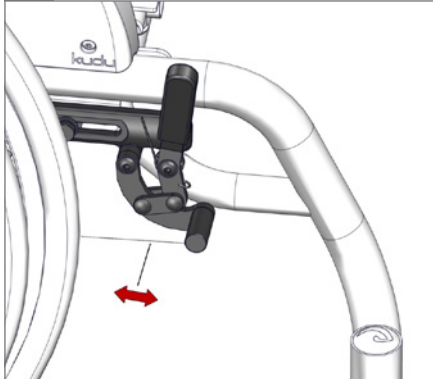
6.


7.


8.

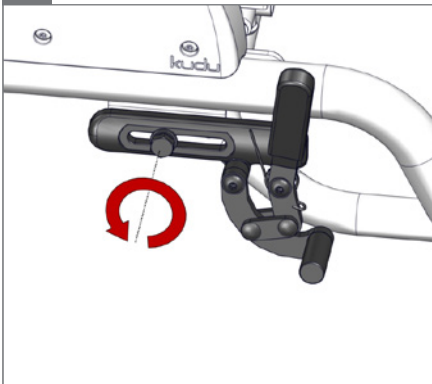13 mm


9.

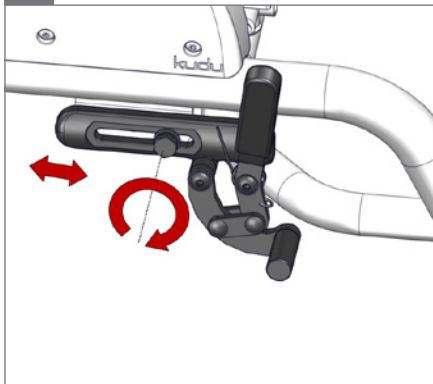
13 mm - 2,5 Nm

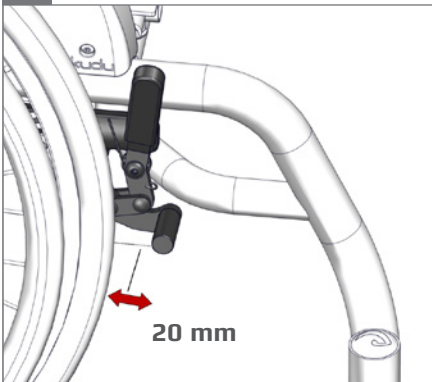10.

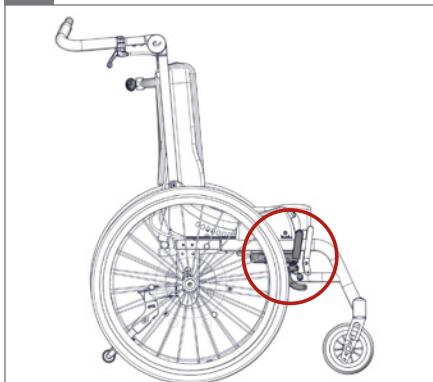11.




	20"	22"	24"	
1	-	-	a 480	
2	-	a 480	c 505	6"
3	a 480	c 505	-	
2	-	-	b 505	7"
3	-	b 505	c 530	


	20"	22"	24"	6"	7"
Min psi	85	85	85	-	-
Max psi	145	145	145	-	36
Handrim diameter	445 mm (17½")	480 mm (19")	530 mm (21")	-	-


				
	12½"	16"		
1	-	a 480		6"
2	a 480	-		

	12½"	16"	6"
Min psi	-	35	-
Max psi	26	65	-

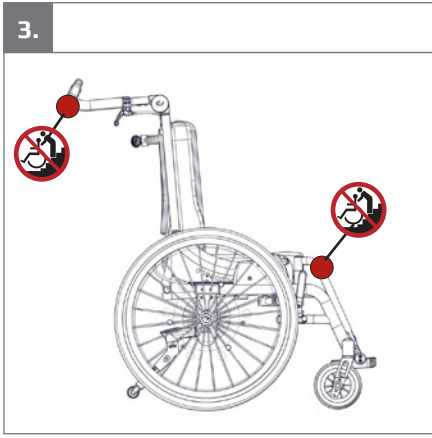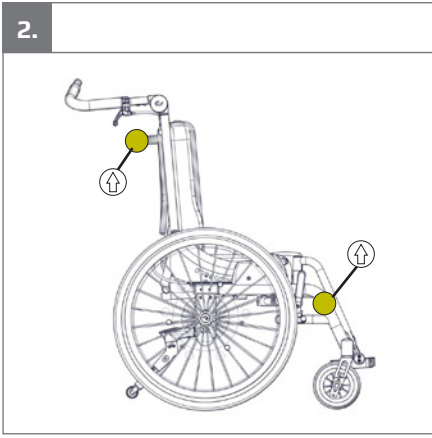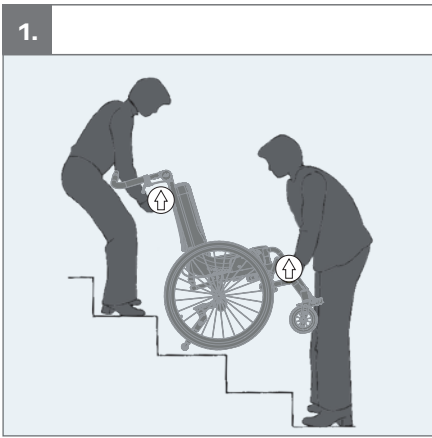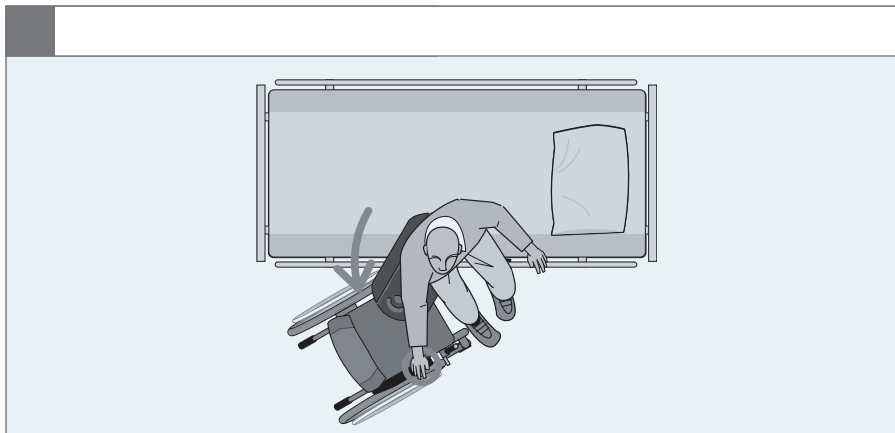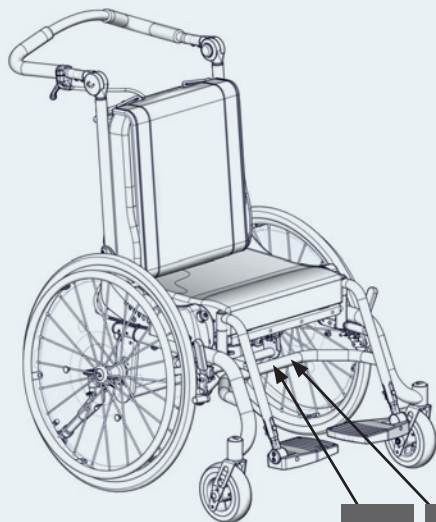A **B**

A

R82[®] 

by Etac

R82 A/S Parallevej 3
 DK-8751 Gedved
 etac.com
 Made in XXXXXXXX YYYY-MM-DD

B

Product: xxxx - Size x
 Art. no.: xxxxxxxx Max user mass: xxx kg
 XXXXXXXX Max user mass: xxx kg
 DMR ver: XXXX Mass device: xxx kg
 Max rated load: xxx kg

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

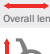
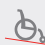
mm (inch)	 1	 2	 3	 4
 Mass device	19 kg (42 lb)	21 kg (46 lb)	23 kg (51 lb)	25 kg (55 lb)
 Max user mass	40 kg (88 lb)	50 kg (110 lb)	60 kg (132 lb)	60 kg (132 lb)
 Max user mass (in transport)	40 kg (88 lb)	50 kg (110 lb)	57 kg (125,5 lb)	57 kg (125,5 lb)
 Max rated load	61 kg (134.4 lb)	71 kg (156.5 lb)	85 kg (187.3 lb)	87 kg (191.8 lb)
 Folded length	825 (32½")	825 (32½")	875 (34½")	875 (34½")
 Folded height	625 (24½")	625 (24½")	625 (24½")	625 (24½")
Mass of the heaviest part	13 kg (28,7 lb)	15 kg (33 lb)	17 kg (37,5 lb)	19 kg (41,9 lb)
 Leg to seat surface angle, centrally mounted footrest	55–180°	55–180°	55–180°	55–180°
 Leg to seat surface angle, parted footrest	105°	105°	105°	105°
 Armrest to seat distance	138–198 (5½–7¾")	138–198 (5½–7¾")	138–198 (5½–7¾")	138–198 (5½–7¾")
 Front location of armrest structure	590 (23")	590 (23")	640 (25")	640 (25")
 Horizontal location of axle	146–196 (5¾–7¾")	146–196 (5¾–7¾")	146–196 (5¾–7¾")	146–196 (5¾–7¾")
 Minimum turning radius	823 (32½")	823 (32½")	823 (32½")	823 (32½")
Required width of right angled corridor	1000 (39½")	1000 (39½")	1100 (43¾")	1100 (43¾")
 Stability sideways	10°	10°	10°	10°
 Static stability downhill	10°	10°	10°	10°
 Static stability downhill	10°	10°	10°	10°

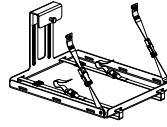9996097001

9996097709

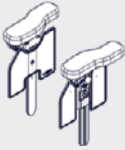
9996097002


9996097010

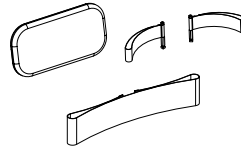
9996097003

9996097011

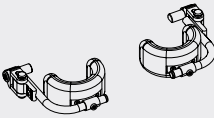9996097004

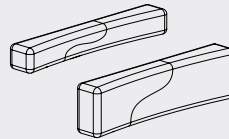
9996097012

9996097005

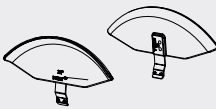9996097013

9996097007

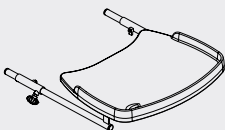
9996097015

9996097006

9996097740

9996097008

9996097137

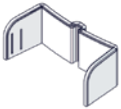
9996097187

N/A

9996097748

N/A

9996097338

N/A

9996097339

N/A

9996097741

N/A

9996097016

N/A

9996097608

N/A

N/A

N/A

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The device is a wheelchair for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position or move around without an assistive device. Thus, the target group for the device is based on functional ability and not a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Contra indications

There are no known contra indications

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- Movement function (e.g. experience involuntary movements), consider using a device with a dynamic backrest.
- Ability to maintain a seated position, use the product with accessories (e.g. side supports, vests, belts or arm supports and a tray).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

The product is for indoor and outdoor use

Enclosed tools:

- 4+5 mm Allen key
- 13 mm spanner

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12183

The CE mark must be removed when:

- Rebuilding the product
 - Using the product outside the intended purpose and declaration of conformity
 - Using other than original R82 spare parts and fittings
- When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may

be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the etac homepage/download.

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- www.etac.com

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Be aware of any MAX mark indications before adjustment
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Removable parts without use of tool: wheels, foot support and cushions. The removal will have adverse effect on the device
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Ease of access to and maneuverability in, motor vehicles can be significantly affected by size of device and turning radius, and the smaller device and/or device with a shorter turning radius will generally provide greater ease of vehicle access and maneuverability to a forward facing position. In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"
- Unoccupied device is suitable for land and air transport
- Attaching additional loads can adversely affect stability. Position of points intended to carry additional loads are specified in the relevant Mounting Guide referred to under section for accessories
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82: Cushions

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

Contact local dealer for assistance, if needed, in setting up, using or maintenance of the device.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Tire pressure

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Washing

The cushion covers can be washed at 60 °C using a mild detergent in a washing machine.

The foam inserts can be washed at 40 °C using a mild detergent in a washing machine.

During and after washing the foam, do not bend, squeeze or wring it as it may negatively affect the shape

Washing machine

This product can be washed at 60 °C using a mild detergent in 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users), every time the product is reissued for use and after long term storage (more than 4 months). The inspection should be carried out by a technically competent person who has been trained in the use of the product.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Seat tilt system is adjusted to run freely and without excess movement
- Do not oil the tilt system
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Quick release wheels are removable/replaceable
- Anti-tips can be fixed in working and resting position
- Tire pressure
- Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard
- Before storage: Always deactivate wheel locks/brakes
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions.

Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

Recommended dimensions	Ready for use (mm)
Overall length - maximum	1200
Overall width - maximum	700
Overall height - maximum	1200
Turning diameter - maximum	1000
Pivot width - maximum	1300
Ground clearance - minimum	30

Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

Wheelchairs transportable by train - limits	
Overall length - maximum	1200 mm + 50 mm for feet
Overall width - maximum	700 mm + 50 mm on each side for hands
Overall height - maximum	1375 mm (including 95th percentile male occupant)
Turning diameter - maximum	1500 mm
Smallest wheels	Accommodate a gap of 75 mm horizontal and 50 mm vertical
Weight - maximum	200 kg for manual wheelchair and occupant (including baggage)
Obstacle height that can be overcome - maximum	50 mm
Ground clearance - minimum	60 mm under foot rest with upward slope angle of 10° on top for going forward
Safe slope on which the wheelchair remain stable - maximum	6° dynamic stability in all directions 9° static stability in all directions, including with brake applied

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