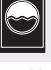
# Swan Curo User Guide

EN 12182  
ISO 17966

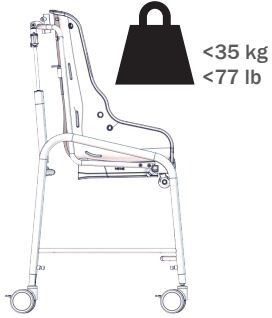
2021-10-rev.1.0

MD CE  
etac.com


	3	ENGLISH DEUTSCH	20 24
	4	NEDERLANDS DANSK	28 32
	5	NORSK SVENSKA	36 39
	6	SUOMI ESPAÑOL	43 47
	7	FRANÇAIS ITALIANO	51 55
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	59 63
	9	РОССИЯ POLSKI	67 71
	10	ČESKY 中國	75 79
	11	日本 ΕΛΛΗΝΙΚΗ	82 86
	11		
	12		
mm (inch)	16		
	19		


### Swan Curo size 1


### Swan Curo size 2



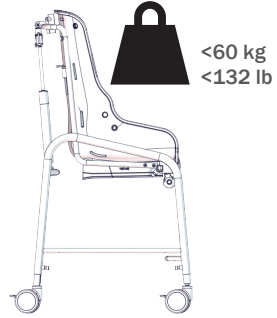
### Swan Curo size 3









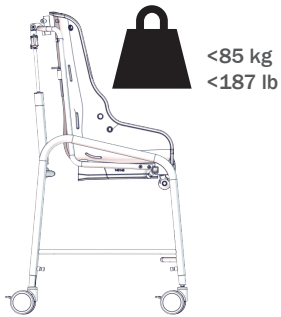
### Swan Curo size 4

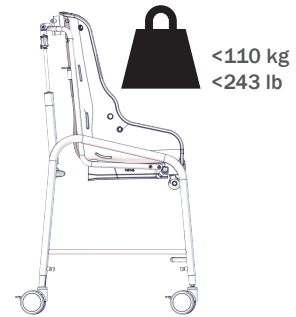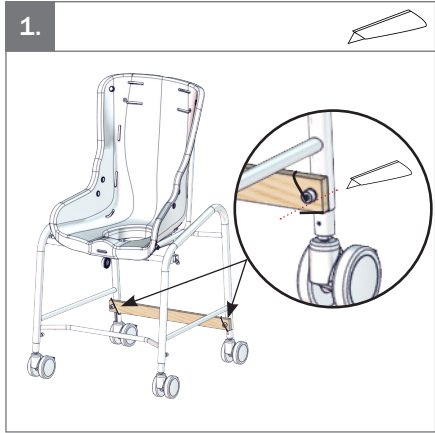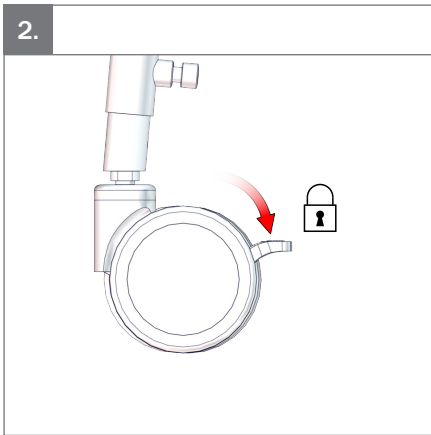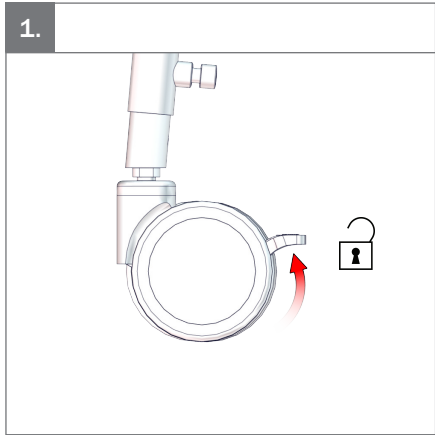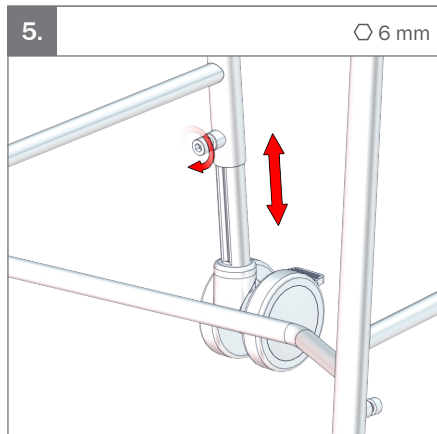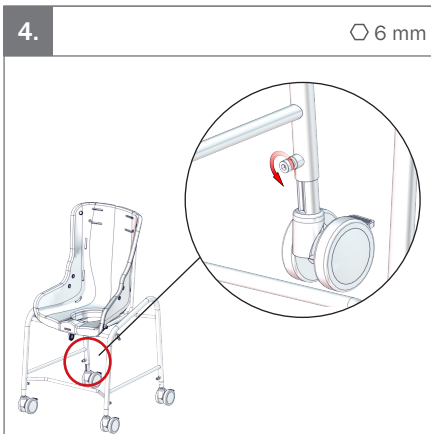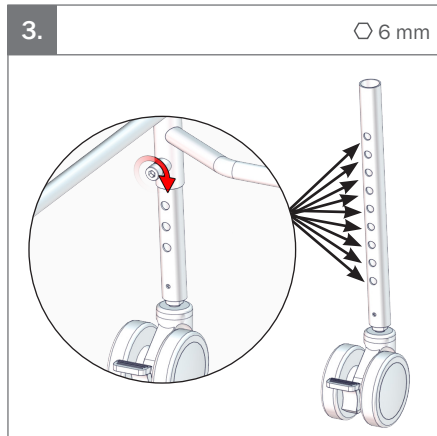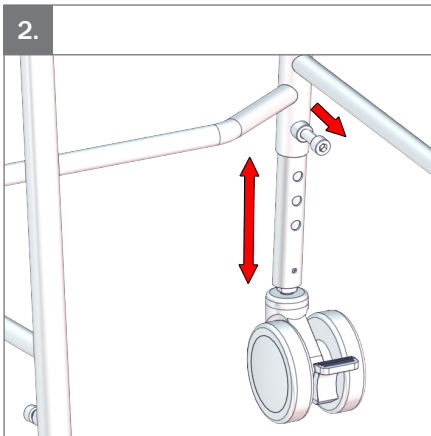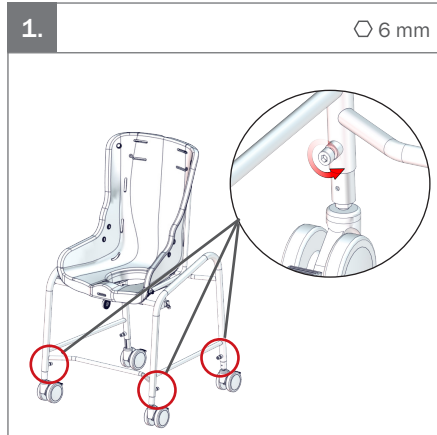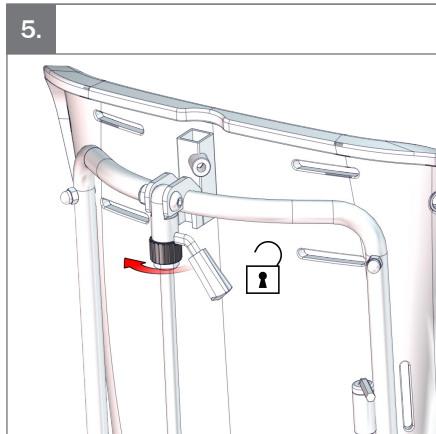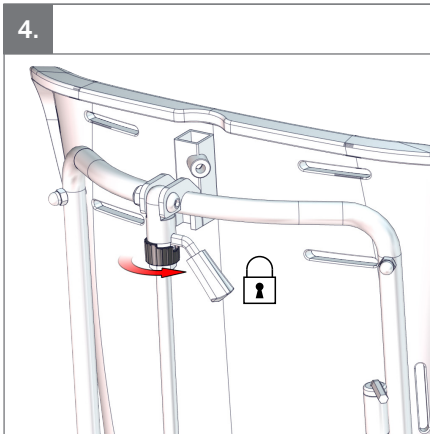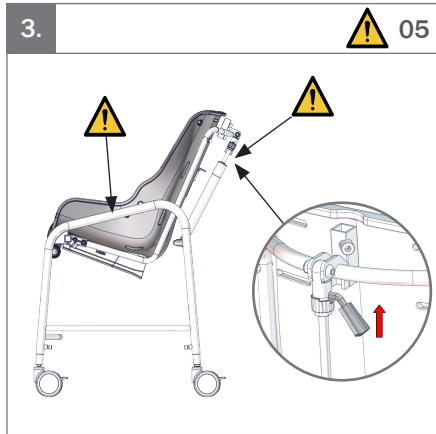
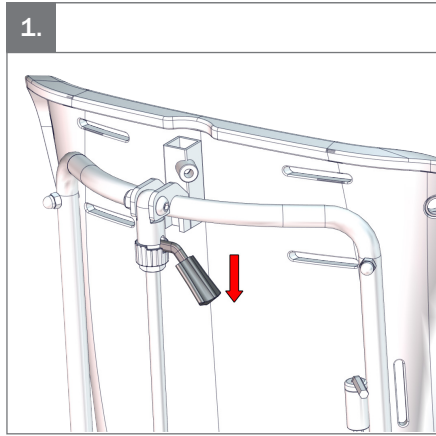
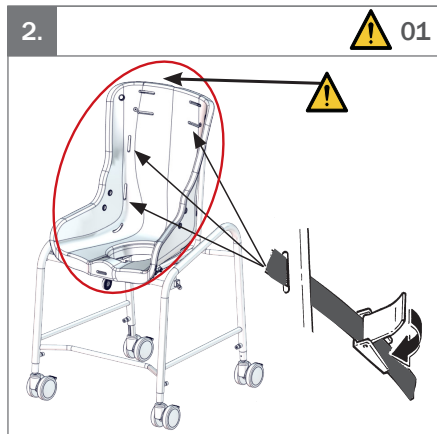
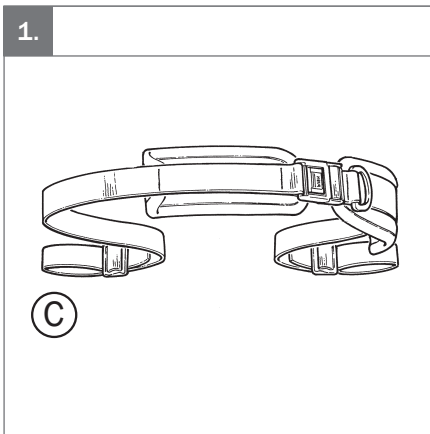
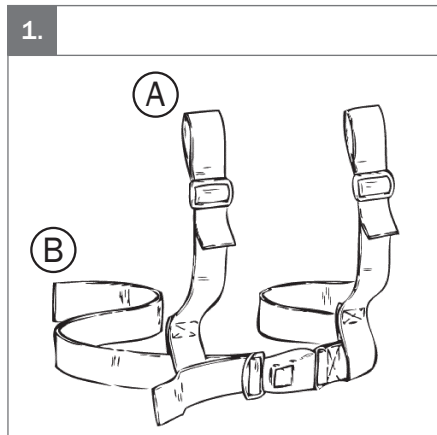
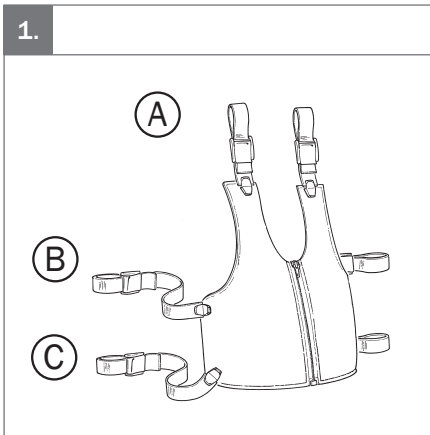
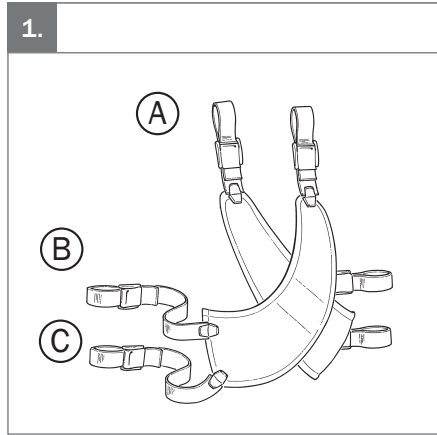
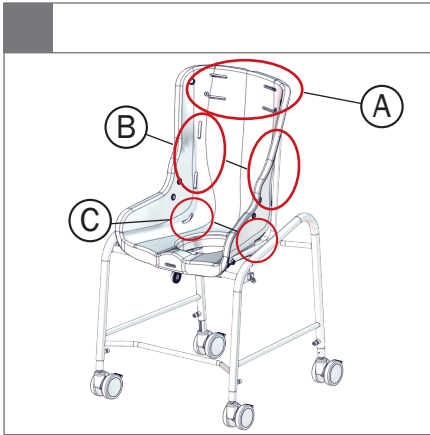
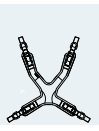
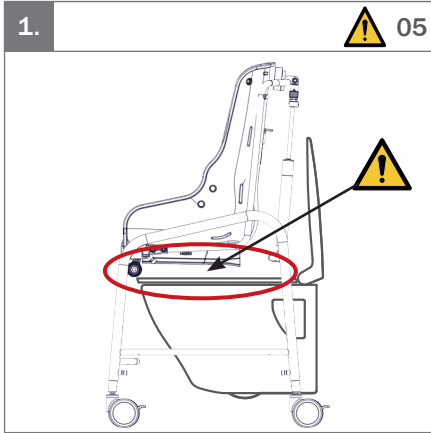
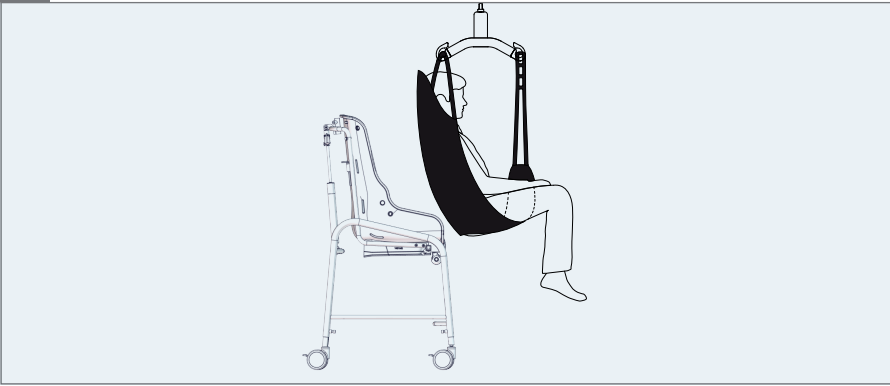
1.

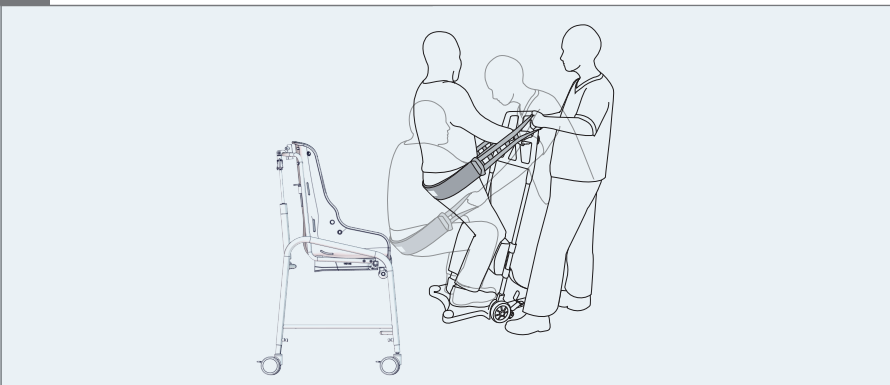
1.

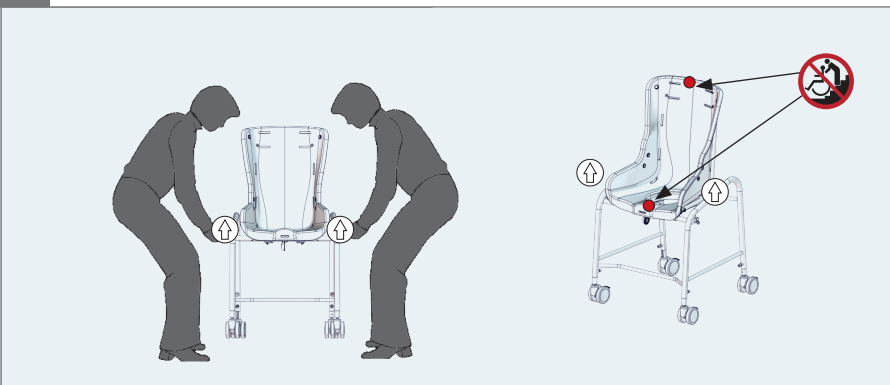1.

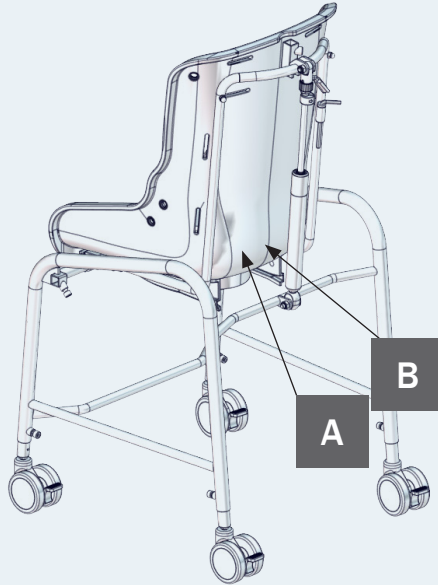1.

A

Product: xxxx - Size x  
 Art. no.: xxxxxxxx Max user mass xxx kg  
 [SN] XXXXXXXX  
 DMR ver.: XXXX Mass device: xxx kg  
 Max rated load: xxx kg

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


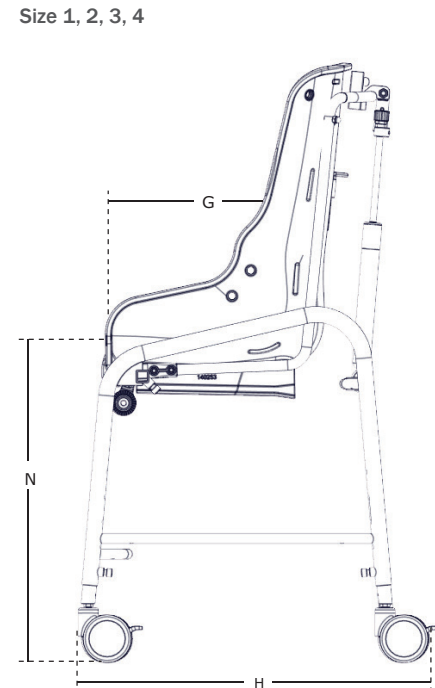
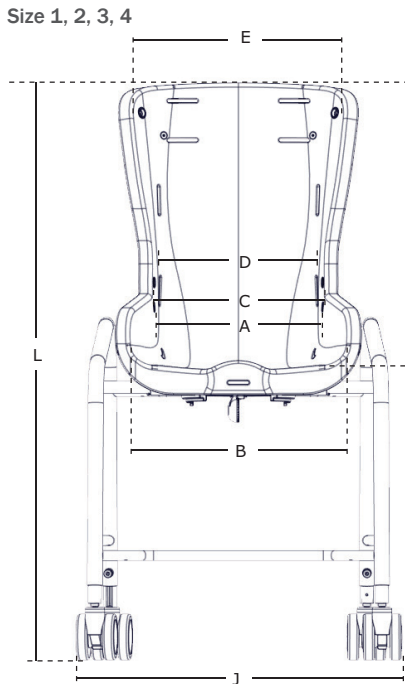
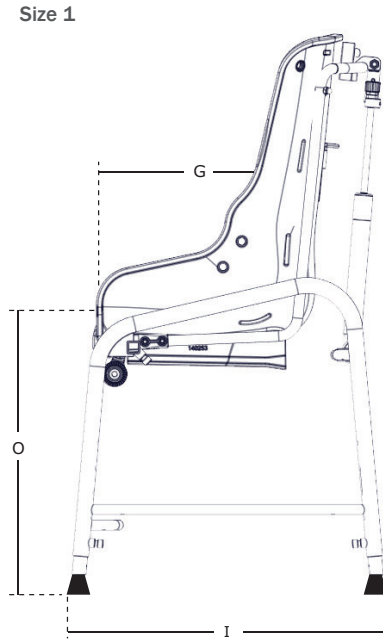
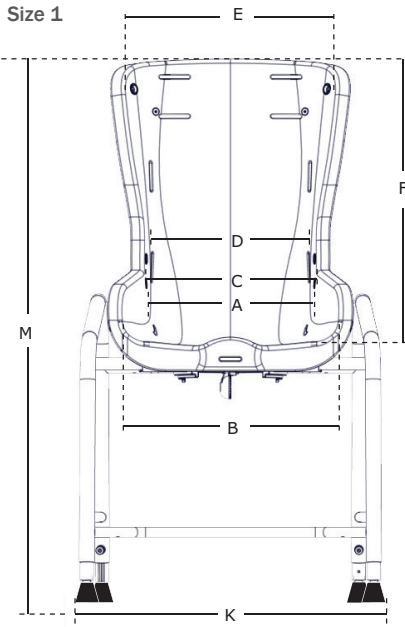
mm (inch), kg (lb) (Inch and lbs is rounded to nearest ¼ value)		1	2	3	4
	Overall height, with castors (L)	760-920 (30,00-36,25)	960-1120 (37,75-44,00)	990-1150 (39,00-45,25)	1060-1220 (41,75-48,00)
	Overall height, without castors (M)	670-830 (26,50-32,75)	N/A	N/A	N/A
	Mass device, with castors	8,50 kg (18,75 lb)	10,50 kg (23,25 lb)	11,75 kg (26,00 lb)	12,75 kg (28,00 lb)
	Mass device, without castors	7,50 kg (16,50 lb)	N/A	N/A	N/A
	Max. user mass	35 kg (77,25 lb)	60 kg (132,25 lb)	85 kg (187,50 lb)	110 kg (242,50 lb)
	Max. rated load, with castors	46 kg (101,50 lb)	73 kg (161,00 lb)	99 kg (218,25 lb)	125 kg (275,50 lb)
	Max. rated load, without castors	45 kg (99,25 lb)	N/A	N/A	N/A
	Aperture (hole dimensions)	185x125 (7,25x5)	220x150 (8,75x6)	280x180 (11x7)	330x180 (13x7)
	Aperture with inlay (hole dimensions)	175x100 (7x4)	120x125 (8,25x5)	275x155 (10,75x6)	320x155 (12,50x6)
	Seat plane angle (tilt)	0° - 20°	0° - 20°	0° - 20°	0° - 18,5°


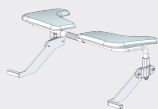
9996098105

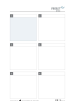
9996098106

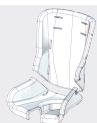
9996098107

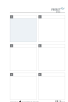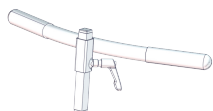
99960978108

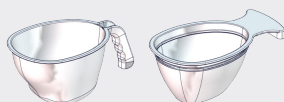
9996098109

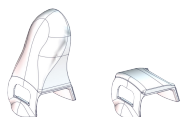9996098111

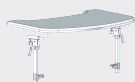
9996098112

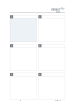9996098113

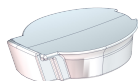9996098114

9996098115

## ENGLISH

### General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

Assistive device for toileting, washing, bathing and showering for children and adults with disabilities.

### Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position without an assistive device or to compensate for the lack of ability to maintain a standing position while washing oneself or showering. The target group for the device is based on functional ability. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The device can be applied more times during the day. Between using the device, the user will be lying down, standing or sitting in another assistive device or a conventional product.

### Contra indications

There are no known contra indications.

### Precautions

Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- Ability to maintain a seated position, use the device with accessories (e.g. vests, belts, or a table).
- Ability to maintain the position of their head, consider using accessories that provide support to the positioning of the head.

### Application areas

- The product is for indoor use
- This equipment is for use in home care environment.

### Enclosed tools

- 6 mm Allen key

## CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity

- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the [etac.com/support](http://etac.com/support).

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon

whether to replace or repair the defective product.  
It is the customer's responsibility to return the item  
being claimed for under warranty to the address of  
purchase.

**Presales information**

- [www.etac.com](http://www.etac.com)

EN

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

## Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Always use the correct lifting techniques and aids for this purpose.
- Ensure the stability of the product is maintained before placing the user in it.
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Be careful not to get body parts trapped, when placing the product over the toilet.
- Activate the brakes before placing the user into the product.
- The user must not occupy the product when it is lifted from the ground.
- All gasspring are to be kept lock, when the device is not being adjusted.
- Be aware of any MAX mark indications before adjustment.

## Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitve skin as they cannot feel the heat or cold. If the surface temperature is above 41 °C then allow the product to cool before use.
- For the purpose of safety and comfort, always check the water temperature before bathing the user.

## User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

## Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Ensure the stability of the product is maintained before placing the user in it.
- Unoccupied device is suitable for land and air transport.
- The device is only designed to travel short distances within the same room.
- Resistance to ignition is not claimed for this product.
- The product might change its resistance to ignition characteristics if using other than original R82: Cushions.

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference

### Before every use

- Disinfect the product. Please check the disinfection section in this User Guide for more information

### After every use

- Wipe the product dry.

### Daily

- Use a cloth with warm water and a mild detergent to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Check all fittings and buckles for signs of worn out parts
- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Ensure that all fasteners are in place and fitted correctly

### Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures

### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Warewashing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

### Before washing

Remove the following items and clean them separately: Cushioning + Accessories

### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months if the product is used very often - daily use for more than 5 times), every time the product is reissued for use and after a long term storage (more than 4 months). The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in

accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime

### Service check list:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Accessories can be inserted and locked in the fittings in the top of the back, using the knob and handles mounted in the fittings.
- Seat is adjustable and can be locked in position.
- Fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

### Materials

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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© 2021 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
## R82 A/S