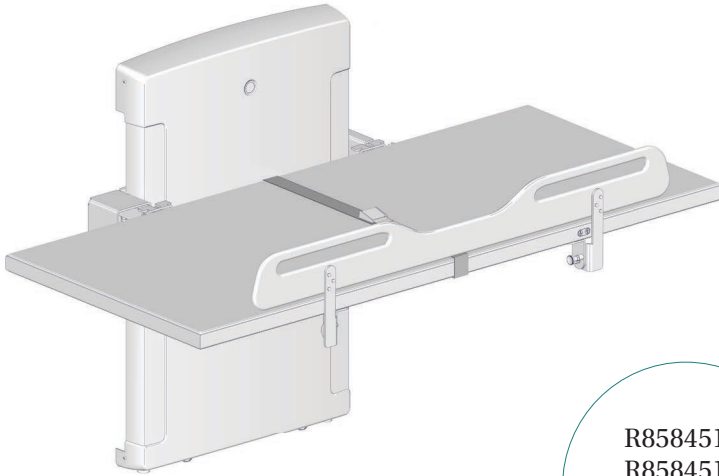
PRESSALIT®

R8584512
R8584514
R8584516
R8584518

Change table – Operation and maintenance manual

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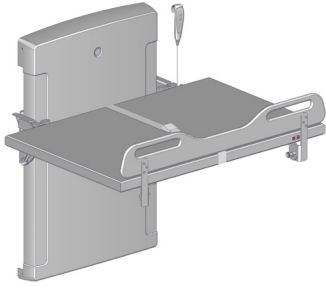
The warranty covers faults or defects in the material of our products within a period of 3 years from date of invoice. Products subject to minor technical modifications and design deviations E. & O.E.

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
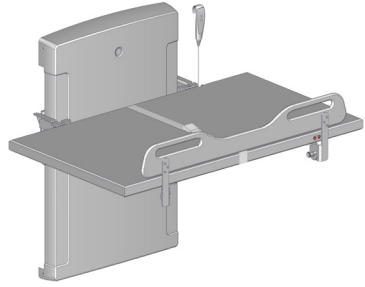
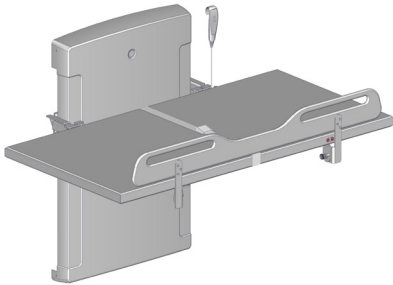
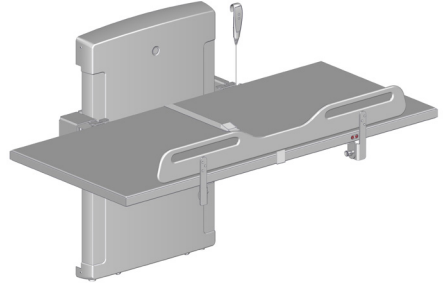
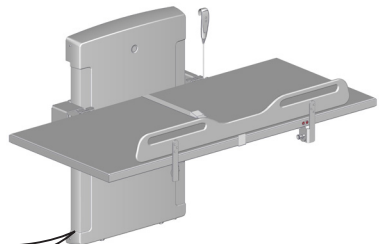
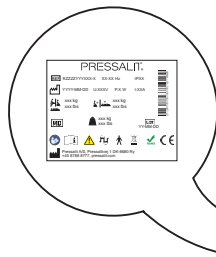
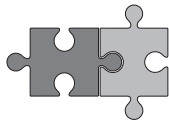
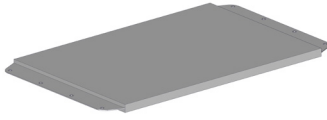
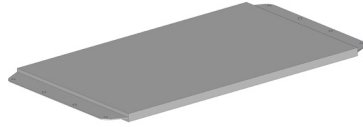
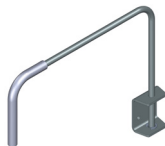
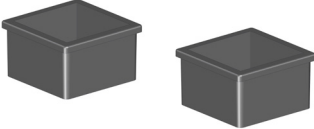
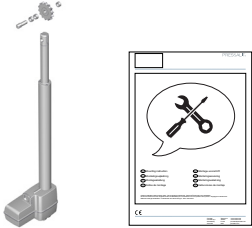
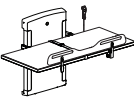
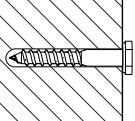
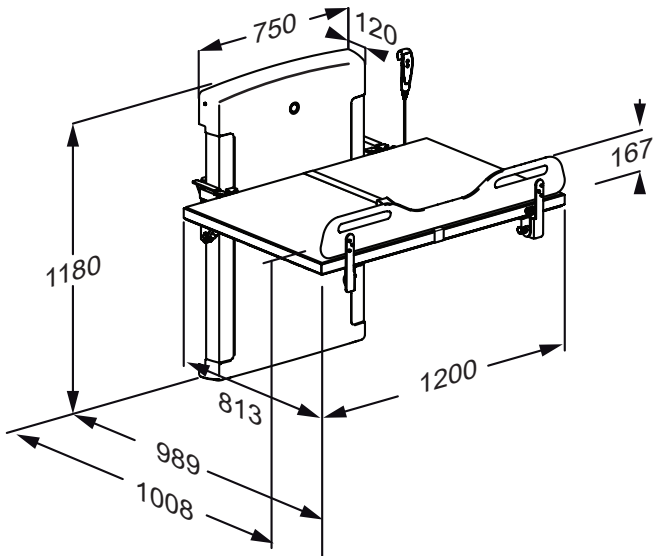
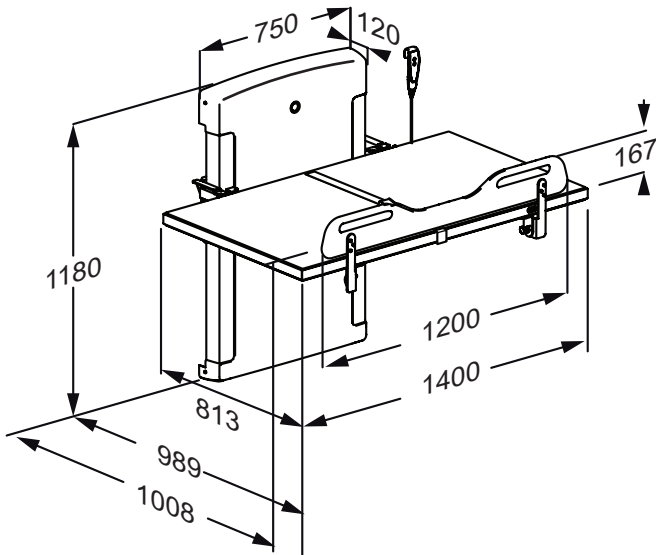
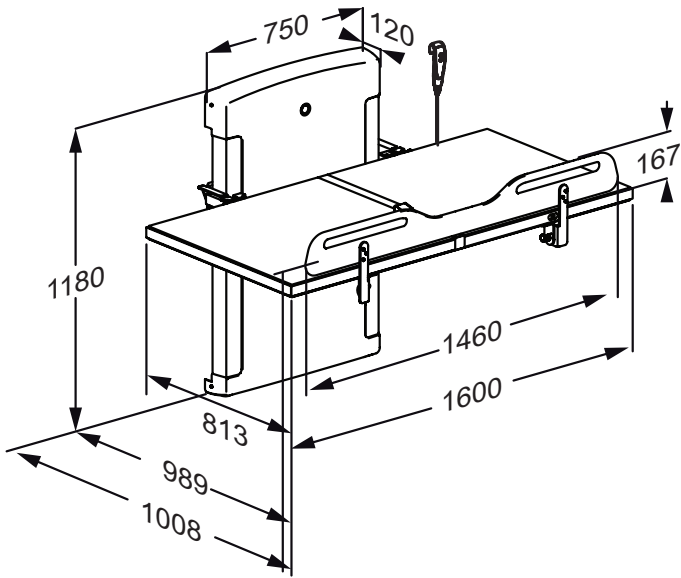
R8584516

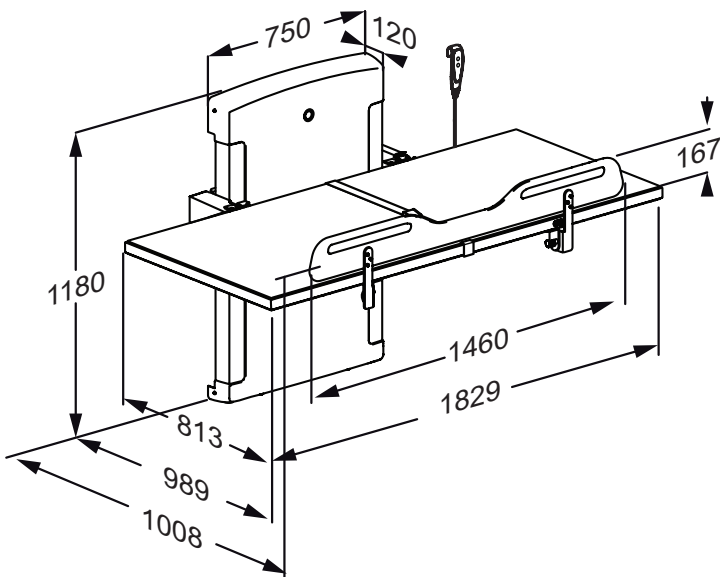R8584518

Symbols used in this manual	10
General safety	11
Intended use	12
Using alone (no helper)	12
Using with a helper	13
Intended operator profile	14
Intended user profile	14
Operating environment	14
Contraindications for use	14
Biocompatibility	15
Resistance to ignition	15
Operating instructions	16
Folding up and down	16
Height adjustment	17
The safety rail	18
The body belt	19
Cleaning	20
Disinfection	20
Maintenance	21
Maintenance inspection	21
Product label	24
Trouble-shooting	26
Electromagnetic compatibility	27
Electromagnetic immunity	28
Electromagnetic emission	31
Technical data	32
Storage and transportation	32
Disposal and recycling	33

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage is included with this product. It can also be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING				<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>
 CAUTION				<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>
IMPORTANT				<p>IMPORTANT indicates important information about handling and use of the product.</p>
NOTICE				<p>NOTICE is used to address practices not related to personal safety.</p>
 <p>Hand injury warning</p>	 <p>Head injury warning</p>	 <p>Electricity / voltage warning</p>	 <p>Other specific warning</p>	
 <p>Compatible products</p>	 <p>Spare parts</p>	 <p>Power specifications</p>	 <p>EMC related symbols</p>	
 <p>Product weight</p>	 <p>Max. user weight</p>	 <p>Article number / versions</p>	 <p>Do not leave patient unattended</p>	

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

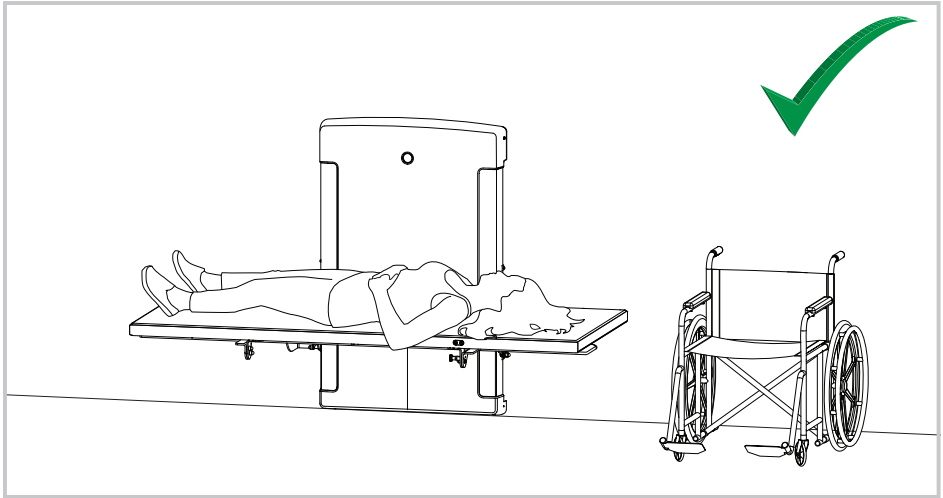
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use


Using alone (no helper)


 **CAUTION**

en

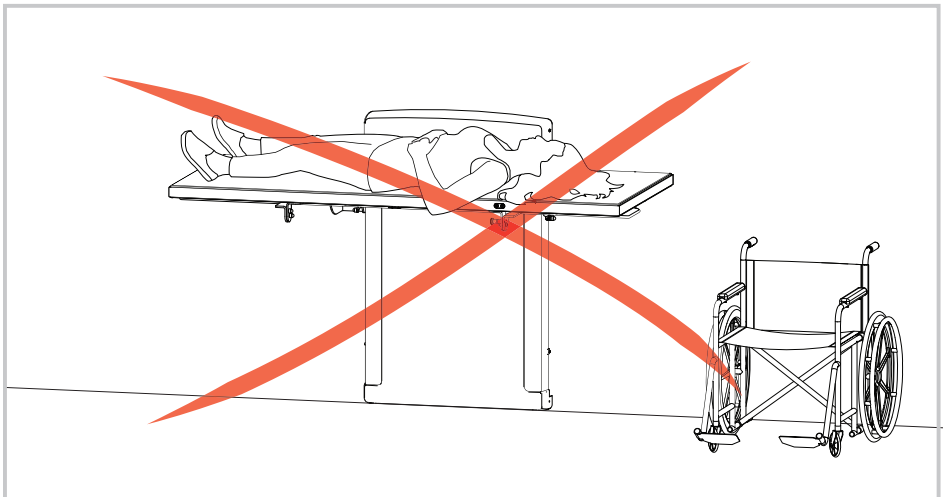This product has no manual lowering function, so in case of a power outage, the lying surface will stay in its current position. It is therefore not recommended to take the lying surface to the highest position when using the product alone.


Using with a helper.

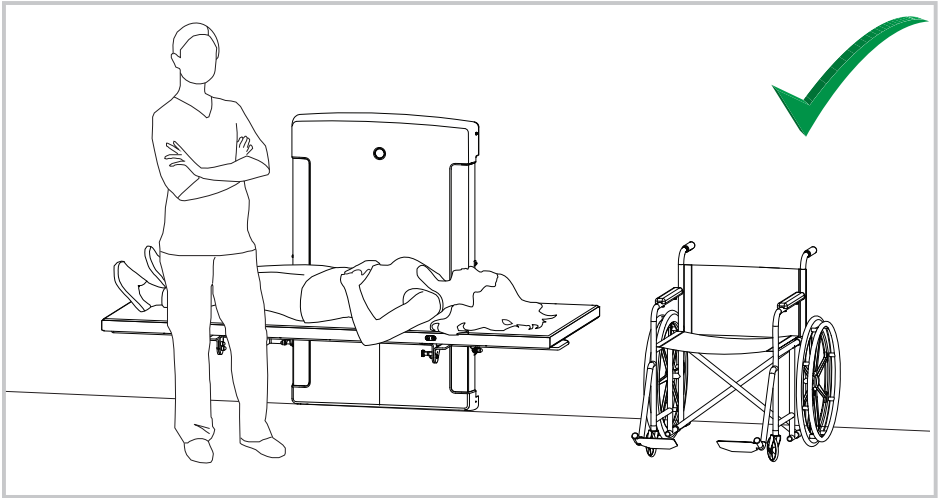 **CAUTION**

It is only recommended to take the lying surface to its highest position when using this product in the presence of a helper.

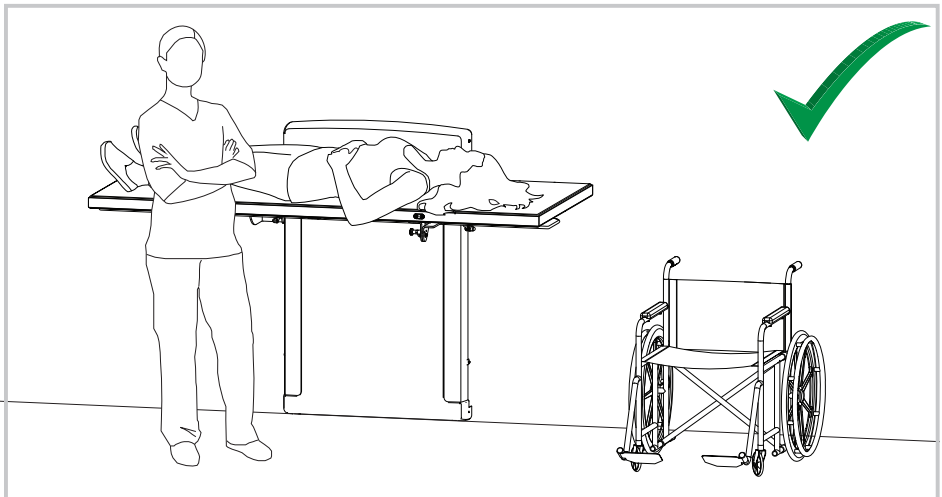en

This product is intended for people who

- need to get changed on a change table
- are above the usual change table age
- are caregivers and need a customized work position
- need to use a change table in combination with person lifts

Intended operator profile

The product is to be used by a person trained in health care and/or trained in changing a person in their care. It can also be used by self-reliant users.

Intended user profile

The product can be used by adults whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

For indoor use in public changing facilities and schools. Can also be used in nursing homes and hospitals under normal working conditions.

Contraindications for use

Use of this product is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the product only after consulting with the attending physician.

Do not use if the patient's weight exceeds 200 kg / 440 lbs.

Environmental factor	Operating range
Temperature	5°C to 40°C (41°F to 104°F)
Relative humidity	20% to 90% @ 30°C (86°F) (non-condensing)
Atmospheric pressure	700 to 1060 hPa

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose. Allergic reactions are not likely to happen based on the use of this product.

Resistance to ignition

The compact laminate lying surface has been tested in accordance with ISO 8191-1 (smouldering cigarette) and ISO 8191-2 (match-flame equivalent) with a satisfactory result.

Always be careful when moving a person to and from the table.

Do not leave the patient unattended.

en

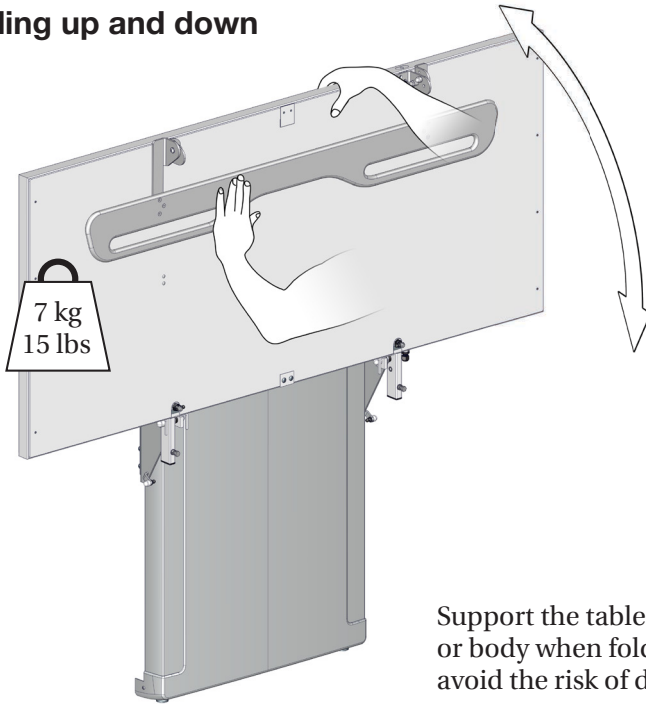
The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

Folding up and down

Support the table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the table up against the wall after use. Be sure to fold the safety rail away first.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

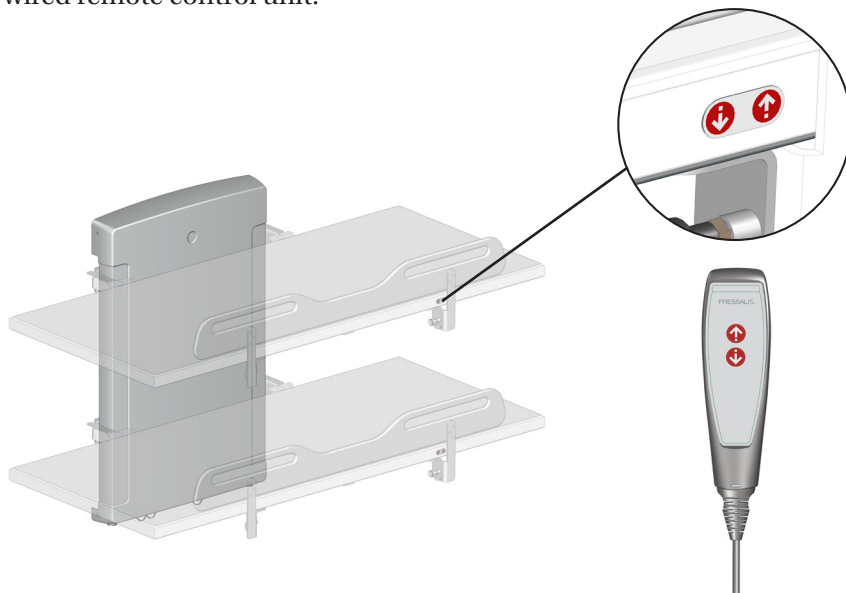
CAUTION

- Never allow children to operate the unit without supervision of an adult. The children are at risk of injuries, and the product may be damaged.
- Keep pets and pests away from the unit. The product may be damaged and/or contaminated.

Height adjustment

When the change table is installed at the recommended height, it can be adjusted between 300 and 1000 mm (11.81 and 39.37 inches) above the floor.

The height adjustment is done by pressing the buttons on the frame or on the wired remote control unit.

en

Press and hold the up button to raise the table, and press and hold the down button to lower the table. The movement stops when the button is released.

The height adjustment may only be used at temperatures between 15 and 30 degrees Celsius / 59 and 86 degrees Fahrenheit..

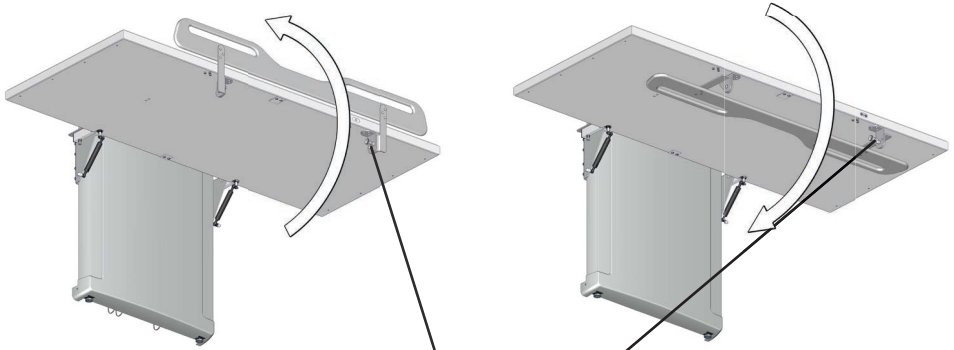
Make sure that there are no objects under the change table when adjusting the height.


To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

The safety rail

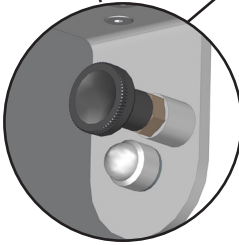
To pull the safety rail up: Release the lock ① and pull the safety rail from the underside of the table, and swing it upwards. The safety rail will automatically lock into position.


To fold the safety rail away: Release the lock ① and swing the safety rail down to the underside of the table. The safety rail will automatically lock into position.

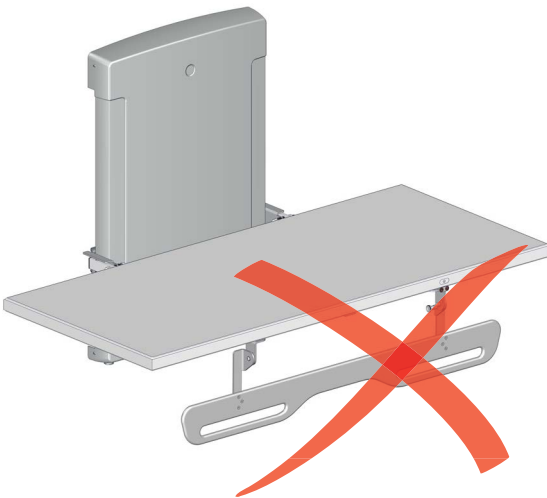en


Make sure that the safety rail never is left unlocked. It must always be locked in an upright position or folded completely under the table.

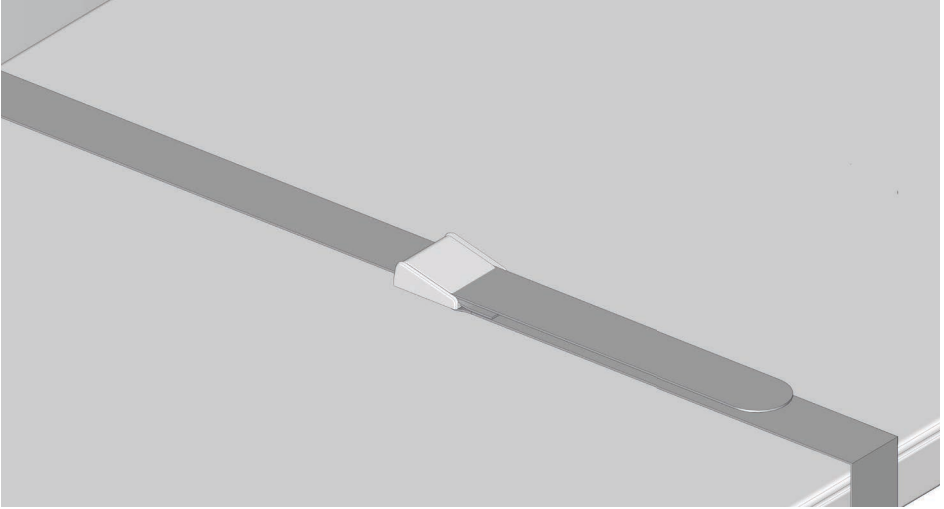The body belt

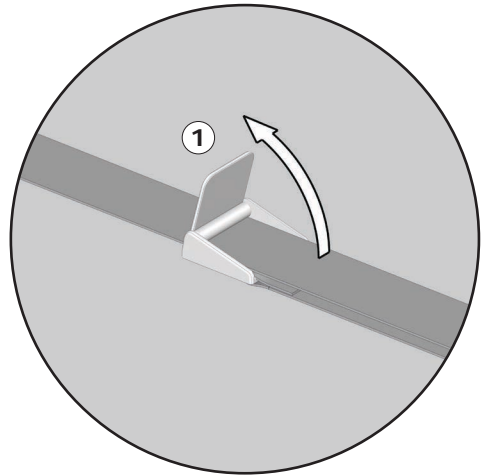The product is equipped with a body belt, that can be used to secure the user. Pull the belt lock lever ① up to release the belt lock. Press the lever down again to lock the body belt.

en

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

Disinfection

When using the change table for multiple patients, it is necessary to both clean and disinfect it between uses on different patients to prevent the risk of cross infection.

Maintenance

Expected service life of the change table is 10 years, provided that maintenance is carried out as described below:

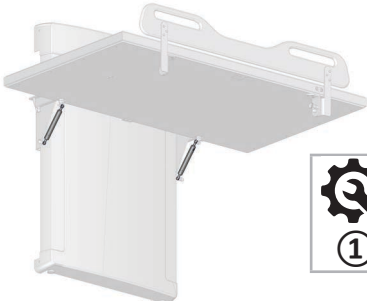
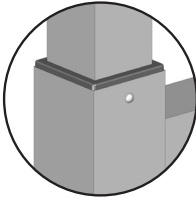
It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that inspection is done once a year.





CAUTION


Do not perform any type of maintenance inspection on the change table while a patient is lying on it.

Maintenance inspection

  	Gas cylinders The nuts must be fastened in both ends and the ball joints must be secure. If the gas cylinders are defective, they must be replaced.
   	Plastic inserts Check that the plastic inserts does not protrude more from the metal frame than shown. If they do, they need to be lubricated and pushed back into the metal part. If the inserts are damaged, they must be replaced.

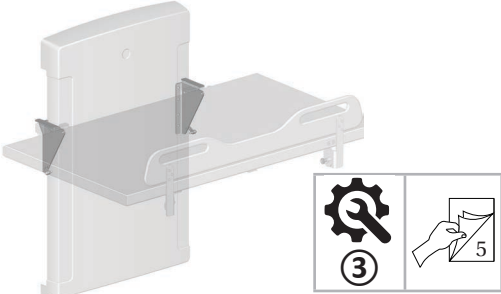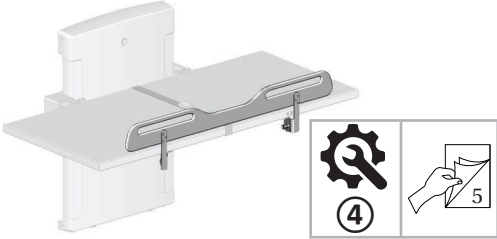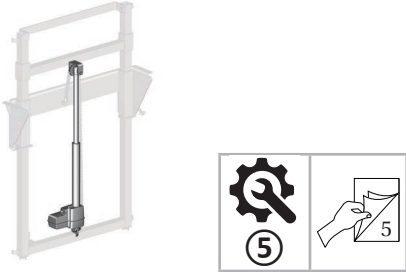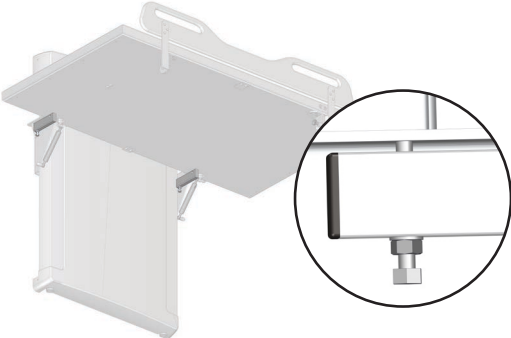
en

	<p>Table bearing brackets</p> <p>Check the two bearing brackets for wear and tear. Tighten the 8 bolts if necessary. If the brackets are damaged, they must be replaced.</p>
	<p>Safety rail</p> <p>Check that the safety rail lock engages correctly. Tighten the screws if necessary. If the safety rail is damaged, it must be replaced.</p>
	<p>Actuator</p> <p>The actuator might need replacement after 3-5 years of using the product. Refer to installation manual T5256.</p>
	<p>Table top</p> <p>Check that the table top is level. If not, tighten or loosen the adjustment screws. The two adjusting screws must be in the same position.</p>

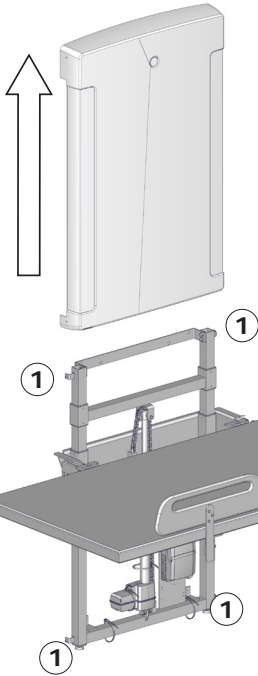
Disconnect mains before removing the covers.

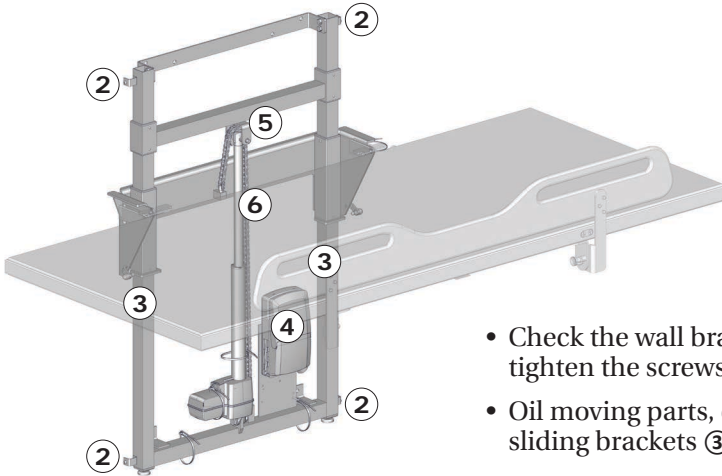Remove the upper and lower retaining screws ① on the lift unit cover and carefully remove the cover.

- Check all electrical connections to the control box ④.
- Check the chain wheel ⑤ for wear.
- Oil the chain ⑥.
- Put the unit cover back again and tighten the retaining screws.

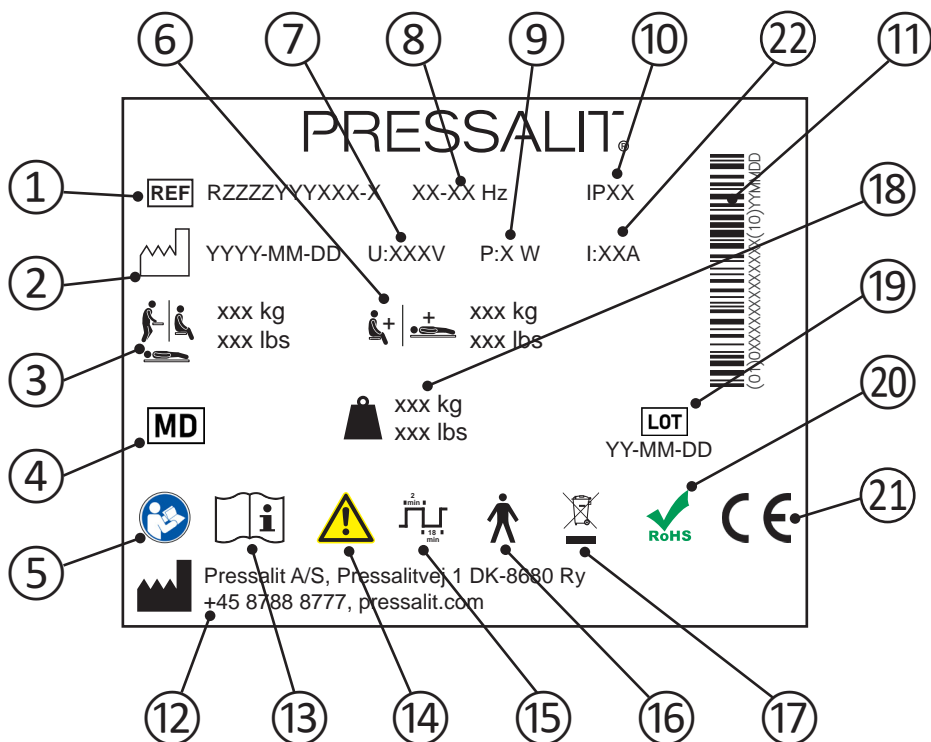Product label

① Article number

② Production date

③ Maximum user weight

④ Medical device

⑤ Consult manual for use

⑥ Max load

⑦ Mains voltage

⑧ Frequency

⑨ Maximum power consumption

10	IP class
11	GS1-128 barcode
12	Name and address of manufacturer
13	Consult manual for use
14	Consult the manual for important cautionary information such as warnings and precautions.
15	Duty cycle
16	Type B applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function. The product complies with requirements of IEC 60601-1 to provide protection against electric shock
17	Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
18	Product weight
19	Batch code
20	This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
21	<p>These products are CE-marked in accordance with relevant requirements in:</p> <ul style="list-style-type: none"> • Regulation (EU) 2017/745 (5 April 2017) on medical devices. • Council Directive 2006/42/EC (of 17 May 2006) on machinery. • Council Directive 2011/65/EC (of 8 June 2011) on RoHS. <p>A declaration of conformity can be found at pressalit.com</p>
22	Electric Current

Trouble-shooting

Symptom	Cause	Action
The change table does not stay in upright position.	One or more gas cylinders are defective.	Contact Pressalit A/S or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply. Defective cables can occur if pests have had access to the cables. Cables can inadvertently be pulled out if children have been playing near the product.	Check that all wires and plugs are intact and correctly connected. Any damaged cables must be replaced. Contact Pressalit A/S or your local dealer.
The change table runs up and down unevenly.	Dust or lint might cause disturbance in the height adjustment.	Wipe off any dust or lint and oil the sliding brackets as described under "Maintenance".
The table top is not level.	Adjustment screws too high or too low.	Adjust the adjustment screws as described under "Maintenance".

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user still can get on and off the product.

Cable	Length
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

en

Electromagnetic immunity

Electrostatic discharge	
	<p>The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-2</p>	<p>+/- 8 kV contact +/- 2, 4, 8, 15 kV air</p>

Radiated RF electromagnetic field	
	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-3</p>	<p>10 V/m 80 MHz to 2,7 MHz 80 % AM 1 kHz</p>

Immunity to proximity fields from RF wireless communication equipment

	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-3*</p>	<p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave) 450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine 710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p>

en


Electrical fast transient/burst

	<p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-4</p>	<p>Signal line: +/- 1 kV, AC power line: +/- 2 kV 100 kHz repetition frequency</p>

Surge immunity


	The product must not be affected to initiate actions or to render components defective as a result of lightning.
Test standard	Immunity test level
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/-1 kV Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV

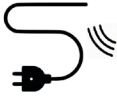Immunity to conducted disturbances induced by RF fields

	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

en

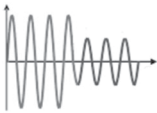Voltage dips

	The product must not initiate or limit actions as a result of a dip in the power supply.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U_T ; 1 cycle at 0°, 70 % U_T ; 25/30 cycles at 0° Single phase at 0°

Voltage interruptions

	The product must not initiate or limit actions as a result of a short-term power supply stop.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 300 cycle. 250/300 cycles

Electromagnetic emission

Conducted RF emission	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Radiated RF emission	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1


Harmonic current emissions	
	The product must not unintentionally draw power from the grid.
Test standard	Emission test class and group
IEC 61000-3-2	Class A

Voltage changes, voltage fluctuations and flicker	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
Test standard	Emission test class and group
EN 61000-3-3	Tested according to the standard.

Technical data

Materials	Powder laquered stainless steel Plastics (PVC, Polystyrene)
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.
Measured sound pressure level	48 dB

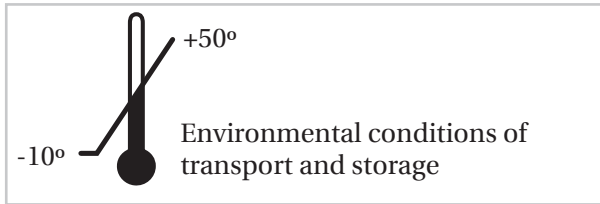Storage and transportation

- The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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