Invacare® Dacapo® EcoGreen

Dacapo
ecogreen

en	Mattress system User Manual	3
da	Madrassystem Brugsanvisning	13
de	Matratzensystem Gebrauchsanweisung	23
nl	Matrassysteem Gebruiksaanwijzing	33
fi	Patjajärjestelmä Käyttöohje	43
sv	Madrassystem Bruksanvisning	53
no	Madrassystem Bruksanvisning	63

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

Yes, you can.®

©2022 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this Document	4
1.3 Compliance	4
1.4 Warranty Information	4
1.5 Limitation of Liability	4
1.6 Service Life	4
2 Safety	5
2.1 Safety Information	5
2.2 Safety Information for Transport	5
3 Product Overview	6
3.1 Product Description	6
3.2 Intended Use	6
3.2.1 Indications	6
3.3 Scope of Delivery	6
3.4 Symbols on the Product	6
4 Usage	7
4.1 Safety Information	7
4.2 Preparing the Mattress for Use	7
4.3 Using the Handles	7
5 Maintenance	8
5.1 Inspection	8
5.2 Cleaning and Disinfection	8
5.2.1 General Safety Information	8
5.2.2 Cleaning Intervals	8
5.2.3 Cleaning Instructions	8
5.2.4 Disinfection Instructions	8
5.3 Replacing Cover	9
6 After Use	10
6.1 Storage	10
6.2 Reconditioning	10
6.3 Disposal	10
7 Technical Data	11
7.1 General Data	11
7.2 Materials	11

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Only use this product if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I. The launch date of this product is stated in the UKCA declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

1.4 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

1.6 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety Information

Education, clinical judgement and action-based planning based on vulnerability are fundamental factors in prevention of pressure injuries.

A range of assessment scales can be used as a formal method of assessing risk from pressure injury development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of fire or explosion!

A cigarette can burn a hole in the bed surface and cause damage to the device. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Failure to observe this warning can result in a severe fire, property damage and cause physical injury or death.

- Take special care in oxygen rich environments.
- Do not smoke.

WARNING!

Risk of Developing Pressure Injuries

The device delivers effective pressure redistribution, when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these is the only item deployed between the support surface and the user.

- Bed sheets must be loosely fitted, with creases smoothed out.
- Ensure that the support surface in contact with the user is kept free from crumbs and other food debris, and that drip cables, stents, and other foreign objects do not become entrapped between the support surface and the user.
- Heated over blankets must only be used in consultation with a qualified healthcare professional, as an increase in temperature can increase the risk of developing pressure injuries.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

2.2 Safety Information for Transport

- Take care when handling the product to ensure no damage to the cover.
- It is recommended that two people lift/carry the product.
- Avoid contact with jewellery, nails, abrasive surfaces, etc.
- Do not drag the product.
- Avoid contact with wall, door frames, door catches or locks, etc.
- Do not transport in roll cages unless completely protected from the sharp edges of the cage.

3 Product Overview

3.1 Product Description

Single-layered support system made from a bio-based high resilience foam. The zoned castellation of the top layer minimizes surface pressure, which allows for optimal pressure redistribution.

3.2 Intended Use

This pressure redistribution mattress is intended to be used in conjunction with an appropriately sized bed frame and side rails. Check the bed's user manual for the appropriate size of the mattress to be used.

This product delivers effective pressure redistribution to users, when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these is the only item deployed between the support surface and the user.


3.2.1 Indications

Suitable both for the prevention of pressure injuries, in cases where there is a low to medium occurrence risk, and for the management of pressure injuries at stage 1 or 2, when combined with an individual and comprehensive pressure injury protocol.

Suitable for use in home care, residential, nursing and acute care settings.

3.3 Scope of Delivery

The following components are included within the scope of delivery:

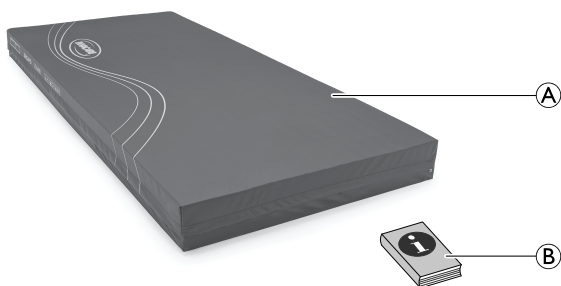(A)	Mattress with cover
(B)	User Manual

3.4 Symbols on the Product

	European conformity		UK Conformity Assessed
	Medical Device		LOT number
	Manufacturer		Date of manufacture
	European representative		Importer
	Machine wash temperature max. 90 °C		Do not dry clean
	Tumble dry low heat		Line dry
	Do not bleach		Do not iron
	User weight limit. See 7 <i>Technical Data</i> , page 11.		Do not put near flame
	Read user manual		

4 Usage

4.1 Safety Information

WARNING!

Insufficient repositioning of the patient can lead to tissue compression and potential injury formation.

To relieve pressure, it is very important for the patients to reposition themselves, or to be repositioned, on a regular basis.

- Before using the product, always consult a qualified healthcare professional for clinical judgement.
- Monitor the patient frequently.

CAUTION!

- Make sure that the printed side of the mattress cover always faces upwards.
- Ensure that the distance between the surface of the mattress and the top of the side rail is at least 220 mm.

CAUTION!

Risk of damage to the mattress

If holes are present in the mattress cover, there is a risk that liquids may ingress and contamination may occur.

- Ensure that the mattress is not jammed or damaged by sharp edges.
- Do not place hypodermic needles, venflons, scalpels or other similarly sharp objects on or under the mattress.
- Do not use electrically heated blankets directly on or under the mattress.
- Ensure that all venflons are taped down correctly with no sharp edges exposed.
- When using bridging boards or other patient transfer aids, check for any sharp edges or burrs before use.
- When using the mattress on a profiling bed ensure that the knee break is used before the backrest.
- Attach medical equipment including infusion pumps and monitors to appropriate bed accessories.
- Avoid cigarette burns and pet claws puncturing the mattress cover.

4.2 Preparing the Mattress for Use

CAUTION!

Risk of Injury or Damage to Property

- Before use, check the product for any defect.
- Do not use a defective product.

NOTICE!

Damage to the structure of the mattress due to incomplete decompression

The mattress may not fully reach its intended size and shape if it is not left to recover.

- Do not keep this mattress compress-rolled for more than 6 months (counted from date of manufacture).
- After unpacking the compress-rolled mattress, leave the mattress to recover for at least 3h before using it.

Ensure all packaging has been removed.


Packaging Disposal

Be environmentally responsible and recycle this product's packaging. Its recycling must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

	Plastic wrapping	Send for plastic recycling
	Cardboard inlay	Send for paper recycling


1. Unroll the mattress directly onto the bed frame.

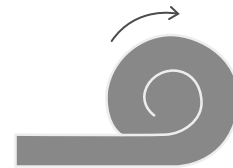2. Shake the mattress.

3. Leave the mattress to recover for at least 3h before using it.

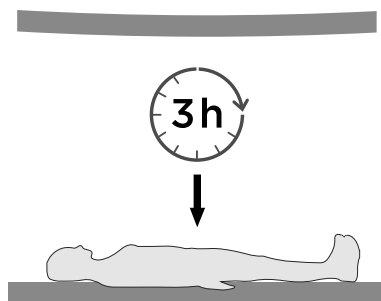4.3 Using the Handles

CAUTION!

Risk of injury

The handles on the cover are meant to transport the mattress only.

- Do not use the handles to move patients on the mattress.

5 Maintenance

5.1 Inspection

Check mattresses (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

Check Mattress

1. Unzip the cover completely.
2. Check for any staining on the white underside of the cover.
3. Check for any staining on the interior foam.
4. Replace any stained items and dispose of as per local authority procedure.

5.2 Cleaning and Disinfection

5.2.1 General Safety Information

CAUTION! Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.), abrasive cleaning agents or solvents (cellulose thinner, acetone etc.). We recommend a mild detergent.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.2.2 Cleaning Intervals

NOTICE!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination. Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

NOTICE!

- Keep a cleaning record as part of cleaning the system.

5.2.3 Cleaning Instructions

Cleaning Covers

(Removal of contaminants such as dust and organic matter)

1. Remove all covers for laundering.
2. Launder the covers at the recommended temperature of 90 °C using a diluted detergent solution (instructions on label).

NOTICE!

- Washing at higher temperatures will cause shrinkage.

Drying Covers

1. Hang covers from a line or bar and drip dry in a clean indoor environment.
or
Tumble dry on a low heat setting.

NOTICE!

- Tumble dry setting must not exceed 40 °C.
- Do not tumble dry for longer than 10 minutes.
- Dry thoroughly before use.

5.2.4 Disinfection Instructions

NOTICE!

- Only use disinfectants and methods approved by your local infection control institution and follow your local infection control policy.
- Follow your local decontamination protocols.

Disinfecting Covers

(Reducing the number of microorganisms)

NOTICE!

- Failure in disinfection process may result in the accumulation of reagent that could damage the polyurethane coating, react with the bed frame, or negate the biocompatibility results.
- Ensure that all cleaning agents, and disinfectants, are thoroughly rinsed off.
 - Dry thoroughly before use.

1. Wipe down the cover with a suitable detergent.
2. Rinse the cover thoroughly with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

Heavy soilage

- Where the mattress is badly soiled, we recommend cleaning with a dilute cleaning solution at maximum temperature in the washing machine (see product label).
- Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a suitable detergent.
 - Large spillages of blood should be absorbed and removed with paper towels first.

**WARNING!**

- In event of contamination, contact your hygiene specialist.
- Remove contaminated foams from use.

**NOTICE!**

- Ensure that the corners of the foam insert are positioned correctly into the corners of the cover.
- Ensure that the castellated foam is facing upwards when inserted into its cover.

**WARNING!**

- In event of contamination, contact your hygiene specialist.
- Remove contaminated foams from use.

5.3 Replacing Cover

1. Unzip cover and remove it carefully from foam insert.
2. Place new cover onto foam insert.
3. Close zipper.

6 After Use

6.1 Storage

**NOTICE!**

- Store mattresses in a dry environment.
- Store mattresses within a protective cover.
- Store items on clean, dry (off-floor) surface with no sharp edges to avoid any possible damage.
- Never store other items on top of a mattress.
- Do not store mattresses next to radiators or other heating devices.
- Protect mattresses from direct sunlight.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 8*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

6.3 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 General Data

The mattresses are available in various sizes, depending on the market. Contact your Invacare distributor for detailed information.

Local requirements / certification available upon request from Invacare distributor.

Product	Dimensions [mm]			Maximum user weight [kg]	Weight of product [kg] ¹
	Width	Length	Height		
Dacapo EcoGreen	800 – 1200	2000	150	150	14

7.2 Materials

Foam	Bio-based polyurethane foam
Wipedown cover	Polyurethane coated bamboo fabric

All product components do not contain natural rubber latex.

1. Weight can vary depending on mattress size. Average weight used as an indication.

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Invacare Limited
Limited