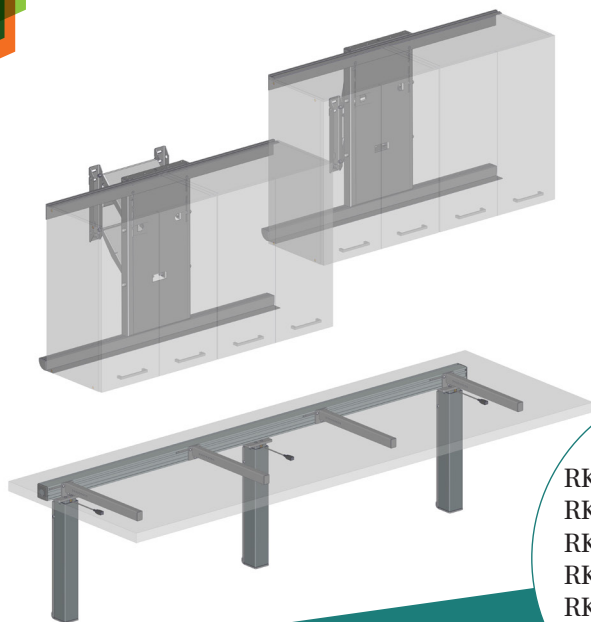
PRESSALIT®

RK1220-21, RK1210-14  
RK1250-54, RK1420-21  
RK1410-14, RK1450-54  
RK1820-21, RK1810-14  
RK1850-54, RK2910-14  
RK2960, RK2920-21

<b>Indivo kitchen</b> – Operation and maintenance manual	<b>en</b>
<b>Indivo-køkkenet</b> – Brugs- og vedligeholdelsesvejledning	<b>dk</b>
<b>Indivo Küche</b> – Gebrauchs- und Pflegeanleitung	<b>de</b>
<b>Indivo cuisine</b> – Manuel d'utilisation et d'entretien	<b>fr</b>
<b>Indivo keuken</b> – Gebruiks- en onderhoudshandleiding	<b>nl</b>
<b>Indivo kök</b> – Bruks- och underhållsanvisning	<b>sv</b>
<b>Indivo kjøkken</b> – Bruks- og vedlikeholdsveiledning	<b>no</b>
<b>Indivo cocina</b> - Instrucciones de uso y mantenimiento	<b>es</b>
<b>Kuchnia Indivo</b> – Instrukcja obsługi i konserwacji	<b>pl</b>
<b>Indivo 厨房</b> - 操作和维护手册	<b>中文</b>

English .....	17
Dansk .....	30
Deutsch .....	43
Français .....	56
Nederlands .....	69
Svenska .....	82
Norsk .....	95
Español .....	108
Jzyk polski .....	121
中文 .....	134

Pressalit's Environmental and Quality Management Systems are certified by LRQA

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


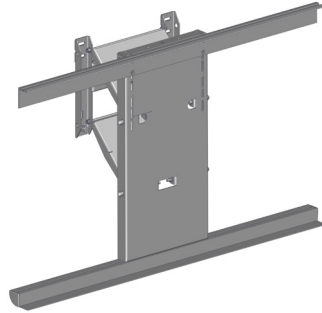
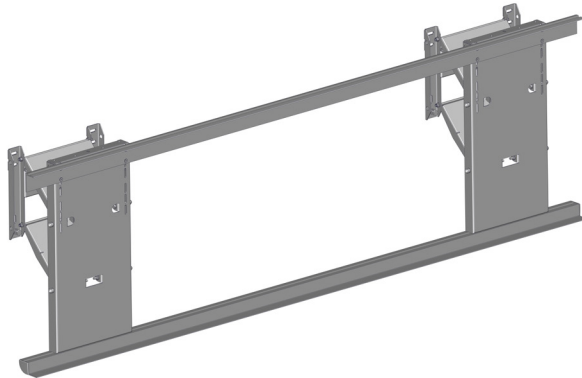
RK1221, RK1421, RK1821, RK2921

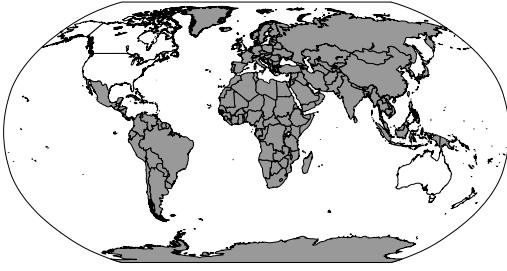
230V

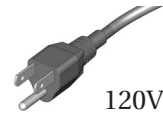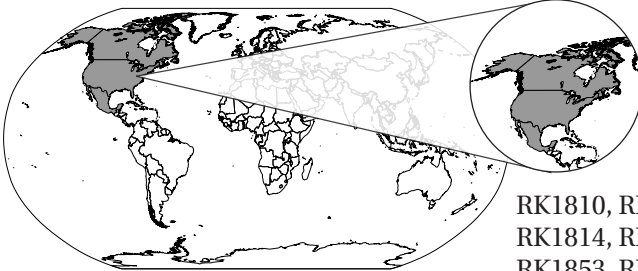RK1410, RK1411, RK1412, K1413  
RK1414, RK1450, RK1451, RK1452  
RK1453, RK1454, RK1420, RK1421  
RK2910, RK2911, RK2912, RK2913  
RK2914, RK2960, RK2920, RK2921

120V

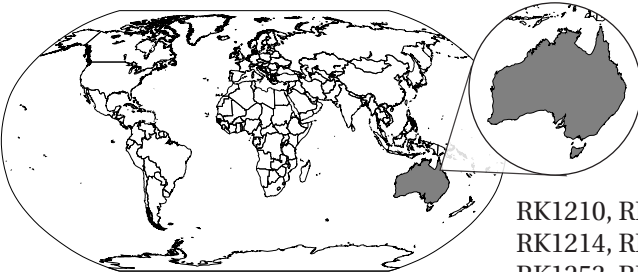RK1810, RK1811, RK1812, RK1813  
RK1814, RK1850, RK1851, RK1852  
RK1853, RK1854, RK1820, RK1821

230V

RK1210, RK1211, RK1212, RK1213  
RK1214, RK1250, RK1251, RK1252  
RK1253, RK1254, RK1220, RK1221


	Max 100 kg / 220 lbs	Max 150 kg / 330 lbs


	Max 110 kg / 242 lbs	Max 220 kg / 485 lbs

	<b>Max P</b>	<b>U</b>	<b>Max I</b>
RK1210, RK1211, RK1212, RK1213, RK1214, RK1410, RK1411, RK1412, K1413, RK1414,	200 W	230 V	0,8 A
RK1810, RK1811, RK1812, RK1813, RK1814	200 W	120 V	1,6 A
RK1250, RK1251, RK1252, RK1253, RK1254, RK1450, RK1451, RK1452 RK1453, RK1454	300 W	230 V	1,3 A
RK1850, RK1851, RK1852, RK1853, RK1854, RK1820, RK1821	300 W	120 V	2,5 A
RK1220, RK1420	100 W	230 V	0,4 A
RK1820	100 W	120 V	0,8 A
RK1221, RK1421	200 W	230 V	0,8 A
RK1821	200 W	120 V	1,6 A

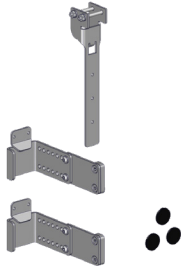
RK1344

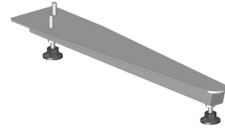RK1342

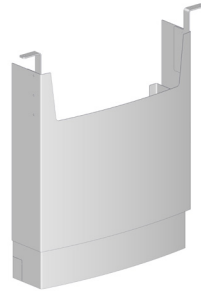
RK1045


RK1043

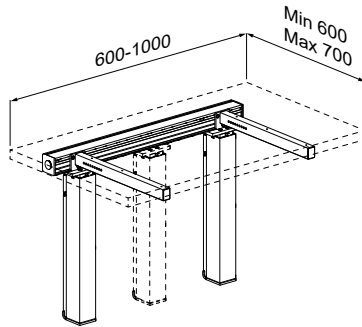
mm

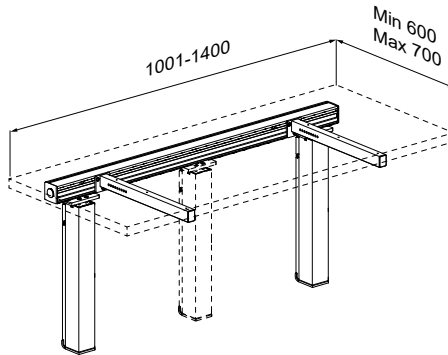
RK1210, RK1410, RK1810, RK2910  
RK1250, RK1450, RK1850

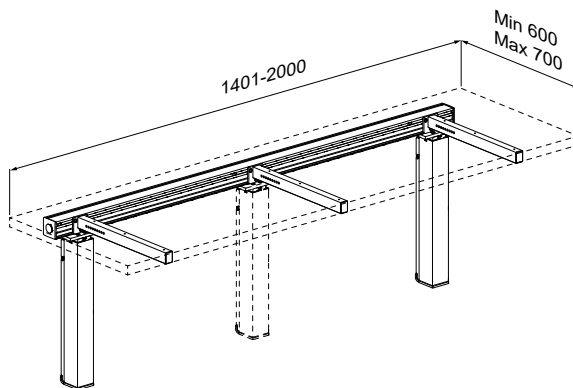
RK1211, RK1411, RK1811, RK2911  
RK1251, RK1451, RK1851

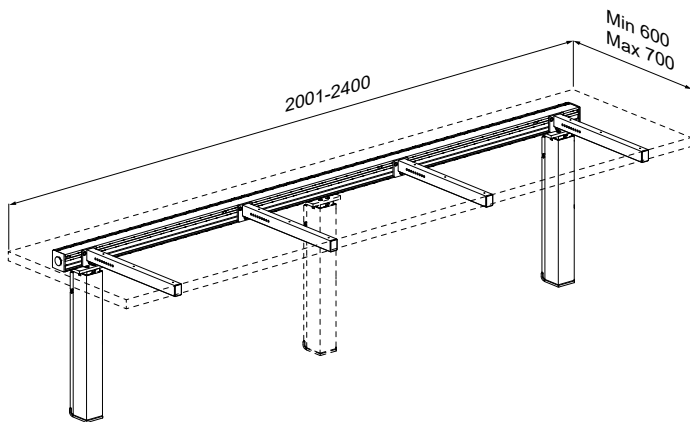
RK1212, RK1412, RK1812, RK2912  
RK1252, RK1452, RK1852

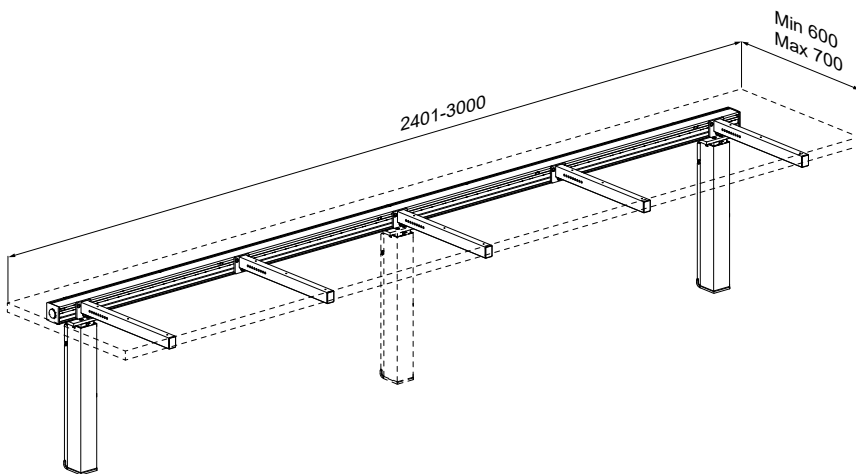
RK1213, RK1413, RK1813  
RK1253, RK1453, RK1853

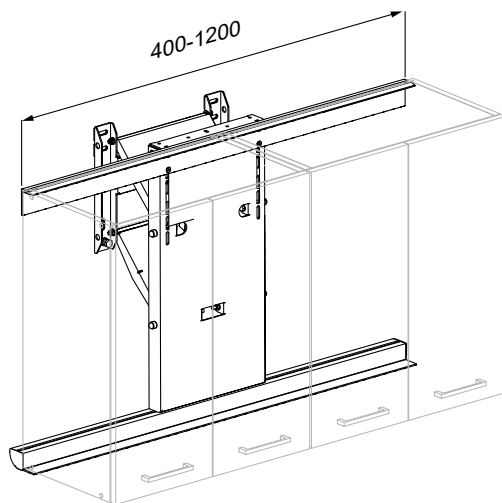RK1214, RK1414, RK1814, RK2914  
RK1254, RK1454, RK1854


RK1220, RK1420, RK1820, RK2920

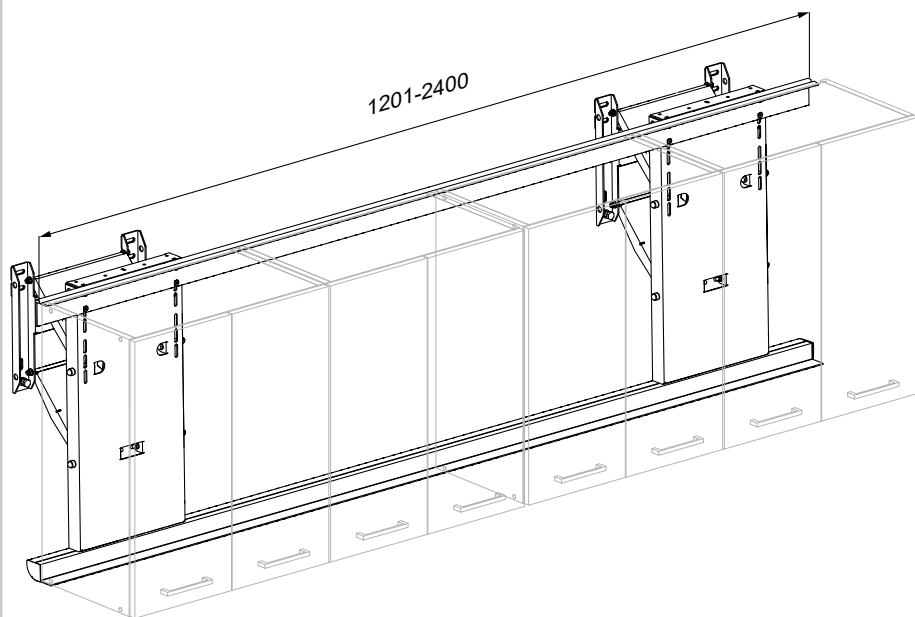
RK1221, RK1421, RK1821, RK2921

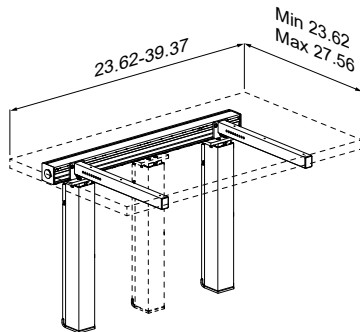
inch

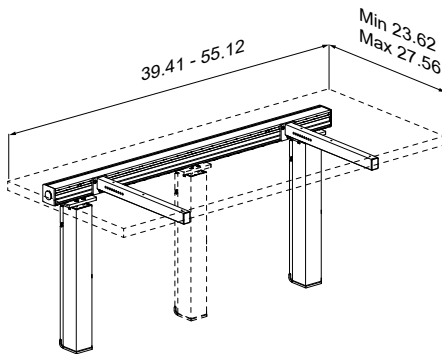
RK1210, RK1410, RK1810, RK2910  
RK1250, RK1450, RK1850

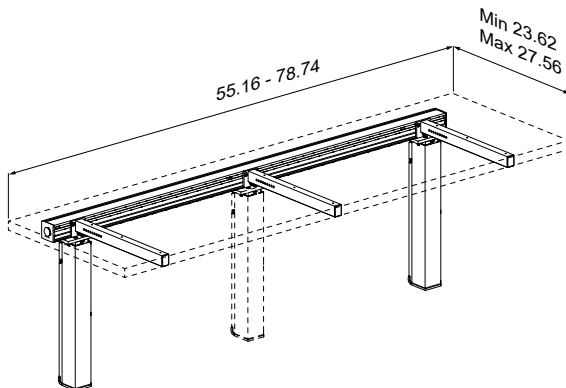
RK1211, RK1411, RK1811, RK2911  
RK1251, RK1451, RK1851

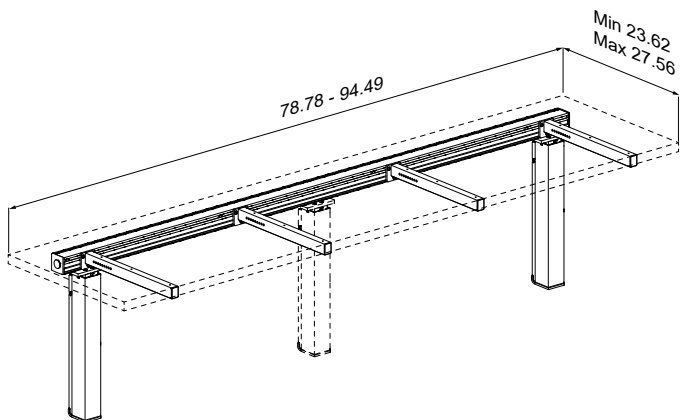
RK1212, RK1412, RK1812, RK2912  
RK1252, RK1452, RK1852

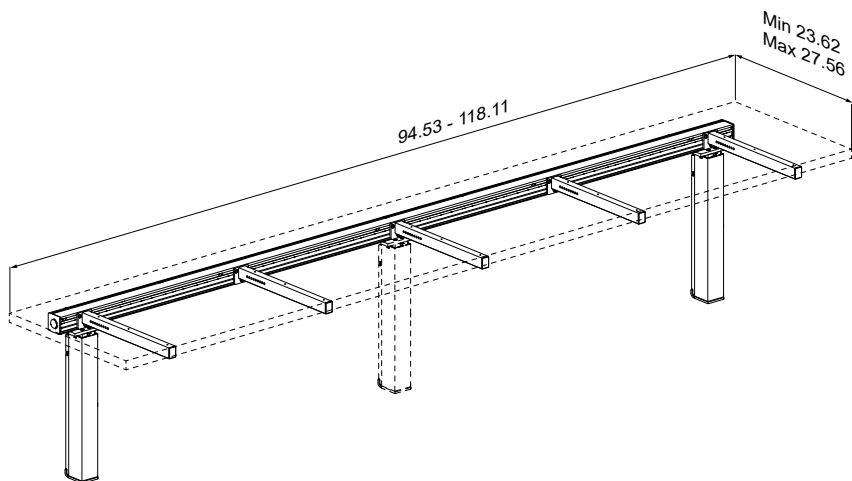
RK1213, RK1413, RK1813, RK2913  
RK1253, RK1453, RK1853

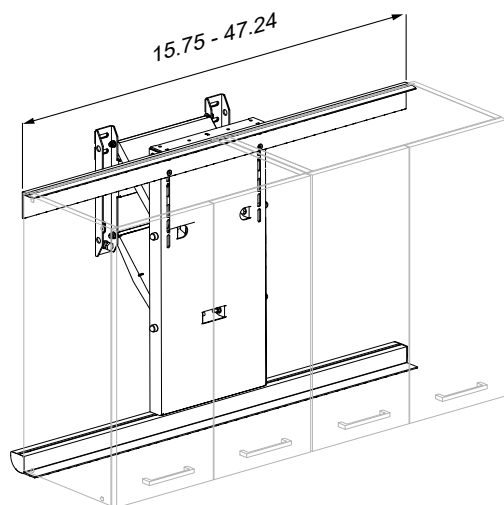RK1214, RK1414, RK1814, RK2914  
RK1254, RK1454, RK1854

RK1220, RK1420, RK1820, RK2920

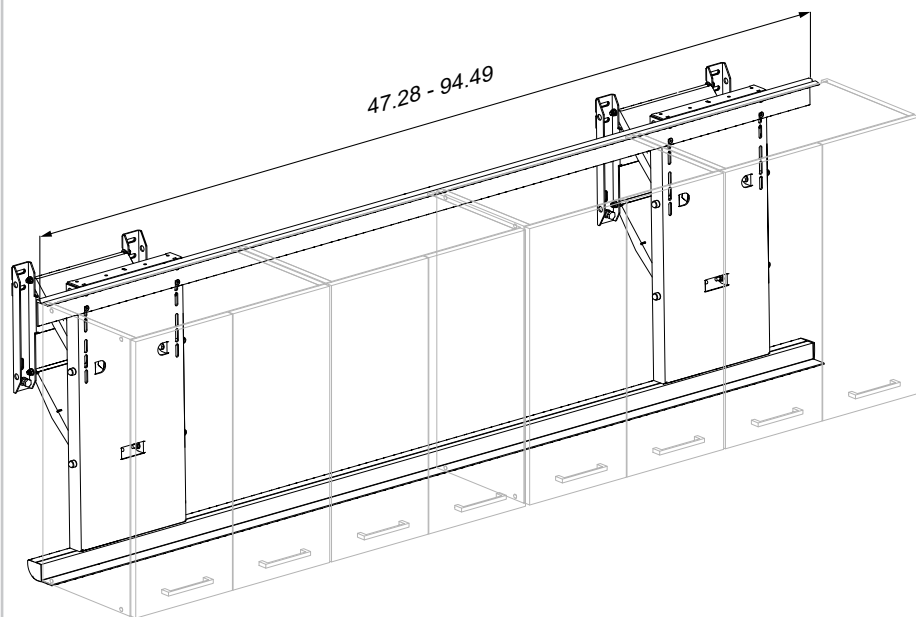RK1221, RK1421, RK1821, RK2921

Mounting instruction .....	17
Safety .....	18
Usage .....	19
Safety labels. ....	19
Operating instructions .....	22
Safety systems. ....	24
Worktop safety rails .....	24
Cupboard safety plates .....	25
Maintenance .....	26
Resetting. ....	26
Cleaning. ....	26
Trouble-shooting .....	27
Disposal and recycling .....	29

## Mounting instruction

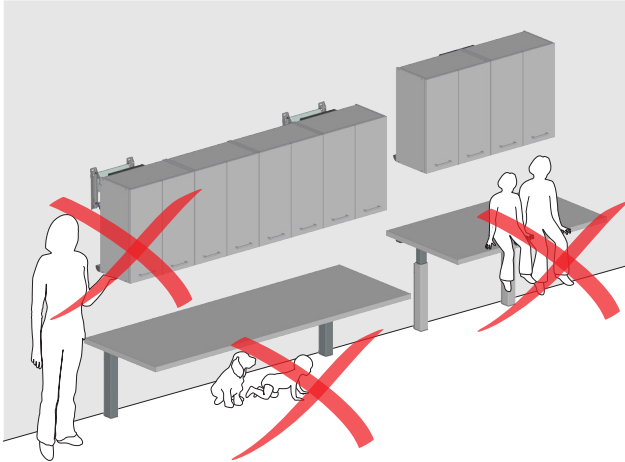
A mounting instruction is included with the product. It can also be downloaded from [pressalit.com](http://pressalit.com).

# Safety

- Read and understand the manual completely before use. Keep the manual close at hand.
- Never use or handle these lifts in other ways than are specified in this manual.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product that are not described in this manual or in the mounting instruction.
- Never allow children to operate these lifts unless under supervision. The lifts are not intended for play.
- Never use a lift if it is defective or damaged. Do not use a defective lift before it has been repaired.
- The information in this manual is based on correct installation in accordance with our mounting instruction.
- Never allow children to play with the lifts.

en

- Never allow children or pets to stay under the table
- Keep hands clear of the lifting mechanism.

Pressalit cannot be held liable if the lifts are used in any way which differs from that described in this manual.


We reserve the right to amend this manual without prior notice.

# Usage

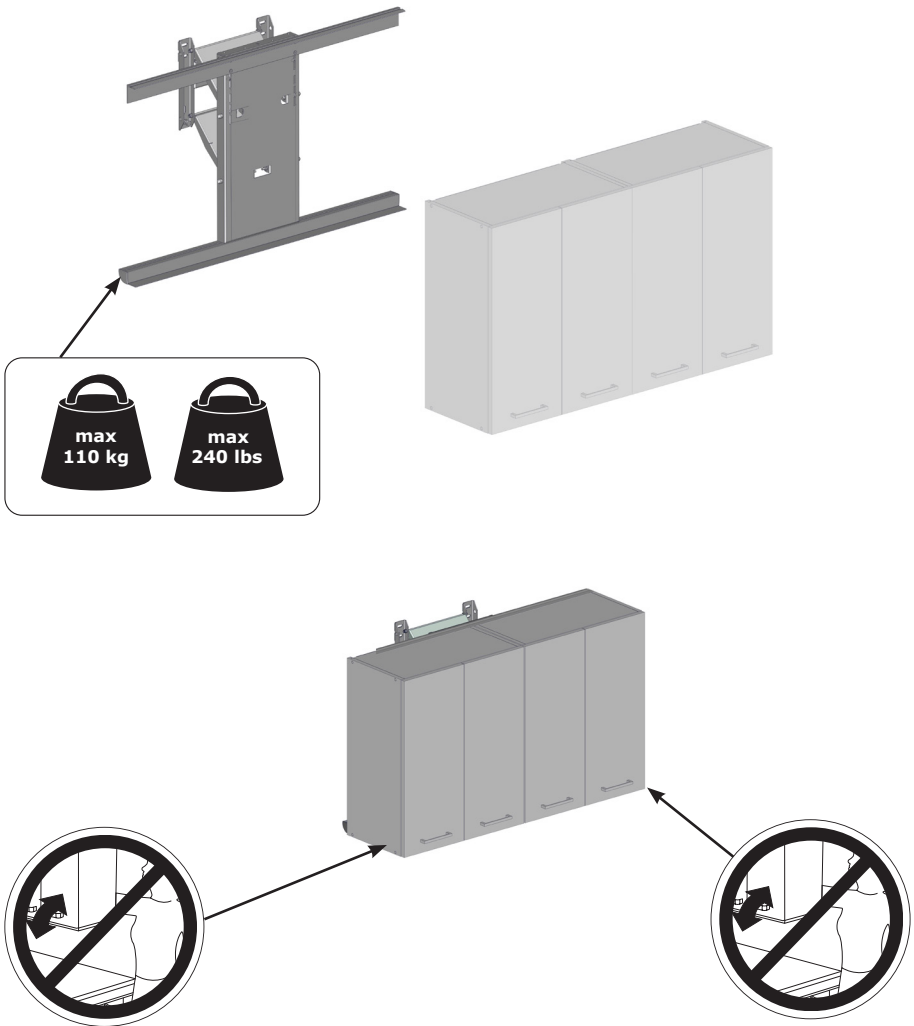
The height adjustable kitchen lifts allow both seated and standing users to work comfortably in the same kitchen. The lifts are for indoor use only. The lifts are not to be used in a bathroom or wetroom.

## Safety labels


Before initial operation, please affix the safety labels supplied along with this manual.



en

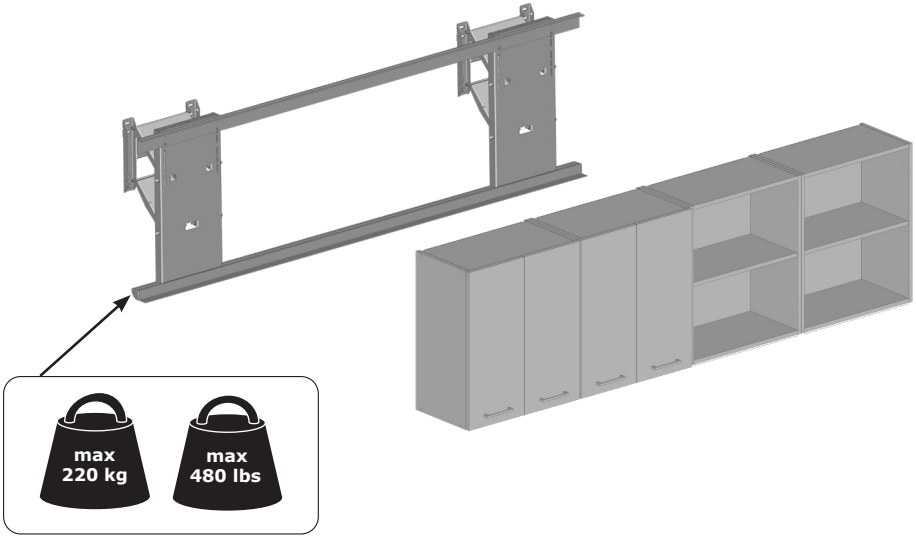
en

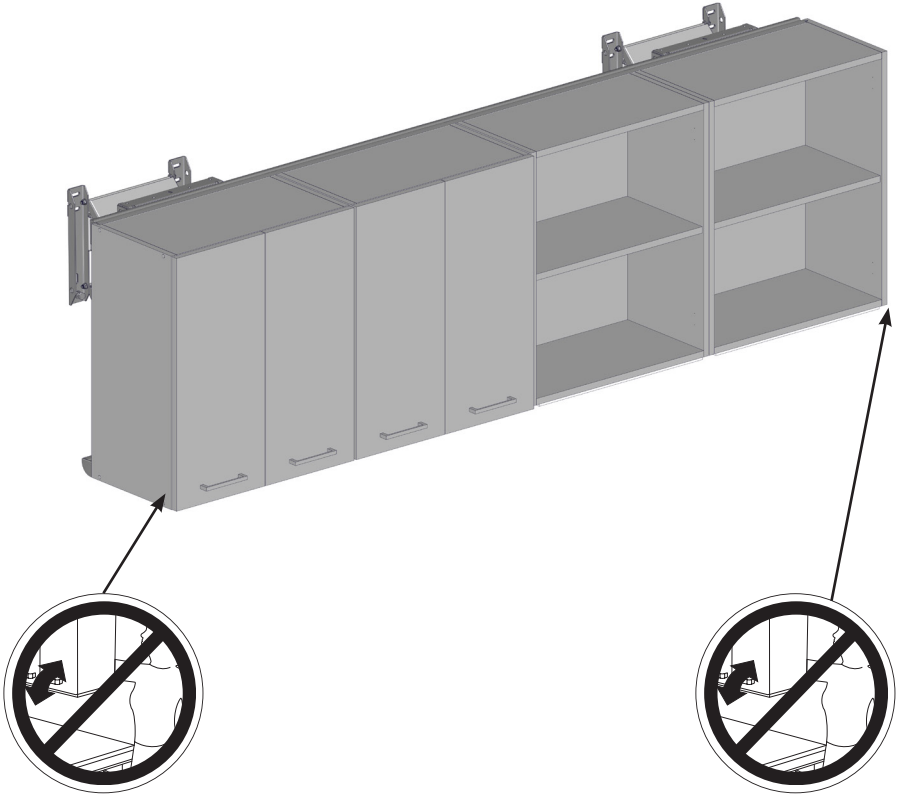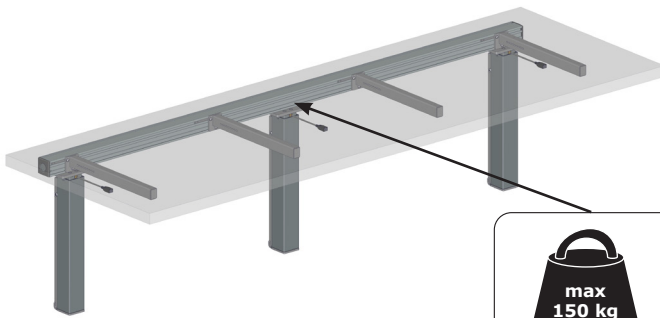
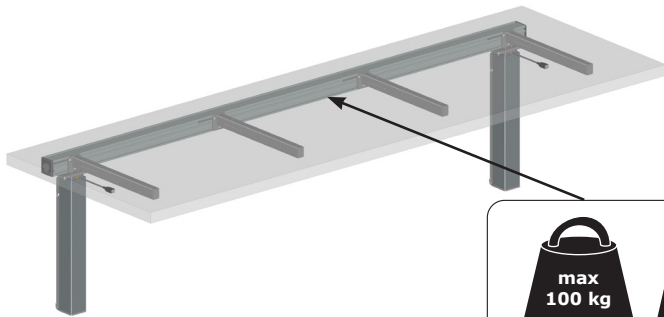
en

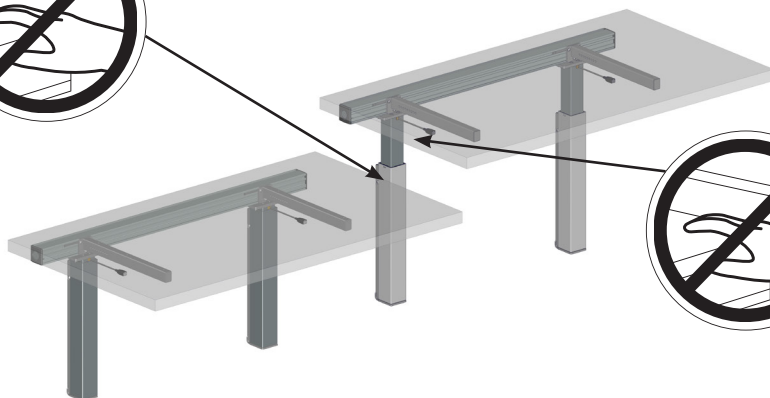
# Operating instructions

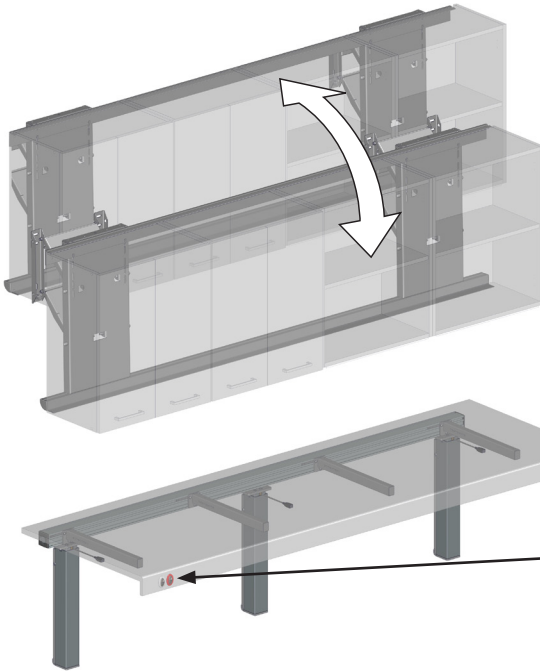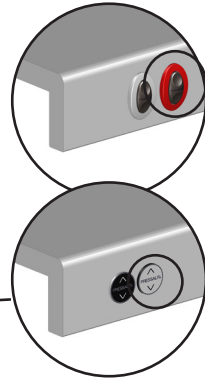The cupboards are height-adjusted with the red or silver button (depending on the type of button installed) on the worktop fascia.

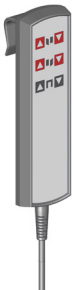
en Press and hold the “up” button to raise the unit. When the button is released, the motion stops. Press and hold the “down” button to lower the unit. When the button is released, the motion stops.

An optional wired remote control may be purchased as an alternative or in addition to the operating buttons.

 **CAUTION**

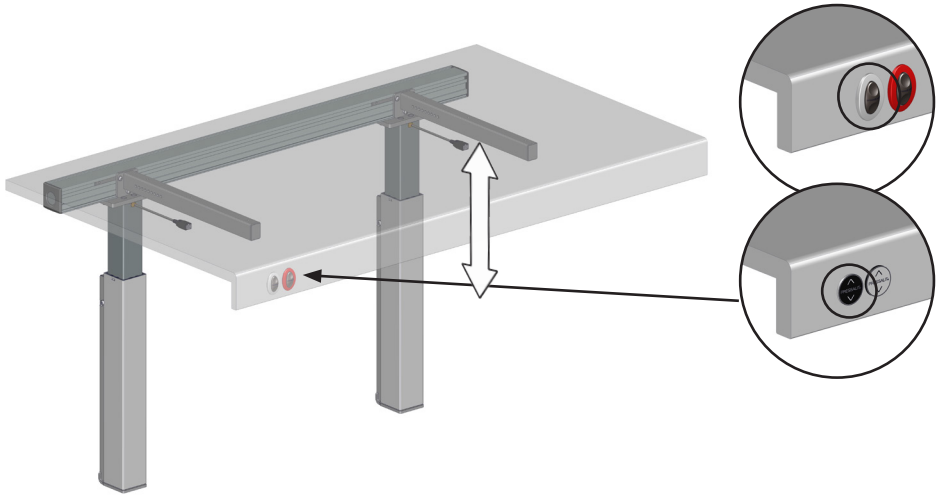Do **NOT** use the safety plates and safety bars under the wall cupboards and the worktop as a stop function!

The worktop is height-adjusted with the aluminium or black button (depending on the type of button installed) on the front of the worktop.

Press and hold the “up” button to raise the unit. When the button is released, the motion stops. Press and hold the “down” button to lower the unit. When the button is released, the motion stops.

en

 **CAUTION**

If two worktops are mounted side by side: Do not allow objects to rest on both worktops at the same time when adjusting the height.

 **CAUTION**

Do not leave kitchen appliances, such as a blender or food processor, running while the worktop is being height-adjusted.

**NOTICE**

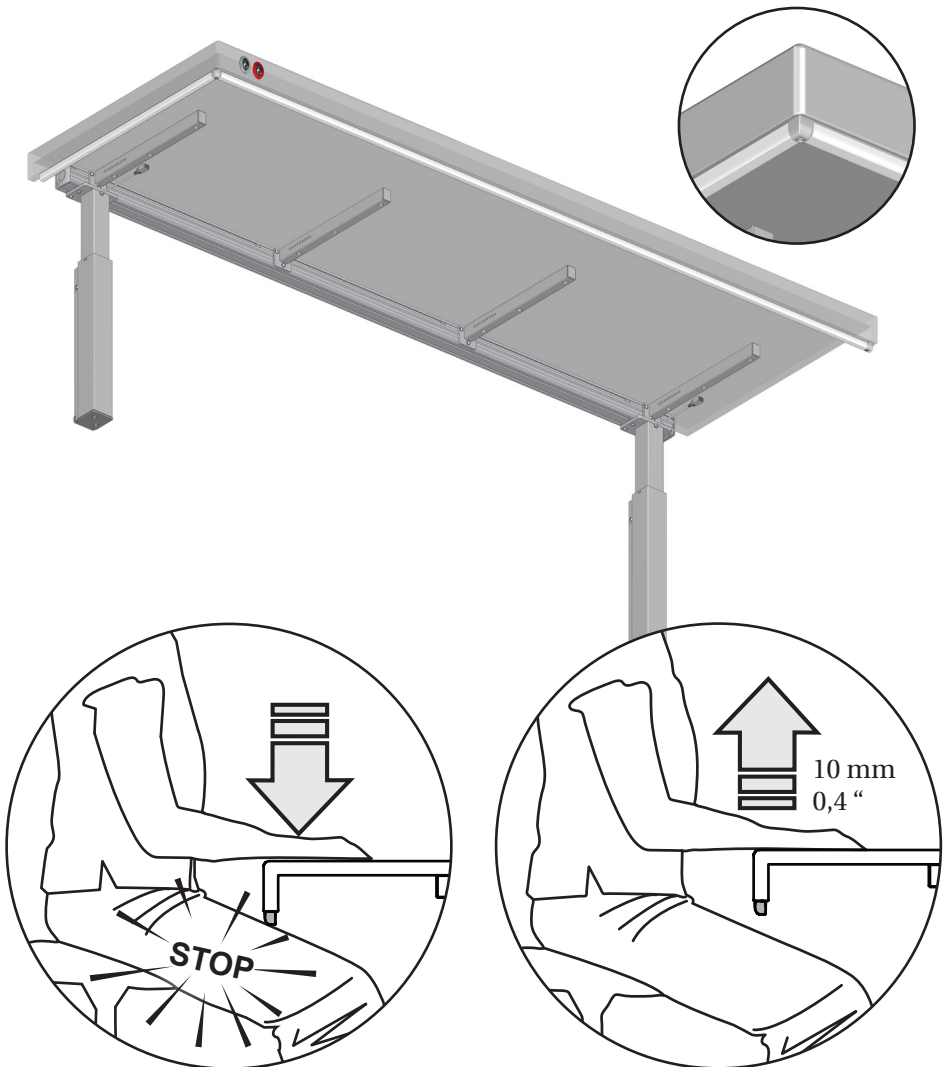
It is recommended that heavy items that are permanently positioned on the work top are placed closest to the wall.

# Safety systems

The lifting units are equipped with an interconnected safety system to ensure safe movement.

## Worktop safety bars

When the worktop lift is lowered, and the safety bar comes in contact with an object, the worktop stops immediately and raises approximately 10 mm/0.4". The safety bars can be placed at the front and/or sides of the worktop.


en



## Wall cupboard safety plates

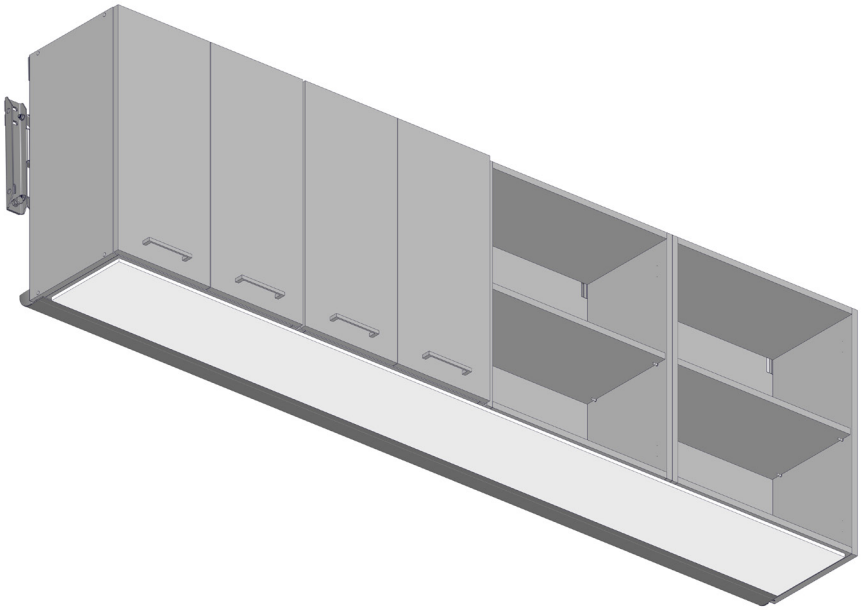When the wall cupboard lift is lowered, and the safety plate comes in contact with an object, the wall cupboard lift stops immediately and raises approximately 10 mm/0.4”.

en
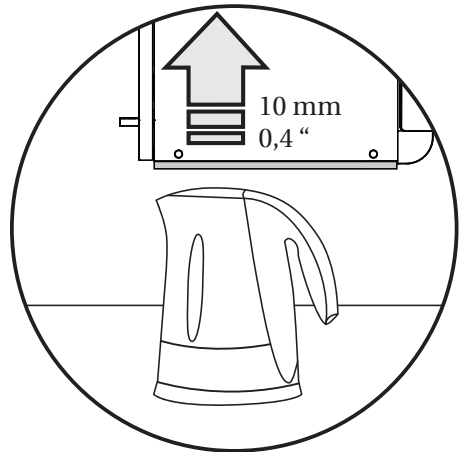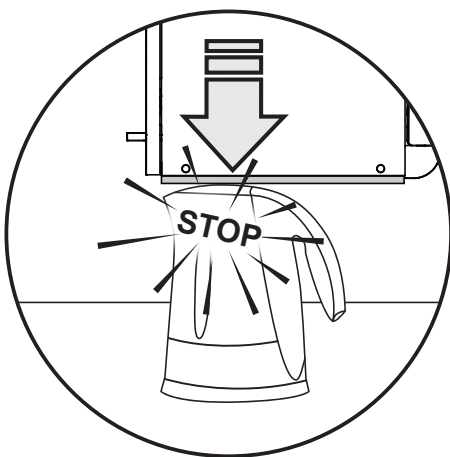
# Maintenance

On a weekly basis it is recommended to run the motors to their maximum positions. Raise the product to its full height and then lower it to its lowest position to lubricate the internal parts.

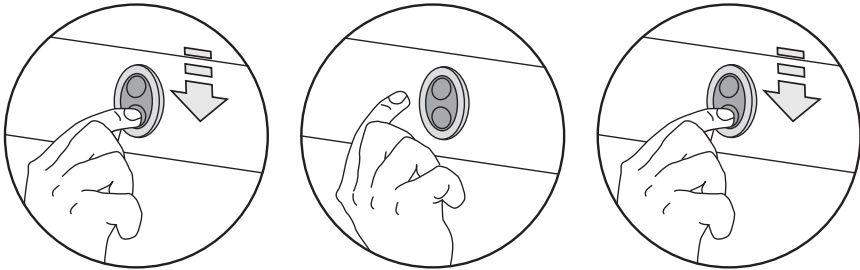
Do not attempt to open the lifting columns.

## Resetting

It is also recommended on a weekly basis to reset the lifting mechanism.

- Press the “down” button until the work top stops in the lowest position.
- Release the button.
- Press and hold the “down” button once more for 5 seconds. Perform the same operation for the wall cupboards.

en

## Cleaning

The product must be cleaned with a soft cloth wrung out in mild soapy water. Avoid the use of abrasive, etching and/or chloride cleaning agents.

# Trouble-shooting

Symptom	Action
<p>It is not possible to height adjust the worktop lift</p>	<p>Make sure that the safety bars under the worktop edge have not been activated. An object pressing on the safety bars will activate them and prevent movement of the lift</p> <p>Try to activate the safety bars by pressing them gently up against the worktop. If they can move approximately 5 mm/0.2", it means that they are functional.</p> <p>If a sink and installation cover have been installed, check that the installation cover does not prevent the downward movement.</p> <p>Check that all operation switches can be operated without obstruction and that they return to a neutral position. Movement may be hampered by dirt or grease.</p> <p>Check that all cables are securely connected and that the electrical circuit is powered. Ensure that all plugs are pressed as far as possible into their corresponding sockets on the control box. The control box will typically be placed underneath the worktop.</p> <p>Disconnect power and reconnect after 10 minutes.</p> <p>Reset the lift – see “Resetting”.</p> <p>It may be necessary to reset the lift several times.</p>

en

Symptom	Action
It is not possible to height adjust the wall cupboard lift	<p data-bbox="353 142 1020 264">Make sure that the safety plates underneath the wall cupboards have not been activated. An object pressing on the safety plates will activate them and prevent movement of the lift</p> <p data-bbox="353 304 981 427">Try to activate the safety plate by pressing it gently up against the bottom of the wall cupboard. If it can move approximately 5 mm/0.2", it means that it is functional.</p> <p data-bbox="353 467 992 590">Check that all operation switches can be operated without obstruction and that they return to a neutral position. Movement may be hampered by dirt or grease.</p> <p data-bbox="353 630 1003 785">Check that all cables are securely connected and that the electrical circuit is powered. Ensure that all plugs are pressed as far as possible into their corresponding sockets on the control box. The control box will typically be placed behind the wall cupboards.</p> <p data-bbox="353 825 964 849">Disconnect power and reconnect after 10 minutes.</p> <p data-bbox="353 888 936 943">Reset the lift – see “Resetting”. It may be necessary to reset the lift several times</p>
Wall cupboard or worktop lift stop and will go no further	<p data-bbox="353 952 936 1110">Make sure that the lifts are not overloaded. Try removing some items from wall cupboard or worktop. Reset the lift – see “Resetting”. It may be necessary to reset the lift several times</p>
Safety bar or safety plate are not working	<p data-bbox="353 1125 622 1155">Contact your supplier.</p>

en

# Disposal and recycling

This product contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after being cleaned and disinfected.

When discarding, we recommend the product be disassembled into different waste groups for proper disposal.

The following chart provides information on the recycling and handling of the materials.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic recycling systems where they are locally available. Plastics can be incinerated for the benefit of generating energy, such as for heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.

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