HEPRO

Kit for svakere tak / Kit for weaker ceilings
Bruksanvisning / User manual

NORSK	2-5
ENGLISH	6-9

NB! Les bruksanvisningen nøye før tilbehøret tas i bruk

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
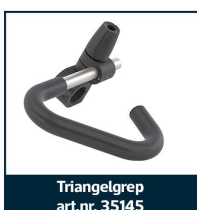
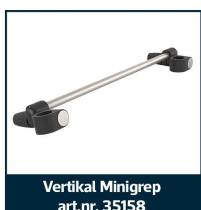
Safety and Warnings

Accessories that can be used in combination with the "Kit for weaker ceilings"

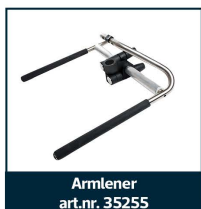Accessories that CANNOT be used in combination with the "Kit for weaker ceilings"

- Maximum user weight: **150 kg**
- Maximum user weight for approved accessories: **110 kg**
- Make sure that the ceiling structure can withstand a load of **50 kg** before installation.
- Clean the surfaces of the floor/ceiling thoroughly with a degreasing cleaning agent before installation; this is especially important when installing on tiles or other smooth surfaces
- Do not use the product if the red safety ring at the top of the pole is visible.
- Always consider the risk of pinching between the pole and other furniture/objects.
- Do not mount the product on loose items that are not securely fixed to the ceiling or floor, such as floor mats, boards, or similar.
- Do not let children play with the product.
- Do not place the product on surfaces that are not level or parallel.
- Make sure that the red marking is not visible during use. If it becomes visible, take down the pole and set it up again.
- Check that the tightening screw on the locking clamp is not loose and that the inner tube is screwed in firmly. If the red marking is still visible, a possible cause may be that the ceiling structure is not strong enough.

NB! This accessory must NOT be combined with other accessories than the ones listed in the **GREEN** box above

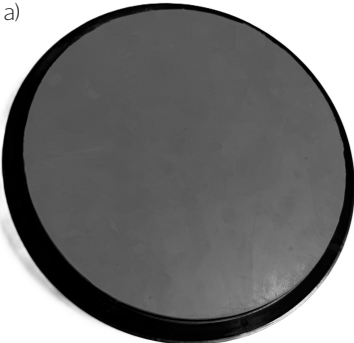
Installation of
**KIT FOR
WEAKER
CEILING**
Art. Nr. 35327

A

Unpack the received parts and check that nothing is missing.

The contents of the package should consist of the following parts:

- a) Large base plate with friction
- b) Ceiling plate including 500N gas spring

b)

B

1. Place a) the base plate with friction coating on the floor.
2. Place the Gripo in the base plate so that it fits into the groove.
3. Place b) the ceiling plate with gas spring on top of the inner tube and push the gas spring towards the red safety mark so that it is secured in place.

C

The Gripo pole is then tensioned between the floor and ceiling in accordance with the instructions for use for Gripo.

Check that the pole is securely fastened before use.

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