GB

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
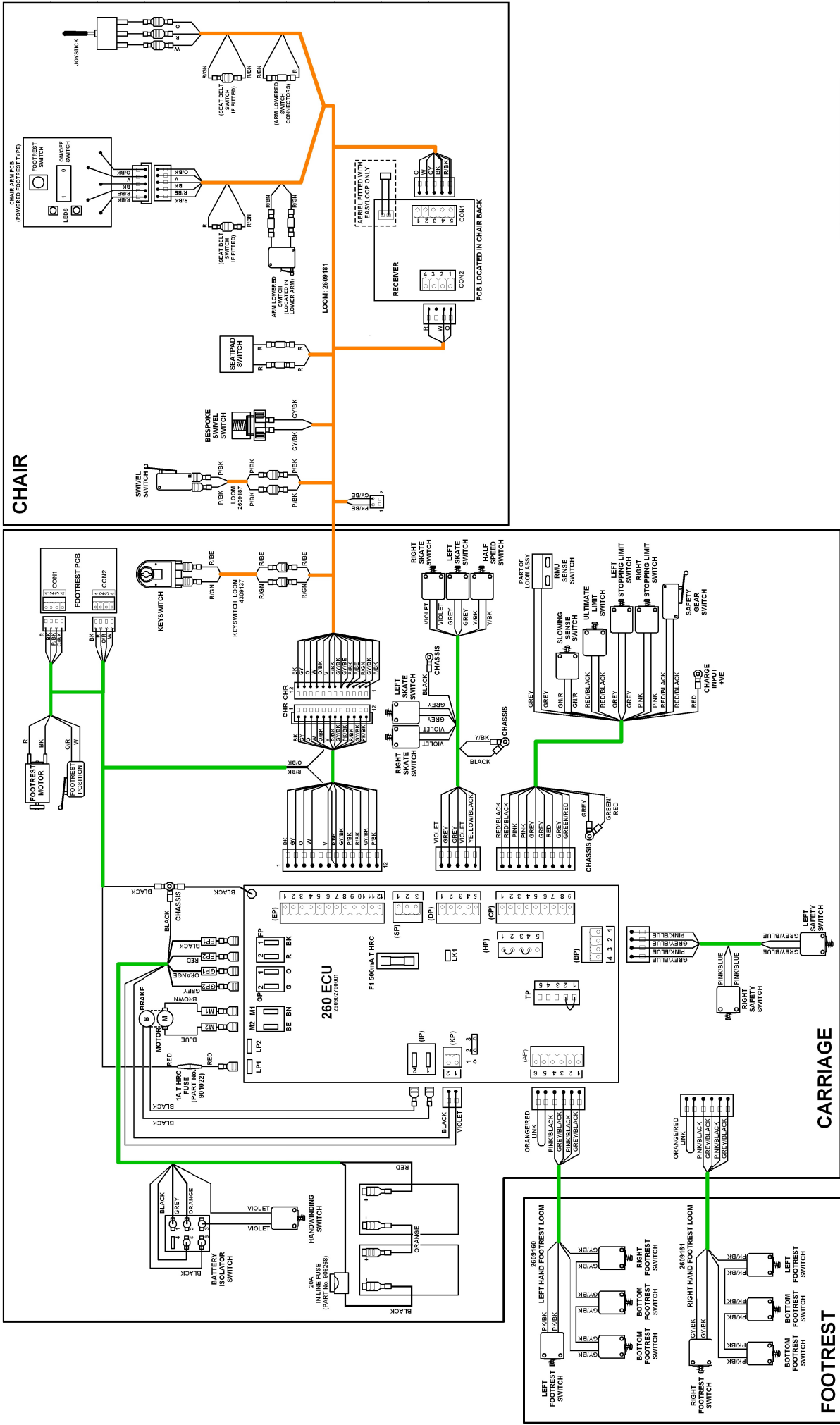
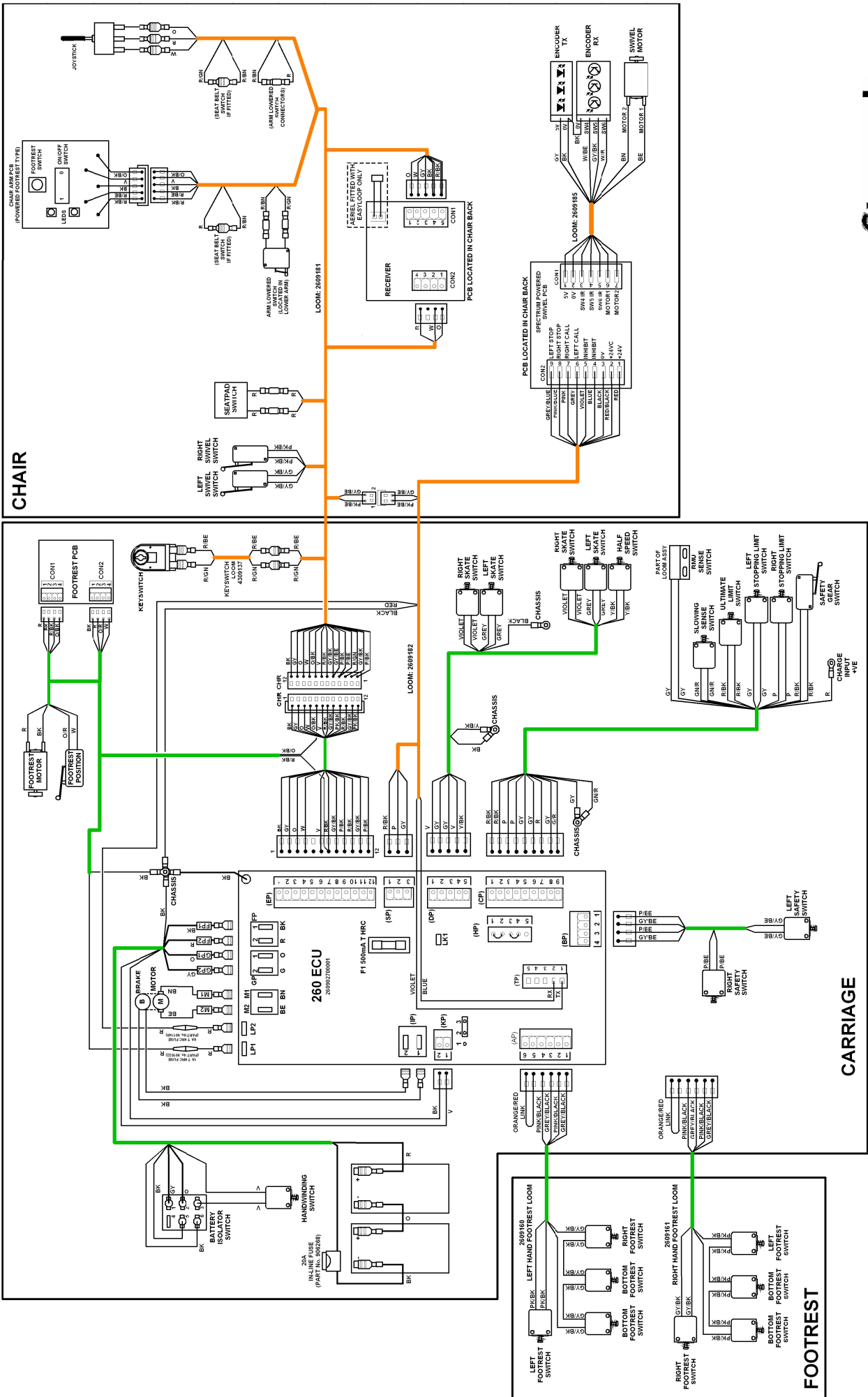
STARLA 260 MANUAL SWIVEL / POWER FOOTREST LAYOUT DIAGRAM 2609372

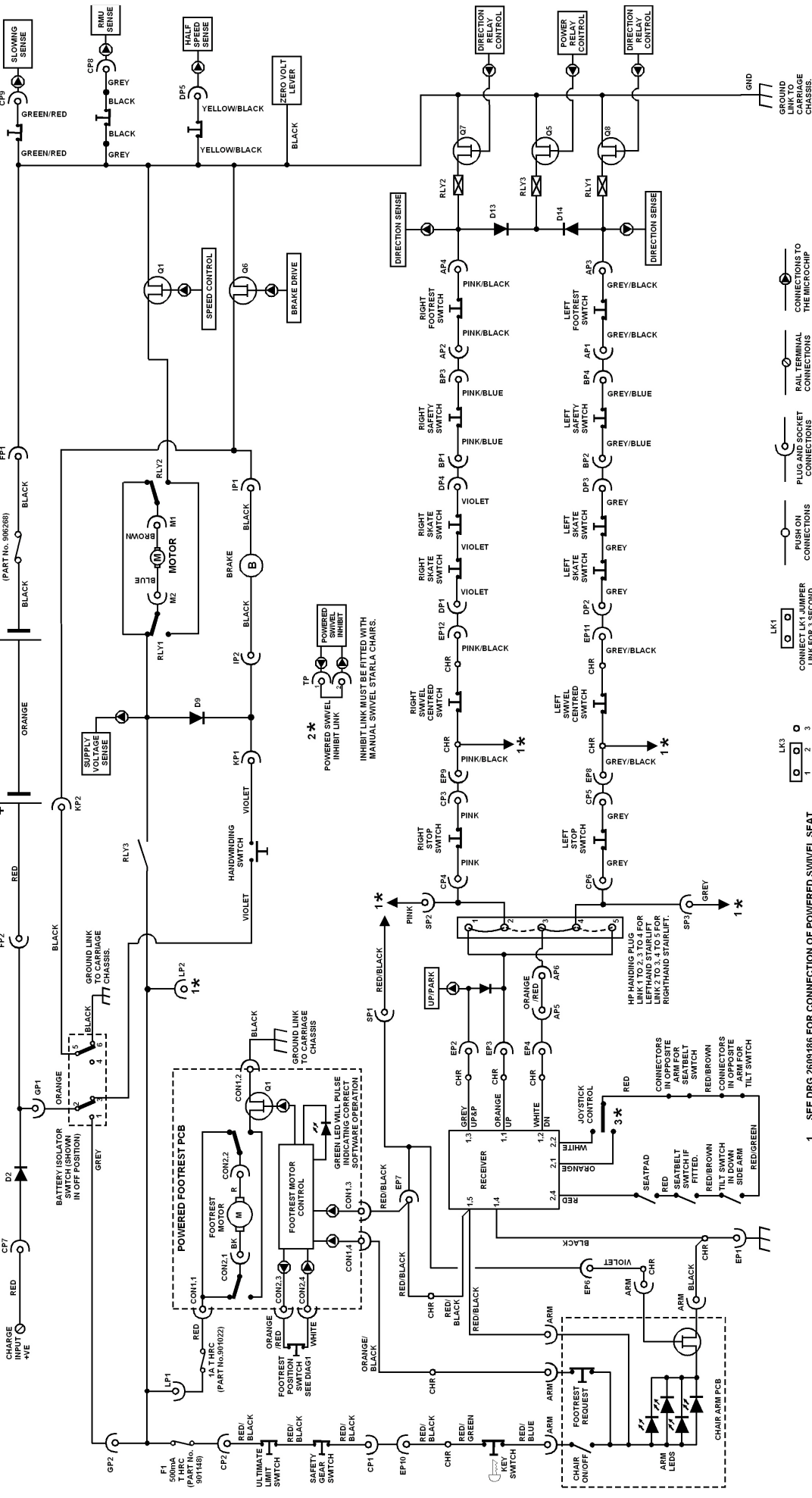STARLA 260 POWER SWIVEL / POWER FOOTREST LAYOUT DIAGRAM 2609373


STARLA 260 POWERED FOOTREST KEY WIRING DIAGRAM 2609042

- 1 SEE DRG 2609186 FOR CONNECTION OF POWERED SWIVEL SEAT PLASTIC. ROTATE BODY 180 DEGREES. RE-INSERT INTO ARM. CORRECT HANDING: ORANGE WIRE MUST BE CONNECTED TO DOWNSIDE TERMINAL OF JOYSTICK.
- 2 * LINK FITTED FOR POWERED SWIVEL SEAT ONLY.
- 3 * TO REHAND JOYSTICK CONTROL: REMOVE JOYSTICK FROM ARM PLASTIC. ROTATE BODY 180 DEGREES. RE-INSERT INTO ARM. CORRECT HANDING: ORANGE WIRE MUST BE CONNECTED TO DOWNSIDE TERMINAL OF JOYSTICK.


260 STARLA KEY WIRING DIAGRAM - POWER FOOTREST

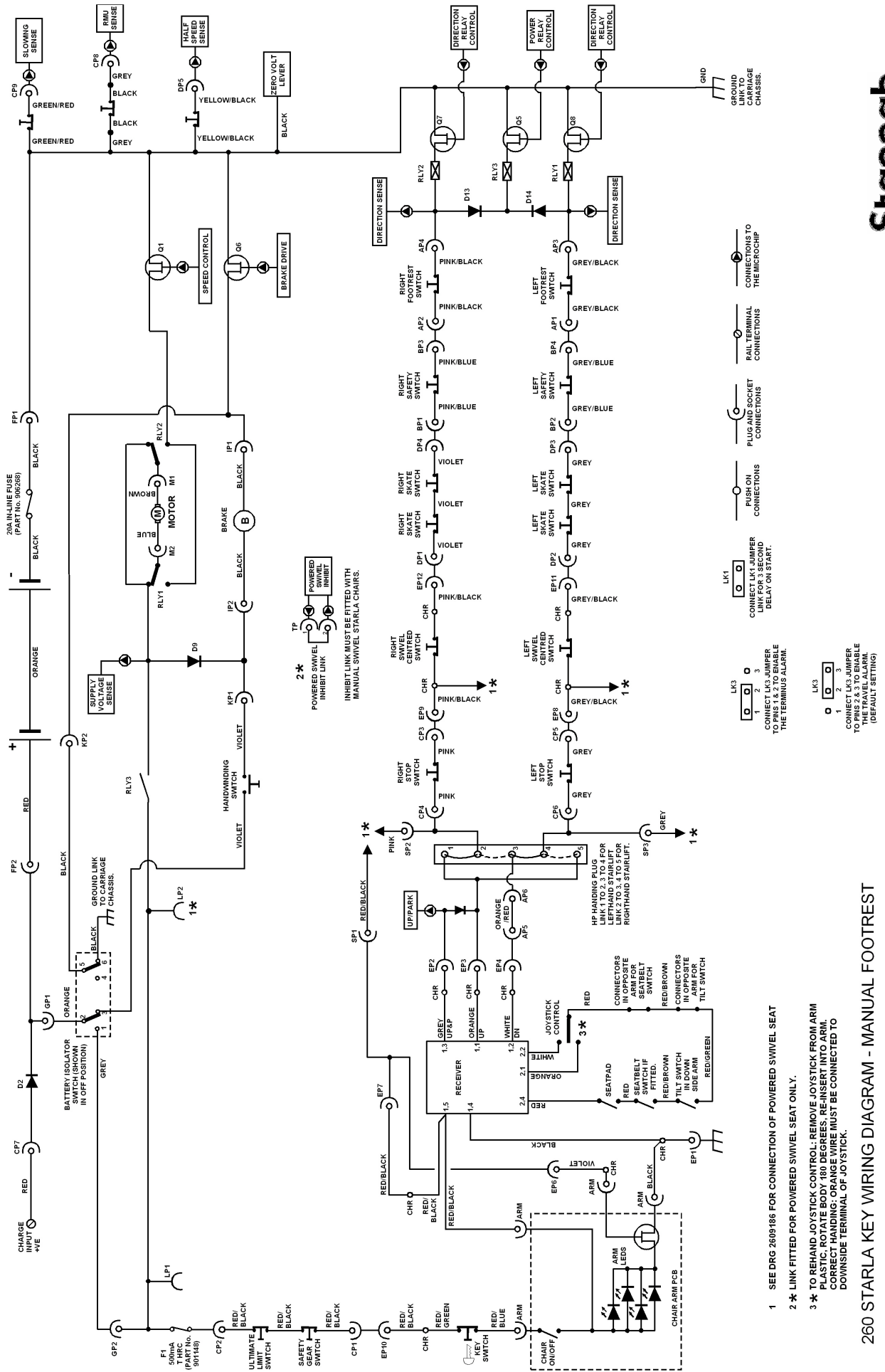STARLA 260 MANUAL FOOTREST KEY WIRING DIAGRAM 2609043

- 1 SEE DRG 2609188 FOR CONNECTION OF POWERED SWIVEL SEAT
- 2 * LINK FITTED FOR POWERED SWIVEL SEAT ONLY.
- 3 * TO REHAND JOYSTICK CONTROL, REMOVE JOYSTICK FROM ARM FOR SEATBELT SWITCHES. CORRECT HANDING - ORANGE WIRE MUST BE CONNECTED TO DOWNSIDE TERMINAL OF JOYSTICK.

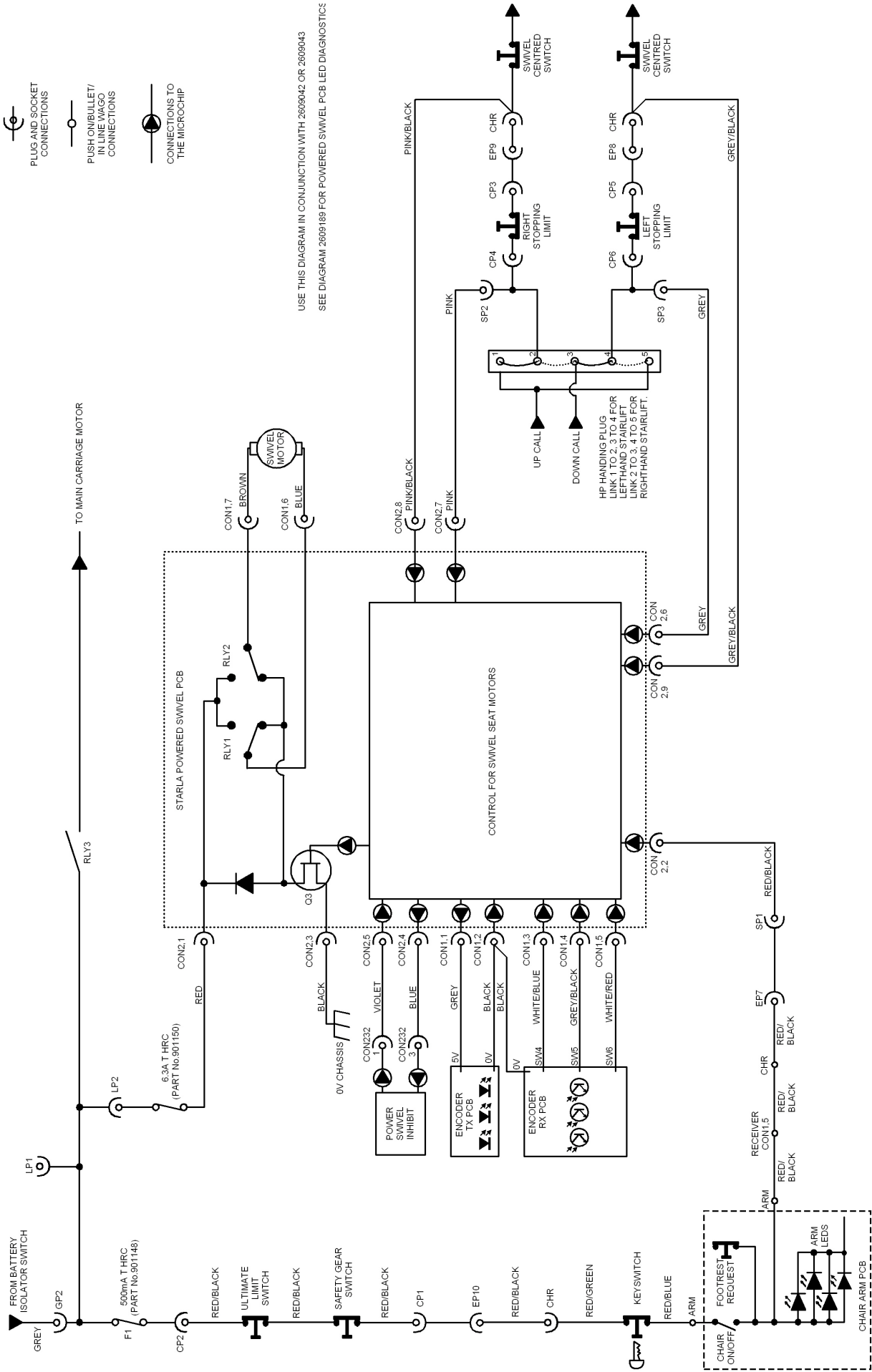260 STARLA KEY WIRING DIAGRAM - MANUAL FOOTREST


USE THIS DIAGRAM IN CONJUNCTION WITH 2609042 OR 2609043
 SEE DIAGRAM 2609189 FOR POWERED SWIVEL PCB LED DIAGNOSTICS

STARLA 260 POWERED SWIVEL KEY WIRING DIAGRAM 2609186

260 STARLA KEY WIRING DIAGRAM - POWERED SWIVEL

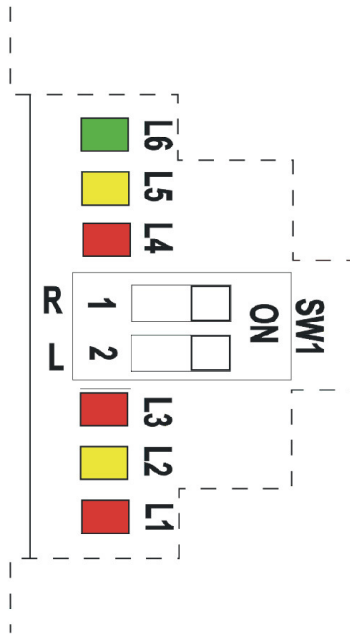Powered Swivel Diagnostic & Configuration Diagram 2609189

DIAGNOSTICS

LED No.	DESCRIPTION	FAULT CLEARING
L1 (Red LED)	FLASHING = MOTOR FAULT ON = PCB FAULT	REMOVE AND RE-APPLY POWER REMOVE AND RE-APPLY POWER
L2 (Yellow LED)	ON = SWIVEL CENTRED	
L3 (Red LED)	FLASHING = OVER CURRENT FAULT ON = LOW VOLTAGE FAULT	REMOVE AND RE-APPLY CALL REMOVE AND RE-APPLY CALL
L4 (Red LED)	FLASHING = ENCODER SEQUENCE FAULT ON = MISSING ENCODER FAULT	REMOVE AND RE-APPLY POWER* REMOVE AND RE-APPLY POWER
L5 (Yellow LED)	ON = VALID CALL EXISTS	
L6 (Green LED)	FLASHING = PCB OK AND COMMS OK ON = NO COMMS WITH CARRIAGE CONTROL PCB	

*If resetting power does not clear encoder sequence fault, this is due to the fault occurring during an inwards swivel operation. To rest fault, use the following method:

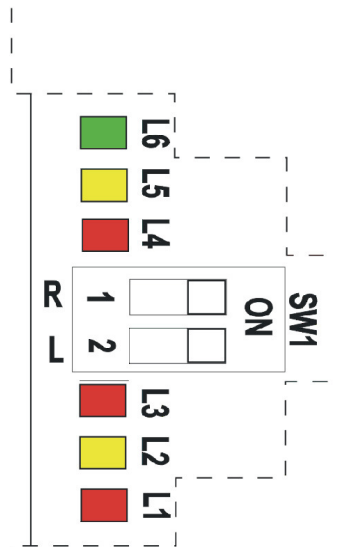
- Switch off power
- Set both switches of SW1 to OFF position
- Switch power ON
- Switch power OFF and reset switches to desired configuration (see direction setting below)
- Switch power ON

SWIVEL DIRECTION SETTING

The default of the swivel board dip switch (SW1) allows outward swivel in both directions. To reconfigure the swivel to operate as required, use the following procedure:

- Switch OFF power
- Set 'R' switch of SW1 to OFF position to disable right outward swivel or set 'L' switch of SW1 to OFF position to disable left outward swivel
- Switch power ON
- Check for correct function of swivel system

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