GB

Model 260 Siena Chair Installation Manual

To be used in conjunction with
260 Installation Manual

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GB

Introduction

The information contained within this manual has been designed for use by Stannah approved Technicians / Engineers who have received 260 product training.

It details the fitting of the Siena chair to the Model 260 and should be used in conjunction with the 260 Installation manual.

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GB

Installation Tools

1. Torque Wrench
2. 6mm Hexagonal bit
3. Hexagonal wrench (Allen Key)
4. Pozi screw driver
5. Flat blade terminal screw driver
6. 7mm Nut Runner
7. T25 Torx driver

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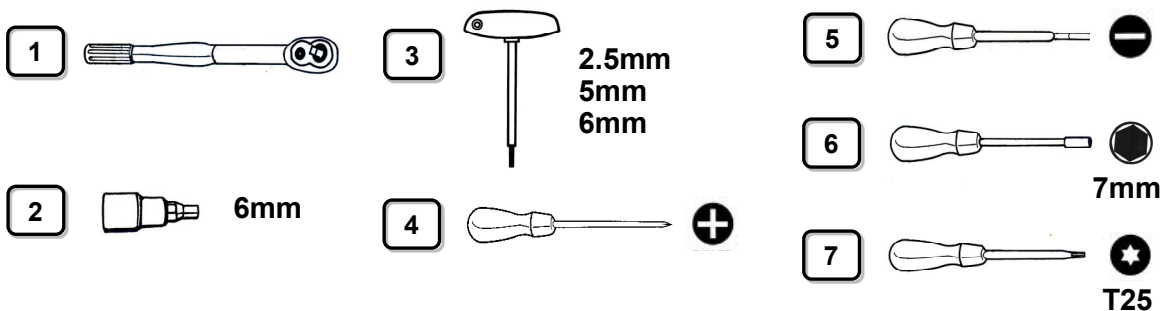
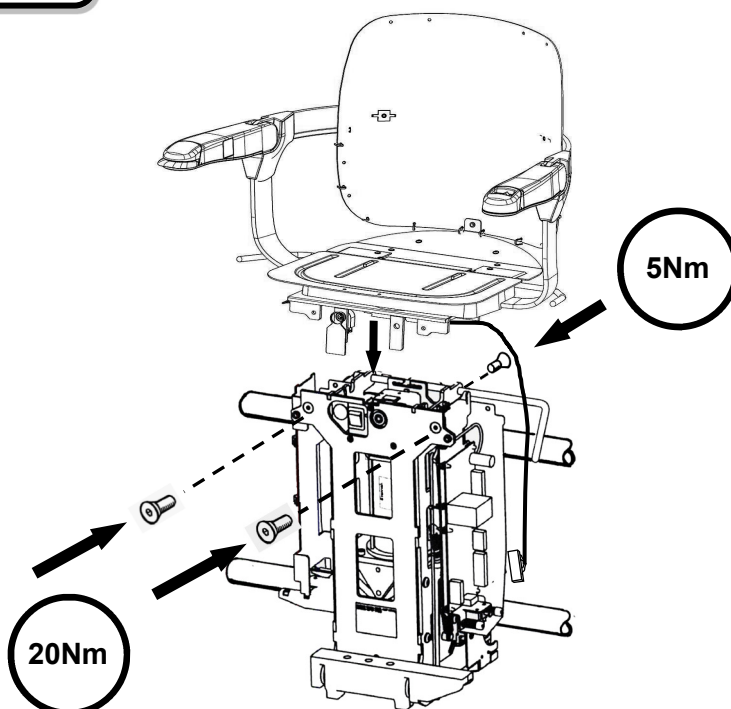
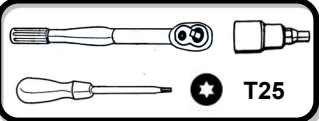
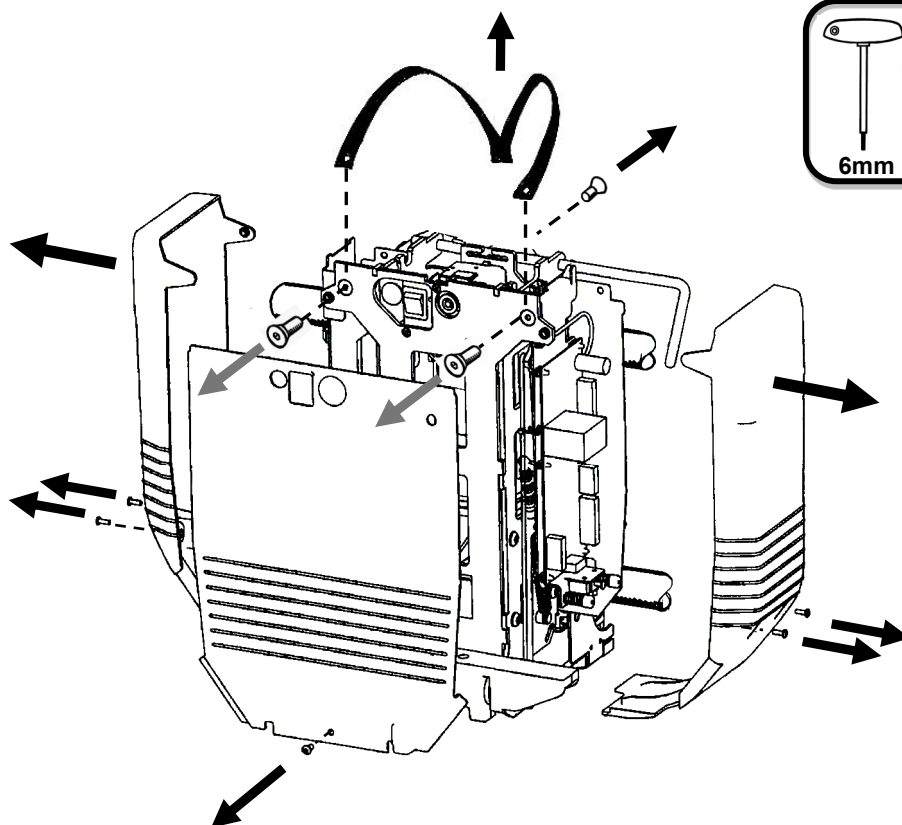
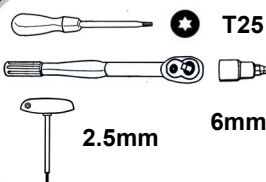
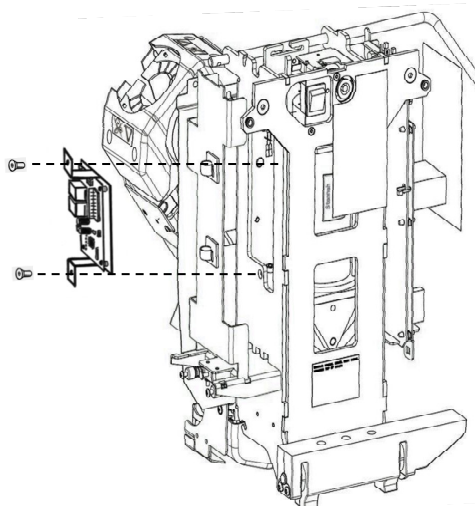
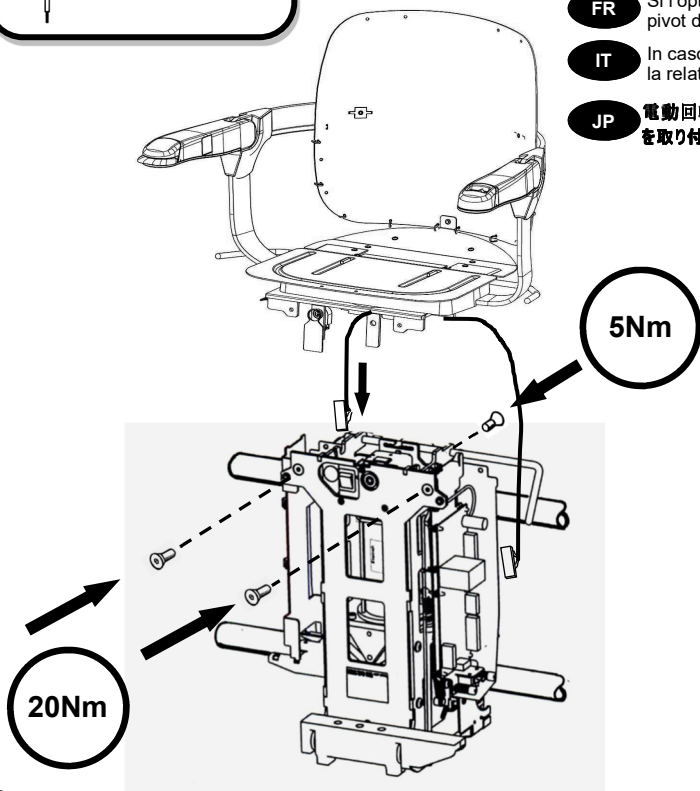
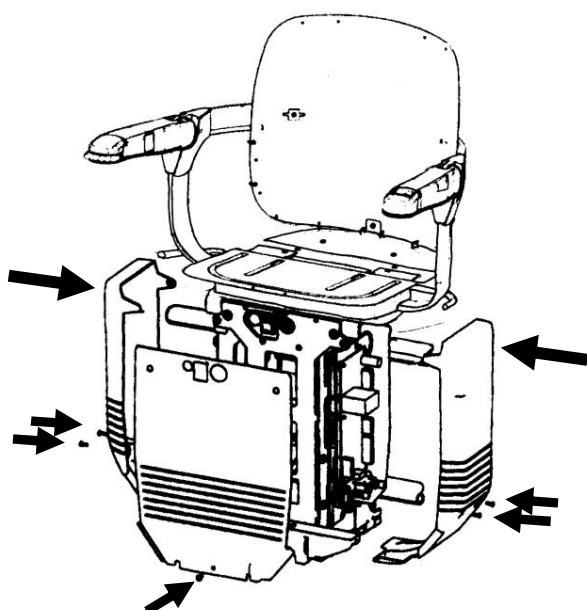
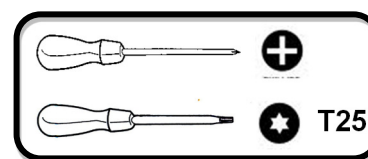
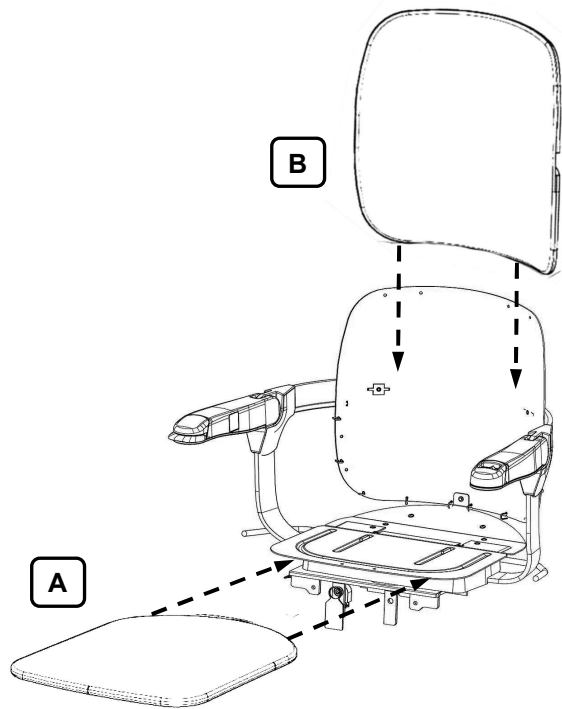
GB

Fitting Upholstery

1. Fit chair base upholstery (A) cover and secure with magnets
2. Slide upholstery cover (B) over chair back, observing cable route. Secure with magnets

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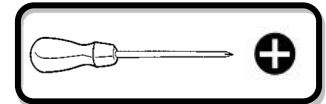
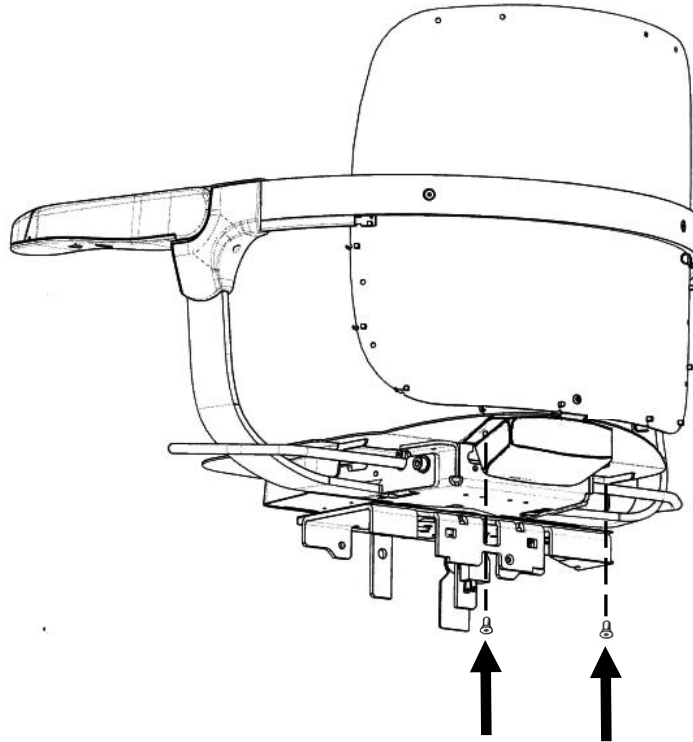
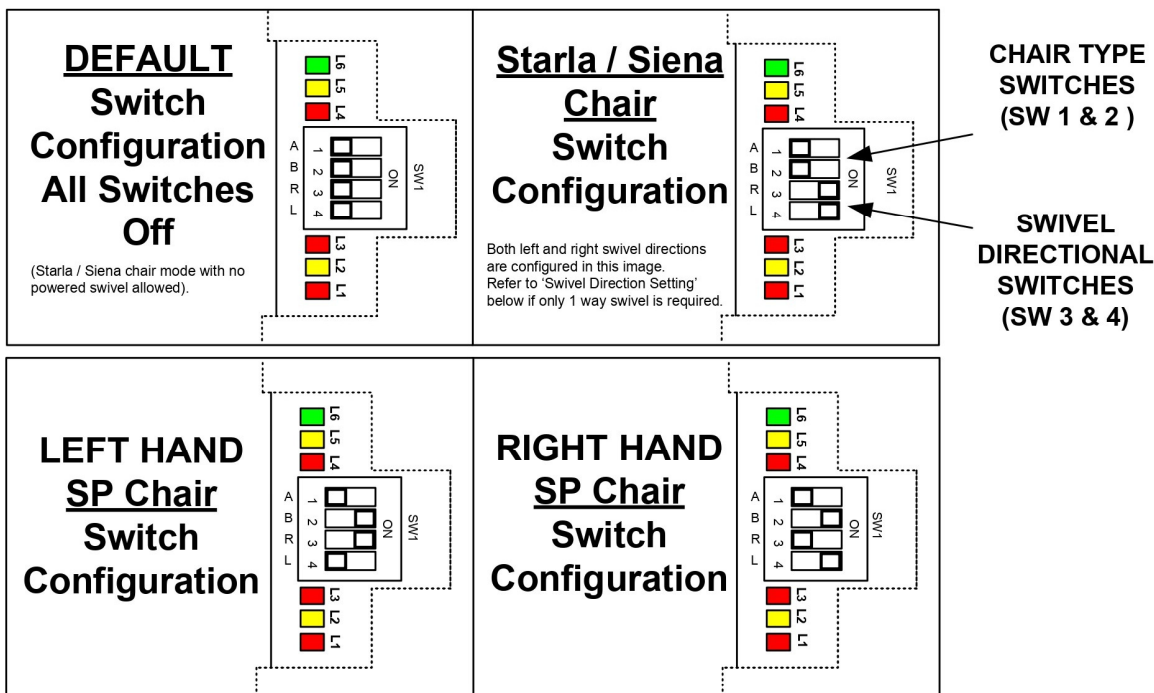
LED NO.	DESCRIPTION	FAULT CLEARING
L1 (RED LED)	FLASHING = MOTOR FAULT ON = PCB FAULT	REMOVE AND RE-APPLY POWER REMOVE AND RE-APPLY POWER
L2 (YELLOW LED)	ON = SWIVEL CENTRED	
L3 (RED LED)	FLASHING = OVER CURRENT FAULT ON = LOW VOLTAGE FAULT	REMOVE AND RE-APPLY CALL REMOVE AND RE-APPLY CALL
L4 (RED LED)	FLASHING = ENCODER SEQUENCE FAULT ON = MISSING ENCODER FAULT OR INCORRECT DIL SWITCH COMBINATION.	REMOVE AND RE-APPLY POWER* REMOVE AND RE-APPLY POWER
L5 (YELLOW LED)	ON = VALID CALL EXISTS	
L6 (GREEN LED)	FLASHING = PCB OK AND COMMS OK ON = NO COMMS WITH CARRIAGE CONTROL PCB	

* IF RESETTING POWER DOES NOT CLEAR ENCODER SEQUENCE FAULT, THIS IS DUE TO THE FAULT OCCURRING DURING AN INWARDS SWIVEL OPERATION. TO RESET FAULT, USE THE FOLLOWING METHOD;

- SWITCH OFF POWER
- SET SWITCHES 'L' AND 'R' OF SW1 TO OFF POSITION
- SWITCH POWER ON
- SWITCH POWER OFF AND RESET THE 'L' AND 'R' SWITCHES TO DESIRED SWIVELLING CONFIGURATION (SEE DIRECTION SETTING BELOW)
- SWITCH POWER ON

SWIVEL DIRECTION SETTING

THE DEFAULT SETTING OF THE SWIVEL BOARD DIP SWITCH (SW1) DOES NOT ALLOW ANY SWIVEL OPERATION IN EITHER DIRECTION. TO RECONFIGURE THE SWIVEL TO OPERATE AS REQUIRED, USE THE FOLLOWING PROCEDURE;

- SWITCH OFF POWER
- SET THE 'L' SWITCH OF SW1 TO ON POSITION TO ENABLE LEFT OUTWARD SWIVEL OR SET THE 'R' SWITCH OF SW1 TO ON POSITION TO ENABLE RIGHT OUTWARD SWIVEL
- SWITCH POWER ON
- CHECK FOR CORRECT FUNCTION OF SWIVEL SYSTEM


CARRY OUT A FULL FUNCTIONAL TEST TO ENSURE THE STAIRLIFT SWIVEL FUNCTIONS OPERATE AS REQUIRED.

NOTE – IF THE SWIVEL SELECTION FOR SP CHAIR IS INCORRECT, THE SWIVEL FUNCTION WILL NOT WORK AND THERE WILL BE NO FAULT LED'S DISPLAYED TO INDICATE THIS.

A	10773	N/A	FIRST ISSUE	A.M.	15/03/18
REV	ECN No.	DPMEA No.	DESCRIPTION	DRN	DATE

DRG TITLE : 260 / 420 OPTIONS SWIVEL PCB DIAGNOSTIC & CONFIGURATION DIAGRAM

Stannah	HEAD OFFICE Watt Close, East Portway, Andover Hampshire SP10 3SD England Tel : +44(0) 1264 364311 Fax : +44(0) 1264 339198	DRAWN BY :	A.M	DATE:	15/03/18
		CHECKED BY :	M.S	DATE:	15/03/18
		DRAWING NUMBER		REV	PAGE 1
		A034560 / C012683		A	OF 1 PAGES 1

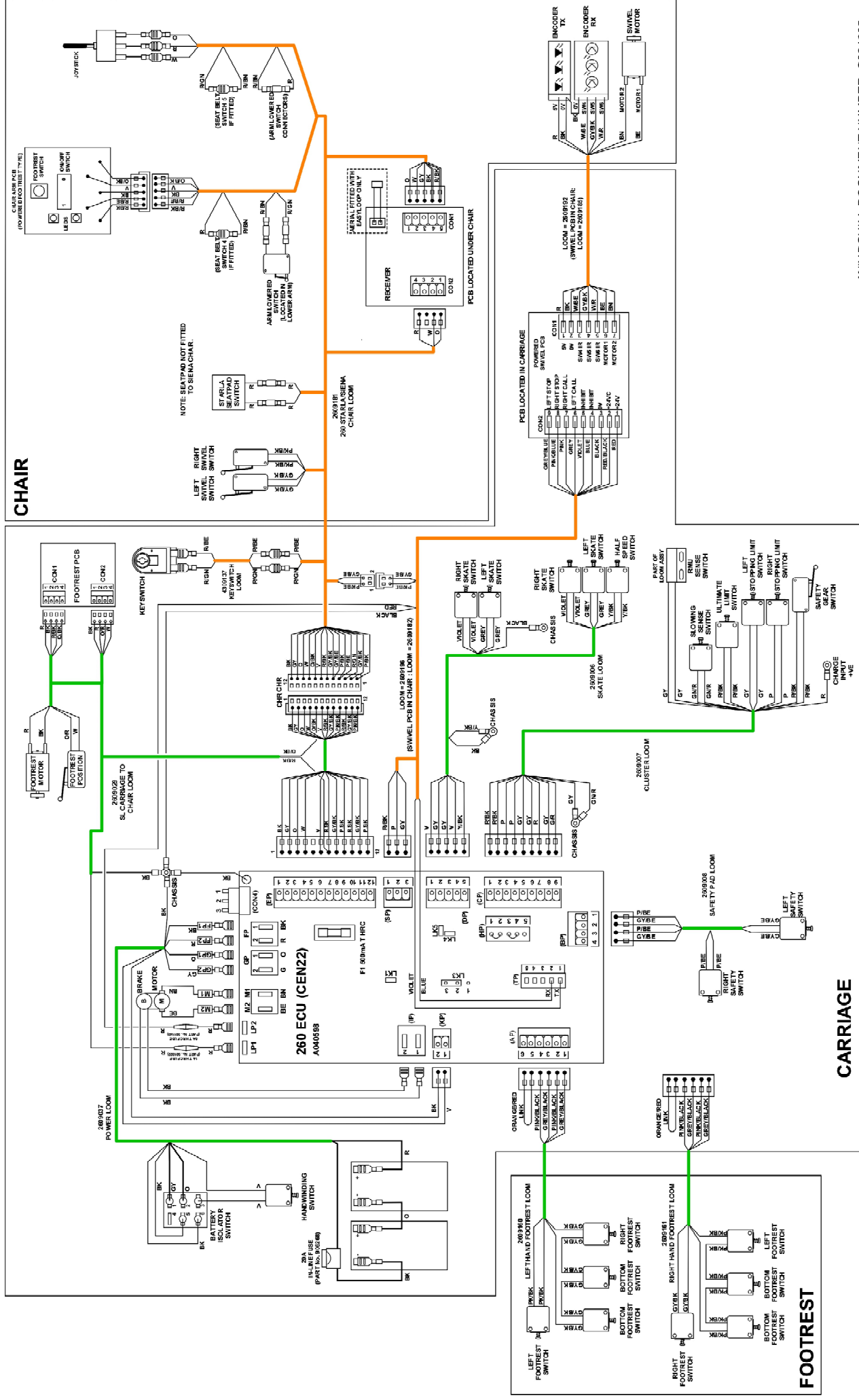
WINDCHILL DOCUMENT NUMBER: C001005

DRAWN BY : W.KLEINJAN	DATE : 16/02/2021
CHECKED BY : A.MCKELLAR	DATE : 17/02/2021
DRAWING NUMBER	REV
2609373	D

Stannah

D	12600	-	KY22 DETAIL S ADDED	WK	190223
B	3472	-	SIENA C-HAIR DETAILS ADDED	FCI	20-06-12
C	3266	-	KE-SWITCH TOWLS, BESPOKE SWIVEL SWITCH & LOCK ADDED	FCI	8-01-11
A	3315	-	RELIEF ASSESSMENT DONE ON	MH	08-2-10
REV			DESCRIPTION	DPN	DATE
			ECN NO.		

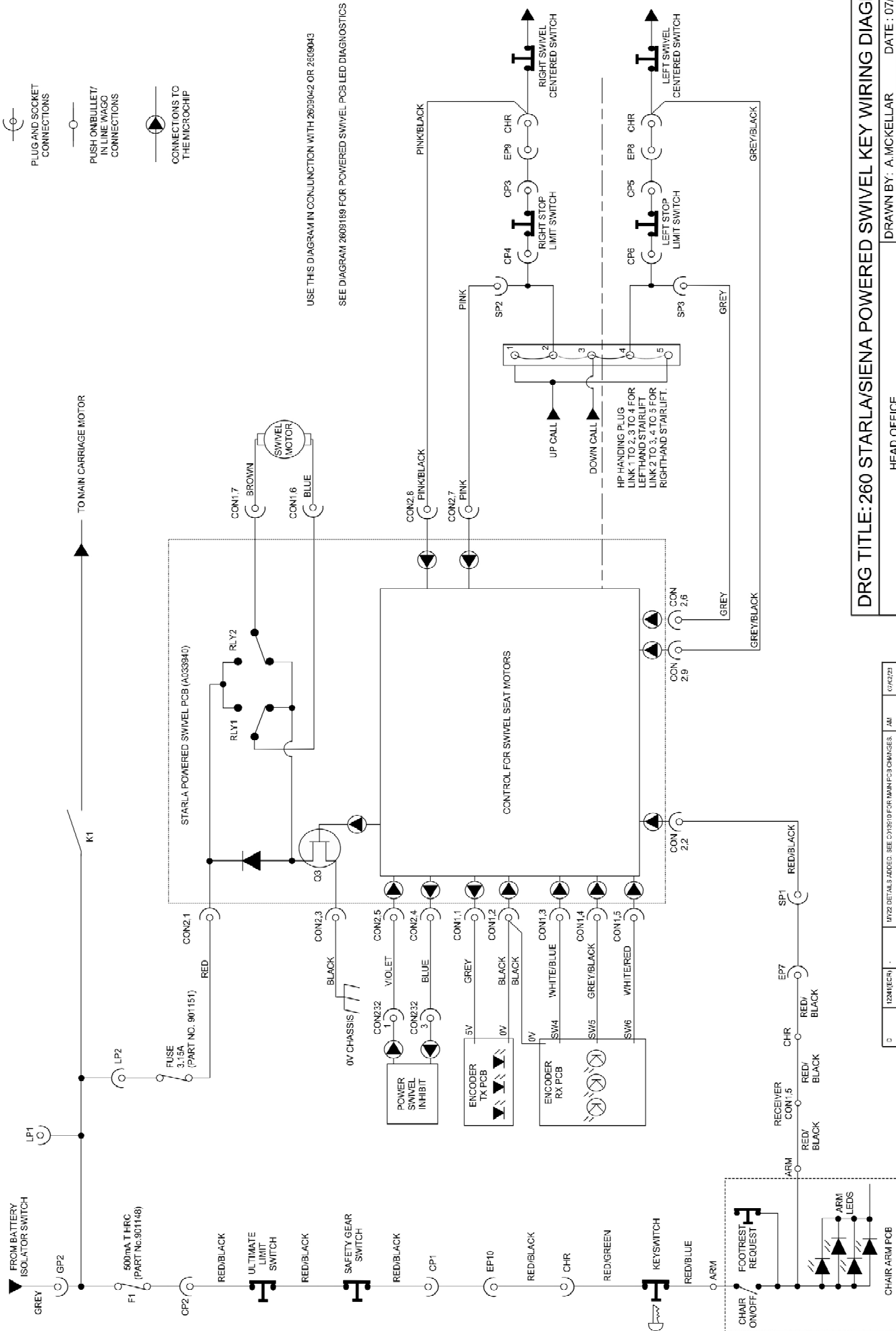DRG TITLE: 260 STARLA/SIENA POWERED SWIVEL KEY KEY WIRING DIAGRAM

Stannah	HEAD OFFICE	DRAWN BY: A. MCKELLAR	DATE: 07/02/23
	Watt Close, East Portway, Andover	CHECKED BY: M. SYMONDS	DATE: 07/02/23
	Hampshire, SP10 3SD England	DRAWING NUMBER	REV
	Tel : +44(0)1264 364311	2609186	D
	Fax : +44(0)1264 339198		

REV	ECN No.	DRIVER No.	DESCRIPTION	DRN	DATE
1	1024/ECN	-	MY22 DETAILS ADDED. SEE CDRS/10 FOR MAIN PCB CHANGES.	AM	07/02/23
2	3765	-	FUSE CHANGED FROM 1.5A TO 3.15A.	BJH	27/02/24
3	3472	-	SIENA ADDED TO TITLE	RCJ	28/06/22
4	3315	-	RELEASED ON EGN	RCJ	09/12/20

'Original Instructions'