



Hurtigguide i-Reader 2

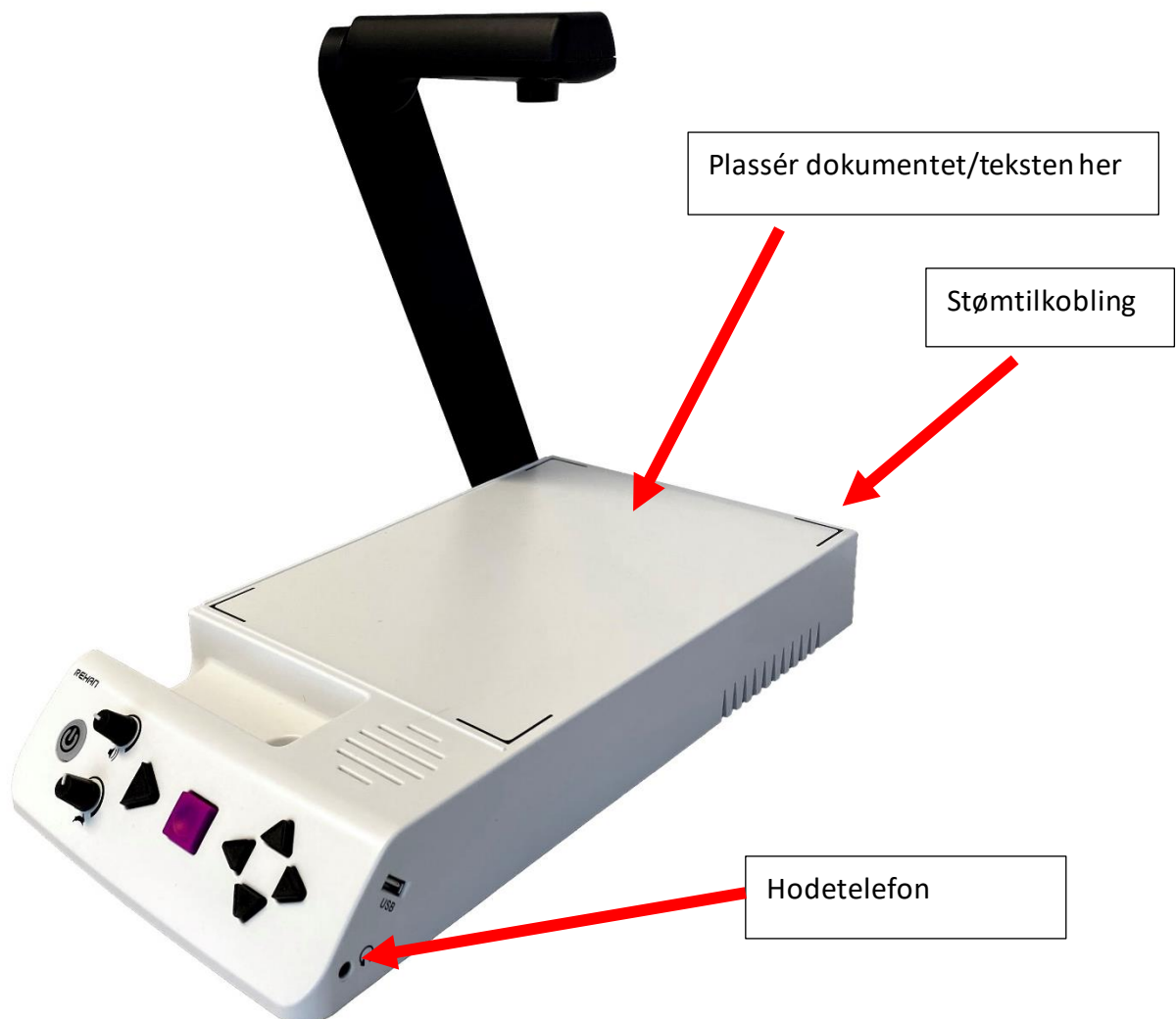

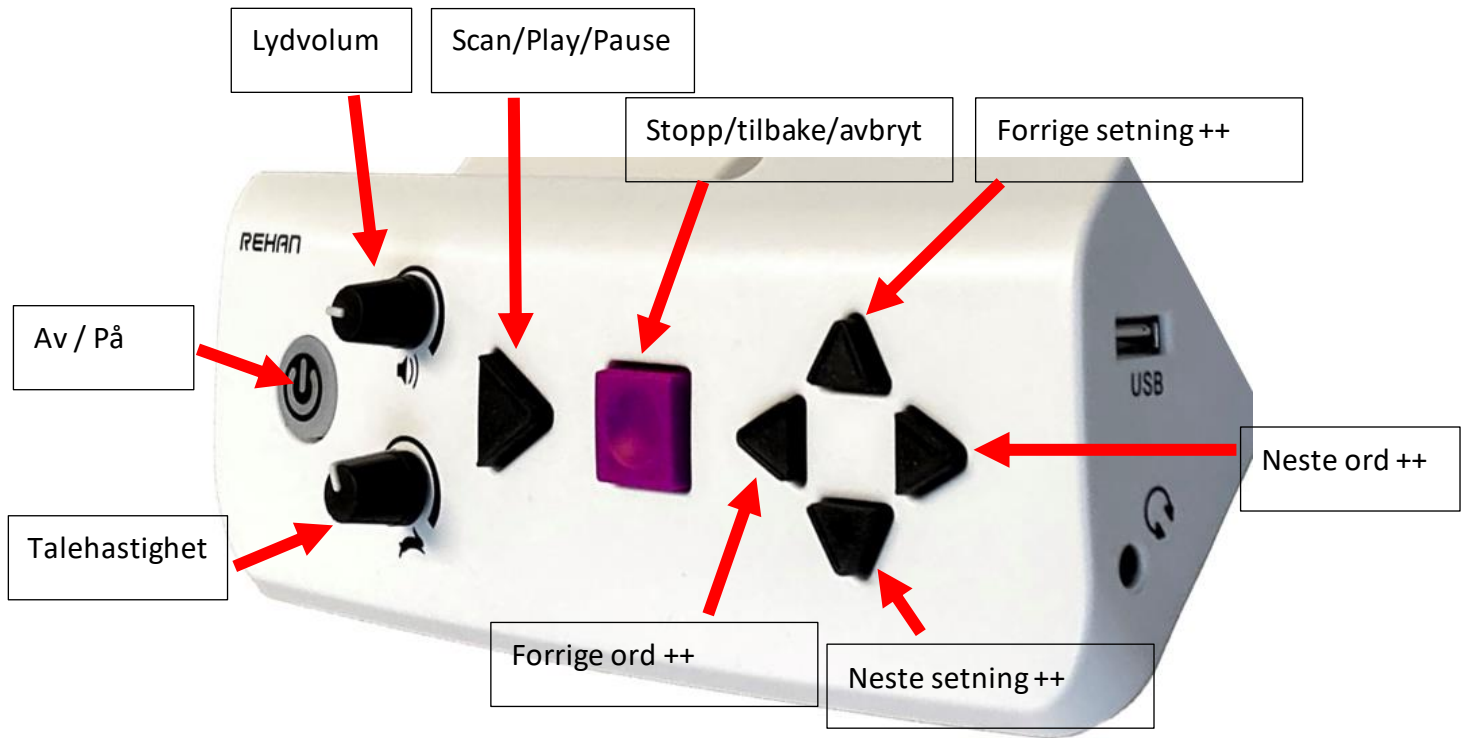

2 sec: Power button
Short press Battery status



Scan / Play / Pause
"OK/continue"



Stop
"Back/cancel"



Save document



Browser



Menu



No function



Date and Time



Capture column mode



E-manual



Next word or zone (hold)
Next page (browser)
Menu item open



Previous word or zone (hold)
Previous page (browser)
Menu item open



Previous sentence or page (hold)
Previous book (browser)
Menu item up



Next sentence or page (hold)
Next book (browser)
Menu item down



Back-up



Re-store back-up



Import selected
document from USB
Export selected document
from internal memory



Automatic language detect ON/OFF



Previous language



next language



Previous voice
Delete document



next voice