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### Disassembly and assembly instructions variana

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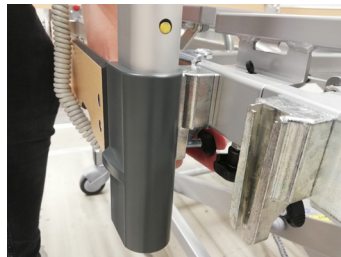
Brake the bed and move it to disassembly position. For this purpose, the cross strut of the scissor frame must be at the level of the marking on the chassis. Then dis-connect the bed from the power supply and hang the power cable on the headboard using plastic hooks.

### Demontering av vertikaldelte grinder | Disassembly split side guards

Løse (men ikke fjern!) de riflede skruene på begge sider av sengegrinden. Grindene kan dermed enkelt løftes ut av holderen.

Loosen but do not remove the knurled screws on both sides of the side guards. The side safety element can then be easily lifted out of the holder.

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Press the release button at the uppermost bar of the side guards and let all spars sink down. Then press a pin at the lower end of the metal guide (e.g. with a screwdriver) to let all spars slide down out of the guide.

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
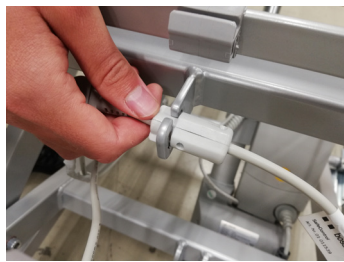
Open mini grid (pull and turn 90°) and remove head and foot section upwards from the holder.

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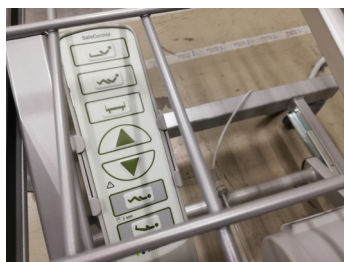Loosen and fold down the wire mesh of the seat section on one side and remove the hand switch cable from the holding rounds on the lying surface.

Sett håndkontrollen i holderen på sakserammen.

Insert the hand switch into the holder provided for this purpose on the scissor frame.

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
Loosen the wire mesh from the thigh rest on one side and fold it over and loosen the motor cover with a screwdriver (or coin).

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
Remove the cover of the control box. The left plug seen from the front (see illustration) must not be pulled out, pull out all other plugs and release them from the cable guide.

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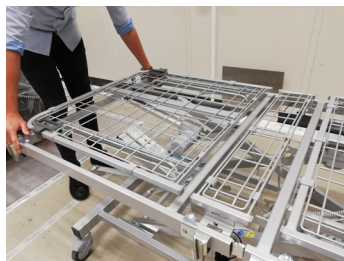
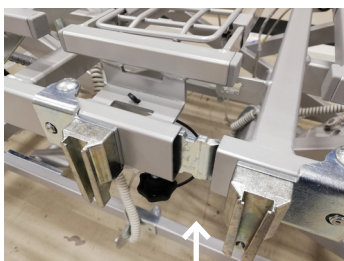
Loosen star grip on both sides of the upper frame. Pull out head-side lying surface element.

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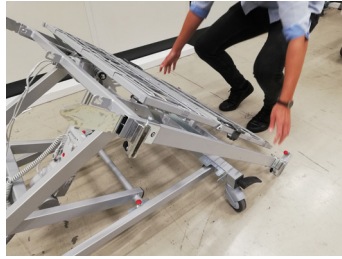
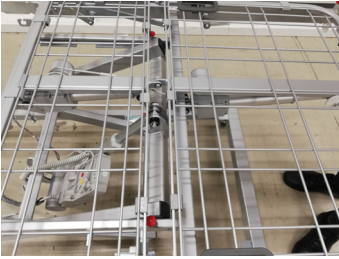
Pull out the lying surface element completely until it is released from the foot element. Then first lower it to the floor and then pull it up on the top side and remove it.

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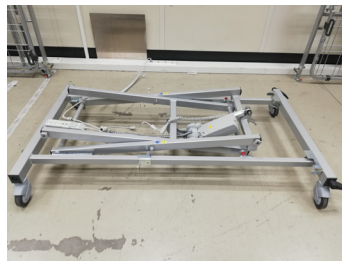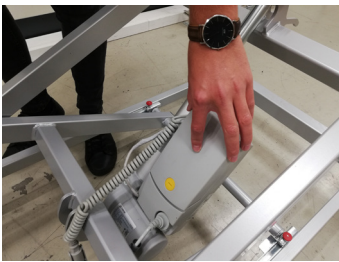Open the two red mini grids on the foot-side lying surface element and pull out, lower and remove it identically to the head-side element.

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Replace the cover of the control box and reconnect the bed to the power supply and move it to its lowest position.

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Optionally, the scissor rack can still be removed by opening the two red mini grids on the lower frame.

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Transport brackets for the lying surface elements are located on the side of the chassis. These are first placed on the middle rail and then pushed into the head or foot-side holder.

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Loosen the transport bracket on the inside of the lying surface elements and hook it in on the opposite side.

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The side rails, head and foot sections can then be hooked into the transport brackets. Hang the power cable on the bracket for transport.

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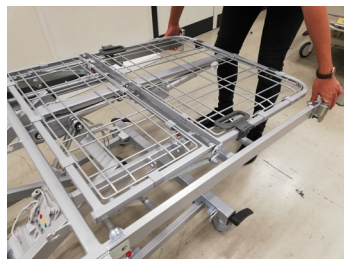
Brake bed, connect power plug and move into assembly position. Remove cover from control box.

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Slide the bolts of the scissor frame into the mounting rails of the foot-side lying surface element. Close the mini grid so that the element snaps into place with an audible click.

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Repeat the step with the head-sided element, put it together with the foot-sided element and screw it with the star grip.

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Loosen wire mesh of thigh rest and seat section on one side from the fastening and fold it down. Insert the plugs of the lying surface motors into the control box accordingly and insert the hand switch cable into the brackets. Click the cap back on the control box and close the wire mesh.

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Hook the head and foot board into the holders and close the mini grid.

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Insert the side safety elements into the receptacles on the upper frame and tighten them with the ratchet screws.

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Using a screwdriver to press the pin at the bottom of the metal guide down to insert all spars into the rail.